hiPeE

USER'S GUIDE

Content

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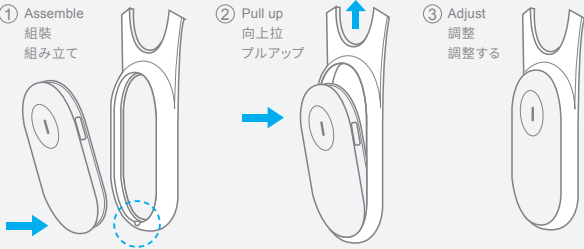
Wear istration

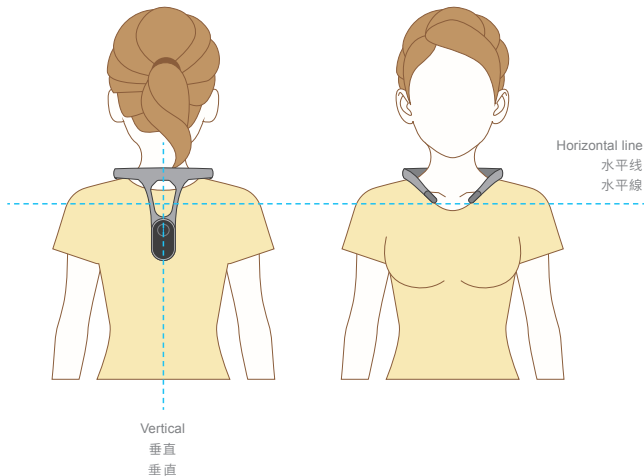
please refer to below image for wearing

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1

Power on / Power off

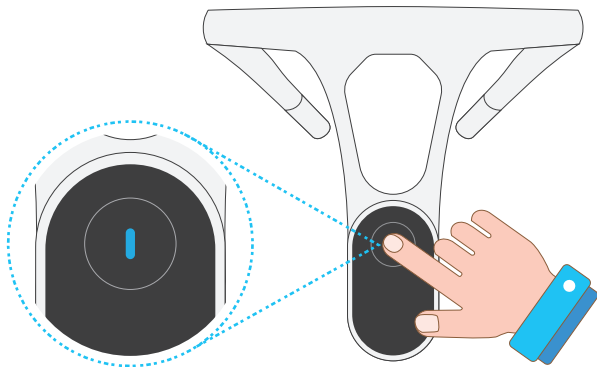
Press the button for 3 seconds to turn on/off the device.

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Connect to mobile phone

Scan the QR code to install the App and operate as instructed.

Noted: Turn on bluetooth and GPS of mobile phone before connecting.

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Personalize reminder

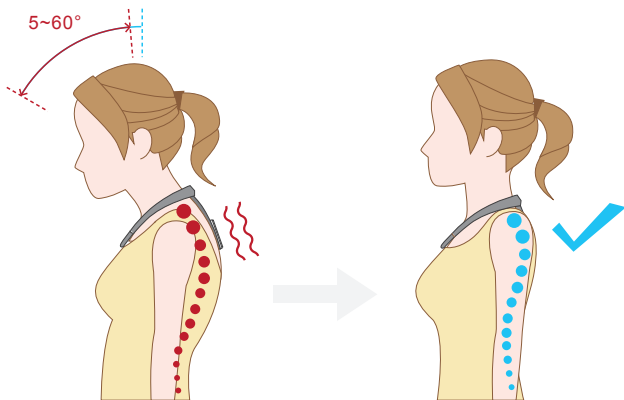
Wear it and set the correct posture, angle and delay the reminding time according to the prompts; when the incorrect posture duration, the device will "Vibretion".

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Rest awhile

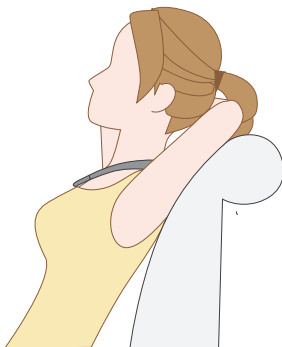
Double-click the button to pause the reminder for 30 min, during which time double-click the button again to resume the reminder.

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Operation and feedback

Device Status	Operating	Vibrations	Indicator lights
Power on	Hold down the button for 3 seconds	Fully charged: turn on after 1 long shock Low battery: Shut down after 1 long shock	Fully charged: Detection mode: blue light keeps on for 5 seconds and then flashes Rest mode: purple light keeps on for 5 seconds and then flashes Low battery: red light flashes Insufficient battery: the device shuts down after the red light flashes twice
Power off	Press and hold the button for 3 seconds after powering on	1 short shock	*
Bluetooth connecting	Bind device in applet	2 short shock	*
Correct posture	Software setting standard posture	Start /Successful : 2 short shocks	Start recording: blue light flashes quickly Record success: blue light flashes
Detecting mode	When bad posture occurs	After 3 s by default Short shock 3 times and repeat	*
Long-sitting reminder	After sitting for 1 hour	3 long shocks	*
Rest mode	Double-click the button	1 short shock	*
Recover detecting mode	In rest mode Double-click the button	2 short shock	*
Check device status	Short press the button	*	Detection mode: blue light on once Rest mode: purple light on once
Charge	*	*	Charging: red light flashes Charging completed: blue light is on

Cautions

1. When app is unavailable, if you want to record your posture, you can press the button for 8 seconds (please don't let go once in the middle), and the equipment vibrates twice to start recording. After recording, the equipment vibrates twice briefly, indicating that the standard posture record is successful.
2. The vibration modes of posture reminder and sedentary reminder can be modified through "Settings" in the software.
3. When Bluetooth is bound, the short shock twice indicates that the device has been found, and click the device button to bind; When entering the software again, the device can be connected directly.
4. Equipment and app are continuously updated and upgraded. Please refer to the actual version for specific operation.

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Charging instruction

1. When charging, please push the back of the machine forward and remove it from the silicone part to see the charging port. (As follows)
2. This device uses a Type-C port for charging, and the power adapter should meet the following requirements, Rated input: 5 V \equiv 1 A
3. After charging, please unplug the power as soon as possible and stop charging, so as not to damage the lithium battery due to overcharging. Please dry your hands before plugging or unplugging the power adapter.
4. The battery cannot be replaced. When the battery life expires and cannot be charged, the product will be scrapped. Discarded products should be disposed of in accordance with the electronic product disposal regulations stipulated by the state.

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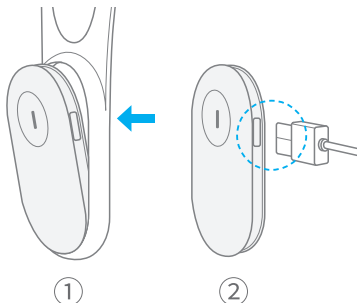
Parameters

Product name: Smart Posture Corrector

Product model: P1

Product size: (Adult) 137×167×32mm
(Kid) 120×132×32mm

Product weight: (Adult) 60 g (Kid) 51 g
battery and neck loop included

Power supply: internal power supply

Rated input: 5V  1A

Operating time: 50 h

Charging time: around 1.5 h

Working environment:

temperature: -10 °C~+55 °C

relative humidity: ≤ 80%R.H.

atmospheric pressure: (70.0~106.0)kPa.

Storage/transportation conditions:

temperature: -10°C~+55°C

relative humidity: ≤ 80%R.H.

atmospheric pressure: (70.0~106.0)kPa.

Safety Precautions

1. In order to use this product safely and correctly to prevent harm to you and others, please read this manual carefully.
2. This product is recommended for children, adolescents and adults 3 years old and above.
3. The silicone accessories are soft, please put them out of the reach of children to avoid accidental eating by children.
4. Do not use this product in strong electromagnetic fields.
5. The use of unspecified accessories, transducers or cables together with equipment and systems may result in an increase in the emission of the equipment or system or a decrease in immunity; even if other equipment meets the emission requirements of the corresponding national standards, the equipment or system will still be interfered by other equipment.
7. Please follow the operating temperature and humidity and storage temperature and humidity recorded in this manual, otherwise the system may not reach the claimed performance specifications.
8. The battery cannot be replaced, please do not disassemble or repair it yourself. If you have any questions, please contact the after-sales unit.
9. This warranty applies to other regions except China.

Warranty description

1. From the date of purchase of this product, within the scope of normal use of the product in accordance with the instructions, if the product has performance failures that are not artificially damaged, the warranty period is 6 months. After 6 months, repairs will be charged.
2. Repairs that are not caused by the quality of the company's products, such as improper use, improper storage, unauthorized disassembly, etc., will be charged for repairs and the freight will be borne by the user.
3. The following conditions are not covered by the free warranty: damage caused by man-made damage, such as damage caused by falling, touching, water immersion, damp, high temperature; damage caused by not following the correct operation of the instruction manual; damage caused by accident; Without the written authorization of the company, the user disassembles and changes the product without authorization, and there is no proof of purchase.
4. This warranty does not apply to China.

Cleaning and maintenance

1. The silicone accessories can be disassembled separately for cleaning and cleaning. The machine is not waterproof, please do not wash it with water; if there is dust or stains, you can wipe the machine with a clean dry cloth or a wrung wet cloth.
2. When cleaning, be sure to shut down and unplug the power adapter.
3. Do not use any cleaning agents or cleaning agents for cleaning. If there is dirt that is difficult to clean, you can wipe it with a disinfectant containing alcohol and other volatile substances. Put it in a cool place and wait for the substance to evaporate before assembling.
4. If the product is not used for a long time, the battery should be charged regularly.

Storage:

1. Do not store the product in high temperature, humidity, direct sunlight, dusty, and salty places; do not place it in a place where it is inclined, vibrated, or impacted; keep away from chemicals or corrosive gases.
2. Avoid hard and sharp materials contacting the silicone fittings to avoid scratches.
3. The cleaned product must be thoroughly dried and stored in an environment that meets the requirements: Storage conditions: temperature: $-10^{\circ}\text{C}\sim+55^{\circ}\text{C}$; relative humidity: $\leq 80\%$; atmospheric pressure: $(70.0\sim 106.0)\text{ kPa}$

Support

The final interpretation right belongs to Tianjin Fruitech Technology Co., Ltd.

If you have any questions, please contact us

Customers in Mainland China can call 400-900-5115 (9:00-18:00)

Email to feedback@fruitech.cn for other regions

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Examiner:

FCC Statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
 - Increase the separation between the equipment and receiver.
 - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
 - Consult the dealer or an experienced radio/TV technician for help.
- The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

©2021 Tianjin Fruitech Technology Co.,Ltd. All rights reserved.

G-1101-15 & G-1101-32, No.6 Haitai Development Ave., Huayuan Industrial Development Zone

Tianjin, Tianjin, 300384

Made in China

www.fruitech.cn

(E-mail) feedback@fruitech.cn

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Manufacturer: Tianjin Fruitech Technology Co.,Ltd.