# Bed Wetting Alarm

### Product Composition

The urine alarm consists of three parts:

- Transmitter (urine detection):** detect urine and send alarm information to receiver by wireless way.
- Receiver:** The receiver is used to receive the alarm information from transmitter and notify the vibration or sound. It has the function of temperature and humidity detection and is used to detect the current ambient temperature and humidity.
- Charging box (storage box):** used for charging transmitter and receiver. When not charging, it is convenient for users to store and carry after being covered.

### Switch on / off

Turn the switch to the right to turn on the receiver, turn it to the left to turn off the receiver, and the transmitter will always turn on by default.

### Receiver alarm mode setting

**Vibration mode setting:** Click the left button, the screen displays the number 000001, and a small icon appears. Press the right button to confirm, and the system will automatically return to the main interface, and the setting will be finished.

**Temperature and humidity sensor:** Left button, Right button

### Sound mode setting:

Press the left button twice, the number 000002 will be displayed on the screen, and a small icon of Celsius will appear. Press the right button to confirm, and the system will automatically return to the main interface, and the setting will be finished.

**Vibration plus sound mode setting:** Press the left button three times, the number 000003 will be displayed on the screen, and a small icon of Celsius will appear. Press the right button to confirm, and the system will automatically return to the main interface, and the setting will be finished.

### Temperature display mode

**Fahrenheit display:** Press the left button four times, the number 000004 will be displayed on the screen, and a small Fahrenheit icon will appear. Press the right button to confirm, and the system will automatically return to the main interface, and the setting will be finished.

**Celsius display:** Press the left button four times, the number 000004 will be displayed on the screen, and a small icon of Celsius will appear. Press the right button to confirm, and the system will automatically return to the main interface, and the setting will be finished.

### Use of urine detection module

Open the button of urine detection module, and buckle the detection module on underwear along the edge of underwear.

### Alarm prompt

When the system gives an alarm, a small icon appears. The alarm vibrates or sounds or vibrates + sounds according to the alarm mode set by the user. The user presses any key to cancel the alarm, and the alarm icon disappears and returns to normal.

### Product charging

Put the receiver and transmitter into the charging box, and plug in USB for charging until the receiver battery is fully charged. When the indicator light of the transmitter turns from red to green, the charging is finished.

### Matters needing attention

1. When using the receiver's temperature and humidity function, the receiver shall not contact or be placed in a special environment such as a pocket, and the hole position of the temperature and humidity sensor shall not be blocked. The receiver's temperature and humidity function is invalid during charging.

2. The product is charged by standard DC5V.

3. After charging, the receiver can be turned off when the product is not in use to save power consumption.

### Importance

It takes at least one week for parents to fully wake up their children after they fall asleep for about one to two hours. And the alarm will help wake up the children who sleep too well. By taking a nap during the day, make sure that the child will not be exhausted when sleeping at night, which will help the alarm to wake up the child. When you hear the alarm every night, remind your child to stop urinating and going to the toilet. Before going to bed at night, it is not necessary to limit children's liquid intake.

### Importance

In the first few weeks, when the alarm sounds, the child may be woken up by the alarm, so the child needs the help of parents: wake up the child and let him go to the toilet. The process of children's participation is indispensable for their success. Don't unplug the alarm for your child. It's better you're your child to notice that the alarm sounds and remove or disconnect the alarm by himself. Parents need to help the child to go to the bathroom to discharge urine. In the first one to two weeks, children may be completely wet in bed, but parents still need to continue to emphasize the importance of going to the toilet and encourage them. Ask the children if they remember the alarm and what they did the next day.

### Importance

Parents' participation is also essential for the success of this program. In the first few weeks, children need parents' help and encouragement. This plan may be slow and frustrating, but parents insist on using the alarm every night, and the alarm can sound normally during every training course, so parents should help to wake up their children. You need to continue using the alarm for 3 to 4 weeks, and make sure that your child will not release after the last alarm. Recurrence refers to a child's continuous wet urine after one or more months. If this happens, please start using the alarm again immediately, so that the recurrence interval will become longer and longer.

### Plan

After the alarm sounds, reconnect the alarm to the child and put on a new pair of underwear. Use an all-night light in the child's room, which will help the child wake up more easily when the alarm sounds and find the way to the bathroom. Be sure to praise your children every time they spend a dry night or notice the alarm. It is also important not to scold children because they don't notice the alarm sound, because it takes some time to change their behavior. In addition, it is helpful to use the progress chart so that children can participate in the progress of the plan.

### Maintenance/cleaning

Remember to clean the sensor clip after every wet part. If necessary, clean the alarm once and for all. Do not immerse the alarm body in water, because it will damage the internal parts, and the sensor clip can be washed in water. Always check the battery to ensure that the alarm is still working. This can be checked by inserting a damp cloth in the middle of the clip. When the alarm is not used for a long time, please take out the battery. The alarm of our company is designed for a single user. If you want another child to use the alarm, please contact to buy another alarm.

### Guarantee

The warranty period of the wearable alarm you purchased is 30 days for defects in materials and processes of the manufacturer, and the warranty period does not include damages caused by misuse or overuse. The functional shelf life is 6 months. In case of man-made damage, our company will charge for repair or replacement of components within the warranty period according to its discretion, and the components must be returned at the expense of customers. If the components are not purchased directly from our company, the customer shall provide proof of purchase, such as sales invoice receipt or invalid cheque.

**Warning: After use, please open the transmitter clip and wipe it with dry paper towel before use.**

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## FCC Information

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates,

uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference

to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does

cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is

encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment .The device has been evaluated to meet general RF exposure requirement.**