

## Instructions for Use

1. Go to the App store. Touch the Search Icon. Type in Accelera, touch Search. You should see Accelera-Control your Accelera Device. Download the app. You may need your Apple ID password to load this app.
2. Open the box and locate the wireless charging unit. Locate the charging cable (under the box insert) and plug it into the charging unit and power supply. Plug the power supply into a standard outlet.
3. Locate the Accelera controller. Make sure the On/Off switch is in the On position.
4. Place the controller on the wireless charging unit. A blue light will come on indicating the unit is charging. when charging is complete, the light will go out. This should take about 1.5 hours.
5. To use the Accelera SR-100, remove the controller from the charger and place it in the pouch on the wrap. Connect the actuator lead to the magnetic connector on the controller and place the On/Off switch in the ON position.
6. Position the wrap on your body as instructed by your clinician.
7. Launch the Accelera app on your phone.
8. You should see a device listed. You can rename the device as you wish, i.e. Left wrist, right knee, personal name.
9. Touch the device name and the control screen should appear. The Stimulation level is controlled by the slider button. Slide it to the right until you can barely feel the vibration. If there is no vibration, make sure there is a blue light lit on the magnetic connector. If not, reposition the magnet to connect to the controller again and use the slider again.
10. Touch Begin. The timer will start documenting the length of use.
11. If you need to pause for any reason, simply touch Pause and then touch Resume when you are ready to continue.
12. When you are finished with your session, place the power unit on the charger.

Accelera, Inc.

PRODUCT NAME: SR-100

PRODUCT MODEL: SR-100

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC ID: 2A99U-SR100

### **FCC Compliance Statement**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

CAUTION: The grantee is not responsible for any changes or modifications not expressly approved by the party responsible for compliance. Such modifications could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.