



Bee+

Quick User Guide

The first Bluetooth Smart insulin injection tracker connecting your insulin pen to your smartphone



1 Charge for your device 1/2

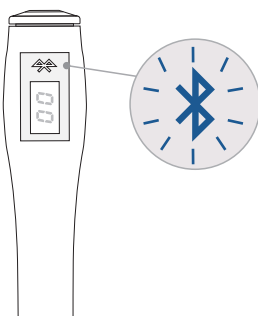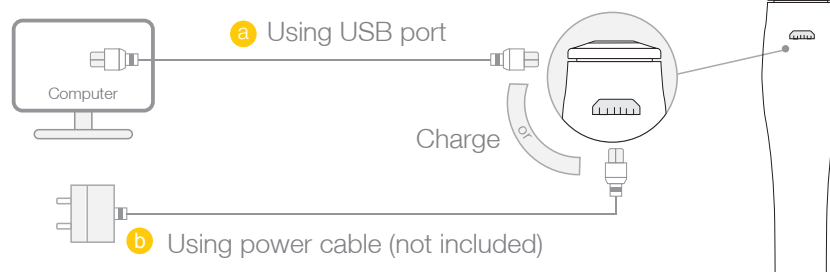

2 Charge for your device 2/2

- a Bluetooth light blinks when recharging.
- b Bluetooth light is steady on when recharging is done.

3 Download and launch the Bee+ App




- b Launch the App



- c Follow the instructions on your mobile App.




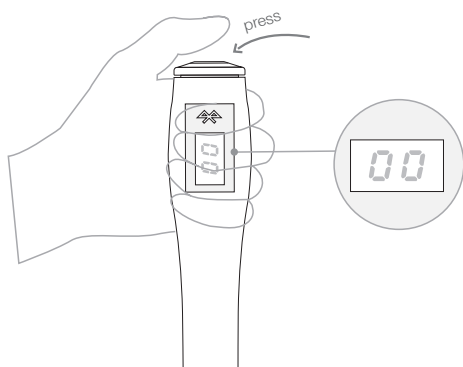


4 Wireless set up preparation

- a Long press the top button
- b The system is ready when the screen shows: "00"

5 Check the Bluetooth connection

- Bee+ is connected to the smartphone when the LED is switched on.

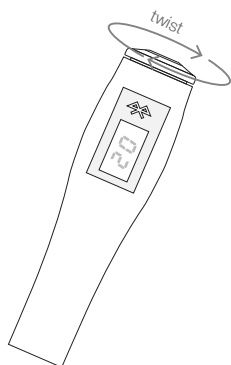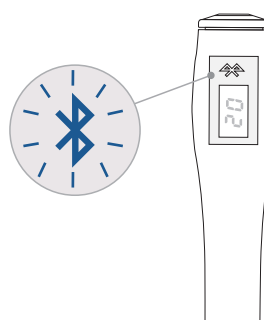


6 unit registration

- Record insulin unit by twisting at the top of the button.

7 Data transmission

- a Short press the top button to send out the data via Bluetooth.
- b Digital figures are blinking while injection data is being transmitted.

* When your Bee+ is not connected to your smartphone, the screen will display: " — — ". your data are recorded and will be sent during the next connection.

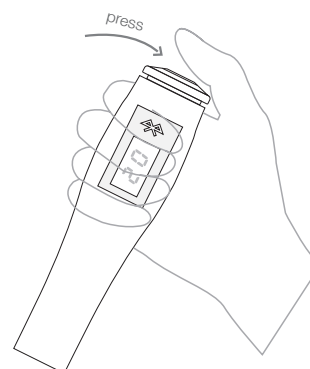

FCC Statement: This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the users authority to operate the equipment