

sound oasis®
sleep better. feel better. live better.®

Instruction Manual

Stereo Bluetooth® Sound Therapy System



FREE
Sound Oasis
APPS

Bluetooth®

www.soundoasis.com
Model: BST-400



Table of Contents

| | |
|-------------------------------|----|
| Introduction..... | 3 |
| Important Safeguards..... | 4 |
| Contents..... | 5 |
| Key Buttons & Controls..... | 6 |
| Operation..... | 9 |
| Free Sound Oasis APPS..... | 13 |
| Technical Specifications..... | 14 |
| Using As A Speakerphone..... | 15 |
| Disposal..... | 16 |
| FCC Information..... | 16 |
| Warranty..... | 17 |

sound oasis®
sleep better. feel better. live better.®

Thank you for choosing Sound Oasis®!

Sound Oasis® is the world leader in sound therapy systems.

We are dedicated to making life healthier and more enjoyable with creative sound solutions that combine superior quality with the very latest technology.

Your Sound Oasis® Stereo Bluetooth Sound Therapy System BST-400 has been designed to help you sleep better by providing sounds proven to help people sleep, relax, block annoying noise, manage tinnitus and increase privacy. You can play the built-in sounds from the included micro SD card or stream new sounds/music from any Bluetooth enabled device (e.g. smartphone, tablet, computer, etc.). You can even add new sounds by inserting a new micro SD sound card.

Your BST-400 includes a rechargeable battery so you can enjoy 5 – 8 hours of wireless playtime or you can keep it plugged in with the included USB charging cable and wall adapter.

Important Safeguards

- Read all instructions.
- To reduce the risk of fire or electrical shock, do not expose this product to rain or moisture.
- Do not expose this product to dripping or splashing, and do not place objects filled with liquids, such as vases, on or near the product. As with any electronic product, use care not to spill liquids into any part of the system. Liquids can cause a failure and/or a fire hazard.
- Do not place any naked flame sources, such as lighted candles, on or near this product.
- Do not use any charging cable other than that specifically provided for use with this product.
- The battery needs to be charged before use. Follow the charging instructions described in this manual.
- After extended periods of storage, it may be necessary to charge and discharge the battery pack several times to obtain maximum performance.
- The battery pack gives its best performance when it is operated at normal room temperature (68°F +/- 23°F or 20°C +/- 5°C).
- In the event fluid from the battery leaks from the product, do not allow the liquid to come in contact with the skin or eyes. If contact has been made, wash the affected area with generous amounts of water and seek medical advice.
- To clean, gently wipe with a soft cloth moistened with warm water or mild detergents only, and then remove all moisture with a soft, dry cloth. Do not use harsh solvents or chemicals for cleaning.

4

Sound Oasis®

Contents

Your BST-400 includes:

1. Stereo Bluetooth Sound Therapy System
2. Micro SD sound card preloaded with Sound Oasis sounds (sounds vary from model to model – please refer to the insert sheet included with this product).
3. USB charging cable
4. Wall adapter
5. User manual
6. Warranty reply card



sleep better. feel better. live better®

5

Key Buttons & Features



Sound Oasis®

6

| Button | Description |
|--------|--|
| | <ul style="list-style-type: none"> • Press and hold for 3 seconds to turn BST-400 on. Press and hold for 1 second to turn BST-400 off. • BST-400 defaults to the sounds contained in your micro SD card when the Power Button is pressed. |
| | <ul style="list-style-type: none"> • Press to replay a sound. |
| | <ul style="list-style-type: none"> • Press to pause and play a sound. |
| | <ul style="list-style-type: none"> • Press to skip to a new sound |
| | <ul style="list-style-type: none"> • Press to decrease the speaker volume incrementally. Press and hold to decrease the volume continuously. |
| | <ul style="list-style-type: none"> • Press to increase the speaker volume incrementally. Press and hold to increase the volume continuously. |
| | <ul style="list-style-type: none"> • Press to alternate between playing sounds from your Bluetooth source or sounds contained in your micro SD card. • When you first turn on your BST-400, it will default to sounds contained in your micro SD card. Pressing the MODE button again will activate the Bluetooth mode and your BST-400 will attempt to reconnect to the last connected device. If there are no paired devices, the BST-400 becomes discoverable (see Playing Bluetooth Streamed Sounds section of this manual). |
| | <ul style="list-style-type: none"> • Blue LED indicates that the BST-400 is in the Bluetooth state and can play sounds/music streamed from your Bluetooth enabled source. • Blinking blue light – BST-400 is discoverable and ready to pair with a Bluetooth enabled device. • Solid blue light – BST-400 is connected to a Bluetooth device. • The solid blue light will turn off after about 10 seconds so the light does not interfere with your sleep. To check if Bluetooth is on, you can press any button and the solid blue light will activate for 10 seconds to confirm the BST-400 is in the Bluetooth state. |

sleep better. feel better. live better®

7

| Button | Description |
|-------------------------|---|
| Charge Indicator | <ul style="list-style-type: none"> • Amber LED indicates sound will play from the micro SD card. • The solid amber light will turn off after about 10 seconds so the light does not interfere with your sleep. • Red LED indicates that your BST-400 is charging. • The red LED will stay lit until your BST-400 is fully charged – then the LED will dim to a very dim level. |
| Microphone | <ul style="list-style-type: none"> • Your BST-400 can operate as a hands-free speaker for your smartphone. |
| USB | <ul style="list-style-type: none"> • Insert included USB charging cable to charge your BST-400's battery or power your BST-400 from a USB source. |
| AUX INPUT | <ul style="list-style-type: none"> • Insert an optional audio cable with 1/8" (3.5 mm) plugs to play music from an external device (e.g. iPhone®, iPod®, MP3 player or CD player) through the unit's speaker system. |
| MICRO SD | <ul style="list-style-type: none"> • Your BST-400 comes with a micro SD card preloaded with Sound Oasis sounds. The micro SD card must be inserted into this slot in order for you to play the sounds on the micro SD sound card. • You may insert new micro SD cards preloaded with other MP3 sounds or content (see the Using Other Micro SD Cards section of this manual). Please check www.soundoasis.com for our download library. |

Operation

Power Sources

Your BST-400 can operate as a wireless, cord free sound machine or a wireless Bluetooth speaker because it includes a built-in rechargeable battery. You can also keep your BST-400 plugged into a USB power source for continuous operation without recharging.

To Power By Built-In Rechargeable Battery:

1. Prior to using your BST-400 in the wireless, cord free mode for the first time, please completely charge the built-in battery.
2. Connect the provided USB Charging Cable to the included AC-charger, computer or laptop USB port. The unit's Charge Indicator is brightly illuminated while charging. It takes about up to 6 hours to charge the battery. When charging is completed, the Charge Indicator will be very dim.
3. Disconnect the USB Charging Cable and operate your BST-400 as described below.
4. When the unit is in low power status, the red Charge Indicator and amber LED will flash. Please recharge the battery of your BST-400 when this occurs.

Note: When the battery is fully discharged, it needs about 30 minutes of charging before you can use the unit again.

To Power By USB Charging Cable:

1. Connect the provided USB Charging Cable to the included AC-charger, computer or laptop USB port.
2. Operate your BST-400 as described below.

Listening To Sounds/Music From The Micro SD Card

You can enjoy sounds or music from your BST-400 from the included preloaded MICRO SD CARD.

Playing Sounds from the Built-In Micro SD Card

Your BST-400 includes a MICRO SD CARD that contains preloaded Sound Oasis sounds.

To play sounds/music from the built-in MICRO SD CARD, please follow these steps:

1. Press and hold the POWER button down for 3 seconds.
2. Your BST-400 will automatically start playing the first sound on the Sound Oasis MICRO SD CARD.
3. Press the - VOLUME + button to increase or decrease volume.
4. Press the >|| button to pause and resume playback.
5. Press the SOUNDS > button to skip to the next sound track.
6. Press the < SOUNDS button to repeat the sound track.

Please note:

- For the exact sounds preloaded on your MICRO SD SOUND CARD, please refer to the separate sound listing sheet included with your BST-400.
- Each preloaded Sound Oasis sound is 8 hours long so it plays throughout the night. After playing through the 8 hour sound track, your BST-400 will automatically replay the same sound track.

10

Sound Oasis®

Playing Bluetooth Streamed Sounds

Your BST-400 can also operate as a Bluetooth speaker so you can stream new sounds/music to your BST-400.

To stream sounds/music from your Bluetooth enabled device (e.g. your mobile phone, music player, PC/laptop) to your BST-400, please follow these steps:

1. Press and hold the POWER button down for 3 seconds.
2. Your BST-400 will play sounds from your MICRO SD CARD by default. Press the MODE button to activate the Bluetooth mode. Your BST-400 will then attempt to reconnect to the last connected device. If there are no paired devices, the BST-400 becomes discoverable (see Playing Bluetooth Streamed Sounds section of this manual).
3. On your Bluetooth enabled device, turn Bluetooth on, locate the Bluetooth device list and select the device named BST-400. Pair your BST-400 with your Bluetooth enabled device.
4. The Bluetooth LED will light up on your BST-400 and you will hear a Bluetooth pairing mode "bip" to indicate that you are in the Bluetooth streaming mode.
5. Press play on your Bluetooth device to play the streamed music/sound if the music/sound does not automatically start playing.
6. Press the -VOLUME + button to increase or decrease volume.
7. Press the >|| button to pause and resume playback.
8. Press the SOUNDS > button to skip to the next sound track on your music list.
9. Press the < SOUNDS button to repeat the sound track playback.

Please note:

- You can only pair one Bluetooth device at a time.
- If pairing fails, please turn your BST-400 off and then on again and retry pairing.

11

sleep better. feel better. live better®

Using Other Micro SD Cards

Your BST-400 includes a MICRO SD CARD that contains preloaded Sound Oasis sounds. You can use any other MICRO SD CARD with your BST-400. Here are a few options:

OPTION 1

1. Download additional sound tracks from our Sound Oasis web site onto your computer and then move/drag these new Sound Oasis sound tracks to a new MICRO SD CARD. Insert the new MICRO SD CARD into the BST-400 and playback those tracks as described above under Listening To Sounds/Music From The Micro SD Card.

Sound Oasis' patent pending tracks are ideal to download and use because they are 8 hours long and will play seamlessly and continuously through the night.

Note: If you drag new Sound Oasis sound tracks to your existing MICRO SD CARD, you will erase the existing sounds if there is not enough memory for the new sounds. Always use a newly purchased blank MICRO SD CARD to install new Sound Oasis sound tracks on.

OPTION 2

2. Purchase custom made MICRO SD CARDS from Sound Oasis. Visit our web site at www.soundoasis.com and choose which specific tracks you would like. We will then preload your own custom MICRO SD CARD with your selected sounds and ship it to you.

OPTION 3


3. Move/drag sound files from your computer's music library to a new MICRO SD CARD. Unless they are Sound Oasis sound tracks, sounds from your music library will not be 8 hours long. As a result, the BST-100 will play each short track and then automatically replay it.

12

Sound Oasis®

Streaming Sound Oasis APPS

Sound Oasis has a growing library of APPS to help you and your family sleep. Please see more information at www.soundoasis.com/apps

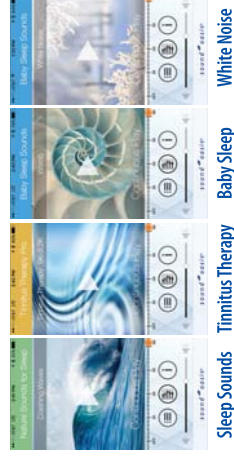

By purchasing this BST-400, you are entitled to use any of our PRO APPS for free. These PRO APPS provide you with many more sound options and they even allow you to customize the sounds with a built-in equalizer. To obtain these PRO APPS, please follow these steps:

1. Visit <http://www.soundoasis.com/apps/> and download the LTE version of your chosen APP. We have four (4) APPS to choose from: Sleep Sounds, Tinnitus Therapy, Baby Sleep and White Noise.
2. Open the LTE APP.
3. Pair your smartphone or tablet to the BST-400.
4. The LTE APP will automatically convert to a PRO APP after pairing is complete. The LTE APP must be open when the BST-400 connects.

When playing these APPS, you can stream them to your BST-400 for extra versatility and convenience.

13

sleep better. feel better. live better®

Technical Specifications

| | |
|--------------------------------|---|
| Bluetooth Specification | Bluetooth 4.0 |
| Bluetooth Profiles | HSP v1.2, HFP v1.5, A2DP v1.2 aptX® |
| Operating Distance | 30' / 10m |
| Run Time | 5 – 8 hours |
| Charging Time | Up to 6 hours |
| Dimensions | 7.87" x 2.36" x 2.52" (200mm x 60mm x 64mm) |
| Weight | 676g/1.5 lbs. |
| Material | Aluminum housing/ABS plastic base |
| Supports USB Sound Card | Data transfer function |
| Supports Reading Micro SD Card | MP3 music format |
| Speaker | 2 x 10 watt speakers with subwoofer |
| Channels | Stereo |
| Distortion THD | < / 1% |
| Signal to Noise Ratio | S/N: > / 80dB |
| Battery | Built-in battery, 7.2V 2000mA |
| Charging Voltage | 5V +/- 0.25V |

Using As A Speakerphone

Your BST-400 can function as a Bluetooth speakerphone to add extra sound fidelity and convenience to your phone call.

Please follow these steps to use as a speakerphone:

1. Press and hold the POWER button down for 3 seconds.
2. Press the MODE button again to activate the Bluetooth mode. You will hear a Bluetooth streaming mode "blip" and see the Bluetooth LED flash.
3. Make sure your BST-400 is Bluetooth paired with your smartphone.
4. Start your call.
5. You will be able to speak into the BST-400 (please speak towards the MICROPHONE) and hear your call through the BST-400 speaker. End the call on your smartphone or by pressing the >|| button. Repeat the call on your smartphone or by pressing and holding the >|| button on your BST-400. Adjust the volume during the call via the - VOLUME + button.

Note: If you are streaming sounds/music to your BST-400 and you receive a call on your smartphone, you can stop the sounds/music and answer the call by pressing the >|| button. To hang up the call, press the >|| button again.

Disposal



Dispose of all product parts in accordance with local regulations. If in doubt, please consult your local authorities.

The battery in the BST-400 needs to be recycled. Do not attempt to replace the built-in battery yourself.

The crossed-out wheeled bin means that within the European Union the product must be taken to separate collection at the product end-of-life. Do not dispose of these products as unsorted municipal waste.

FCC Information

This equipment has been tested and found to comply with the limits for a class B digital Device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning: Changes or modifications made to this device not expressly approved by HEADWATERS, INC. may void the FCC authorization to operate this device. Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

RF exposure statement:

This device complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. The device is installed and operated without restriction.

1 Year Limited Warranty

Sound Oasis Company Warrants that this product is free from defects in materials or workmanship for a period of one (1) year from the date of original purchase.

If the Product fails to operate in accordance with the manufacturer's specifications, simply return the product to the retail location where you purchased it or call our toll free number below within one (1) year of the date of purchase. Proof of date of purchase is required.

Sound Oasis Company will, as its option, repair or replace the product at no charge to you.

This warranty does not cover damage resulting from accident, misuse or abuse, lack of reasonable care, the affixing of any attachment not provided with the product, or loss of parts, or subjecting the product to anything but the specified voltage or batteries.

This warranty is in lieu of any other warranty, express or implied. All implied warranties, including any implied warranty of merchantability or fitness for any particular purpose, are limited in duration to one (1) year from the date of original purchase. In no event will Sound Oasis Company be responsible for incidental, consequential, or special damages resulting from the use of this product.

Some states do not allow limitation on how long an implied warranty lasts and / or the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

To Speak with a Sound Oasis Customer Service Representative,
Please call: 1-866-625-3218 or email us at info@soundoasis.com

ISED Statement

This device complies with Industry Canada's licence-exempt RSSs. Operation is subject to the following two conditions:

(1) This device may not cause interference, and (2) This device must accept any interference, including interference that may cause undesired operation of the device.

In order to avoid the possibility of exceeding the ISED radio frequency exposure limits, human proximity to the antenna shall not be less than 15mm (8 inches) during normal operation.

Cet appareil est conforme à la licence de RSSs d'industrie Canada. L'exploitation est soumise à les deux conditions suivantes:

(1) ce dispositif ne peut causer d'interférence et (2) ce dispositif doit accepter toute interférence, y compris les interférences qui peuvent causer un fonctionnement indésirable de l'appareil. Afin d'éviter la possibilité de dépasser les limites d'exposition aux radiofréquences de l'isedc, La proximité humaine de l'antenne ne doit pas être inférieure à 15mm en temps normal opération

Thank you for choosing Sound Oasis®!

sound  *oasis*®
sleep better. feel better. live better.®