


# DYNAMIC THERMOMETER



Let it help !

User Manual

## PRODUCT OVERVIEW

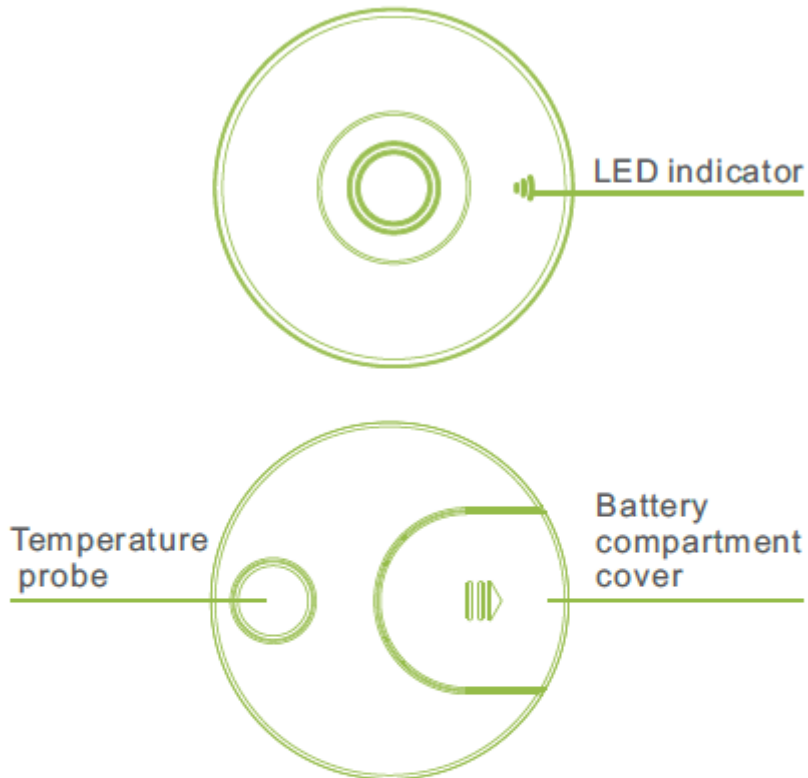

Model No : DS-04

Name : Dynamic Thermometer

Main components : case, temperature probe, battery

Valid transmission distance ( Bluetooth ) : up to 5 meters

Battery : CR2032

Applicable body part : armpit

Application : medical facilities or household

**Note: Bluetooth transmission distance may vary due to obstacles.**

## TECHNICAL SPECIFICATIONS

Product type : wearable device

Temperature sensor : temperature probe

Operational mode : 24 hours continuously

Bluetooth module : Bluetooth 4.0

Dimension : 46 × 46 × 6.7mm

Switch : touch switch

Operation conditions (temperature and humidity):

5°C ~ 55°C, ≤95%RH

Measurement Range : 33°C ~ 43°C

Accuracy : ± 0.05°C (35°C~38.5°C)

± 0.1°C (33°C~34.99°C and 38.51°C~43°C)

Temperature display : 4 digit display

Storage condition : -25°C ~ 55°C, ≤95%RH

Battery work time : 1000 hours

# UPLOAD BATTERY

## 1、 Open battery compartment cover

Follow instructions below, push the battery compartment cover to the direction of the arrow.



## 2、 Put in the battery

Please make sure the positive pole of the battery is facing up.



## 3、 Please make sure the positive pole of the battery is facing up

Push the cover to the opposite direction of the arrow



**DANGER!**

**SWALLOWING THE BATTERY MAY CAUSE LIFE RISKS!**

Please keep the battery out of reach of children.

**DO NOT THROW THE BATTERY INTO FIRE, IT MAY CAUSE EXPLOSION!**

# OPERATING INSTRUCTIONS

## 1、Download APP

Option A: Scan the QR code on the rightside or on packaging, download and install the APP



Option B: download the APP from **91 Assistant, Peascod or 360**

## 2、Wearing

a、take out a sticker from the packaging box



b、take off the protective paper on one side



c、apply the sticker onto the back of the device, Make sure the probe is uncovered



d、take off the protective paper on the other side



e、paste the device into armpit



# OPERATING INSTRUCTIONS


## 3、Registration

Please follow the instructions and register when log in for the first time.



## 4、Check results

Turn on the APP and wait for around 10 minutes to see the temperature becoming stable.

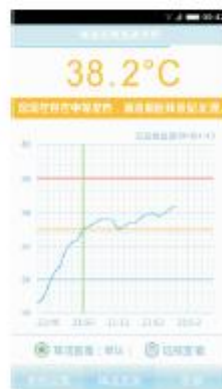

\* Measurement time may vary due to the change of surrounding environment.

## NOTIFICATIONS

- 1、 APP is for Android system only and it requires version 4.3 or above.
- 2、 This product is not suitable for children with allergic constitution.
- 3、 Please do not over exercise before measuring your temperature.
- 4、 Please make sure the embedded probe touches skin directly during the measurement.
- 5、 Please keep this product away from chemical solvent or diluents.
- 6、 Please do not store or use this product in the strong electromagnetic environment.
- 7、 Please sterilize DS-04 with medicinal alcohol, not high-temperature treatment should be used.
- 8、 Please clean the probe and the case after each use and store it properly.
- 9、 For children under age of three, device should be used under parents' guidance.
- 10、 A device can only transmit data to ONE mobile.

## INSTRUCTIONS

- 1、 ON: The product will power on automatically when it touches skin.  
After a long 'light on' for 10 seconds, the LED light will flash once every 10 seconds.
- 2、 OFF: Detach the product from the skin, the LED light will flash twice and it will power off after a long 'light on' for 10 seconds.

## FAQ

- **How to purchase more stickers?**  
Please contact our Customer Services or go to our official website to purchase more stickers.
- **How safe is the device?**  
All materials used are up to medical equipment standards, and the device meets RoHS and SGS's standard.
- **Would Bluetooth transmission harm children?**  
Our ultra-low power consumption Bluetooth technology is safe for children.



# WARRANTY

- 1、 If there is any malfunction on the product within 15 days after purchase, you can request for a replace of product with the same model, under the condition that both the product and packagingshould be in good condition (no scratch). This does not apply if the malfunction is caused byartificial damage, and we reserve the right to examine the product and explain this term.
- 2、 The warranty period is one year since purchase. We provide free repair or replace services if there is malfunction caused by defects of parts or manufacturing.
- 3、 All artificial damage, including disassembling, remove seal and wrong operation, are not covered by the warranty. Without the card, the company reserve right to charge the maintenance and parts changing during warranty period.

## **FCC Caution.**

### **§ 15.19 Labelling requirements.**

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

### **§ 15.21 Changes or modification warning**

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

### **§ 15.105 Information to the user.**

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**\* RF warning for Portable device:**

The device has been evaluated to meet general RF exposure requirements. The device can be used in portable exposure condition without restriction.