We're committed  
to your health  
and your safety.

Thank you for choosing an Oura Ring. We want you to get the most out of your experience. Please read this leaflet and the accompanying documentation carefully before using your ring, mobile app, Oura Services and any other related services. Failure to follow the instructions below could result in product damage or malfunction, or even injury.

## Support and Questions

If you have any questions about getting started with or using your ring, mobile app and any other Oura Services, or you have any issues with your product, please refer to the Frequently Asked Questions (FAQs) at [ouraring.com/faq](http://ouraring.com/faq). If you can't find what you're looking for, send us an email at [help@ouraring.com](mailto:help@ouraring.com).

## Safety Information

Please note that Oura Services are not intended to diagnose, treat, cure, or prevent any disease or medical condition. The information and guidance in Oura Services are there for informational purposes only and cannot replace the services of health professionals or physicians. You should always consult a physician if you have any questions regarding a medical condition or any changes you intend to make to your sleep or activity based on information or guidance from Oura Services. Never disregard or delay in seeking professional medical advice because of something you've read from Oura Services.

We are not responsible for any health problems that may result from information or guidance you receive from Oura Services. If you make any change to your sleep or activity based on Oura Services, you agree that you do so fully at your own risk. It is important to be sensitive to your body's responses. For example, if you feel unexpected, repeated or long term pain, fatigue or discomfort due to having made changes to your sleep or activity, it is recommended that you consult a physician before continuing with such changes. The information and guidance in Oura Services may be misleading if your physiological functions and responses differ significantly from population averages due to medical conditions or rare natural differences.

Please be cautious that the ring or any other Oura product you wear does not get caught on fixed structures or heavy objects when moving yourself or said heavier objects.

**If you experience redness or skin irritation on your finger due to the ring or any other Oura product, remove it immediately. If symptoms persist longer than 2-3 days of not using your Oura product, please contact a dermatologist.**

## Use, care and maintenance

Finger size can vary depending on the time of the day, and sometimes it may be difficult to remove the ring from your finger. In case the ring gets stuck:

- Use cold water and gentle soap to wet your finger, and slowly twist the ring to remove it.
- Hold your hand up above your heart until the blood pressure gets lower, and then try to remove it.
- **In cases of emergency and/or discomfort, when you can't remove the ring yourself, seek immediate medical attention.**

Use a soft cloth or hand wash with mild soap and water to clean the ring.

Your Oura Ring can be worn during showers, baths, swimming, and snorkeling.

Try to avoid wearing the ring when strength training, working with a shovel or other heavy tools, or carrying heavy objects made of metal, ceramics or stone. In addition, try to avoid wearing the ring next to other rings or objects which are made of metal, ceramics, stones or diamonds. The Oura Ring may get scratched and can itself scratch softer metal jewelry or other objects in close contact with the ring like phone covers made of gold, silver, or aluminum. Some ceramic phone covers with a soft coating may also get scratched.

Keep the ring away from children. This product is not intended for individuals under the age of 18. Seek immediate medical attention if you know or suspect that a child has swallowed the ring. Do not leave the ring exposed to heat, such as in a vehicle or in the sun. Do not puncture the ring or its battery.

Please avoid handling batteries, or working on devices and with machinery that contain batteries while wearing your Oura Ring. In certain cases, where both the cathode and the anode of another battery touch the ring, there is a risk of a short circuit which is similar to standard metallic rings. This can result in a potentially dangerous shock. Please take the proper precautions to avoid these situations.

## Safety and Product Information

### Specifications

- **Outer surface materials** Titanium with physical vapor deposition (PVD) coating in Gold color. Titanium with PVD coating in Silver color. Titanium with Diamond-like Carbon Coating (DLC) in other models.
- **Inner surface materials** Non-allergenic, non-metallic seamless inner molding
- **Memory capacity** Recommended to sync ring data daily with Oura application. Ring is capable of storing up to 3 weeks of data depending on type and frequency of ring use
- **Battery** Rechargeable 15mAh (US6) - 22mAh (US13) Lipo battery, non-replaceable
- **Battery life** Typically between 4-7 days. Battery life changes depending on the type and frequency of ring use.
- **Charging** Wireless charging with Oura charger
- **Charging time** Typically between 20 - 80 minutes depending on the ring starting charge level
- **Red and green light-emitting diodes (LED)** used in the product are in the visible colour range. They are safe to the user and to the people around.
- **Connectivity** Bluetooth Low Energy (Bluetooth Smart<sup>®</sup>)
- **Ring firmware updateability** Automatic firmware updates via Oura app
- **Oura app availability** App Store, Google Play
- **Oura app compatibility** See ouraring.com/compatibility
- **Security** Encrypted Bluetooth communication used between the ring and the app
- **Ring is water resistant** To 100 m / 328 ft

- **Charger is not water resistant** and is intended for indoor use only. Do not use or store the charger in humid environments like bathrooms.
- **Operating temperature range** -10–54 °C / 14–129 °F
- **Charging temperature range** +10–40 °C / 50–104 °F, room temperature recommended.
- **Charger input connector and voltage** USB Type-C connector. USB nominal 5V.
- Equipment intended to be supplied by an external 5Vdc supply that complies with IEC 62368-1 standard. The power available to the equipment shall never exceed 15W after 3 seconds.
- Charger complies with EN/IEC 62368-1 international safety standard for ICT and AV equipment.
- **Dispose of the ring** (and internal battery) in accordance with applicable local laws and regulations. Special handling may be required. The charger is an electronic device and local legislation for electronics recycling should be followed.
- **Check instructions** in support.ouraring.com to perform a factory reset before ring disposal.

## Warranty Disclaimer

EXCEPT FOR THE LIMITED WARRANTY FOR THE RING SET FORTH BELOW, THE RING AND THE OURA SERVICES ARE PROVIDED BY OURA AND ITS AFFILIATES "AS IS."

NEITHER OURA NOR ITS PARTNERS, SUPPLIERS, OR AFFILIATES MAKE ANY REPRESENTATIONS OR WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, AS TO THE OPERATION OF THE OURA SERVICES (INCLUDING THE RING), ITS CONTENTS, OR ANY INFORMATION MADE AVAILABLE BY OR THROUGH THE OURA SERVICES. IN ADDITION, OURA AND ITS PARTNERS, SUPPLIERS AND AFFILIATES DISCLAIM ALL WARRANTIES WITH RESPECT TO THE OURA SERVICES (INCLUDING THE RING), EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY, TITLE, FITNESS FOR A PARTICULAR PURPOSE AND NON-INFRINGEMENT.

FURTHERMORE, OURA DOES NOT WARRANT THAT USE OF THE OURA SERVICES WILL BE UNINTERRUPTED, AVAILABLE AT ANY TIME OR FROM ANY PARTICULAR LOCATION, SECURE OR ERROR-FREE, THAT DEFECTS WILL BE CORRECTED, OR THAT THE OURA SERVICES (INCLUDING THE RING) ARE FREE OF VIRUSES OR OTHER POTENTIALLY HARMFUL COMPONENTS.

LIMITED WARRANTY FOR THE RING. OURA WARRANTS TO THE ORIGINAL END USER PURCHASER THAT THE RING IS FREE FROM DEFECTS IN MATERIAL AND WORKMANSHIP FOR TWO (2) YEARS FROM THE DATE OF PURCHASE. If your local law in effect at the time of purchase requires a warranty period longer than two (2) years, this warranty shall be extended to the extent required by such law. Within the warranty period, Oura shall repair or replace at no charge to you any components of the ring that fail the limited warranty provided. You shall be responsible for any related transportation charges. Replacement products may be new or refurbished at our discretion. This limited warranty does not apply to (i) normal wear and tear, including scratches and dents; (ii) consumable parts included in ring, such as batteries, unless product damage has occurred due to a defect in materials or workmanship; (iii) damage resulting from your failure to use the ring in accordance with the instructions accompanying the ring or available at our website; (iv) damage resulting from an accident, flood, fire, misuse, or abuse; (v) damage resulting from service performed, or damage resulting from tampering or alterations to the ring, by anyone not authorized by Oura; or (vi) use of the ring with any other application or software than the Oura mobile app.

Oura retains the exclusive right to repair or replace the ring, or offer a full refund, at its sole discretion. Such remedy shall be your sole and exclusive remedy for any breach of warranty. Warranty repairs and replacements have a new warranty which is the longer of ninety (90) days or the balance of the original two-year warranty.

## Limitation of Liability

IN NO EVENT WILL OURA OR ANY OF ITS AFFILIATES BE LIABLE FOR DIRECT, INDIRECT, INCIDENTAL, PUNITIVE OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION THOSE RESULTING FROM LOST PROFITS, LOST DATA OR BUSINESS INTERRUPTION) ARISING OUT OF THE USE, INABILITY TO USE, OR THE RESULTS OF USE

OF THE RING AND THE OURA SERVICES, WHETHER SUCH DAMAGES ARE BASED ON WARRANTY, CONTRACT, TORT OR ANY OTHER LEGAL THEORY AND WHETHER OR NOT OURA HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THE AGGREGATE LIABILITY OF OURA, ARISING FROM OR RELATING TO THE RING AND OURA SERVICES (REGARDLESS OF THE FORM OF ACTION OR CLAIM, E.G. CONTRACT, WARRANTY, TORT, STRICT LIABILITY, NEGLIGENCE, OR ANY OTHER LEGAL THEORY) IS LIMITED TO \$100. In some locations applicable law may not allow certain of the limitations described above, in which case such limitations will apply to the maximum extent allowed by such applicable law.

## Intellectual Property Rights

ÛURA, Õ, and OURA are trademarks of Õura Health Oy and may not be used without permission. Pat. www.ouraring.com. © Õura Health Oy (Oura) 2021. All rights reserved.

Manufactured by Õura Health Oy (Oura), Elektriikkatie 10, FI-90590 Oulu, Finland.

## Terms of Use

Your use of the Oura products and services is subject to our Terms of Use, available at <https://ouraring.com/terms-of-use/>, and Privacy Policy, available at <https://ouraring.com/privacy-policy/>. The Content and Warranty Disclaimer and the Limitation of Liability set forth above are from our Terms of Use and are subject to change if we change our Terms of Use.

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## 2014/53/EU

In order to satisfy the essential requirements of 2014/53/EU Directive, the product is compliant with the following standards:

RF spectrum use	EN 300 328 v2.2.2
EMC	EN 301 489-1 V2.2.3 EN 301 489-17 V3.2.4 EN 55011:2016/A:2020 EN 61000-6-1:2007
Health & Safety (RED art. 3.1a)	EN 62368-1:2014 + AC:2015 + A11:2017 + IEC 62479:2010

The conformity assessment procedure of Directive 2014/53/EU has been followed with the involvement of the following Notified Body:

DEKRA Testing and Certification, S.A.U.  
Parque Tecnológico de Andalucía

## 2012/19/EU

This product is compliant with Directive 2012/19/EU. Please dispose used electrical and electronic equipment (WEEE) according to national laws and regulations with electrical waste. It should not be mixed with general household waste.

## USA

Oura Health Oy has not approved any changes or modifications to this device by the user. Any changes or modifications could void the user's authority to operate the equipment.

## California Proposition 65

WARNING: This product may contain chemicals known to the State of California to cause birth defects or other reproductive harm.

## FCC regulatory information

This device (2AD7V-OURA2101) complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses

and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### RF exposure safety

This product is a radio transmitter and receiver. It's designed not to exceed the emission limits for exposure to radio frequency (RF) energy set by the Federal Communications Commission.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

### Canada

Regulatory notices for Canada are available in this website: [ouraring.com/regulatory-notices-ca](http://ouraring.com/regulatory-notices-ca)

Innovation, Science and Economic Development Canada (ISED) regulatory information

This device (20635-OURA2101) complies with Innovation, Science and Economic Development Canada's licence-exempt RSS standard(s). Operation is subject to the following two conditions:

1. This device may not cause interference; and
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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### RF exposure safety

This product is a radio transmitter and receiver. It is designed not to exceed the emission limits for exposure to radio frequency (RF) energy set by the ISED.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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### CAN ICES-3 (B)/NMB-3(B)

This Class B digital apparatus complies with Canadian ICES-003.

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OURA、Ô、OURAは、Oura Health Oyの商標であり、無断で使用することはできません。Pat. www.ouraring.com. © Oura Health Oy (Oura) 2021. All rights reserved.