dreem²

Product Information / Informations produit

Contents


First steps...	04
What's in the box?	05
Headband's components	06
Charging your headband	07
How to position your headband	08
The adjustment strips	09
Headband's maintenance	10
Need help?	11
Français	13

First steps...

1. Read this Product Information booklet.
2. Download the Dreem app onto your smartphone. Available for free on the App Store (iOS) or Google Play (Android).
3. Open the app and follow the instructions.

Note: Your headband will go through an automatic update which can last up to 30 minutes.

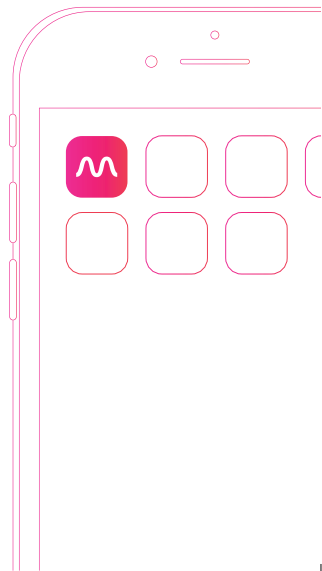
What's in the box?

1. Headband
2. Adjustment strips (sizes S, M and L)*
3. Charger
4. Adaptors (EU, US, UK)
5. USB cable


*An adjustment strip (size M) is already in position on the headband.

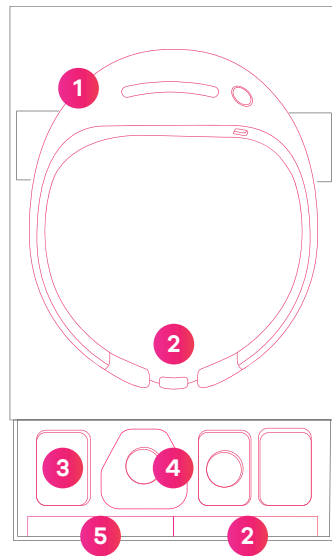Components

1. Power button
2. Brain activity sensors (EEG)
3. Cardio sensor (pulse oximeter)
4. Speaker (bone conduction)
5. Charging plug
6. Audio jack (3.5mm)
7. Adjustment strip
8. Touchpad*

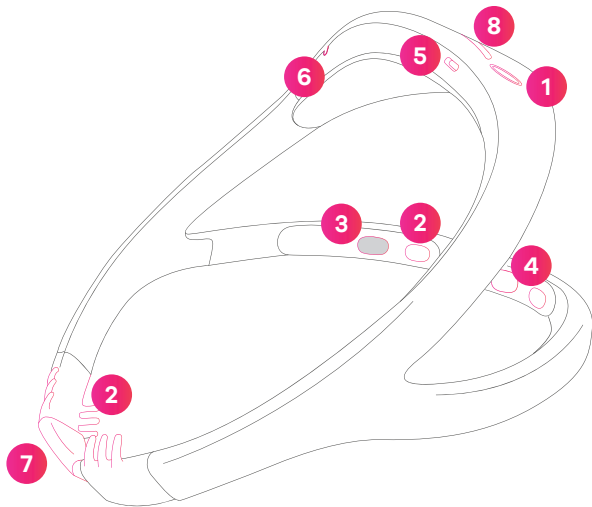
*Tactile zone that allows you to interact directly with the headband without using the app.

Charging

1. Choose the adaptor plug you need and attach it to the charger cable.
2. Plug in your headband. When the indicator light becomes blue and stops flashing, your headband is charged enough for your next night.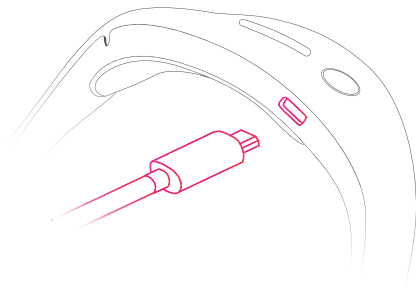

 A Dreem charging dock is also available for wireless charging.


Positioning

1. Place the front band (the thin part of the headband with the sensors) against the middle of your forehead.
2. Place the back sensors (situated either side of the elastic) against your scalp.
3. Gently pull the top arch back while making sure that the front band stays against the middle of your forehead.

If you have long hair, we recommend placing the back of the headband under your hair to ensure the signal quality of the back sensors.

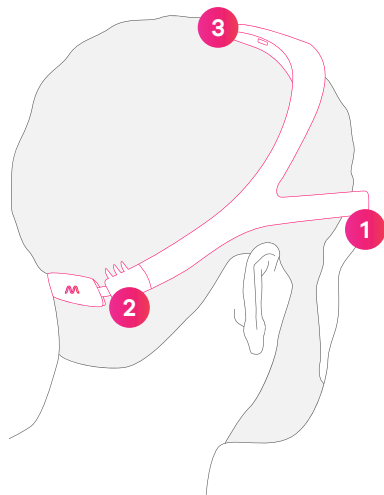

Adjustment strips

You can adjust the size of your headband if it's too tight or moves when you shake your head. There are 3 sizes of adjustment strips: S, M and L. By default, the M adjustment strip is already on your headband.

How to attach an adjustment strip:

Open the adjustment strip and place the headband's two velcro dots face to face inside it (see diagram). Then close the adjustment strip.

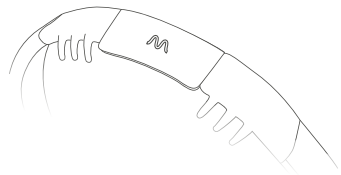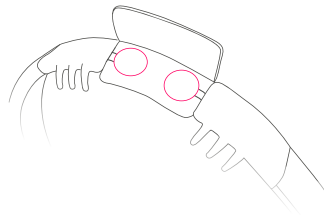Maintenance

After the first use, we recommend you use a soft, damp cloth to clean the interior of the headband.

After that:

You can clean the sensors of your headband from time to time to ensure good signal quality.


If you travel with the headband, make sure to carefully position it in your luggage so it doesn't get squashed out of shape.

We're here for you

Questions on how to start or use your headband? Head to our Information Center via [the Dreem app](#) or at support.dreem.com.

Alternatively, our team are available at support@dreem.com to answer any questions you may have.

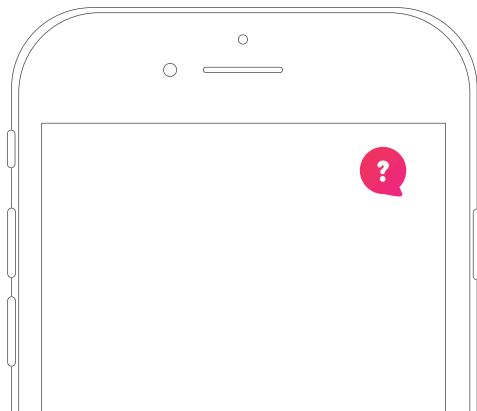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