

FitSleep

User's manual

Thanks for using FitSleep' s product.For safety's sake,please read FitSleep's Manual for information to install and use the product. And please follow the instructions while installing and using.

Package contents

FitSleep sleep tracker 1

Charger 1

USB cable 1

User's manual;Warranty card;Certificate of conformity


Features

1. Reset:when the equipment is down, press and reset)
2. Power Port: plug type
3. Status lamp: state the equipment battery:red is low and blue is fine.
4. Status Switch:turn on and see the battery.

## Installation

### Equipment Installation:

1. Turn on FitSleep.
2. Settle FitSleep right under your pillow.
3. Connect with APP and the light is on.

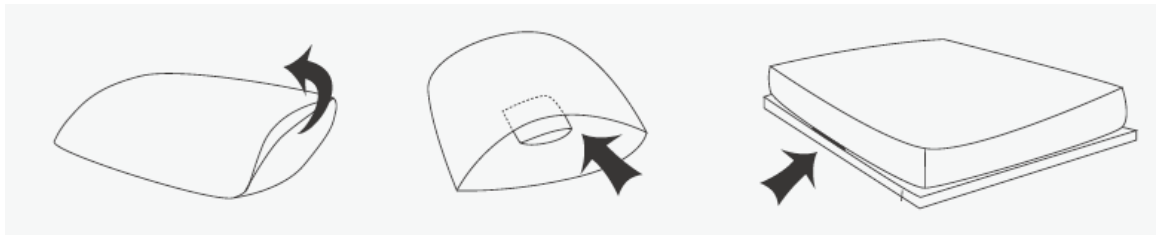

### APP Downloading:

Method 1: Scan the QR Code on the user's manual or [www.fitsleep.net](http://www.fitsleep.net) and download.

Method 2: Search ShuiShui in Appstore for iPhone users and search FitSleep in Smart Store for Android users.

### Equipment Connection:

Method 1: Scan the QR Code of equipment on the user's manual. Then turn on Bluetooth to bind.

Method 2: Scan the QR Code on FitSleep Sleep Tracker to bind.

### SPECIFICATIONS:

Product name: FitSleep

Material: ABS

Dimension: 92mm×92mm×16mm

Weight:150g

Power: 3.8V lithium ion rechargeable battery

Battery:2050mAh

Communication Mode: Bluetooth 4.0

Battery life: 30 days

- Please avoid any environment of high temperature or high pressure.

#### IMPORTANT

FitSleep is not a medical device. Users shall not rely on this equipment totally. For sick people, his/her family should check his/her physical conditions from time to time even he/she is using FitSleep.

FitSleep is not suitable for those who wears a cardiac pacemaker.

FitSleep is for adult (older than 18 years old)ONLY.

Shenzhen OneThird Sleep Technology Co., Ltd.

Hot-line:400-0888-662

Website: [www.onethird.com.cn](http://www.onethird.com.cn)

Address:Room 405,Nanshang Building,Zhuguang Street,Nanshan District,Shenzhen.

## **FCC STATEMENT :**

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

**Warning:** Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.

- Increase the separation between the equipment and receiver.

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

- Consult the dealer or an experienced radio/TV technician for help.

## **RF warning statement:**

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

# **CE 2200**