



---

# WEARABLE ANTENNA ASSEMBLY

## USER MANUAL

Caution: Federal law restricts this device to sale by or on the order of a physician.














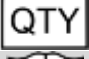





---

WEARABLE ANTENNA ASSEMBLY KIT (PDBT-915-2K)

WEARABLE ANTENNA ASSEMBLY  
(PDBT-915-2A)

# EXPLANATION OF SYMBOLS ON PRODUCT OR PACKAGE

Refer to the appropriate product for symbols that apply.

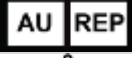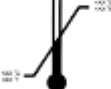Symbol	English – EN
	Device reference identification
	Lot number
	Quantity of product included in package
	Consult instructions for use
	Do not reuse
	Do not resterilize
	Do not use if package is damaged
	Store in a cool, dark, dry place
	Caution
	Warning
	MR Unsafe
	Use by
	Manufacturing date
	Manufacturer
 Length	Device length
	Sterilization: ethylene-oxide gas
	European Authorized Representative
	Australian Sponsor
	Temperature limits

# TABLE OF CONTENTS

<b>EXPLANATION OF SYMBOLS ON PRODUCT OR PACKAGE .....</b>	<b>2</b>
<b>HOW TO USE THIS MANUAL .....</b>	<b>4</b>
INDICATIONS FOR USE .....	4
<b>SAFETY INFORMATION.....</b>	<b>5</b>
CONTRAINDICATIONS.....	5
WARNINGS .....	6
PRECAUTIONS .....	11
ADVERSE EVENT SUMMARY.....	14
<b>PARTS OF YOUR STIMULATION SYSTEM.....</b>	<b>15</b>
OVERVIEW OF THE USER CONTROLS.....	16
POSITIONING THE WAA.....	17
STARTING STIMULATION .....	18
SELECTING A STIMULATION PROGRAM .....	18
INCREASING OR DECREASING AMPLITUDE .....	18
<b>MAINTENANCE .....</b>	<b>19</b>
BATTERY CHARGING.....	19
WHEN TO CALL YOUR CLINICIAN.....	21
CLEANING AND CARE PRECAUTIONS .....	21
SAFETY AND TECHNICAL CHECKS .....	21
WAA DISPOSAL .....	21
PATIENT IDENTIFICATION CARD .....	22
<b>SPECIFICATIONS.....</b>	<b>22</b>
<b>WIRELESS INFORMATION .....</b>	<b>23</b>
<b>TROUBLESHOOTING.....</b>	<b>24</b>
<b>CONTACT INFORMATION .....</b>	<b>26</b>

## HOW TO USE THIS MANUAL

This manual will help you understand how to use and care for your neurostimulator system. It also provides you with warnings and precautions you should know about. You should discuss with your clinician any questions or concerns you have after reading this manual. Please refer to the Freedom Spinal Cord Stimulator (SCS) System Product Safety Guide for EMC related safety information.

Please refer to the Freedom Spinal Cord Stimulator (SCS) System Product Safety Sheet for EMC related safety information per IEC 60601-1-2:2007. The Product Safety Sheet is used in conjunction with this manual as the instructions for use.

This device complies with Part 18 of the FCC Rules. Per FCC 18.213, See Warnings section in this document for the interference potential of the device and methods for correcting interference. See Maintenance section in this document for the instructions on system maintenance.

Per FCC 15.21, changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Per FCC 15.19(a)(3) and (a)(4) This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. FCC ID: 2AHXAPDBT2

## INDICATIONS FOR USE

The Freedom Spinal Cord Stimulator (SCS) System is intended as the sole mitigating agent, or as an adjunct to other modes of therapy used in a multidisciplinary approach for chronic, intractable pain of the trunk and/or lower limbs, including unilateral or bilateral pain. The Freedom-8A Trial Lead Kit is only used in conjunction with the Freedom-8A Stimulator Receiver Kit, and the Freedom-4A Trial Lead Kit is used for either the Receiver Kit Freedom-4A Stimulator or the Receiver Kit Freedom-8A Stimulator. The trial devices are solely used for trial stimulation (no longer than 30 days) to determine efficacy before recommendation for a permanent (long term) device.

# SAFETY INFORMATION

## CONTRAINDICATIONS

- **Poor surgical risks** – Spinal cord stimulators should not be used on patients who are poor surgical risks or patients with multiple illnesses or active general infections. This includes patients who need anticoagulation therapy that cannot be temporarily halted to accommodate the implantation procedure.
- **Pregnancy** – Safety and effectiveness of the Freedom SCS System for use during pregnancy and nursing have not been established.
- **Inability to operate System** – Spinal cord stimulators should not be used on patients who are unable to understand or operate the System.
- **Exposure to shortwave, microwave, or ultrasound diathermy** – Diathermy should not be operated within the vicinity of a patient implanted with a Freedom Stimulator or when wearing the Wearable Antenna Assembly (WAA). The energy from diathermy can be transferred through the stimulator or WAA and cause tissue damage, resulting in severe injury.
- **Implanted cardiac systems** – Patients who have implanted cardiac systems should not use the Freedom SCS System. Electrical pulses from the Freedom SCS System may interact with the sensing operation of an implanted cardiac system, causing the cardiac system to respond inappropriately.
- **Occupational exposure to high levels of non-ionizing radiation that may interfere with therapy** – Users who regularly work in environments with elevated levels of non-ionizing radiation should not be implanted with the Freedom SCS System. The energy in high-level areas can be transferred through the stimulator and cause tissue damage, resulting in severe injury. Examples of environments having high level non-ionizing radiation includes radio or cell phone transmission stations, facilities using radiofrequency heat sealers or induction heaters, electric power infrastructure controlled environments (i.e. step down transformers or high voltage power lines).

## WARNINGS

**Electromagnetic interference (EMI)** – EMI is a field of energy generated by equipment found in the home, work, medical or public environments. EMI that is very strong can interfere with System. The device includes features that provide protection from EMI. Most electrical device and magnets encountered in a normal day will not affect the operation of the System. However, strong sources of EMI could result in the following:

- Serious patient injury resulting from heating of the implanted device and damage to surrounding tissue.
- System damage, resulting in a loss of, or change in, symptom control and requiring additional surgery.
- Operational changes to the WAA. This may cause either external device to turn on, turn off, or to reset to factory settings. If this occurs, the WAA need to be reprogrammed.
- Unexpected changes in stimulation, causing a momentary increase in stimulation or intermittent stimulation. Some patients have described as a jolting or shocking sensation. Although the unexpected change in stimulation could feel uncomfortable, it does not damage the device or cause a patient direct injury. In rare cases, as a result of the unexpected changes in stimulation, patients have fallen down and been injured.

If you suspect that your Freedom SCS System is being affected by EMI then you should:

- Immediately move away from the equipment or object.
- Remove the WAA from the vicinity.

**Machinery or heavy equipment** – Machinery and heavy equipment (including vehicles) should not be operated while using the Freedom SCS System. Malfunction of the System could result in loss of body control, body function, or a feeling that could render the patient incapable of controlling the system.

**Stimulator fracture** – If the Stimulator insulation is ruptured or pierced due to extensive forces, unexpected changes in stimulation could result.

**Electromagnetic equipment/environments** – Avoidance of high electromagnetic equipment radiators or environments is highly encouraged. Examples of equipment and/or environments include the following:

- High-power amateur transmitters/antennas or citizen band (CB) radio or Ham radio used for private recreation, communication, and wireless experimentation.

- Electric arc welding or resistance welding equipment used for melting and joining metals or plastics.
- Industrial electric induction furnace/heater or electric arc furnace/heater used for melting metals and plastics.
- High-voltage areas identified by fenced areas, restricted access signs, and caution signs (safe if outside the fenced area).
- Microwave transmitters identified by fenced areas, restricted access signs, and caution signs (safe if outside the fenced area).
- Television and radio towers identified by fenced areas, restricted access signs, and caution signs (safe if outside the fenced area).
- Linear power amplifiers used for increasing the power output of radio transmitters, wireless communication applications, audio equipment or other electronic equipment.
- Radio telemetry equipment used for tracking location of vehicles, equipment or animals.

**Active Implantable or Body Worn Medical Devices** – Safety has not been established for patients who use the Freedom SCS System with other active implantable or body worn medical devices. These devices include other neurostimulation systems, insulin pumps, automated external defibrillators (AED), cochlear implants, and wearable medical sensors. Malfunction and/or damage could occur to either system that could result in harm to the patient or other people nearby.

**Magnetic Resonance Imaging (MRI) – The Freedom-8A/4A SCS System is MR Unsafe.** Since the Freedom-8A/4A SCS System is MR Unsafe, the strong magnetic field of the MR system could attract or otherwise damage the System, and in the process cause serious harm to the patient or other people or damage to the MR system.

**The WAA component is MR Unsafe; ensure that the WAA does not enter the MR system room.** Since the WAA is MR Unsafe, the strong magnetic field of the MR system could attract or otherwise damage the WAA, and in the process cause serious harm to the patient or other people or damage the MR system.

**Radiofrequency (RF) ablation** – Safety has not been established for radiofrequency (RF) ablation in patients with a Stimulator. RF ablation may cause induced electrical currents that result in heating and tissue damage. Do not use RF ablation anywhere near the Stimulator. If RF ablation is used, ensure that ablation is not performed over or near the Stimulator.

**Electrocautery** – If electrocautery tools are used near the Stimulator then the insulation can be damaged. The Stimulator may fail or conduct induced currents. Induced electrical currents can cause heating that results in tissue damage.

When electrocautery is necessary, these precautions must be followed:

- The WAA should be taken off.
- Bipolar cautery should be used.
- If unipolar cautery is necessary:
  - Only low-voltage modes should be used.
  - The lowest possible power setting should be used.
  - The current path (ground plate) should be kept as far away as possible from the Stimulator.
  - Full-length operating room table ground pads should not be used.
- After electrocautery, confirm that the Stimulator is working as intended.

**High-output ultrasonics / lithotripsy** – Safety has not been established for high-output ultrasonics or lithotripsy when implanted with the Freedom SCS System. Use of lithotripsy may result in damage to the device or harm to the patient.

**Psychotherapeutic procedures** – Safety has not been established for psychotherapeutic procedures using equipment that generates electromagnetic interference (e.g., electroconvulsive therapy, transcranial magnetic stimulation) in patients who have spinal cord stimulators. Induced electrical currents can cause heating that may result in tissue damage.

**Radiofrequency Identification (RFID) Emitters - Theft detectors, electronic article surveillance (EAS) systems, and radiofrequency identification systems** – Tests have been performed with an array of simulated RFID emitter systems, and have demonstrated that the Freedom SCS System (implanted device and WAA) can be affected by separation distances between the Freedom SCS System and the RFID emitter of less than 3m (~10 ft.). More powerful RFID Emitters might cause effect at farther distances. RFID emitters can be hidden or portable and not obvious to the Stimwave user. Any RFID emitter may temporarily interrupt stimulation, or cause elevated levels of stimulation. It is recommended that if a patient feels a change in stimulation near a potential RFID emitter, they promptly move away from the area and remove the WAA from the body.



When possible, it is best to avoid RFID emitters or remove the WAA while passing near RFID emitters. Patients with an implanted device should inform the attendant who may be able to assist them in bypassing any RFID emitter. If unavoidable, the patient should walk through the RFID emitter and promptly move away from the area. Patients should not lean on scanners or linger in the area of RFID emitters.

**Computed Tomography (CT) Scanning** – Safety has not been established for CT scanning of patients with a Stimulator. X-rays from the scan could cause unintended shocks or malfunctions of the Stimulator.

The CT operator should use CT scout views to determine if implanted medical devices are present and their location relative to the programmed scan range. For CT procedures in which the device is in or immediately adjacent to the programmed scan range, the operator should:

- Remove the WAA from the CT scan range.
- Minimize X-ray exposure to the implanted device by:
  - Using the lowest possible X-ray tube current consistent with obtaining the required image quality.
  - Making sure that the X-ray beam does not dwell over the Stimulator for more than a few seconds.

After CT scanning directly over the implanted device:

- Place the WAA on body and turn on stimulation.
- Check for proper stimulation, and that indicator lights are operating as expected.
- Shut off the WAA if it is suspected that the device is not functioning properly.

**Bone growth stimulators** – Safety has not been established for bone growth stimulators within the vicinity of the Freedom SCS System. Use of a bone growth stimulator may result in damage to the device or harm to the patient.

**Dental drills and ultrasonic probes** – Safety has not been established for dental drills or ultrasonic probes within the vicinity of the Freedom SCS System. Use of dental drills or ultrasonic probes may result in damage to the device or harm to the patient.

**Electrolysis** – Safety has not been established for electrolysis within the vicinity of the Freedom SCS System. Use of electrolysis may result in damage to the device or harm to the patient.

**Laser procedures** – Safety has not been established for lasers within the vicinity of the Freedom SCS System. Use of lasers may result in damage to the device or harm to the patient.

**Radiation therapy** – Safety has not been established for high radiation sources such as cobalt 60 or gamma radiation when implanted with the Freedom SCS System. Use of radiation therapy could cause damage to the device or harm to the patient.

**Transcutaneous electrical nerve stimulation** – Safety has not been established for use of transcutaneous electrical nerve stimulation (TENS) when implanted with the Freedom SCS System. Use of TENS could cause the device to turn off or intermittent/increased stimulation.

**Other medical procedures** – EMI from the following medical procedures is unlikely to affect the device:

- Diagnostic ultrasound (e.g., carotid scan, Doppler studies)
- Diagnostic x-rays or fluoroscopy
- Magnetoencephalography (MEG)
- Positron emission tomography (PET) scans
- Therapeutic magnets (e.g., magnetic mattresses, blankets, wrist wraps, elbow wraps) – Keep the magnet away from the stimulator site. Magnetic fields will generally not affect the Stimulator.

**WAA Skin Contact** – Do not place the WAA directly on the skin. Direct skin contact may cause irritation and/or sensitivity to the materials. The WAA must be placed overtop a thin layer of clothing at all times.

**Painful Stimulation** – If the patient experiences painful stimulation, the amplitude on the WAA should be decreased immediately and/or removed from the patient's body. Contact your clinician if this continues to occur.

**Aircraft Usage** – Safety has not been established for use of the Freedom SCS System on aircrafts. Use of the Freedom SCS System on a commercial aircraft may result in damage to the device or harm to the patient.

## PRECAUTIONS

**Physician training** – Prescribing clinicians should be experienced in the diagnosis and treatment of chronic intractable pain and should be familiar with using the Freedom SCS System. Implanting clinicians should be experienced in spinal procedures and should review the procedures described in the Instructions for Use.


**Keep the WAA dry** – The WAA is not waterproof. Keep it dry to avoid damage. Do not use the WAA when engaging in water activities.

**Clean the WAA** – Clean the outside of the WAA with a damp cloth when needed to prevent dust and grime. Mild household cleaners will not damage the device or labels.


**Handle the WAA with care** – The WAA is a sensitive electronic device. Avoid dropping the device onto hard surfaces. Keep the WAA out of the reach of children and pets.

**Storage temperatures** – The Freedom SCS System should be kept within the storage temperatures listed on product packaging. Exceeding the storage temperature could cause harm to you or the component. Please contact the manufacturer if a storage temperature is surpassed.

Stimulator Receiver  
Storage Temperature

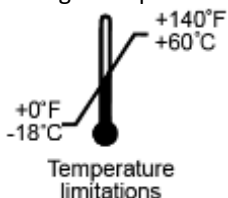

Wearable Antenna Assembly  
Storage Temperature

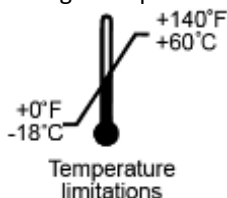

**Medical tests and procedures** – Before undergoing medical tests or procedures, contact the clinician to determine if the procedure will cause damage to the patient or to the System.

**Physician instructions** – Always follow the programs and therapy instructions established by the clinician. Failure to do so may cause the therapy to be less effective in providing pain relief.

**Use the WAA as directed** – Use the WAA only as explained by the clinician or as discussed in the User Manual. Using the WAA in any other manner could result in harm.

**Do not dismantle the WAA**– Do not dismantle or tamper with the device. Tampering with the device could result in harm. If the device is not working properly, contact the clinician for help.

**Flammable or Explosive Environments** – Do not use the WAA in flammable or explosive environments. Using the WAA in one of these environments could result in harm.

**Use of another patient's WAA** - Never use another patient's WAA. The therapy programmed is a unique prescription for each patient. Use of another patient's WAA could result in overstimulation.

**Activities requiring excessive twisting or stretching** – Avoid activities that potentially can put undue stress on the device. Activities that include sudden, excessive, or repetitive bending, twisting, bouncing, or stretching can cause your stimulator to fracture or migrate. This can result in a loss of stimulation, intermittent stimulation, and additional medical procedures.

**Scuba diving or hyperbaric chambers** – Do not dive below 13 meters (45 feet) of water or enter hyperbaric chambers above 1.5 atmospheres absolute (ATA). These conditions can damage the device. Before diving or using a hyperbaric chamber, discuss the effects of high pressure with the clinician.

**Skydiving, skiing, or hiking in the mountains** – High altitude should not affect the System. However, take care to not put undue stress on the Stimulator. During skydiving, the sudden jerking that occurs when the parachute opens can dislodge or fracture the Stimulator. This can result in a loss of stimulation, intermittent stimulation, and additional medical procedures.

**Unexpected changes in stimulation** – Electromagnetic interference, changes in posture, and other activities can cause a perceived increase in stimulation. Some patients have described this as a jolting or shocking sensation. You should reduce your amplitude to the lowest setting and turn OFF your System before engaging in activities that could become unsafe. Discuss these activities with your clinician.

**Interference during programming** - If interference is suspected during programming of the WAA, the clinician should confirm that the Bluetooth® data transmission is operating properly. Bluetooth® data communication is confirmed by the solid blue light on (when device is paired) and a fast blinking yellow light (only when data is transmitted). If during the programming session the light indicator is not blinking then the clinician should do the following:

- Terminate the current programming session and shut down the WaveCrest Application.
- Check for sources of Bluetooth® interference in the surrounding area.
- Remove or turn off the source of interference.
- Re-establish the Bluetooth® link with the WAA through pairing.
- Resume programming by opening the WaveCrest Application.
- Confirm the light indicator is now blinking.

## ADVERSE EVENT SUMMARY

Implantation of a spinal cord stimulation system is similar to any surgical procedure. Risks include the following:

- Allergic or immune system response to implanted material
- Infection
- Leakage of cerebrospinal fluid
- Epidural hemorrhage, hematoma, or paralysis

Therapeutic use of the Freedom SCS System incurs the following risks:

- Undesired change in stimulation, including uncomfortable chest wall stimulation
- Stimulator migration, erosion through the skin, or fracture causing a loss of therapeutic effect
- Electromagnetic interference causing a change in System performance
- Loss of therapeutic effect despite a functioning system

Adverse events that could occur with the Freedom SCS System:

- Stimulator migration, resulting in altered stimulation therapy that may be uncomfortable
- Stimulator fracture, resulting in loss of stimulation
- Infection, resulting in tissue sensitivity, redness and swelling

Adverse effects of stimulation are usually mild and go away when stimulation is turned off. Contact your clinician immediately if you experience any problems. Over time there could be changes in the level of pain control. Contact your clinician if you experience a change in stimulation that you believe is a result of the Stimulator slipping from the implant site.


## PARTS OF YOUR STIMULATION SYSTEM



**Freedom Stimulator** – Also known as an “implant” or “receiver”. The Stimulator is a set of thin wires and a miniature receiver, covered with a protective casing. The Stimulator has small metal electrodes near the tip that are set to different electrical polarities. An electrical field of energy is created when power is applied to the electrodes. The electrical field aids in blocking the pain signals coming from certain nerves near the spinal column. The Stimulator receives energy wirelessly from the external unit (WAA).





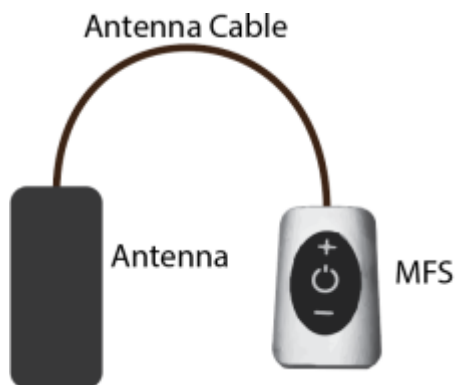







**Wearable Antenna Assembly (WAA)** – The WAA is an electronic device used to power the Stimulator. The device contains a rechargeable battery with an antenna and micro USB port (for charging only). The WAA is worn near the area where the Stimulator-Receiver is implanted.

The WAA communicates with your Stimulator-Receiver by sending radiofrequency signals through the antenna. Your Stimulator-Receiver only accepts communication from your WAA. The clinician programs your WAA with your specific Stimulation parameters from a wireless Bluetooth® programming tablet.



**Antenna and Antenna Cable** – The antenna sends out the wireless signal. The antenna is permanently attached to the antenna cable. The antenna is permanently connected to the Microwave Field Stimulator (MFS) with the antenna cable.

## OVERVIEW OF THE USER CONTROLS

BUTTONS			
Key	Action Description		
	<u>Increase Amplitude Button</u> – Used to increase the stimulation strength. Holding the button down for more than 2 seconds ramps the amplitude to the highest level, 24.		
	<u>Power ON/OFF Button</u> – Used to turn the WAA ON or OFF. Green LED is a default indicator and will blink to indicate power status. A solid Yellow light indicates that the antenna is disconnected and is not functioning.		
	<u>Decrease Amplitude Button</u> – Used to decrease the stimulation strength. Holding the button down for more than 2 seconds ramps the amplitude to the lowest level, 1.		
Change Program Selection – Depressing the Increase Amplitude Button ('+') and the Decrease Amplitude Button ('-') at the same time will alternate the programmed setting to either '1' to '2' to '3' and back to "1".			
LEDS			
Action Description		Pattern and Color	Description of Pattern
	Charging		Solid Blue
	OFF, Not Charging		None
	Charge Complete Battery FULL		Blinking Blue
	Low Battery		Blinking Yellow
	ON Setting 1		Blinking Green (single)
	ON Setting 2		Blinking Green (double)
	ON Setting 3		Blinking Green (triple)
	Minimum Index		Blinks Once Yellow
	Maximum Index		Blinks Once Yellow



## POSITIONING THE WAA




### WARNING:

- Do not place the WAA directly on your skin. Direct skin contact may cause irritation and/or sensitivity to the materials. The WAA must be placed overtop a thin layer of clothing at all times.

### Steps:

- The antenna must be placed over the general region of the Stimulator-Receiver in order to transfer the optimal amount of energy.
- The MFS and antenna cable does not need to be placed over the Stimulator-Receiver. Your clinician will assist you to find the optimal placement for effective therapeutic relief.

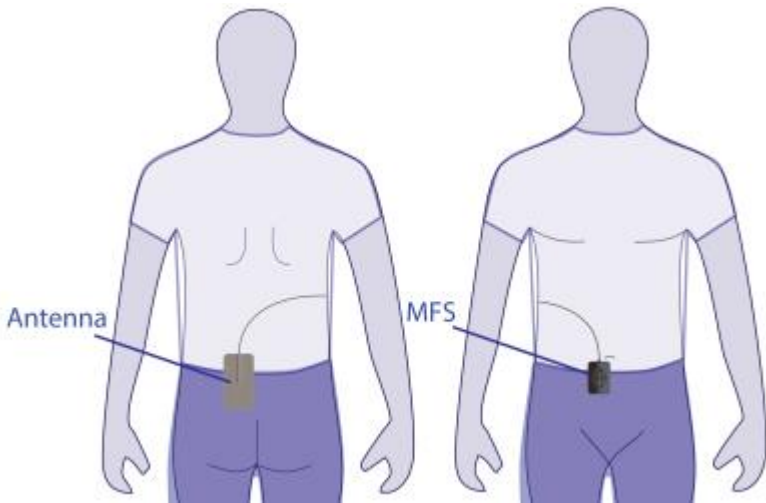

## STARTING STIMULATION

### Notes:

- *WAA automatically starts at lowest amplitude setting when turned on.*

### Steps:

1. Place the WAA Antenna directly over top of the Stimulator-Receiver.
2. Turn on the WAA by pressing the Power ON/OFF Key until the green Power Indicator Light activates.
3. Adjust the amplitude as directed by your clinician by using the Increase or Decrease Amplitude Key.

## SELECTING A STIMULATION PROGRAM

The WAA has three program options that may be set. The green indicator light is used to identify which program is currently active on the device.

- Program 1 = One Blink
- Program 2 = Two Blinks
- Program 3 = Three Blinks

### Steps:

1. After powered “On”, the WAA defaults to the last program used.
2. Press “+” and “-” buttons at the same time to switch programs.
3. The light pattern will blink to signify which program is active.

## INCREASING OR DECREASING AMPLITUDE



### CAUTION:

Turn the power off or decrease the amplitude before changing the position of the WAA Antenna to prevent possible uncomfortable stimulation.

### Notes:

- *WAA must be turned on to increase or decrease the amplitude.*
- *WAA does not need to be placed over the Stimulator to change amplitude.*

### Steps:

1. Press the “Up” arrow key to increase the amplitude.
2. Press the “Down” arrow key to decrease the amplitude.

To receive the most effective therapy you will need to adjust your stimulation throughout the day. Your clinician will provide guidelines about when you may want to adjust your stimulation. The following table provides general guidelines on how to adjust your stimulation.

Situation	Action
Stimulation is too strong	Decrease amplitude with the Decrease Stimulation Key
Stimulation is not strong enough	Increase amplitude with the Increase Stimulation Key
You have unexpected changes in stimulation	<ol style="list-style-type: none"> <li>1. Remove the WAA</li> <li>2. Decrease amplitude</li> <li>3. Put the WAA on your body</li> <li>4. Adjust amplitude to desired level</li> </ol>
You have tried adjusting stimulation but are unable to find an effective setting	Contact your clinician

## MAINTENANCE


### BATTERY CHARGING

#### Notes:

- *Use only the USB Charger and cable provided to perform charging.*
- *The battery is built into the WAA and does not need to be removed. Contact your clinician if you experience poor battery life.*
- *You will need the USB Charger Kit to recharge the internal battery.*
- *The antenna does not need to be connected during charging.*

#### Steps:

1. Remove the WAA from your body.
2. Connect the USB cable to the power adapter.
3. Plug the power adapter into a wall outlet.
4. Connect opposite end of USB cable to the micro-USB port on the WAA.
5. Charging Indicator Light will stay solid on while the battery is charging.
6. Allow the battery to charge for at least four hours.
7. Charging Indicator Light will blink when the battery is fully charged.
8. WAA is now ready to be used again.

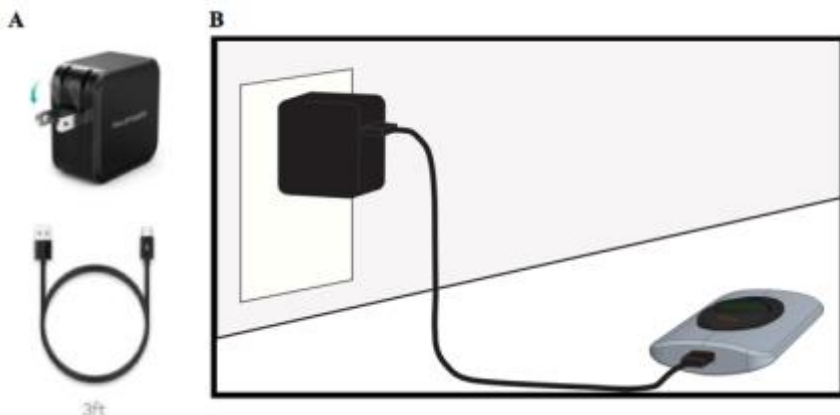

Common Questions	Response
How long will it take to recharge a “dead” or depleted battery?	It normally takes an average of three (3) hours to recharge the battery.
When is the battery near depletion, and how will I know?	The yellow indicator light will begin blinking. Eventually the WAA will turn off and not respond to user controls. You should connect the WAA to the charger as illustrated above.
What happens if I deplete the battery completely?	You cannot damage the WAA by running the battery completely empty. The device has safeguards to prevent this from harming the battery. Connect the WAA to the charger as described.
How long will a fully charged battery provide power?	Eight (8) hours on average. The battery performance is affected by the amount of total power used on average.
When is the battery done charging?	The battery is done charging when the Charging Indicator Light (blue) blinks on/off.
Can I charge while the device is turned on?	No, you cannot charge the device while it is turned on. The stimulation cannot be used while the device is connected the charger.
Must I disconnect the device from the charger when full?	It is not necessary to disconnect the device from the charger when full.
Will the WAA feel warm during or after charging?	The WAA may feel warm during or immediately after charging.

## WHEN TO CALL YOUR CLINICIAN

Contact your clinician if any of the following events occur:

- You have pain, redness, or swelling later than 6 weeks after the implant procedure.
- The stimulation is causing you to have pain or discomfort.
- The system is not working properly.
- You cannot adjust stimulation using the user controls.
- You cannot place the WAA Antenna in the optimal position to communicate with the stimulator.
- If you lose your WAA device.

Your clinician will schedule follow-up visits to make sure your device is working properly and that the stimulation is managing your pain.

## CLEANING AND CARE PRECAUTIONS

### **Notes:**

- Clean the outside of the device with a damp cloth when necessary. Mild household cleaners will not damage the device or labels.
- The device is not waterproof.
- Keep the device out of the reach of children and pets.
- Use the device only as explained to you by your clinician or as discussed in this manual.
- Handle the device with care. Do not drop, strike, or step on the device.
- Do not dismantle or tamper with the device.

## SAFETY AND TECHNICAL CHECKS

Periodic safety checks or maintenance of the WAA are not required. The WAA contains no user-serviceable parts. If repair or service is needed, contact your clinician or a Stimwave representative for a replacement. Refer to the contact information at the end of this manual.

## WAA DISPOSAL

The WAA should be returned to your clinician or a Stimwave representative. Do not dispose of your WAA in the garbage.

# PATIENT IDENTIFICATION CARD

A patient identification card (ID) is mailed to you after Stimwave receives your registration form. The patient ID card supplies information about you, your stimulator system, and your doctor. Your identification card may allow you to bypass security devices. Carry this card with you at all times. If you move, change doctors, or lose your card, contact Stimwave for a replacement card. Refer to the Stimwave contacts at the end of this manual.

## SPECIFICATIONS

Item	Specification
Amplitude	0 to 12.8 mA per electrode pair
Pulse Width	50 to 500 $\mu$ s
Frequency (of therapy)	5 to 1,500 Hz
Number of Channels	2
Number of Programs (selectable)	3
Transmit Carrier Frequency	915 MHz
Bluetooth Frequency	2.402 GHz
Charger Power Source	120-240V 50-60 Hz power line
Operating/storage temperature, relative humidity	-18° C to 60° C (0° F to 140° F) 20% to 90%
Operating/storage atmospheric pressure	70 kPa to 150 kPa (20.7 in Hg to 44.3 in Hg)
Size (approximate)	7.6 cm x 5 cm x 2 cm (3 in x 2 in x 0.8 in)
Weight (approximate)	0.5 kg (1 lb.)
WAA Material (Do not place directly on skin, to be worn over a thin layer of clothing at all times)	Aluminum

Parameter	Min	Typical	Max	Units
Supply Voltage	120	---	240	V <sub>A/C</sub>
Supply Voltage Frequency	50	---	60	Hz
Output Voltage (DC)	4.5	5	5.5	V
Input Current	---	---	1.0	A
Output Current (each)	---	---	2.4	A
Battery Recharge Time (assuming 3.7V battery)	2	3	4	hours
Modulation	---	CW	---	N/A

## WIRELESS INFORMATION

The Freedom SCS System uses wireless technology to program the WAA and to power the Stimulator. The WAA is programmed utilizing Bluetooth® data communication protocol. Bluetooth is used only during programming sessions and only by a trained clinician. Various programs are stored in memory within the WAA, which can be selected as needed.

WAA Wireless Specifications	
FCC ID	2AHXAPDBT2
Transmission Frequency	915 megahertz (MHz)
Bandwidth	149 kilohertz (kHz)
Tissue Depth	Up to 2 cm (0.80 in.)
Quality of Service	In order for the Freedom SCS System to operate, WAA should be centered over Stimulator-Receiver within 0.8 inches or 2 cm. WAA should be worn in the same position as when it was originally fitted. When the wireless link between the WAA and Stimulator is broken, stimulation will cease. The wireless link may not function in the presence of large magnetic or radio fields.
Bluetooth Version	4.1 (Low Energy Only)
Bluetooth Version of WaveCrest Application	4.0
Frequency of Bluetooth	2.4 gigahertz (GHz)
Bandwidth of Bluetooth	1 MHz
Power of Bluetooth	100 milliwatt (mW)
Bluetooth Operating Distance	4 to 5 meters
Wireless Link Performance	Wireless link active less than 10% of the time when WAA is 1 inch or closer to the implant.
Wireless Security	Stimulator will only operate within a short distance of WAA. The WAA uses encryption and proprietary data protocols to reduce the likelihood of inadvertent control or malicious “hacking” through Bluetooth®. Only the WaveCrest Application is able to communicate with the WAA via Bluetooth®. No identifiable personal data is stored or transmitted by the WAA.
Bluetooth Quality of Service	Typical Bitrate: 360 bps Maximum Data Latency: 100 ms Maximum Operating Distance: 1 to 3 meters

# TROUBLESHOOTING

**NOTE:**

- *If a problem is not solved after several attempts, or if a problem is not described here, contact your clinician or Stimwave representative.*

This information can help you to solve problems that may arise with the WAA. It also provides information on when to call your clinician. See the following table for a list of common problems and actions to remedy the issue.

Problem	Causes and Actions
Uncomfortable stimulation: You are too uncomfortable with the current stimulation to think about how to change it.	Selected parameter settings are not suitable for your activity or posture. <ul style="list-style-type: none"><li>▪ Reduce the amplitude of the WAA.</li><li>▪ Remove WAA from your body.</li></ul>
Intermittent stimulation: You feel stimulation only some of the time.	WAA may have a poor connection with the implanted Stimulator. The antenna may not be placed over the area of the implanted Stimulator. <ul style="list-style-type: none"><li>▪ Place antenna directly over the implanted Stimulator.</li><li>▪ If you are not receiving adequate pain relief, contact your clinician or Stimwave.</li></ul>
No stimulation: You do not feel stimulation but you think stimulation should be on.	Stimulation is off. <ul style="list-style-type: none"><li>▪ Turn the power OFF and wait 5 seconds before turning the power back ON.</li></ul> Antenna is not placed over the Stimulator. <ul style="list-style-type: none"><li>▪ Place the antenna directly over the implanted Stimulator.</li></ul> WAA amplitude is set too low to feel stimulation. <ul style="list-style-type: none"><li>▪ Increase the amplitude.</li></ul>



Problem	Causes and Actions
<p>The WAA is unresponsive: The indicator light does not turn on. The stimulation amplitude keys do not respond.</p>	<p>The WAA is not powered on.</p> <ul style="list-style-type: none"> <li>▪ Turn the power ON.</li> </ul> <p>The WAA system has “frozen”.</p> <ul style="list-style-type: none"> <li>▪ Turn the power OFF and wait 5 seconds before turning the power back ON.</li> </ul> <p>The battery is not charged.</p> <ul style="list-style-type: none"> <li>▪ Recharge the battery by connecting the WAA to the charger.</li> </ul>
<p>Dropped WAA: Your WAA falls off a cabinet or table.</p>	<p>WAA is designed to withstand a short drop on a hard surface and still operate normally. Try powering ON the WAA and allow it to transmit for 10 minutes while not worn on the body. If the WAA appears to be functioning properly and is not excessively heating, try using it. On its next charge cycle, monitor the WAA’s temperature while it is charging to prevent the battery from getting too hot. If the WAA seems to be heating up excessively while charging, then unplug the charging cable from the WAA and let it sit in a well-ventilated area for a few minutes to cool down. Contact your clinician.</p>
<p>Fluid on the WAA: Fluid was spilled onto the WAA or the WAA was dropped into water.</p>	<p>WAA is not waterproof, and water can damage the device.</p> <ul style="list-style-type: none"> <li>▪ Immediately remove the WAA from the water, then dry the WAA with a towel dampened with clean tap water. Unplug the power cord and allow the WAA to dry at room temperature for 24-48 hours.</li> </ul>
<p>WAA is heating up during operation.</p>	<p>The WAA will heat up during heavy operation, just like your cell phone will be warm during heavy usage. Make sure to keep the WAA ventilated during operation. Do not place the WAA directly on your skin. Do not cover the WAA with layers of clothing or fabric. If the temperature becomes uncomfortable then discontinue use, remove the WAA from near your body and contact your clinician.</p>

## CONTACT INFORMATION



### MANUFACTURER

Stimwave Technologies Incorporated  
1310 Park Central Boulevard South  
Pompano Beach, Florida 33064  
United States  
Telephone: +1.800.965.5134  
Fax: +1.800.965.5134  
Internet: [www.stimwave.com](http://www.stimwave.com)

05-00523-5 Published June 2017



### EUROPEAN AUTHORIZED REPRESENTATIVE

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands



### AUSTRALIAN SPONSOR

Emergo Australia  
Level 20  
Tower II, Darling Park  
201 Sussex Street  
Sydney, NSW 2000  
Australia