


User Manual

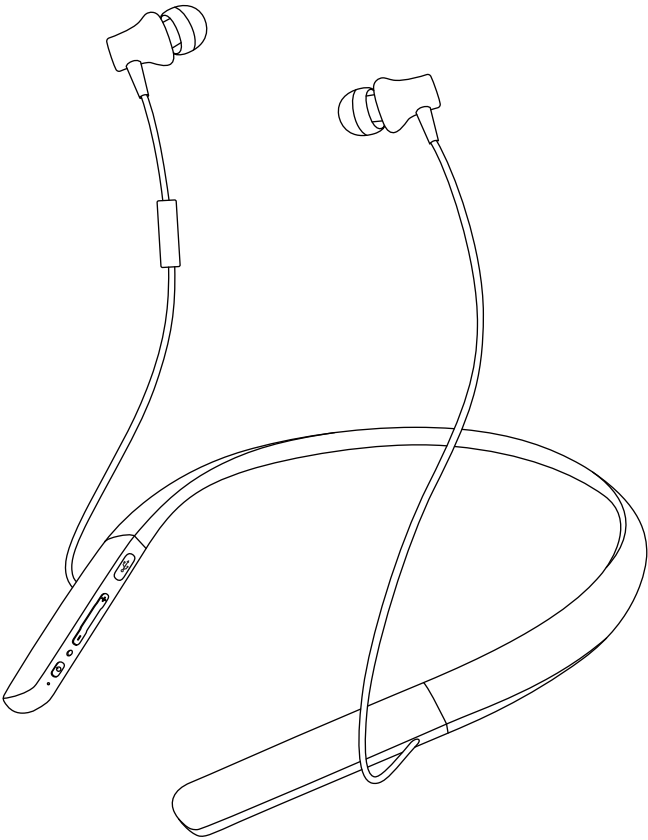




THANKS

Congratulations! Welcome to a New Life created by wireless neckband hearing amplifier. Please read this manual carefully before use.

Intended use

This package includes wireless neckband hearing amplifier . Our neckband hearing amplifier is designed for every occasion strong noise filter feature.

Configuration

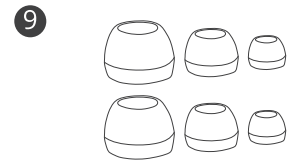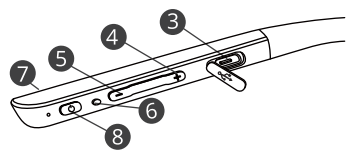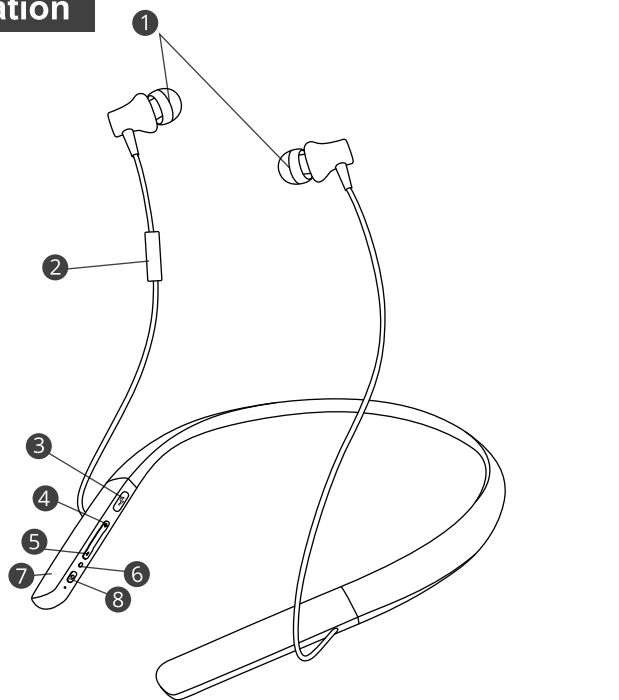


Neckband Hearing Amplifier

- ① Earphone
- ② Microphone
- ③ Charging port
- ④ Volume up
- ⑤ Volume down
- ⑥ Indicator light
- ⑦ On/off switch
- ⑧ Multi-function button
- ⑨ Dome



How to Use

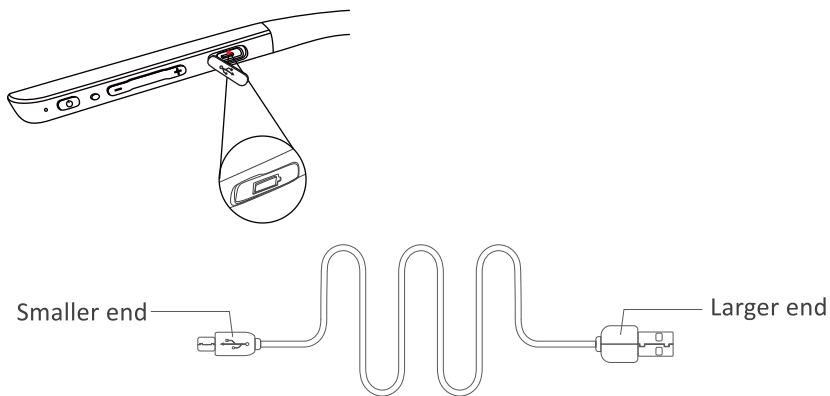1. Charge with Micro-USB cable



Charging the hearing amplifier is similar to charging a cellphone.

- Plug the larger end of the charging cable into the adaptor.
- Plug the smaller end into the USB port on the device.
- Plug the adaptor into a power outlet.

	Charging time	Charging indicator	Fully charged indicator
Neckband hearing amplifier	≤ 3 hours		

Low battery indication

The device will emit voice indicator “please charge” to remind you to charge the device when it is in low battery condition.

Notice:

- The devices contain built-in and non-removable rechargeable lithium-ion battery.
- Before using your amplifier for the first time, it is recommended to have the device fully charged.


2. Power on and off

Power on


Turn on the neckband hearing amplifier by pushing up the On-off button. You will hear voice indicator “Power on” when the device turned on.

Power off

Turn off the neckband hearing amplifier by pushing down the On-off button. You will hear voice indicator “Power off” when the device turned off.

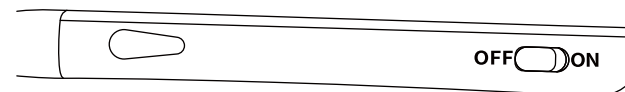

3. How to wear

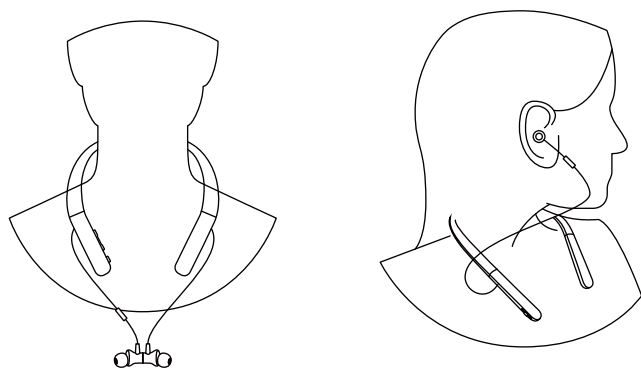


- 1) As shown above, slightly lift both ends of the device over the head, wear around the neck with control panel facing up.
- 2) Insert the earphones into both ears.



Notice:

The volume was preset after production. It's recommended to firstly adjust volume according to your individual need.

4. Adjust volume

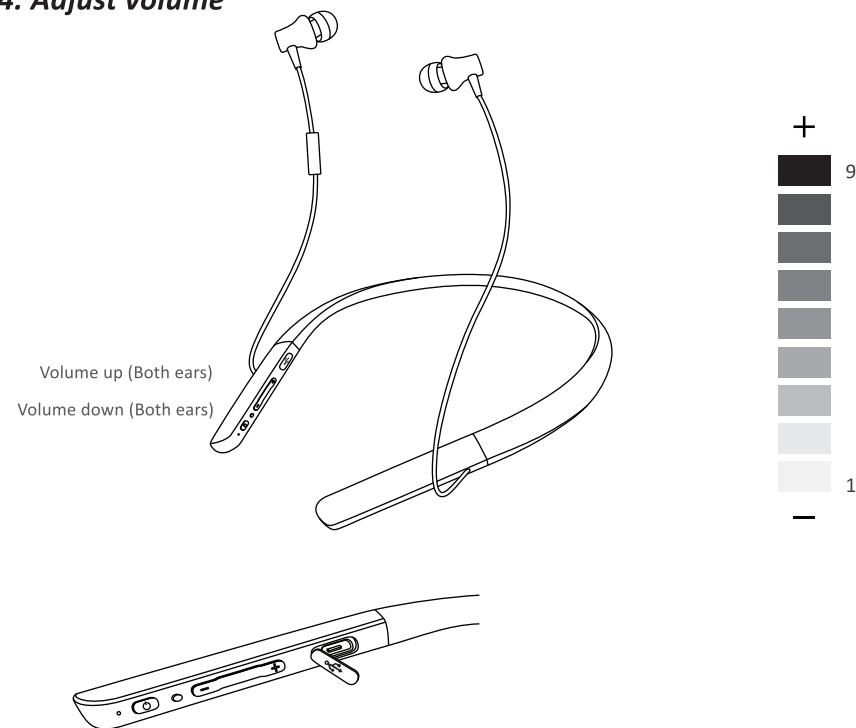


As shown above, short press “volume+” to increase volume, short press “volume-” to decrease volume.

Attention: There are total of nine levels of volume. When reach the minimum or maximum volume level, you will hear three beeps indicator.

5. How to connect with your smart phone:

- 1)Automatically enter the connecting state after booting.
- 2)Enable the Bluetooth on smart phone and connect with “BW52”.
- 3)If unsuccessful connection within 1 minute, need to restart.

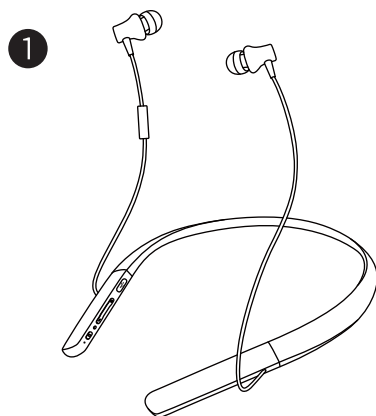


Pair automatically when take out

2




Enable Bluetooth on your phone

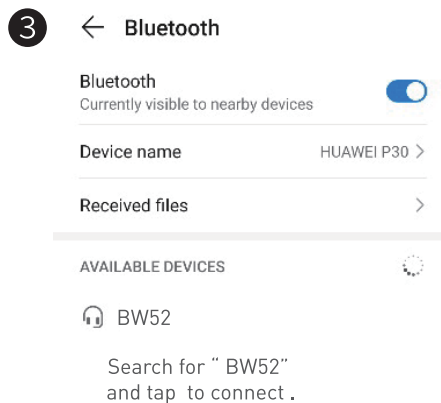

6. How to use Multi-function button:

- 1)Can only be activated when connected to a smart phone;
- 2)When connected to mobile phone, single click to start/pause music;
- 3)Phone incoming call, single click to answer/hang up the call; long press for 2s to reject the call;
- 4)Connected to the mobile phone, short press twice to call the nearest outgoing call.

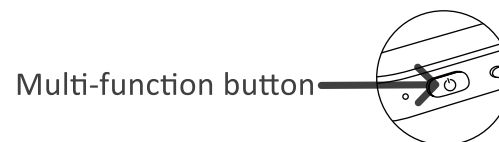

Quickly operate the core function		
	Press once	Music listening play/pause
	Press once	Answer/Hang up phone call
	Hold for 2 seconds	Reject call

FCC Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

RF warning for Portable device:

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.