



Know about the hearing devices and accessories

1. Hearing devices

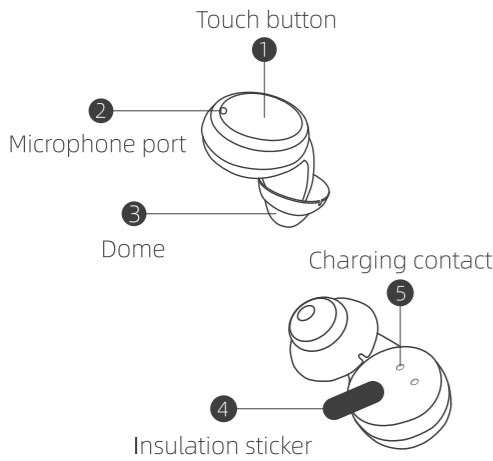

2. Charging case

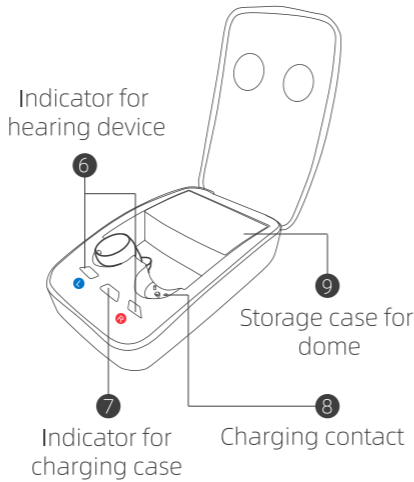

3. About the dome

Magic II dome

More comfortable to wear and more effectively reduce the whistling sound.


Round top dome (Power upgrade)

Effectively reduce the feedback/whistling sound.


Notice:  
The quantity may differ from different configuration.

How to assemble and wear the devices

4. How to assemble

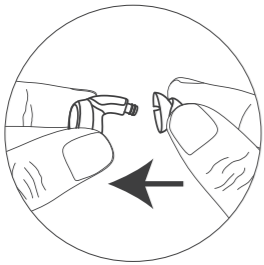

Step 1  
Choose a dome.

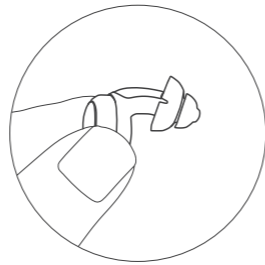

Step 2  
Mount onto the connector firmly.

Note: Please tear off the insulation sticker and start using. The devices come with pre-assembled domes. You can replace one to fit your special needs.


5. How to wear




Step 1  
Hold the device.



Step 2  
Insert into ear canal.

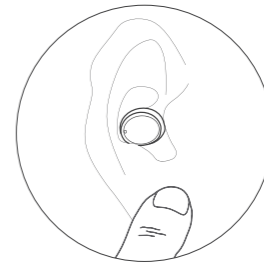

Step 3  
Gently tug on your ear lobe.

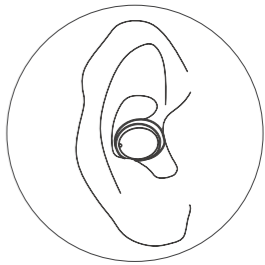


Step 4  
Push the device into best fit position.

TIPS: Correct position: Microphone port facing the auricle. Do not touch the microphone port during wearing to prevent howling. Whistling may happen if not placed into place, keep patience and try again.

How to charge the devices


6. How to charge the devices

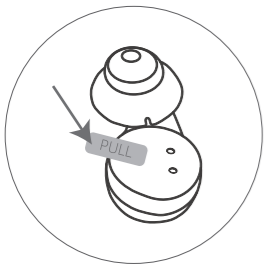


Step 1  
Please tear off the insulation sticker before charging.




Step 2  
Place the devices into charging case

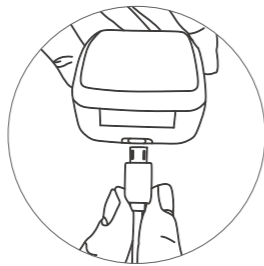


Step 3  
Plug the small end of cable into charging port of charging case.

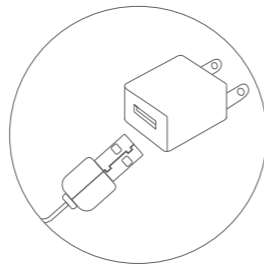

Step 4  
Plug the big end of cable into adaptor.

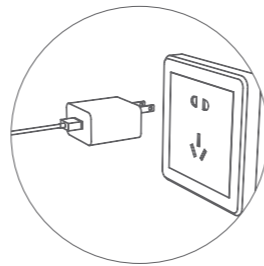

Step 5  
Connect with power.

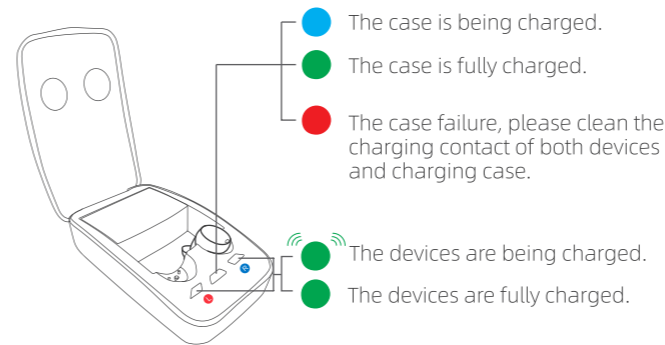


Notice:

- 1) Always put your hearing device into the charging case without a dome attached.
- 2) Always put your hearing device into the charging case when you do not use them.
- 3) We strongly recommend you fully charge the devices when you first time received.
- 4) After the charging cycle, the case will start drying process for 3 hours. The drying process will stop once devices are removed. And drying time will be recalculated when devices are replaced in again.

## How to charge the devices


### 7. Without external power connection

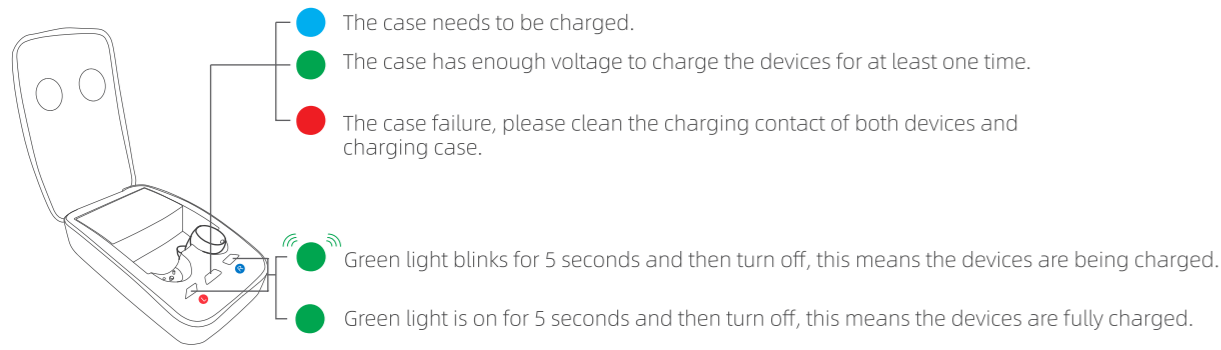

**Notice:**  
The devices will emit 5 beeps every 5 minutes when they are in low battery.

### 8. ON-OFF switch

To turn on: Just take the devices out of the charging case, they will be automatically turned on.  
To turn off: Place the devices into corresponding port of the charging case.

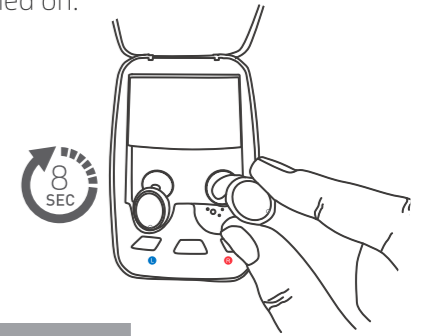


**Notice:**  
The devices are equipped with power-on delay function. It needs about 8 seconds to get started. Whistling maybe heard when the devices are turned on but not positioned securely in your ear. Once properly inserted, the whistling will stop.

## How to use the devices

### 9. Memory Switch and Volume Control

There are totally 4 listening programs and 6 level sound under each program.  
Follow the guide below to change the program and volume according to your needs.

	Specifications		Operation	Indicator
	Program	Sound		
Program	Program 1	Normal	Long press the touch button for 2-3 seconds.	1 beep sound
	Program 2	Restaurant		2 beeps sound
	Program 3	Traffic		3 beeps sound
	Program 4	Outdoor		4 beeps sound
Volume	From 1 to 6		Short press the touch button	1 beep sound indicator for each change; 3 beeps when reach the highest volume level

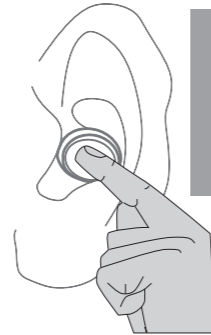

**Notice:**  
The device will be in program 1 when you first time use it. After that, it will be in program and volume you set last time.

## Maintenance

### 10. Troubleshooting


No/ low sound amplification	
POSSIBLE CAUSE	SOLUTION
Low battery	Recharge the device
Built-up wax in the dome	Clean the dome or replace the old dome
Excessive ear wax	Consult your physician
Improper dome insertion	Re-insert the dome
Hearing device damaged	Contact your provider
Hearing change	Contact your hearing care professional
Whistling or feedback	
Dome is wrong size or damaged	Replace with new or better fitting dome
Improper dome insertion	Re-insert the dome into your ear so that it is flush with the wall of the ear canal

## Maintenance

### 12. Care (You should inspect and clean your hearing devices everyday)


#### 1. Clean the exterior & charging contact

To clean, gently wipe the surface with a soft, dry cloth.  
Use the brush to clean the receiver port.  
Do keep the charging contact and charging port clean and dry.  
It is recommended to use dry box or electronic dry box for drying every day to reduce moisture and wax buildup.

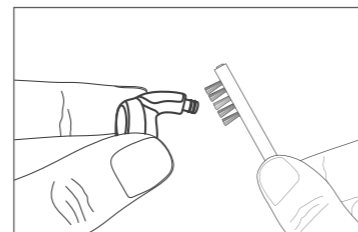

#### 2. Clean the dome



Use the brush to clean the dome.  
It's recommended that replace with new dome every 2-3 months.

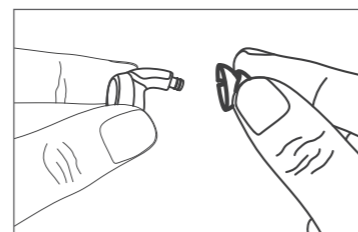

The hearing instruments and charger must be clean and dry in any case.

## How to use the App

Step 1: Wear on your device and turn on it.  
Step 2: Enable the Bluetooth on your phone.  
Step 3: Search the device.  
Step 4: Tap to connect your device with your phone.  
Step 5: You can adjust the volume, switch listening program and many customized adjustment through the APP.  
Kindly refer to the APP manual for better using.

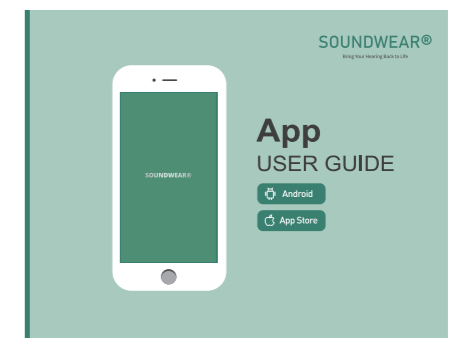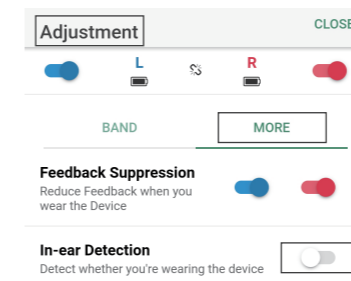

**Notice:**  
Only Available on iOS 9 or later, iPhone 5 or above;  
Android 6 or later, screen size 4" or above.

## **FCC Statement**

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

## **RF warning for Portable device:**

**The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.**