R10

20

128

186

300GC2S

BACK

EN Warnings / Important Safety Information

! **WARNING:** The Owlet Smart Sock is not a medical device. It is not intended for use as a medical device or to replace a medical device. It does not and is not intended to diagnose, cure, treat, alleviate or prevent any disease or health condition or investigate, replace or modify anatomy or any physiological process.

Warranty
Owlet proudly stands behind its products. Our warranties are found at: <https://owletcare.com/pages/warranty>.

Terms and Conditions
The use of Owlet's products is subject to the terms and conditions found at: <https://owletcare.com/pages/terms>. By using an Owlet product, you agree to those terms.

For contact information specific to your region, please visit <https://support.owletcare.com>.

Safety
Your safety and the safety of others are very important to us. There are many important safety messages in this manual and in the Owlet App. Always read and obey all safety messages.

! FOR YOUR SAFETY AND THE SAFETY OF THE BABY:
RESPOND IMMEDIATELY WHENEVER A NOTIFICATION SOUNDS.

! ALWAYS KEEP THE BASE STATION AT LEAST 3 FEET AWAY FROM YOUR BABY'S CRIB BECAUSE THE CORD CAN POSE A STRANGULATION HAZARD.

THE OWLET SMART SOCK IS AN AID. IT IS NOT A SUBSTITUTE FOR ADULT SUPERVISION.

FOR PREMATURE BABIES, BABIES ON SUPPLEMENTAL OXYGEN, OR BABIES WITH HEALTH CONDITIONS, PLEASE CONSULT YOUR HEALTH PROFESSIONAL TO DETERMINE WHETHER OR NOT THE OWLET SOCK IS RIGHT FOR YOU.

FC Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. Please ONLY use the supplied power adapter. Failure to do so may void our limited warranty. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the manufacturer's instructions, may cause interference harmful to radio communications.

IC: 21386-OBL3A (Base Station)
IC: 21386-OSS3A (Sensor)
This device complies with part 15 of the FCC rules and with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions: (1) this device may not cause interferences, and (2) this device must accept any interferences, including interferences that may cause undesired operation of the device.

CAN ICES-3 (B)/NMB-3(B)

CE Owlet Baby Care declares that the Owlet Smart Sock is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: <https://owletbabycare.co.uk/pages/ce-compliance>

2402 - 2480MHz (63mW Output Power) - Base Station Bluetooth
2412 - 2462MHz (75mW Output Power) - Base Station Wi-Fi
2402 - 2480MHz (6.3mW Output Power) - Sensor Bluetooth

! Owlet is committed to meeting the requirements of directive 2012/19/EU on waste electrical and electronic equipment (WEEE). Please visit our website at owletcare.com to find out more about the proper disposal of Owlet products in the EU and UK.

INPUT:100-240V-0.2A 50-60Hz
OUTPUT:5V→1A

Conforms to UL 62368-1 and CAN/CSA-C22.2 No. 62368-1

U.S. Pat. Nos. 9,693,730 and 10,449,837. Other patents pending.

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