


Neeuro User Manual

Understand the product

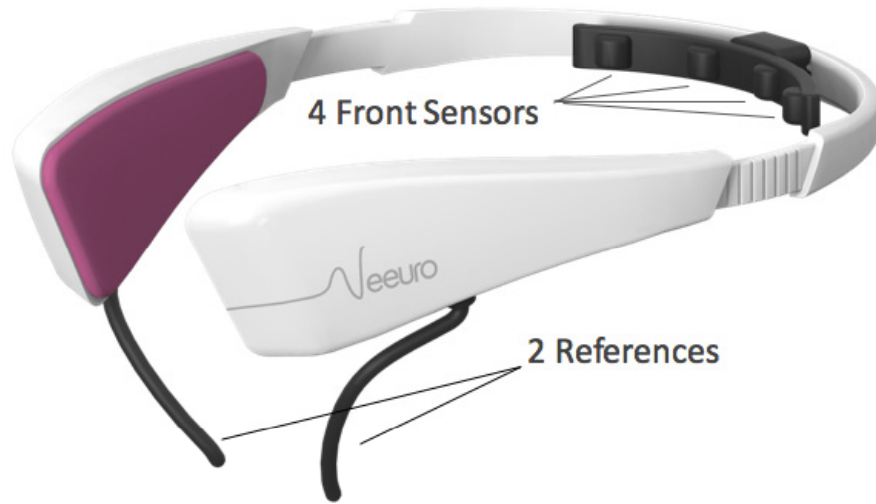

EEG Sensors

Proprietary elastomer sensors. 4 channels and 2 references at 250 samples per second per channel with a resolution of 0.8 uV.



Accelerometer

Monitors for any head movements that can be used to detect motion.



Bluetooth Syncing

Synchronize wirelessly with leading smartphones and tablets using Bluetooth 4.0 BLE.



Mental Performance

Measures and tracks your attention, focus, mental workload, relaxation and stress levels.



Compatibility

Compatible with leading iOS and Android smartphones and tablets.



Battery

Internal 350mAh lithium polymer battery, with up to 4 hours of continuous usage.



Patented Design

Stylish, yet designed with comfort in mind. Made with high quality materials to ensure the best experience.



Extensibility

Neeuro SenseBand is extendable to pair with different applications.

Before you start:

Step 1: Download the App



You can download the APP from Apple App Store or Google Play Store



Step 2: On your mobile device, turn bluetooth On


For iOS devices:

1. Go to Settings.
2. Select "General"
3. Then select "Bluetooth"
4. Ensure that the Bluetooth option is turned on.

For Android devices:

1. Go to the Settings menu.
2. Select the **Bluetooth** menu.
3. Ensure that the Bluetooth option is turned on.


Step 3: Turn on your Neeuro SenzeBand

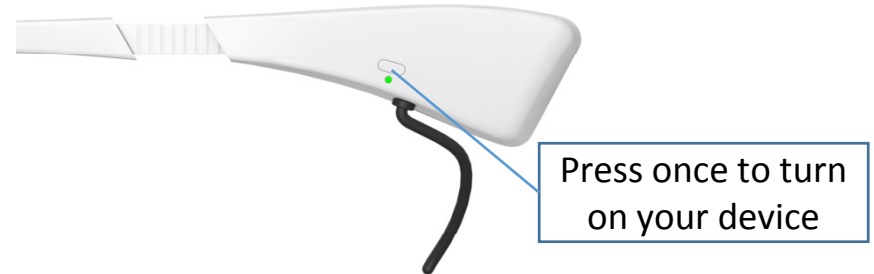

Step 4: Launch the App

1. Select the "Settings" tab.
2. Select the Device option.
3. Tap on the lower side (below the white line) of the screen
4. Click on the "Add Device" button and follow the instructions.

Powering On and Off

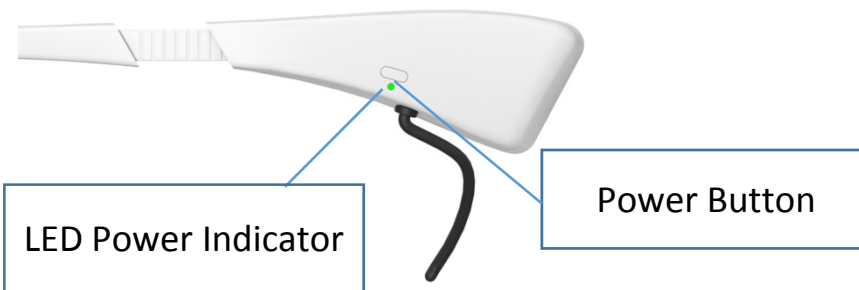


Powering On Neeuro SenzeBand

1. Press the power button once.
2. Wait the green light to blink 3 times.
3. The green light will stay on as an “On” indicator.

Powering Off Neeuro SenzeBand

1. Press the power button once while the green light is on.
2. The green light will turn to red light and blink 3 times before it turned off.

Charging

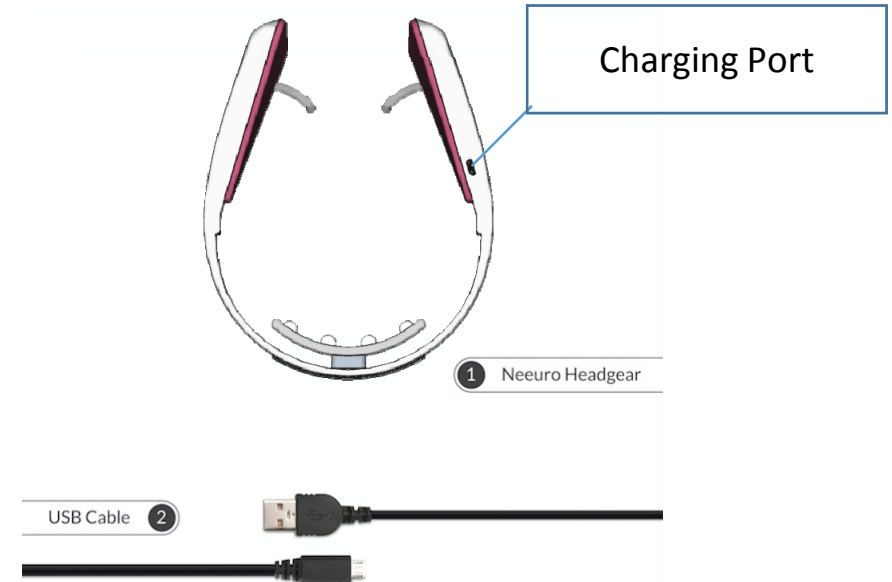


Charging Neeuro SenzeBand

Plug in the charging cable that is included in the package to the Neeuro SenzeBand charging port.

Wearing Instruction


Headband Strip

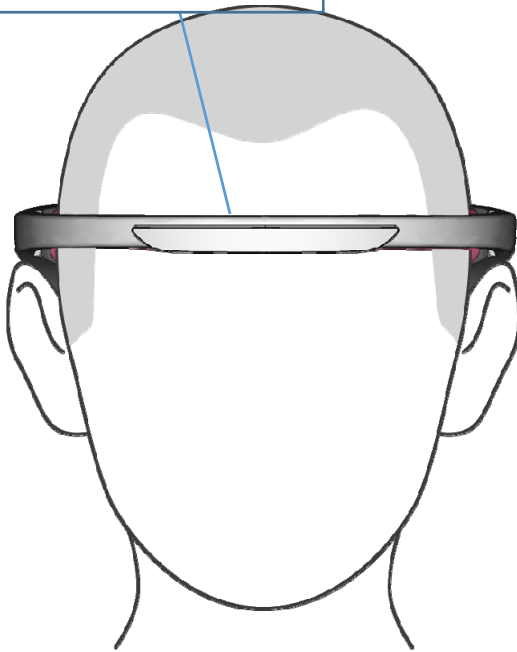

Frontal View

- Make sure that the headband strip sits about 1 inch above the eye brows.


Reference Sensors

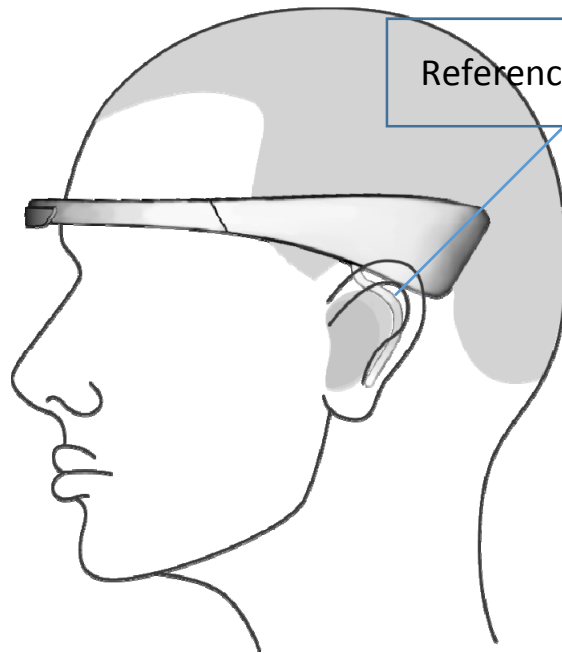

Side View

- Make sure that the 2 reference sensors touch the back part of the ear completely.

4 Front Sensors

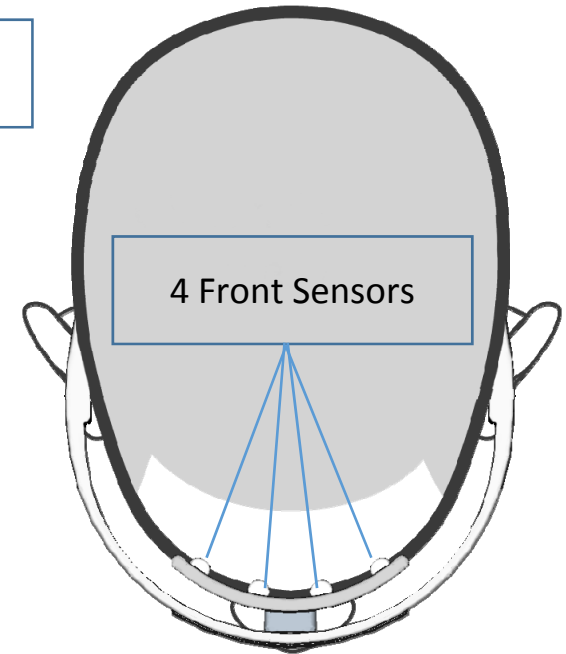

Top View

- Ensure the 4 front sensors touch the skin of the forehead properly.

- **FCC Caution:**

-

- Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

- This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

- **FCC Radiation Exposure Statement:**

-

- This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment .

-

- This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.