




Conduit SMARTCAP

The Conduit SMARTCAP event capture device is designed to record the time of an insulin injection event and transmit it directly to a receiving smartphone application. The device is intended to replace the cap of commercially available insulin pens & is capable of detecting when the cap is removed or recapped on the insulin pen. The cap has the ability to store & transmit the time of the event wirelessly, to a bonded smartphone with the Conduit application installed.

Device setup:

1. Install Conduit smartphone application.
2. Select matching Conduit SMARTCAP for your insulin pen.
3. Remove device from outer package.

4. In the space provided, place appropriate color label that matches your insulin type onto the Conduit SMARTCAP. Discard remaining labels. If no matching label is available, leave the space blank.

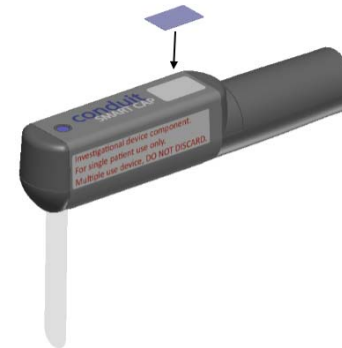

5. Supply power to device by removing plastic tab from Conduit SMARTCAP.



6. Remove existing pen cap from your current insulin pen.
7. Follow in-app instructions for bonding the Conduit SMARTCAP device to your smartphone.
8. Once the Conduit SMARTCAP is bonded to the smartphone application, it will wirelessly transmit data without the need for additional intervention.
9. Leave Conduit SMARTCAP on insulin pen until your next injection

General use:

- Continue to use your disposable insulin pen according to your doctor's recommendations.
- The SMARTCAP is intended to remain on your insulin pen between injections.
- When the times comes to take an injection, remove

SMARTCAP from the insulin pen and perform injection as instructed by your doctor or caregiver.

- When finished with the injection, immediately recap your disposable insulin pen with the Conduit SMARTCAP device. Be sure the cap is fully seated on the insulin pen.
- The Conduit SMARTCAP will automatically begin to transmit the injection event time to the Conduit app.
- The event will only be recorded as an injection by the smartphone app if the Conduit SMARTCAP is removed from the insulin pen for greater than 6 seconds, but no longer than 10 minutes.
- When transitioning from an empty pen to a new pen, follow the same injection procedure and place Conduit SMARTCAP on new pen after the injection.

Disclaimers & Maintenance:

- Investigational device component.
- For single patient use only.
- Multiple use device, DO NOT DISCARD.
- Do not use Conduit SMARTCAP with any other disposable insulin pen or transition cap between different insulin types.
- Handle device with care. Significant damage may occur to critical electrical and plastic components if device is dropped or crushed.
- Do not store device in refrigerator.
- Exterior of Conduit SMARTCAP may be cleaned with pre-saturated isopropyl alcohol swabs.

- Do not submerge, wash, lubricate, or rinse device.



Technical Support:

For additional support with the Conduit app or Conduit SMARTCAP please call 1-800-919-3314

CAUTION -- Investigational device. Limited by Federal (or United States) law to investigational use.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

Any changes/modifications to this equipment not approved by Becton, Dickinson and Company could void the user's authority to operate the equipment.



FCC ID: 2AJJ5-CHRONOS