6. Garanzie implicite

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USA

FCC ID: 2APV7-AVA02

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference.
- (2) this device must accept any interference received, including interference that may cause undesired operation.

The Ava Fertility Tracker has been designed and complies with the safety requirements for portable RF exposure in accordance with FCC rule part §2.1093 and KDB 447498 D01.

Canada

IC: 23955-AVA02 Model 2

EN: This device complies with ISED's license-exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device

This device complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 for portable use conditions.

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Europe (EU/EFTA)

EN: Declaration of conformity with regard to the EU Directive 2014/53/EU.

Ava AG is authorized to apply the CE mark on its product, thereby declaring conformity to the essential requirements and other relevant provisions of Directive 2014/53/EU and other applicable directives. The complete declaration of conformity can be found in the specific product section at www.avawomen.com/quality

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ABBREVIATIONS

ISED: Innovation, Science and Economic Development Canada

FCC: Federal Communications Commission

ESD: Electrostatic Discharge / Descargas Electrostáticas

PCOS: Polycystic Ovary Syndrome / Polyzystisches Ovar Syndrom / Sindrome dell'Ovaio Policistico

www.avawomen.com/getstarted

www.avawomen.com/de/esgehtlos

www.avawomen.com/fr/commencer

www.avawomen.com/es/empieza

www.avawomen.com/it/siinizia

avao

Ava App Guide

Version 1.2

©2018 AVA AG. all rights reserved.

Ava and Ava logotype are trademarks of Ava AG.

All other trademarks are property of their respective owners.

Patent Pending.

Ava AG

Gutstrasse 73, CH-8055
Zurich, Switzerland
www.avawomen.com

Representative in the USA

Ava Science, Inc.
77 Geary Street
San Francisco, CA 94108, USA

TABLE OF CONTENT

Indication of use	3
Intended use	3
<i>Using the App</i>	
Pairing	4
Trying to conceive / Cycle tracking	5
Navigation	6
Dashboard & Calendar	7
Health data	8
Logging	9
Additional features	10
Profile	11
Cycle Report	11
Alarm	12
History view	13

Regulatory information	14
Settings	15
Pregnancy	16
Navigation	17
Dashboard & weekly content	18
Health data, Logging & Profile	19

INDICATION FOR USE

The Ava Fertility Tracker is for women who want to conceive or gain a better understanding of their bodies. It is for women with cycles between 24-35 days. It is not for women with PCOS or cycles outside of a 24-35 day range. Ava is not a contraceptive and cannot be used to prevent pregnancy.

INTENDED USE

The Ava Fertility Tracker is a non-invasive device intended to measure and display physiological parameters to aid women in ovulation prediction to facilitate conception. Additionally, parameters are collected to improve the quality of the prediction, and to provide general information on health and wellness.


Temperature is the most important measurement in support of the Ava Fertility Tracker's prediction of the fertile window. This, in combination with historical menstrual cycle data, is the basis for the Fertility Tracker technology and the intended use of the device.

The Ava device (bracelet) is combined with smartphone applications to display the physiological parameters and to gather user inputs.

Mobile device compatibility



Ava syncs with the following operating systems and devices:

-  iOS 10.0 or later
-  Android 5.0 or later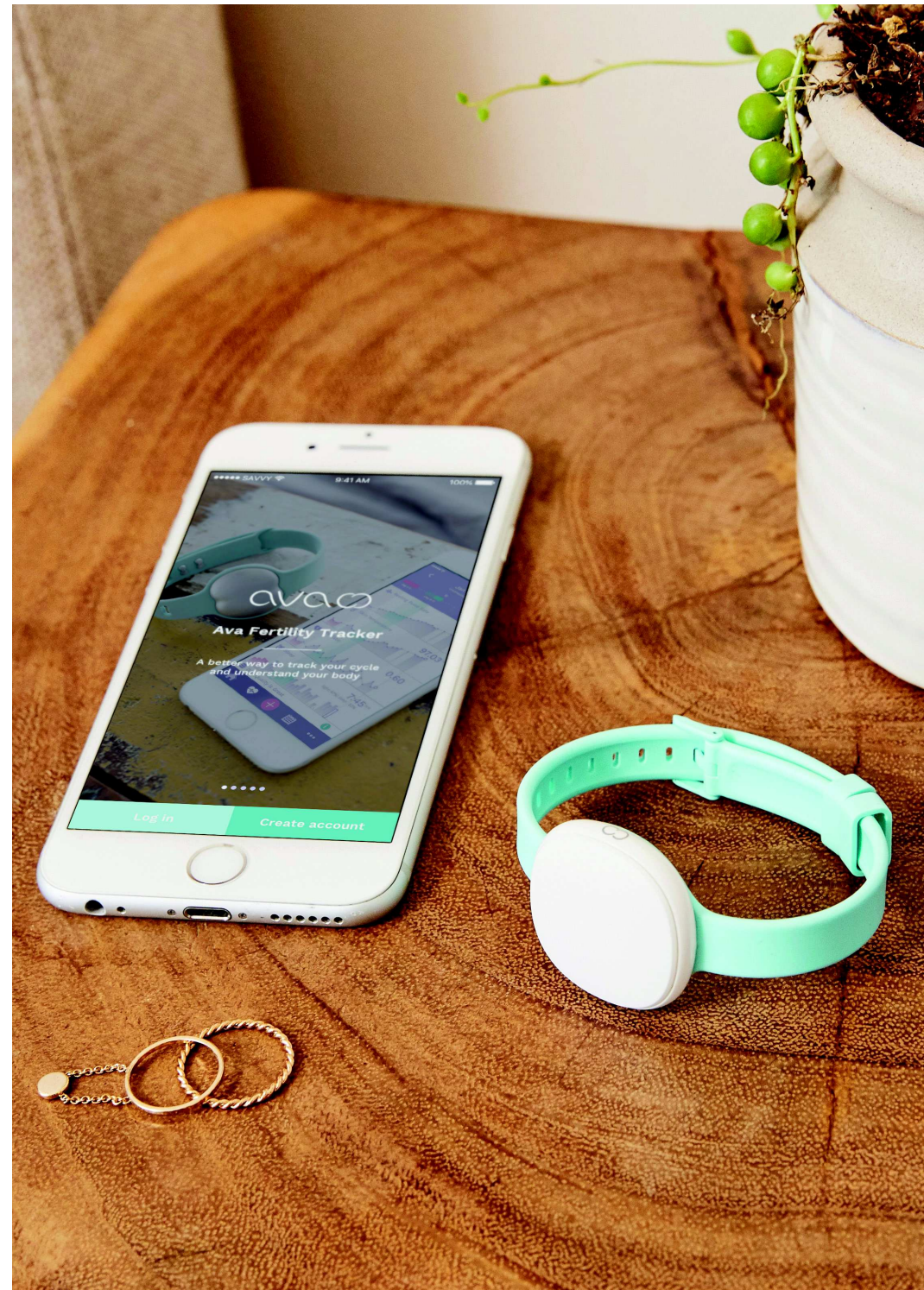


PAIRING

After you set up your account you will be asked to pair your Ava with the app. Pair the Ava before you wear it for the first night. In order to do so, follow the instructions on the screen.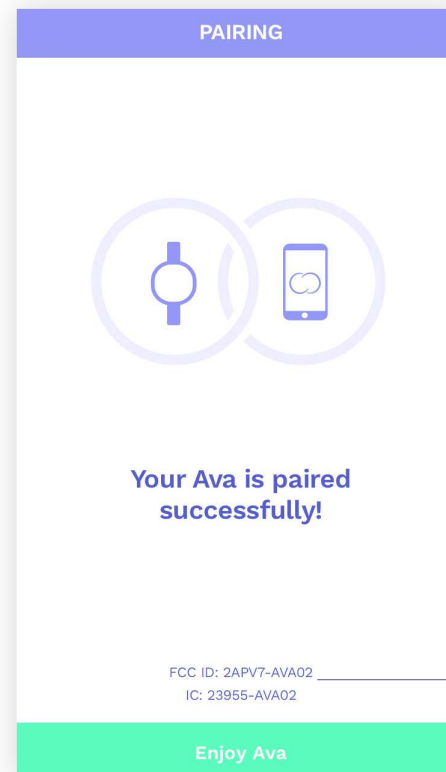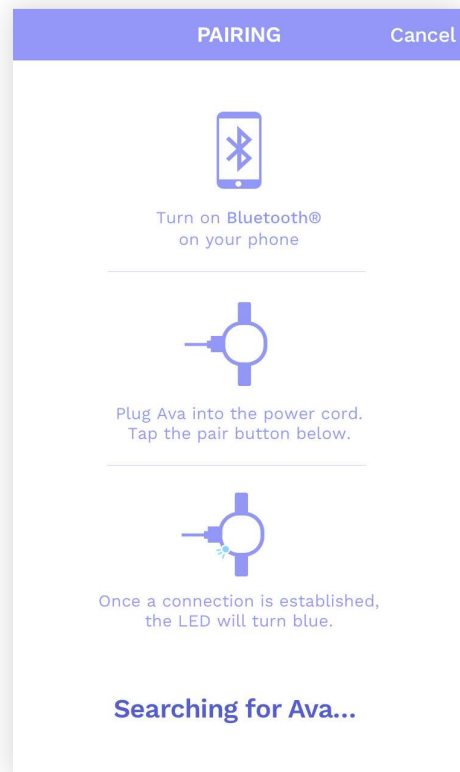

FCC ID: 2APV7-AVA02
IC: 23955-AVA02

Regulatory information
[See [Regulatory](#) - p. 13]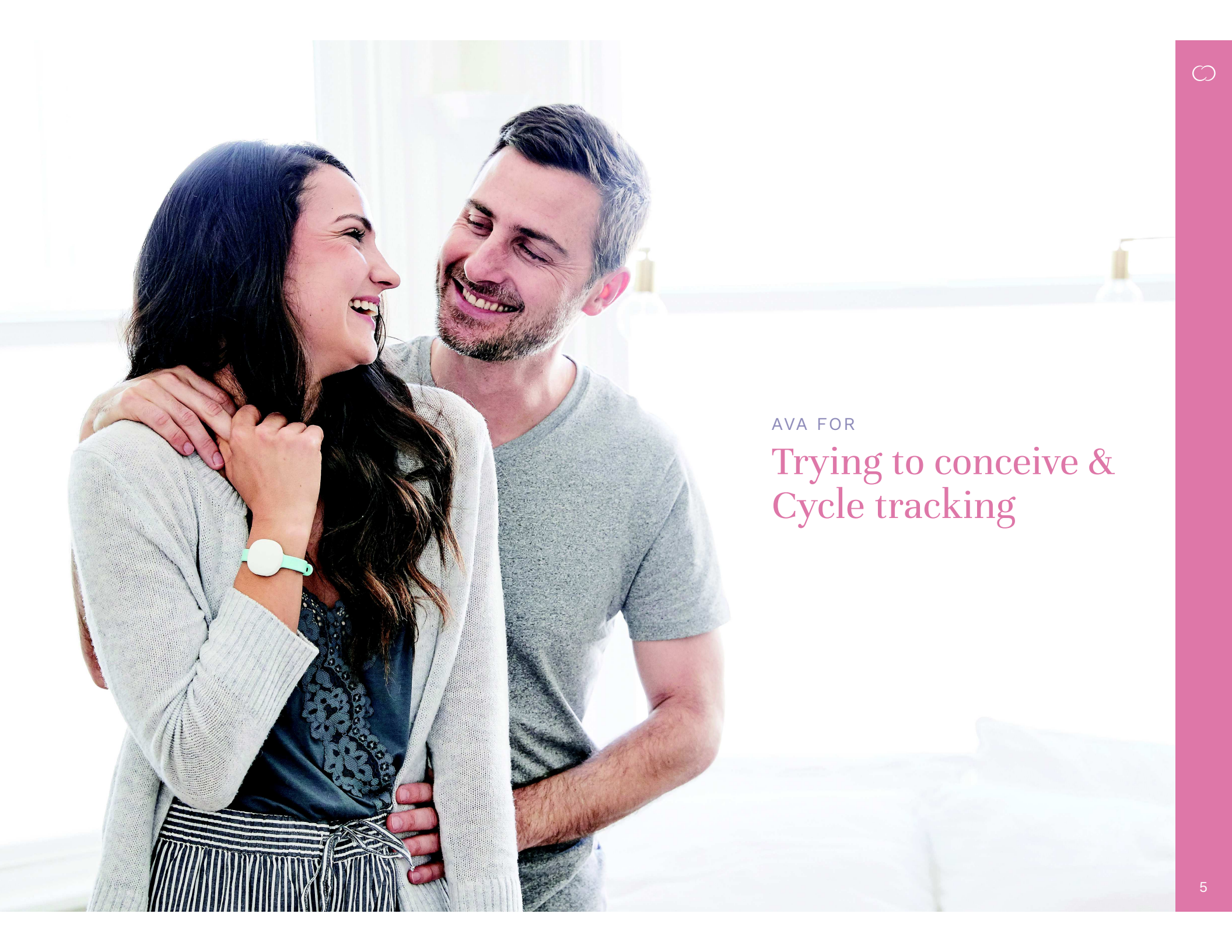


AVA FOR


Trying to conceive & Cycle tracking

NAVIGATION


iOS Devices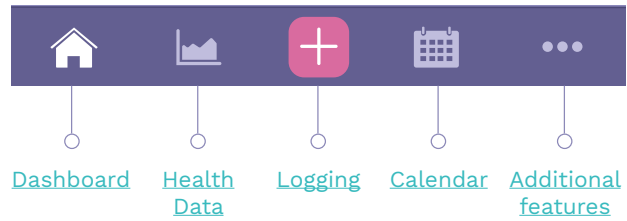

Logging

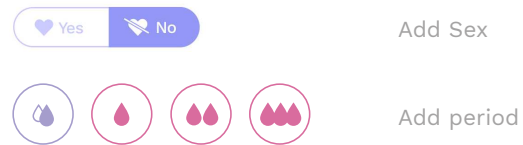
Calendar

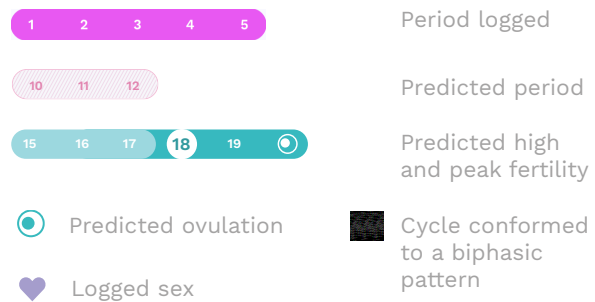Health charts


Android Devices

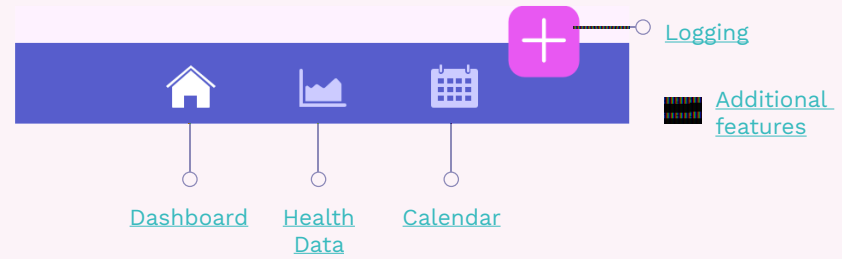Logging


Calendar

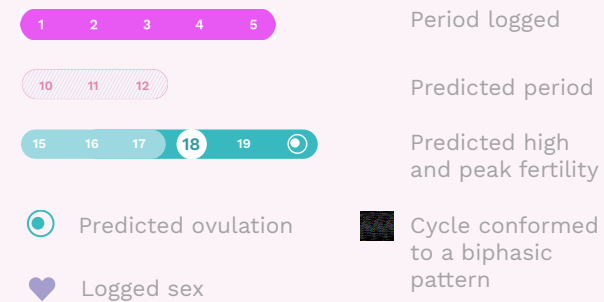Health charts

DASHBOARD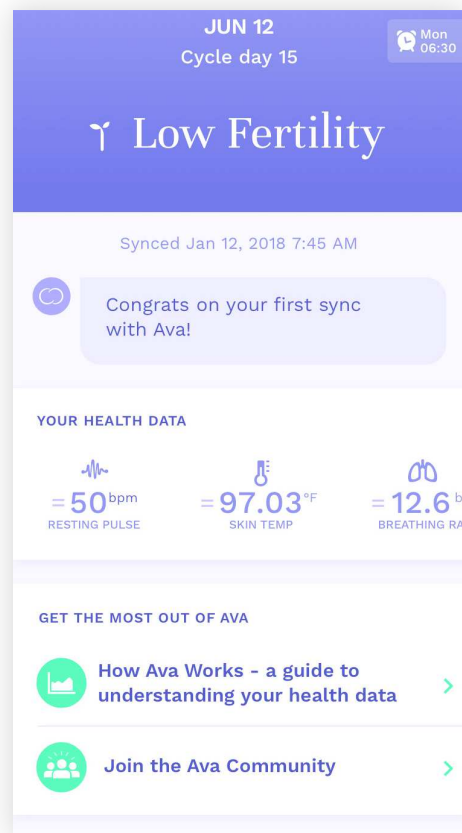

Different cards you can see in this section:


Measurement of the day

Swipe down to see more

CALENDAR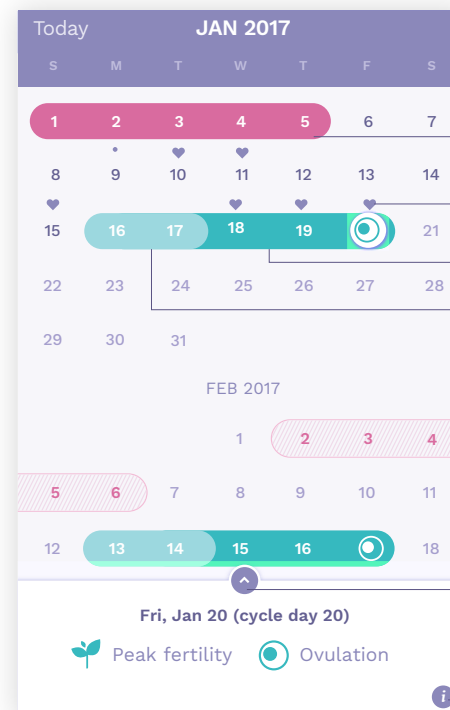

Period logged

Sex logged

Peak fertility

High fertility


Period predicted

Tap to see logged items

Tap for calendar key

HEALTH DATA


The Health data view enables you to see your parameter values for a specific day and visualize your data throughout your cycle. You can also see what you've logged in correlation to your health data.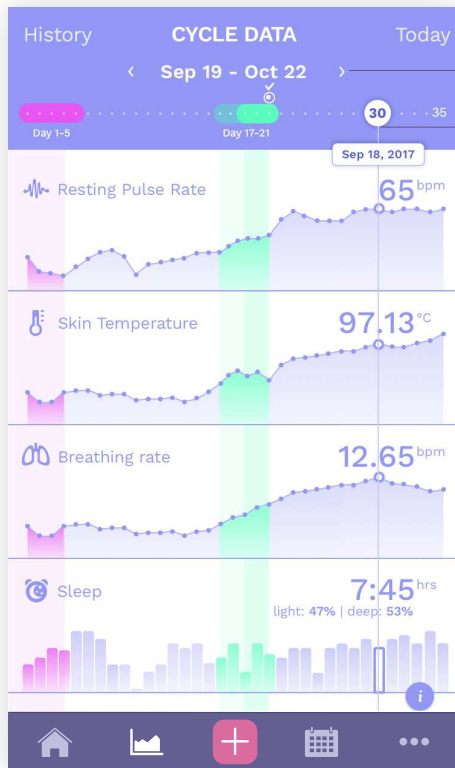

○ Navigate through your cycles

○ Drag it left or right to see more data values for different days

○ Swipe down to see more data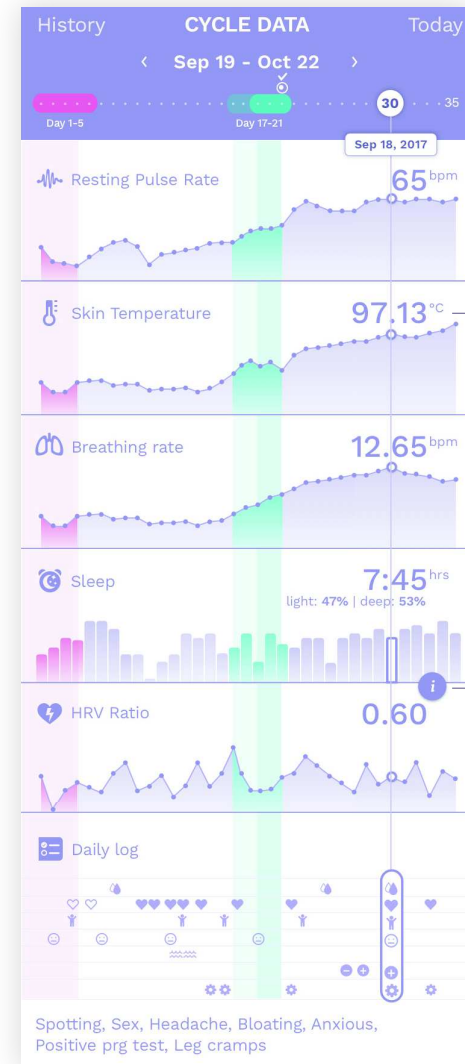


Complete view

LOGGING

Log items during your cycle and see your logging pattern in the calendar and health data view.


Make sure you always log your period in order for Ava to make accurate predictions.

iOS Devices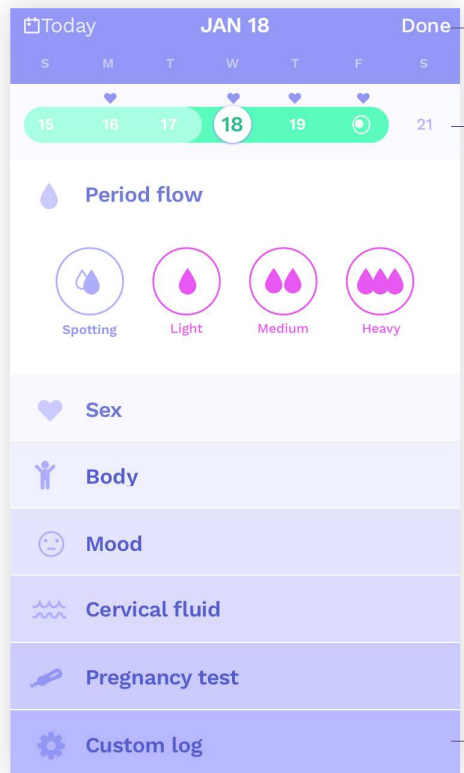

- Confirm log
- Swipe left or right to navigate through days
- Add your period by tapping on the icon. Tap again to remove it.
- Additional logging options
- Tap to create your own customized log. You can create up to 10 logs.

Android Devices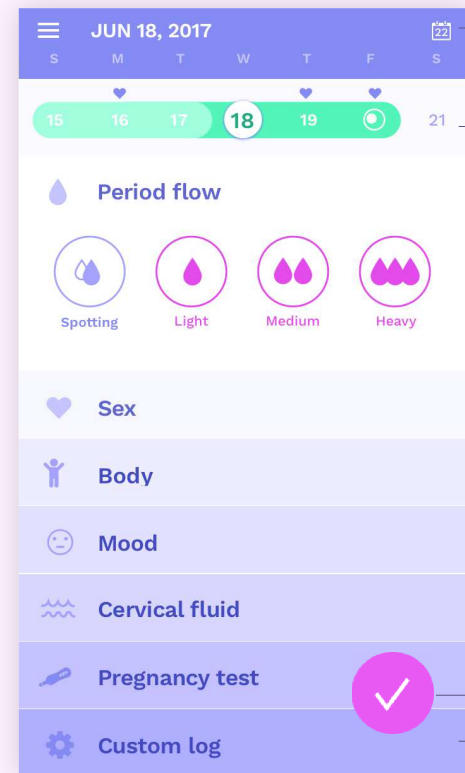


- Go to today
- Swipe left or right to navigate through days
- Add your period by tapping on the icon. Tap again to remove it.
- Additional logging options
- Confirm log
- Tap to create your own customized log. You can create up to 10 logs.

ADDITIONAL FEATURES


iOS Device

 Tap on the more icon to access additional menu items.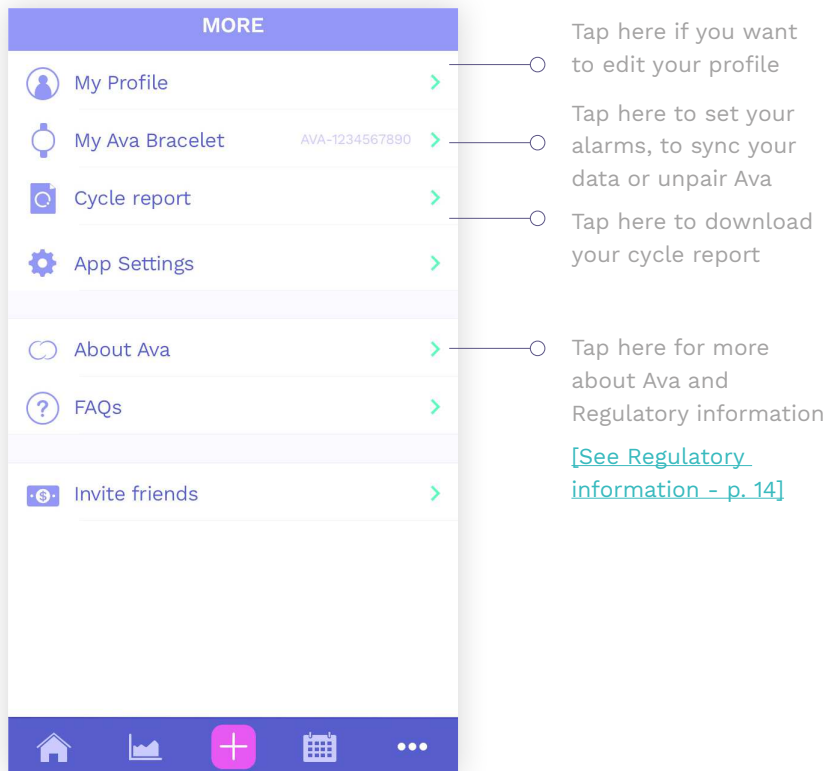

Android Device

 Tap on the menu icon to access additional menu items.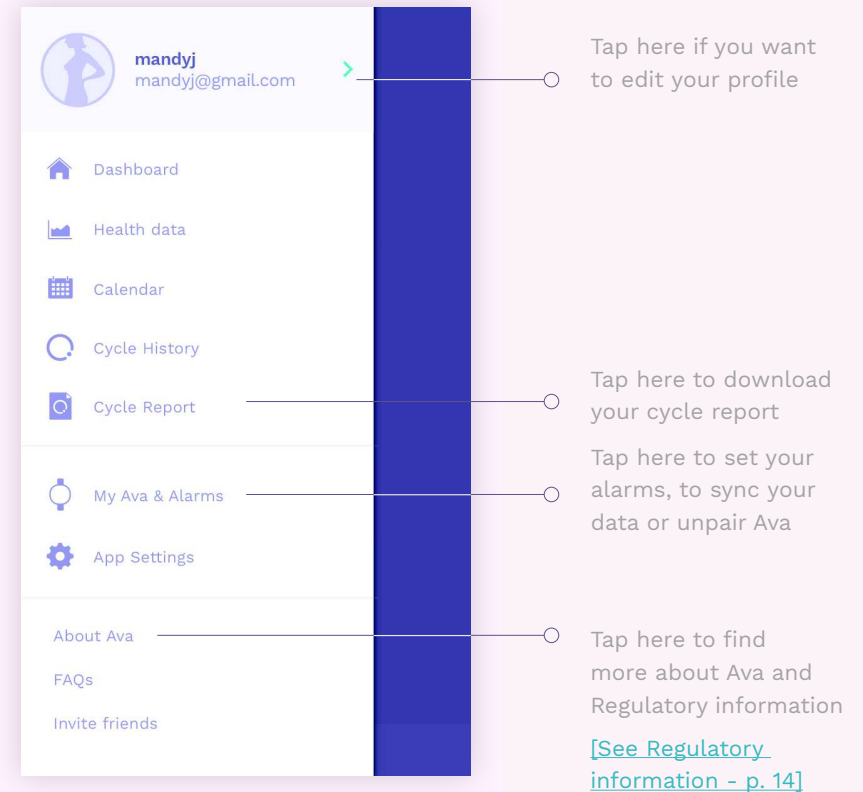

PROFILE

< More
MY PROFILE

mandyj

mandyj@gmail.com

MY CURRENT STATUS

☐

I'm tracking my cycle

☒

I'm trying to conceive

☐

I'm pregnant

PREVIOUS PREGNANCIES

Feb 10, 2017 - Sep 12, 2017
remove

ACCOUNT DETAILS

First name
Amanda

Last name
Jones

Password
change

Log out

Delete account

Change your status

Tap here to remove a pregnancy from your data

Log out or delete your account

CYCLE REPORT

Once you have completed one cycle with Ava, a link to your cycle report will appear within the [additional features](#). By tapping this link, you can email a personal cycle report (PDF) to yourself or your doctor.

< Profile
CYCLE REPORT

Download your personal cycle report

The cycle report provides an overview of your last 12 cycles in a printable PDF format. Save it for yourself or share it with your doctor.

Only cycles in which you've synced data will be included in this report.

Email
annsmith@gmail.com


Email report

Tap here to send the report to your email

ALARM

The Ava bracelet gently wakes you up with a vibration. Once you've woken up, tap the bracelet once to stop the alarm from vibrating.

You can set up to 10 alarms.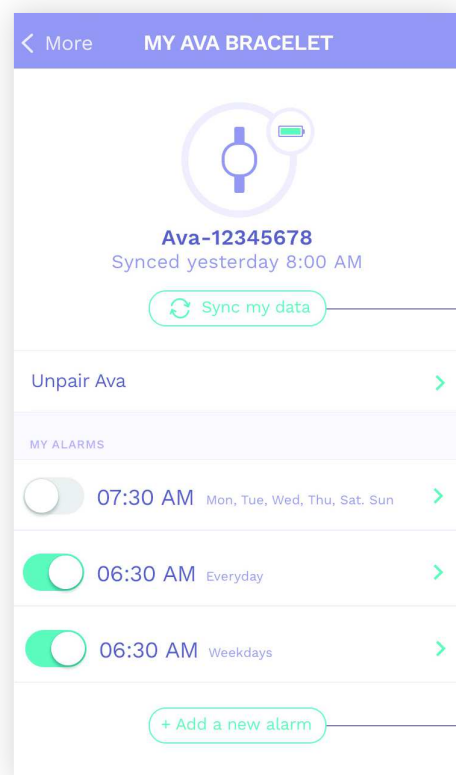


Tap here to sync your data and alarms

Here you can delete an alarm or switch it off

Tap to add a new alarm

ALARM - NEW TIMEZONE

When Ava detects a new timezone it will warn you with a message.

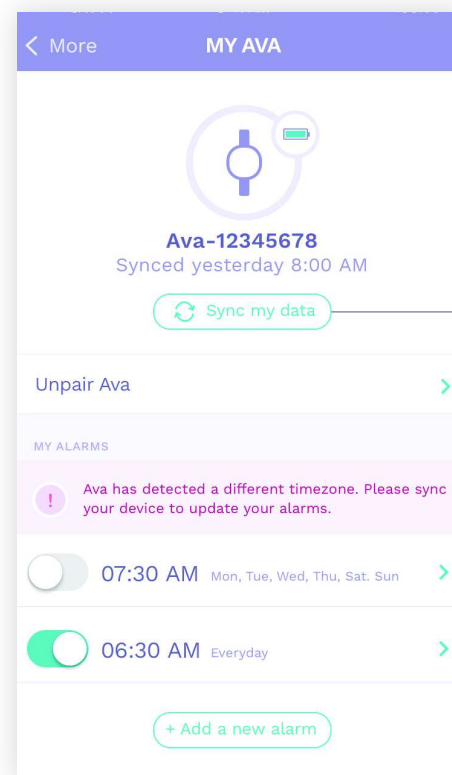Tap here to sync your device to update your alarms

If you enter a new timezone, you need to force sync in order to adjust your alarms to the new timezone. Before you sync and error message will appear.

HISTORY VIEW

🍏 iOS Device

History Tap “history” in the upper left of the [Health Data View](#) in order to see a complete history of your cycles and their corresponding health data. You will need to complete at least one cycle to see this view.

Navigate through your cycle details and parameters (Resting Pulse Rate, Skin Temperature...)

Your current cycle, tap on it to go to the corresponding health data view

Example of a Pregnancy

Tap here to see your averages

🤖 Android Device

Cycle History Tap Cycle history in the main navigation drawer to see cycle history. [\[See "Additional features" - p. 10\]](#)



Navigate through your cycle details and parameters (Resting Pulse Rate, Skin Temperature...)

Your current cycle, tap on it to go to the corresponding health data view

Example of a Pregnancy

Tap here to see your averages

REGULATORY INFORMATION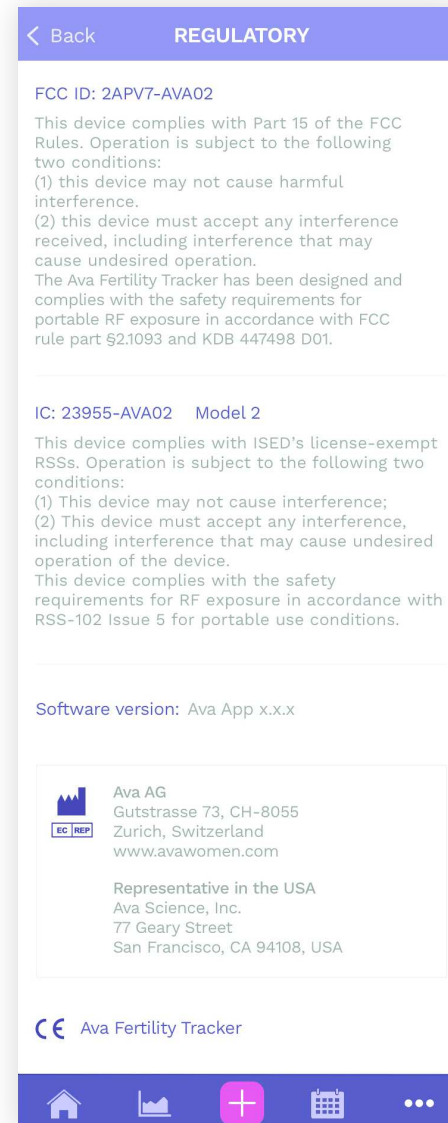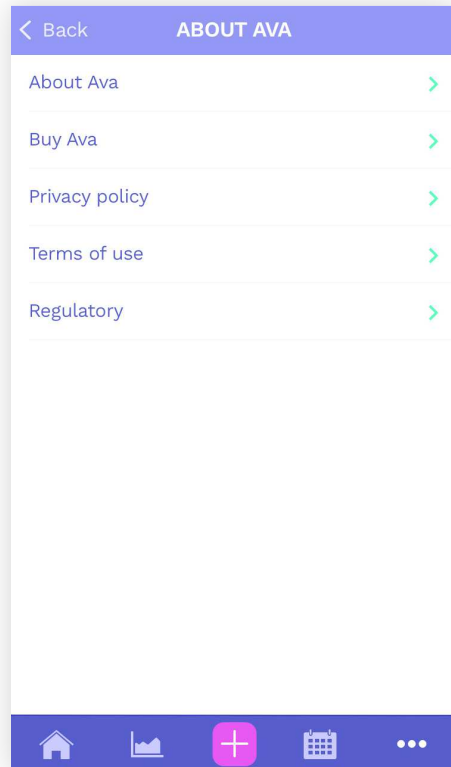

SETTINGS

Here you can select which parameters you would like to display on the health data view and dashboard.

Back
SETTINGS

PREFERENCES

Units
Inches/Pounds

Temperature
Fahrenheit

HEALTH DATA

Choose the parameters you would like Ava to display. You can hide and show parameters any time without losing data.

Skin temperature

Resting pulse rate

Weight

HRV

Breathing rate

Sleep

Daily log


PRIVACY

Enable Ava customer service to access my data


AVA FOR
Pregnancy

NAVIGATION


🍏 iOS Devices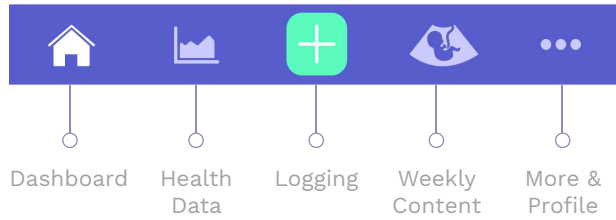

🤖 Android Devices

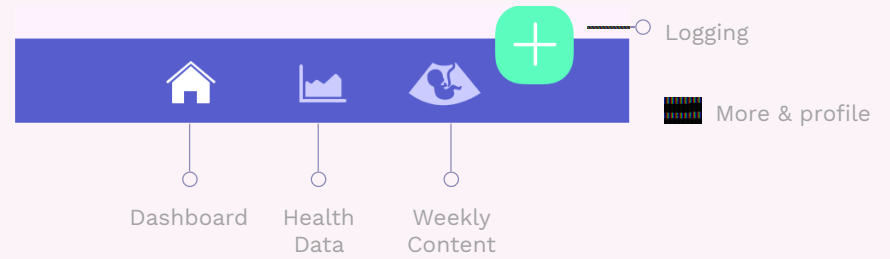DASHBOARD

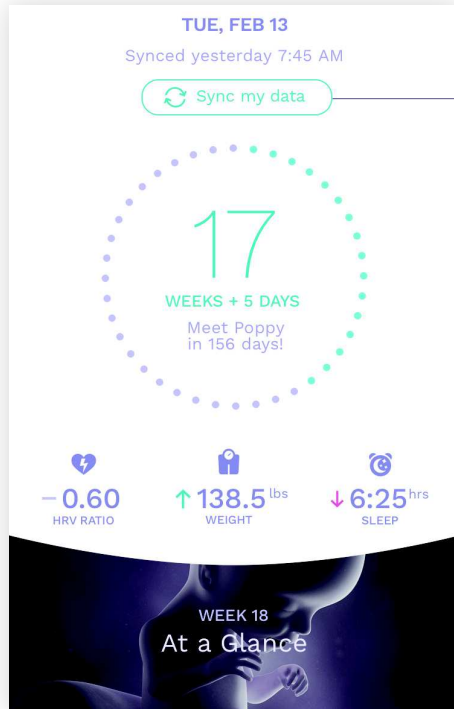Sync my data

17

WEEKS + 5 DAYS

Meet Poppy
in 156 days!

-0.60
HRV RATIO

↑138.5 lbs
WEIGHT

↓6:25 hrs
SLEEP

WEEK 18
At a Glance

Tap to sync


Week count to due date.
If you want to change your due date go to the profile section.

[\[See Profile - p. 19\]](#)

Measurement of the day

Tap to see the weekly content

WEEKLY CONTENT

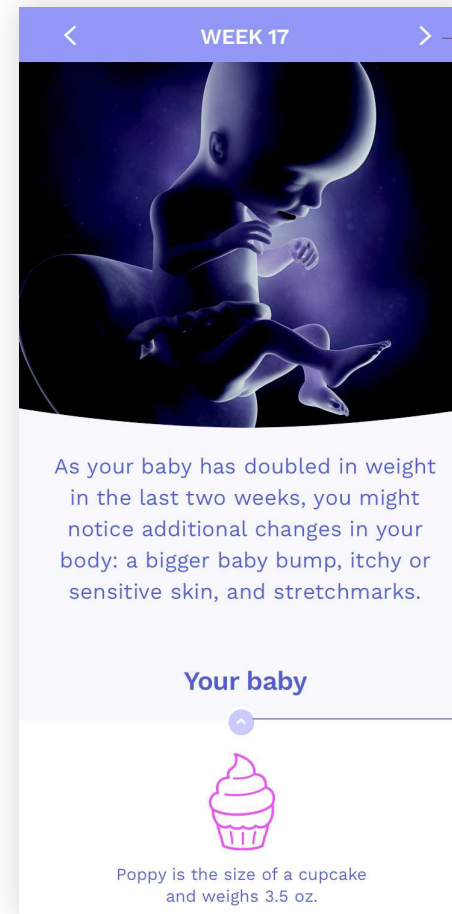WEEK 17

Navigate through weeks

As your baby has doubled in weight in the last two weeks, you might notice additional changes in your body: a bigger baby bump, itchy or sensitive skin, and stretchmarks.

Your baby

Poppy is the size of a cupcake and weighs 3.5 oz.

Tap to expand

Swipe down to see more

HEALTH DATA AND LOGGING

The Health data view enables you to see your parameter values for a specific day and visualize your data throughout your pregnancy. You can also log and track your weight.

PROFILE

The screenshot shows the 'MY PROFILE' screen. It features a user profile card with a silhouette icon, the name 'mandyj', and the email 'mandyj@gmail.com'. Below this, the 'MY CURRENT STATUS' section shows three options: 'I'm tracking my cycle', 'I'm trying to conceive', and 'I'm pregnant' (which is selected with a green checkmark). The 'MY PREGNANCY' section includes fields for 'Due date' (May 20, 2017) and 'Baby nickname' (Poppy). The 'ACCOUNT DETAILS' section shows 'First name' (Amanda), 'Last name' (Jones), and 'Password' (change). At the bottom, there are buttons for 'Log out' and 'Delete account'.

You can remove a pregnancy by switching to "Trying to conceive" or "tracking my cycle".

After you've switched back to "Trying to conceive" mode, you can remove a pregnancy on the profile screen.

[\[See TTC Profile - p. 11\]](#)

Edit / adjust your due date and your baby nickname

Ava Science, Inc.
77 Geary Street - 5th Floor
San Francisco, CA 94108
USA

Ava AG
Gutstrasse 73
8055 Zurich
Switzerland