

# Instructions for use


★This product is not a medical device, please do not engage in medical conduct or provide first aid!!

- Please read the manual and instruction carefully before use.
- User must be instructed by experienced person before use. User must be accompanied by someone during use.
- Patients who are diagnosed or suspected of breast disease, please do check-up in the hospital and follow doctor's advice.
- Using the product on the following parts are strictly prohibited:
  - ① Artery position on both sides of the neck
  - ② Artery position on both inner sides of the armpit
  - ③ Area around eyes and eyeballs
  - ④ Temple on both sides of the head
  - ⑤ Trachea position around the neck
  - ⑥ Area around ears and ear canals
- Those who are unfit and sick are not allowed to use the product, i.e.
  - ① Cancer or tumour patients
  - ② Those with micro-invasive plastic surgery on breast and other parts
  - ③ Heart disease patients
  - ④ Pregnant women
  - ⑤ Diabetes patients
  - ⑥ Surgical patients
  - ⑦ Those with mental illness
  - ⑧ Those with hematopoietic and blood disorders
  - ⑨ Hypertensive patients
  - ⑩ Skin disease patients
  - ⑪ Those with bone trauma and osteoporosis
  - ⑫ Others
- Due to differences in individual physique, your skin may exhibit hair growth on certain parts of the body. Careful consideration should be taken before use!
- Please follow doctor's regular treatment when you are sick.


- This instrument is used for massage and it is not a replacement of any treatment.
- Please do not allow children to use, operate or contact with the instrument (strong suction easily causes danger).
- Please stop using the instrument immediately if you feel unwell in the process.
- Please be sure to use the designated essential oils and skin care products from special ingredients and original supplies to avoid eroding the cup material and the phenomenon such as blisters, broken skin, bruises, and pore blockage from happening.
- Please begin with a low pressure (level 3-5) and gradually adjust to a comfortable strength.
- Overly strong suction may damage skin and cause blisters, bruises or bone fractures, please focus on comfort first and gradually increase the strength and use carefully.
- Do not exceed 30 minutes per use on breasts, abdomen and buttocks, the interval of usage should be more than 6 hours. Other than body parts mentioned above, it is recommended to move from a body part to another body part every 3 minutes for repeating use.
- Bruises are caused by exudation of blood from blood vessels into the skin, exhibiting purple red colour. Human body has self-repairing function, skin gradually recovers when the bruises absorbed by body naturally in a week or two.
- Healthy Cup must be disinfected before and after use, do ensure vibrating part does not contact with water during cleaning.
- After use, sweat and waste discharged from the body part may cause skin discomfort. Please cleanse or wipe the skin immediately with water.
- After use, be sure to remove the power plug for safety.

DC12V→  
←100~240V~

EN-1

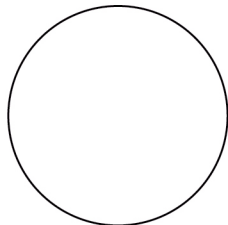

2-OUT

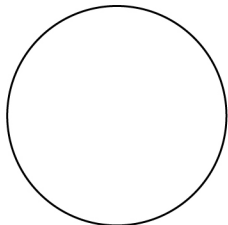

1-IN

## Serial Number

Product name: BIBOTING Healthcare Massager  
 Product model: BB-333  
 Rated voltage: 100~240V ~ or 12Vdc  
 Rated frequency: 50~60Hz  
 Rated power: 35W  
 Executive standard number: GB4706.1-2005  
 Researched & developed, designed and manufactured in Taiwan



碧波庭



www.biboting.com.tw

### International Certification

FC US FCC International Certification

ISO9001 International Production Certification

E114694

MET UL 1647, CAN/CSA C22.2 No. 68

[Product has the following patents] China: ZL03134640.5, ZL201720119476.6, ZL201730035353X, ZL201620101685.3, ZL201630224841.0, ZL201420453111.3  
 Taiwan: M486420, I285103, I241918, M537481 USA: US7,377,890 B2  
 France: 2778570 Singapore: P-No.65770 Thailand: 26845

IC: 24296-BB333 FCC ID: 2AQ2M-BB-333

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

CE EU CE International Certification

CB CB International Certification

Design and fabrication: Taiwan

BIBOTING International CO., LTD  
 8F., No.66-7, Sec. 2, Nankan Rd., Luzhu Dist., Taoyuan City 338, Taiwan