

# Yushan Series X-Ray Flat Panel Detector

## Operation Manual



# INNOLUX Corporation

Radiologist or any other practitioners licensed should ensure that they have adequate knowledge of the operation prior using the INNOLUX Yushan X-Ray Flat Panel Detector.

This Operation Manual should be studied and understood before proceeding to operate the equipment on patients; it describes details on how to operate the INNOLUX Yushan X-Ray Flat Panel Detector and cautions to be observed when operating it.

After reading this manual, store it nearby the INNOLUX Yushan X-Ray Flat Panel Detector so that you can see it whenever necessary.

"Caution: Federal Law restricts this device to sale by or on the order of a radiologist or any other practitioners licensed by the law of the state in which that person practices to use or order the use of the device."



This Operation Manual contains confidential and proprietary information of the Manufacturer.

Before using the device, please check local regulations. If any local legislation is violated, use cannot be authorized.

If additional training or material for training is needed, please directly contact our official dealer or INNOLUX Representative.

---

## **Trademarks**

Company names described in this manual are the trademarks or registered trademarks of INNOLUX Corporation or their respective holders.

Windows is the registered trademark of US Microsoft Corporation in the U.S.A. and other countries.

Copyright © 2013 INNOLUX Corporation. All rights reserved.

Document Number: YS-OM-01

Version: A1

Issued date: 2018/09/18

# Table of Content

<b>Chapter 1 Introduction</b> .....	6
<b>1.1 Product Introduction</b> .....	7
<b>1.2 Product Series</b> .....	7
<b>1.3 Product Intended Use</b> .....	7
<b>1.4 Product Significant Performance Characteristics</b> .....	7
<b>1.5 Product Intended Patient Population</b> .....	8
<b>1.6 Product Intended Part of the body or type of tissue applied to or interacted with</b> .....	8
<b>1.7 Product Intended user profile</b> .....	9
<b>1.8 Product Intended conditions of use</b> .....	9
<b>1.9 Product Features</b> .....	10
<b>1.10 Clinical Guide</b> .....	10
<b>Chapter 2 Safety and Regulatory</b> .....	11
<b>2.1 Compliance Standards and Classification</b> .....	11
<b>2.1.1 Compliance Standards</b> .....	11
<b>2.1.2 Classification</b> .....	12
<b>2.2 Contraindications</b> .....	12
<b>2.3 Power Supply Hazard</b> .....	12
<b>2.4 Electric Shock Hazard</b> .....	12
<b>2.5 Abnormalities Hazard</b> .....	14
<b>2.6 Installation Precautions</b> .....	14
<b>2.7 Connection Instructions</b> .....	15
<b>2.8 External Network Connection</b> .....	15
<b>2.9 System Isolation Instructions</b> .....	16
<b>2.10 Software Precautions</b> .....	17
<b>2.11 Disinfection Instructions</b> .....	17
<b>2.12 Charging the Battery Pack</b> .....	18
<b>2.13 Battery Pack Instructions</b> .....	19
<b>2.14 Other Precautions</b> .....	22
<b>2.15 Disposal of Waste</b> .....	24
<b>2.16 Product Labels</b> .....	24
<b>2.16.1 Identification Label</b> .....	25
<b>2.16.2 Battery Polarity Label</b> .....	25
<b>2.16.3 Battery Label</b> .....	26
<b>2.16.4 Packing Carton Label</b> .....	26
<b>2.16.5 Adapter Specification Label</b> .....	26

2.16.6 Safety Symbols .....	27
3.1 How to Connect FPD .....	28
3.2 Unit Names and the Functions .....	29
3.2.1 Flat Panel Detector .....	29
3.2.2 Accessories .....	30
3.3 LED indicators.....	33
Chapter 4 Basic Operation .....	34
4.1 Preparing the Flat Panel Detector.....	34
4.1.1 Type of Flat Panel Detector .....	34
4.1.2 Number of the Connectable Flat Panel Detector .....	34
4.1.3 Connecting/Disconnecting the Flat Panel Detector .....	34
4.1.4 Charging the Battery Pack for the Flat Panel Detector.....	35
4.1.5 Installing/Removing the Battery Pack for the Flat Panel Detector .....	35
4.1.6 LED Indicator to show battery capacity.....	36
4.2 Detector Connection.....	37
4.2.1 Wire connection .....	37
4.2.2 Wireless connection.....	38
4.2.3 AP mode connection.....	39
4.3 Starting Up and Shutting Down the Yushan FPD.....	41
4.3.1 Starting Up the Yushan .....	41
4.3.2 Overview of User Interface.....	41
4.3.3 Search and connect FPD .....	42
4.3.4 Enter the operation and select the mode .....	44
4.3.5 Performing an Exposure and Image Acquisition .....	45
4.3.6 Download image from FPD.....	48
4.3.7 Shutting Down the Yushan.....	49
4.4 Correction procedure.....	50
4.4.1 Collect images .....	50
4.4.2 Correction process.....	51
4.4.3 Storage of calibration data .....	53
5.1 Daily User Inspection and Maintenance.....	54
5.1.1 Periodical Inspection .....	54
Chapter 6 Troubleshooting .....	55
Appendix A Specifications of FPD.....	56
A.1 Specifications .....	56
A.1.1 Reduced Equivalent.....	56
A.1.2 Power Supply Conditions when Using Back-up Cable.....	56

A.1.3 Power Supply to Battery Charger .....	57
A.1.4 Power Consumption and Battery Specification.....	57
A.1.5 Environmental Conditions .....	57
A.1.6 Image Performance.....	59
A.1.7 Radio Waves .....	60
A.1.8 Reliability and Lifetime .....	60
Appendix B Electromagnetic Compatibility (EMC) .....	62
B.1 Yushan FPD.....	62
B.2 Further Information for IEC 60601-1-2 (EN 60601-1-2).....	63
Appendix C Radio Frequency (RF) Compliance Information.....	64

# Chapter 1 Introduction

This manual is intended to provide the operator with an overview of the operation and safety requirements for the INNOLUX Yushan X-Ray Flat Panel Detector. This manual is not intended to provide instructions on actual treatment procedures and it is expected that users should have official radiologist or any other practitioners license prior to using the device. The Manufacturer and Distribution organization assume no liability through the use of the device.

All care has been taken in the preparation and checking of this manual however there is no guarantee provided that all information is correct. The information provided in this manual is subject to change without notice.

Only CE or FDA approved or authorized accessories may be used in this device. The Manufacturer and Distribution organization shall not be held liable or responsible for damages or injury caused as a result of using non-approved accessories. Installation may only be conducted by authorized service personal. Some maintenance and service work also must be carried out by authorized service agents and only those procedures outlined in the operator and service manual are allowed. Any service work carried out by unauthorized person will void all warranties. No circuit diagrams or component part lists are to be provided for this device. If you require technical documentation that is not provided in this manual then please contact the manufacturer or your local distributor in writing with your reasons for wanting them and then a copy of the service manual may be provided. Before using the INNOLUX Yushan X-Ray Flat Panel Detector, the operator should read this manual carefully and pay particular attention to the sections of Safety, Operation and Maintenance.



## Caution

INNOLUX Corporation shall not be liable for malfunctions and damages resulting from use under environment conditions outside the range of using conditions for this product such as power supply, installation environment, etc. contained in this manual.

INNOLUX Corporation shall not be liable for malfunctions and damages resulting from natural disasters such as fires, earthquakes, floods, lightning, etc.

## 1.1 Product Introduction

INNOLUX Yushan X-Ray Flat Panel Detector is an X-ray image acquisition device that is based on flat-panel. This device should be integrated with an operating PC and an X-ray generator. It can do to utilize as digitalizing X-ray images and transfer for radiography diagnostic. Each X ray photon will be converted to electronic signal by scintillator and the sensor array. Electronic signal in the pixel will be readout by driver IC and then become an image before send to PC.

## 1.2 Product Series

Model	Description
Yushan V14C	Glass version 14"x 17" FPD with CsI
Yushan V14G	Glass version 14"x 17" FPD with GOS

## 1.3 Product Intended Use

The Wireless/Wired INNOLUX Yushan X-Ray Flat Panel Detector is intended to capture for display radiographic images of human anatomy. It is intended for use in general projection radiographic applications including pediatric and neonatal exams wherever conventional film/screen or CR systems may be used. The INNOLUX Yushan X-Ray Flat Panel Detector is not intended for mammography, fluoroscopy, tomography, and angiography applications. INNOLUX Yushan series provide either raw X ray image or corrected image for system integrator to do further image process.

## 1.4 Product Significant Performance

### Characteristics

Items		Characteristics
Image resolution		2500x3052
Data		16 bit data
MTF (Modulation transfer function) (Typical)	CsI	0.69 @ 1 lp/mm 0.39 @ 2 lp/mm 0.22 @ 3 lp/mm
	GOS	0.52 @ 1 lp/mm 0.23 @ 2 lp/mm

		0.11 @ 3 lp/mm
DQE (Detector quantum efficiency) (Typical)	Csl	0.51 @ 0 lp/mm
		0.48 @ 1 lp/mm
		0.39 @ 2 lp/mm
		0.29 @ 3 lp/mm
	GOS	0.31 @ 0 lp/mm
		0.27 @ 1 lp/mm
		0.16 @ 2 lp/mm
		0.09 @ 3 lp/mm
Maximum linear dose (Typical)		88 uGy
Sensitivity (Typical)	Csl	574 lsb/uGy
	GOS	592 lsb/uGy
Dark noise (Typical)	Csl	4.5 LSB
	GOS	7.9 LSB
Lag		<0.5% after 1 minute

## 1.5 Product Intended Patient Population

Considerations	Requirement Description
Age	- Not restricted
Height	- Not restricted
Weight	- The uniform load of the device is 300kg - The local load on 4 cm diameter of the device is 120kg. *Note: Please evaluate the patient's condition before using.
Health	- Not restricted
Nationality	- Multiple
Sex	- Not restricted

## 1.6 Product Intended Part of the body or type of tissue applied to or interacted with

- Measurement site: body
- Condition: Intact or wound skin



## 1.7 Product Intended user profile

Considerations		Requirement Description
Education	Minimum	- Graduate of radiology college
	Maximum	- No maximum
Knowledge	Minimum	- Read and understand 'westernized Arabic' numerals when written in Arial font - Can distinguish of human body - Understands hygiene
	Maximum	- No maximum
Language Understanding	Minimum	- Local language
	Maximum	- Understand the operation manual that is writing in English
Experience	Minimum	- Physician or legally certified operator
	Maximum	- No maximum

### Note

Patient should not operate the device by its own, in case result in malfunction of the equipment.

## 1.8 Product Intended conditions of use

Considerations	Condition
Environment including hygienic requirements	- Non-sterile - Multiple patient use - Less than ten minute contact - Indoor use only - Ambient luminance range: 100 to 1500 lx - Viewing angle: normal to the display $\pm 20^\circ$
Frequency of use	- Reusable - 1day: 200 shot
Location	- In hospital environment
Mobility	- Portable ME equipment to be used on a patient

## 1.9 Product Features

- 140 micron pixel pitch
- Wide image
- 16-bit dynamic range output
- Exposure times up to 3.0 seconds
- Wireless communication mode or wired communication mode is available. When used in wireless communication mode, an access point and battery pack (optional) are required.

## 1.10 Clinical Guide

Users may require to attend the training courses held by INNOLUX before using. A complete training should consist of the following elements:

- All safety precautions for practitioners and patients when using the device.
- A review of published clinical literature should be conducted to understand the indications and effects of the X-Ray Flat Panel Detector.

---

### Note

It is strongly recommended for INNOLUX Yushan practitioners to attend organized meetings, workshops, seminars, and conferences on use of X-Ray Flat Panel Detector.

---

# Chapter 2 Safety and Regulatory

This INNOLUX Yushan X-Ray Flat Panel Detector has been designed and tested to function in a safe and correct when used as indicated in this manual. Do not use this device before reading and completely understand this Operation Manual. Always observe precautions for safety and only operates the device in a qualified room that provides protection.

## 2.1 Compliance Standards and Classification

### 2.1.1 Compliance Standards

- FDA Standards 21 CFR 1020.31 for ionizing radiation emitting products
- FDA Standards 21 CFR 892.1680 for stationary x-ray system
- European Medical Devices Directive (93/42/EEC)
- EN ISO 13485:2016
- EN ISO 14971:2012
- EN 60601-1: 2006+A1: 2013+A12: 2014
- ANSI/AAMI ES60601-1:2005/(R)2012 and A1:2012, C1:2009/(R)2012 and A2:2010/(R)2012
- CAN/CSA C22.2 No. 60601-1:14
- EN 60601-1-2:2015
- IEC 62304:2006/Amd 1:2015
- EN 60601-1-6: 2010/A1:2015
- EN 62366-1: 2015
- EN ISO 10993-1:2009/AC:2010
- EN ISO 10993-5:2009
- ISO 10993-10:2010
- EN 1041:2008
- EN ISO 15223-1:2016
- RED Directive (2014/53/EU)
- RoHS Directive (2011/65/EU)
- REACH Directive (EC/1907/2006)
- WEEE Directive (2012/19/EU)

## 2.1.2 Classification

- Type of protection against electrical shock: Class I Equipment
- Degree of protection against electrical shock: Type B Equipment
- Degree of protection against harmful ingress of water: IPX6 (An IPX6 rating does not indicate compliance with, and the detectors have not been tested for compliance with, any other IPX\_ ratings.)
- Degree of safety of application in the presence of a flammable anesthetics mixture with air or with oxygen or nitrous oxide: Equipment not suitable for use in the presence of a flammable anesthetics mixture with air or with oxygen or nitrous oxide.

## 2.2 Contraindications

No contraindications present.

## 2.3 Power Supply Hazard



The power supply to the Yushan is AC 100 to 240V.

To avoid electric shocks, users should always take the following precautions:


- Do not open any covers when it is not necessary.
  - Install the equipment in a location where it will not be exposed to water.
  - Check that the equipment is securely earthed.
  - Check that all of the cables are completely and securely connected.
  - Keep the control cabinet out of reach of patients.
- 

## 2.4 Electric Shock Hazard



Observe the following precautions when using the cables:

- Do not touch the plug and connector with wet hands. Otherwise, electric shock may result causing death or severe injury.
- Hold the plug or connector when removing the cable.
- Pulling the cable or carrying by holding it may damage the cable, causing fire or electric shock.
- Do not damage or remodel the cable.
- Do not place a heavy object on the cable or lay it under the flat panel detector. Do not step on, pull, forcibly bend, or bundle the cable. Otherwise, fire or electric shock may result.
- Do not use the flat panel detector for the radiographic examination stand if its cable becomes overloaded. Otherwise, the cable may be damaged, causing fire or electric shock.
- Do not touch the back-up cable connector of Yushan, otherwise it may result in electric shock or malfunction of the equipment.

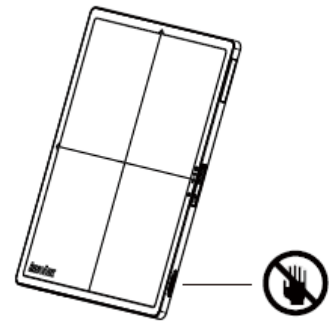


Do not use the equipment in a location where metal particles could come into the equipment. This may cause an electric shock.

Do not disassemble or remodel the equipment. Otherwise, fire or electric shock may result. Keep away from the parts inside the product, which may cause electric shock. If you touch them accidentally, death or severe injury may result.

Do not hit or drop the equipment or subject it to severe shock. Otherwise, the equipment may be damaged. If the damaged equipment is used, fire or electric shock may result.

Do not use the flat panel detector to lift up patient.

In addition, do not apply strong pressure onto the flat panel detector. If applied, the flat panel detector deforms and the waterproof function may be invalid.

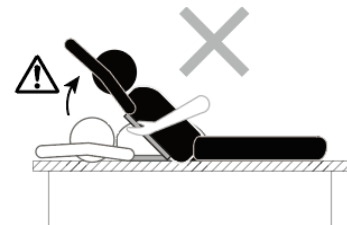

Have the patient take a fixed posture and do not let the patient touch parts unnecessarily. If the patient touches connectors or switches, it may result in electric shock or malfunction of the equipment.

Do not use the flat panel detector without the battery packs. If the battery

packs are not attached, an electric shock may result.

Make sure to use the optional parts and accessories provided by innolux. Failure to use the optional parts and accessories recommended by us may result in damage to the equipment and/or electric shock and injury.

Keep the equipment away from patient's body fluids, chemicals, water, etc. Otherwise, it may be damaged, causing fire or electric shock. If necessary, protect the flat panel detector by covering it with a disposable bag.

---

## 2.5 Abnormalities Hazard

---



If any of the following occurs, immediately turn off the power of each unit, unplug the power cable from the outlet, and then contact our official dealer or INNOLUX Representative.

- When smoke, strange odor, or abnormal sound is present.
  - When a foreign object (such as a metal object) or liquid enters the product.
  - When the equipment is dropped or hit and is damaged.
- 

## 2.6 Installation Precautions

---



Do not install the equipment in a location with the following conditions:

- Where the temperature changes sharply.
- Close to heat sources such as a heater.
- Where the equipment may be exposed to water due to water leakage or ingress.
- Where corrosive gas may be generated.
- Where there is excessive dust.
- Where the equipment is subject to frequent or excessive vibration/shock.
- Where the equipment is exposed to direct sunlight.

Use the equipment on a flat place. If the equipment falls, it may cause damage to the equipment or personal injury.

When you move the equipment, place it in the cassette storage box of a mobile X-ray unit or hold it by hand to prevent it from falling. If the cart is used to move the equipment, place it horizontally.

For veterinary or mobile applications, contact our official dealer or INNOLUX Representative.

When the devices are used outdoors in wireless communication mode, contact our official dealer or INNOLUX Representative.

Do not place any object in a place where removal of the power cable is prevented.

To ensure optimal image quality, it is recommended that you do not use the flat panel detector near devices (motor, transformer, switching supply, etc.) that generate electromagnetic noise.

To ensure optimal image quality, it is recommended that you do not place the cables (power cable, back-up cable, etc.) of the equipment near devices (motor, transformer, switching supply, etc.) that generate electromagnetic noise and their cables.

---

## 2.7 Connection Instructions

---



### Warning

Make sure that the devices to be connected to the equipment are authorized for connection.

Connect the panel unit Yushan only to the access point or DR system.

---

## 2.8 External Network Connection

---



**Caution**

When a setting of the network to which the equipment is connected has been changed, check that the change does not affect the system operation and take measures if necessary.

The setting change may include the following:

- Change of connection destination
- Addition of devices
- Removal of devices
- Update of devices
- Upgrade of devices

After connecting this system to the network with other systems, confirm that the other systems are not affected. If they are affected, take countermeasures such as network separation.



**Warning**

Make sure to use the optional parts, accessories and networks recommended by us. Failure to use the optional parts, accessories and networks recommended by us may result in damage to the equipment and/or electric shock and injury.

Connect to the Ethernet Network of 100BASE-TX or 10BASE-T prescribed in the IEEE standard 802.3. Do not connect telephone lines to LAN connector. Only UTP-type straight LAN cables of 4-pair Category 5 cable (CAT 5E) or higher are appropriate for connection to this connector.

---

## 2.9 System Isolation Instructions

---



**Warning**

To ensure complete system isolation, never install any unauthorized accessories or other such items.

When it is necessary to install authorized accessories or optional items,



contact our official dealer or INNOLUX Representative.

Keep equipment other than those used for patients out of their reach to ensure appropriate system isolation.

In normal use, have a patient take a proper positioning for exposure. The operator should operate the system in a place where safety from radiation is ensured. The operator should also make sure before exposure that no one but the patient is in the exposure area and the operating area of the system.

---

## 2.10 Software Precautions

---



### Caution

Do not install additional software to the system. Do not uninstall any of the software preinstalled in the system. The system is preinstalled with the appropriate software. If other software is installed or if the existing software is uninstalled, various operational errors may result.

#### Open-Source Software Contained in This Product

- This product contains third party's software that is made available as open source software or free software.
  - This software is provided "as is" with no warranty of any kind as to its merchantability or fitness for any particular purpose.
  - If you would like to receive such source codes, please contact INNOLUX dealer or the service representatives at the agency from which you purchased this product. (Please be noted that any inquiries concerning the contents of source codes should be directed to original licensors of open source software.)
- 

## 2.11 Disinfection Instructions

---



### Warning

Do not use the following disinfectants or sterilizers at the time of disinfection,

which the quality, performance and safety of the equipment cannot be assured.

- Chloric disinfectant which is strongly corrosive to metals and rubber parts.
- Disinfectant whose uses on metals, plastics, and coating are forbidden according to the instructions supplied with the disinfectant.
- Formalin gas and disinfectant sprays that may get inside the equipment.
- Ultraviolet sterilizers.

Disinfectant ethanol is recommended for disinfection. Carefully read the instructions and cautions supplied with the disinfectant before use.

For details on the disinfectant, contact a INNOLUX dealer or the service representatives at the agency from which you purchased the disinfectant.



If flat panel detector is not disinfected, it may lead secondary infection.

Be sure to disinfect with ethanol after use.

Clean the flat panel detector with ethanol for disinfection, etc. for each patient to prevent infection.

---

## 2.12 Charging the Battery Pack

---



Charging battery only by the back-up cable provided by INNOLUX when battery is installed on Yushan flat panel detector or use the battery charger provided by INNOLUX Corporation. For details on operations, refer to the instruction manual for the battery charger.

Do not charge the battery pack near fire or under strong sunshine. If the built-in protection mechanisms are activated by a high temperature, the battery pack cannot be charged. Also, if the built-in protection mechanisms are damaged, the battery pack may be charged with extremely high current and voltage, and abnormal chemical reactions may occur inside the battery pack, causing it to overheat, emit smoke, explode or ignite.

To charge the battery pack, be sure to use the designated battery charger and to observe the charging conditions specified by INNOLUX Corporation. If the battery pack is charged in other conditions (temperature or voltage/current higher than specified, remodeled battery charger, etc.), the battery pack may be overcharged or charged with extremely high current, and abnormal chemical reactions may occur inside the battery pack, causing it to overheat, emit smoke, explode or ignite.

Immediately stop charging the battery pack, if charging is not completed within the specified time. Otherwise, the battery pack may overheat, emit smoke, explode or ignite.

Do not use the flat panel detector near the power cable.

Do not use a faulty or broken battery charger or AC adapter.

---

## 2.13 Battery Pack Instructions

---

### **Warning**

Battery pack requires regular checkup and replacement. Battery capacity begins to warm after a period of time.

If this equipment is not in use for while, store it with the battery pack removed. Not removing the battery pack may cause malfunction.

### **Caution**

The battery pack is used with the flat panel detector. Do not use them in other combinations.

Charge the battery pack only with the designated battery charger. If the battery pack is charged under the charging conditions (voltage, current and charging method) different from those specified by INNOLUX Corporation, the battery pack may emit smoke, ignite, explode or leak fluid.

Store the battery pack in a cool and dark place. Recharge the stored battery pack every six months or every year. Otherwise a decrease in battery capacity or other problems may result.

Do not leave the removed battery pack in the car or other places exposed to high temperature. If the battery pack is used or stored in a place where it is exposed to high temperature, the battery pack may emit smoke, ignite, explode or leak fluid.

Use or store the battery pack only in the environmental conditions specified by INNOLUX. If the battery pack is used or stored in a place where it is exposed to high temperature, the battery pack may emit smoke, ignite, explode or leak fluid.

When disposing of the battery pack, consult our official dealer or INNOLUX Representative.

Do not disassemble or remodel the battery pack. The battery pack is equipped with built-in safety and protection mechanisms. If they are damaged, the battery pack may overheat, emit smoke, explode or ignite.

Be careful not to drop the battery pack. The patient may be injured.

Do not touch the terminal of the battery pack directly. There is a risk of electric shock.

Do not connect the positive (+) and negative (-) terminals with a wire or any metal object.

Do not carry or store the battery pack together with metal objects such as necklaces or hairpins. Otherwise, the battery pack may short-circuit and overcurrent may flow, causing the battery pack to overheat, emit smoke, explode or ignite. Metal objects such as necklaces or hairpins may also become hot.

Do not throw the battery pack into fire or expose it to excessive heat. Otherwise, its insulator may melt, its gas release vent or safety mechanisms may be damaged, and/or its electrolyte may catch fire, causing the battery pack to overheat, emit smoke, explode or ignite.

Do not use or leave the battery pack in a place where it is exposed to high temperature (60C or higher), such as fire or a heater. If the resin separator is damaged due to heat, the battery pack may short-circuit, causing it to overheat, emit smoke, explode or ignite.

Do not immerse the battery pack in water or seawater, and do not allow it to become wet. If the built-in protection mechanisms are damaged, the battery

pack may overheat, emit smoke, explode or ignite.

Do not pierce the battery pack with a nail, hit it with a hammer, or step on it. Otherwise, the battery pack may be damaged or deformed and short-circuit, causing it to overheat, emit smoke, explode or ignite.

Do not subject the battery pack to strong impact or throw it. If the built-in protection mechanisms are damaged, the battery pack may be charged with extremely high current and voltage, and abnormal chemical reactions may occur inside the battery pack, causing it to overheat, emit smoke, explode or ignite.

Do not use an apparently damaged or deformed battery pack. Otherwise, the battery pack may overheat, emit smoke, explode or ignite.

Do not solder the battery pack directly. Otherwise, its insulator may melt, or its gas release vent or safety mechanisms may be damaged, causing the battery pack to overheat, emit smoke, explode or ignite.

Do not reverse the positive (+) and negative (-) terminals. Otherwise, the battery pack may be reverse-charged during charging. As a result, abnormal chemical reactions may occur inside the battery pack, or extremely high current may flow during discharging, causing it to overheat, emit smoke, explode or ignite.

The battery pack has a predetermined polarity. If you cannot connect the battery pack to the battery charger or other equipment, do not connect the battery pack forcefully. Make sure that the terminals are correctly oriented. If the battery pack is connected in reverse, it will be reverse-charged, and abnormal chemical reactions may occur inside the battery pack, causing it to overheat, emit smoke, explode or ignite.

Do not connect the battery pack to an electrical outlet or cigarette lighter socket in a car. Overcurrent may flow to the battery pack due to high voltage applied, causing the battery pack to overheat, emit smoke, explode or ignite.

Do not use the battery pack for equipment other than those specified. Otherwise, the guaranteed performance will be reduced and/or the service life will be shortened. Depending on the equipment to which the battery pack is connected, extremely high current may flow, causing the battery pack to be damaged, overheat, emit smoke, explode or ignite.

If the electrolyte leaked from the battery pack enters the eyes, do not rub them.

Wash the eyes immediately with clean water such as tap water, and consult a doctor. Otherwise, eye injury may result.

Do not use the battery pack in combination with a primary battery such as a dry battery or other battery of a different capacity, type and/or brand. Otherwise, the battery pack may be overcharged during charging, and abnormal chemical reactions may occur inside the battery pack, causing it to overheat, emit smoke, explode or ignite.

Do not put the battery pack in a microwave oven or high-pressure container. Otherwise, the battery pack may be rapidly heated or damaged, causing it to overheat, emit smoke, explode or ignite.

If the battery pack leaks or emits an unusual odor, remove it from fire immediately. Otherwise, the leaked electrolyte may catch fire, causing the battery pack to overheat, emit smoke, explode or ignite.

If you notice an unusual odor, heat, discoloration, deformation or any other abnormality during use, charging or storage, remove the battery pack from the equipment or battery charger, and stop using it. Otherwise, the battery pack may overheat, emit smoke, exploder ignite.

Do not use the battery pack exposed to a strong magnetic field of an MRI system, etc.

Do not use the battery pack immersed in liquid.

Keep the battery away from fire. Avoid the action of short circuit. Never attempt to disassemble the battery pack.

Use the designated chargers to charge the battery pack. Risk of explosion if battery replaced by an incorrect type.

Dispose of used batteries according to the instructions.

---

## 2.14 Other Precautions

---



No modification of this equipment is allowed.

Because this equipment is not explosion-proof, do not use combustible and explosive gases near the equipment.



Do not hit or drop the equipment. Otherwise, injury or damage to images, etc. may result.

Be sure to inspect the system periodically.

To assure optimum performance of the equipment, it is necessary to systematically perform maintenance and inspection. For information on maintenance and inspection, contact our official dealer or INNOLUX Representative.

Do not perform maintenance and inspection while the equipment is used for a patient.

The institution where the equipment is installed is responsible for its use and maintenance. In addition, this equipment should not be used by persons other than doctors or suitably trained staff.

Be careful not to expose the flat panel detector to X-ray without a subject.

Although the flat panel detector conforms to IPX6, no warranty is given as to the prevention of water intrusion in the flat panel detector. If the flat panel detector is splashed with water, wipe off moisture and ensure that the flat panel detector is completely dry before use.

As the cables of the equipment are long, be careful not to entangle the cables during use. Also, be careful not to trip over the cables. Falls could result in injury.

Follow the specified procedure when turning off the equipment. Otherwise, the flat panel detector could be damaged by thermal shock.

External equipment intended for connection to signal inputs, signal outputs or other connectors shall comply with the relevant product standard e.g. IEC 60950-1 or IEC 62368-1 for IT-equipment and the IEC 60601-series for Medical Electrical Equipment. In addition, all such combinations – Medical Electrical Systems – shall comply with the safety requirements stated in the general standard IEC 60601-1, edition 3, clause 16. Any equipment not complying with the leakage current requirements in IEC 60601-1 shall be kept

outside the patient environment i.e. at least 1.5 m from the patient support or shall be supplied via a separation transformer to reduce the leakage currents.

Any person who connects external equipment to signal inputs, signal outputs or other connectors has formed a Medical Electrical System and is therefore responsible for the system to comply with the requirements. If in doubt, contact qualified medical technician or your local representative.


---

## 2.15 Disposal of Waste

It is illegal to place the device in the trash. Please contact the qualified local recycling provides before disposing the device, The disposal procedure should comply with local law.

## 2.16 Product Labels


The information of each label can be found in this section and the corresponding location are also presented as the following figure. The following labels are adhered to the INNOLUX Yushan X-Ray Flat Panel Detector.

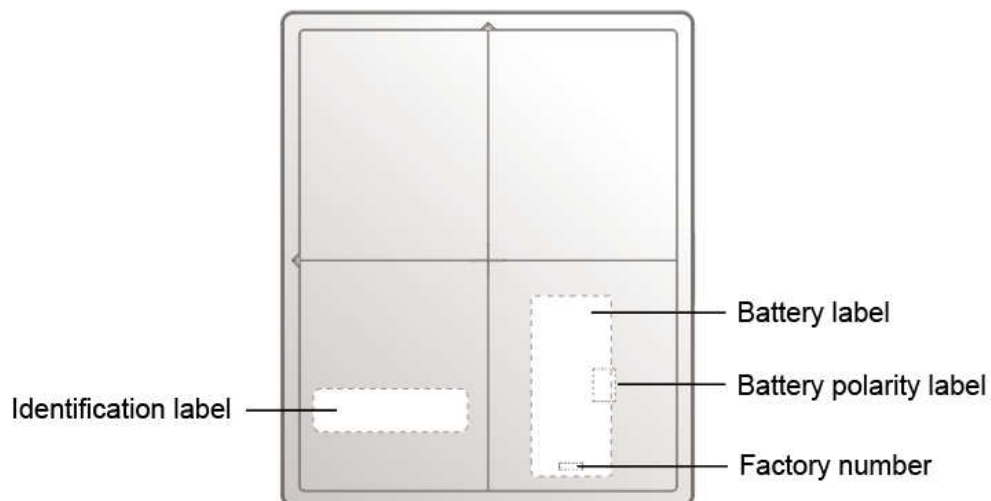




## 2.16.1 Identification Label


Yushan V14C

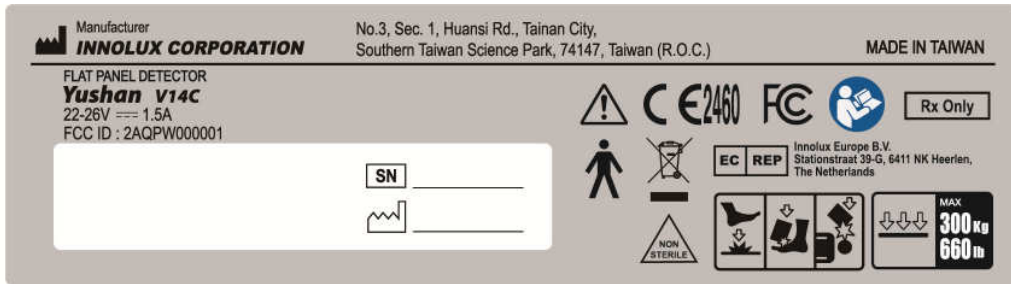

Yushan V14G

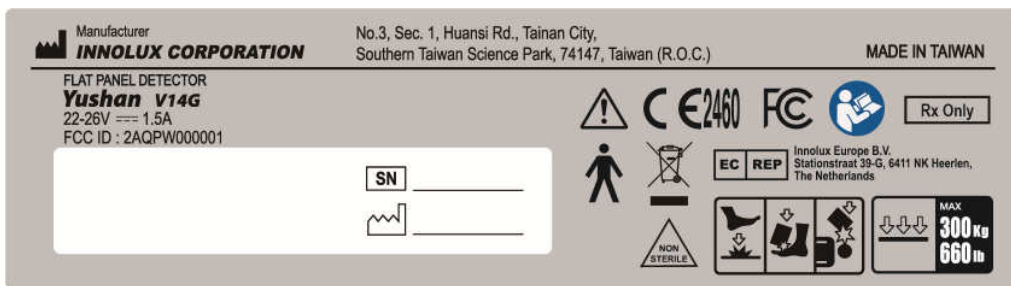


## 2.16.2 Battery Polarity Label

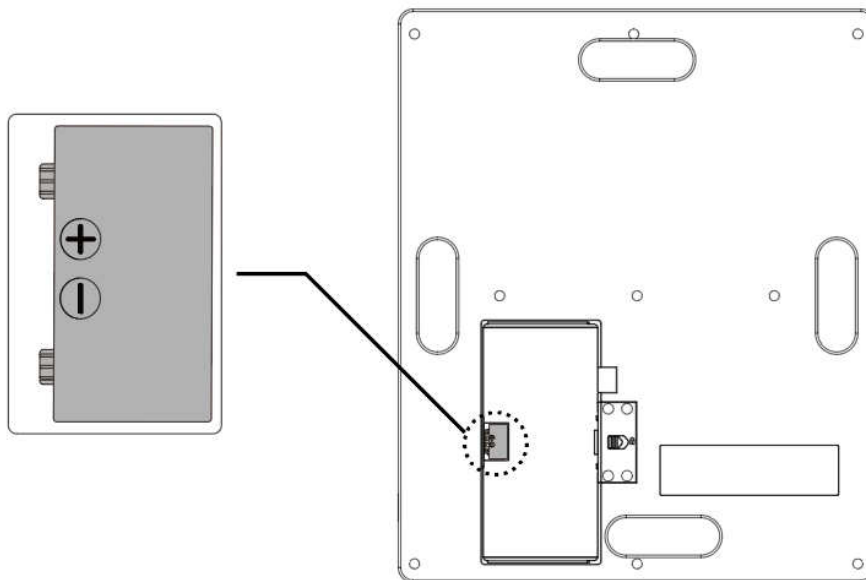

### 2.16.3 Battery Label



### 2.16.4 Packing Carton Label

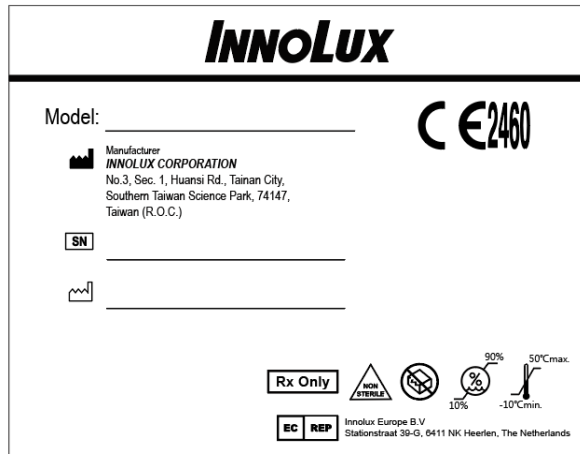











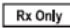






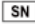



### 2.16.5 Adapter Specification Label



## 2.16.6 Safety Symbols

The following safety symbols are used in the labels or on its body.


Symbol	Description
	Caution
	This symbol indicates compliance of the equipment with Directive 93/42/EEC.
	Federal Communications Commission FCC Marking.
	This symbol indicates that this product is not to be disposed of with your household waste, according to the WEEE Directive (2002/96/EC) and your national law. This product should be handed over to a designated collection point. Improper handling of this type of waste could have a possible negative impact on the environment and human health due to potentially hazardous substances that are generally associated with EEE. At the same time, your cooperation in the correct disposal of this product will contribute to the effective usage of natural resources. For more information about waste, please contact our official dealer or INNOLUX Representative.
	This symbol indicates that the equipment is a Type B Applied Part. IEC 60417/Graphical symbols for use on equipment.
	Refer to instruction manual/booklet
	Serial number
	Manufacturer
	Date of manufacture
	Direct current, DC
	Protective earth (ground)
	Authorized representative in the European Community.
	Prescription use only. Federal Law restricts this device to sale by or on the order of a radiologist or any other practitioners licensed by the law of the state in which that person practices to use or order the use of the device.
	Non-sterile
	Caution for local load
	Do not drop the flat panel detector to the user/patient.
	Entire surface load
	Power on / off
	Battery
	Link
	Status

# Chapter 3 System Configuration

## 3.1 How to Connect FPD

Yushan FPD can be operated either in wireless connection mode or wired connection mode. When using wired mode, back-up cable is necessary to provide the link from FPD to network switch hub or to PC. User can also connect back-up cable to the adapter provided by INNOLUX to give the power to operate or charging its battery. If the adapter is not connected, FPD will be powered by its own battery.

Wireless Mode

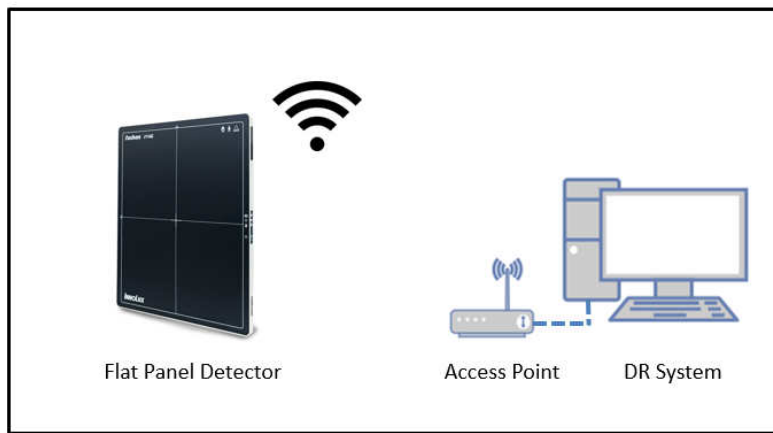


---


 **Caution**

Do not touch the back-up cable connector of Yushan when in wireless mode, otherwise it may result in electric shock or malfunction of the equipment.

Wireless mode

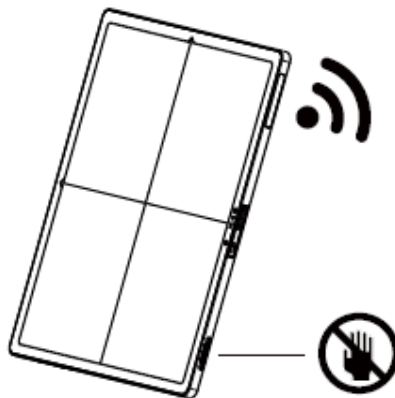


Wired connection mode

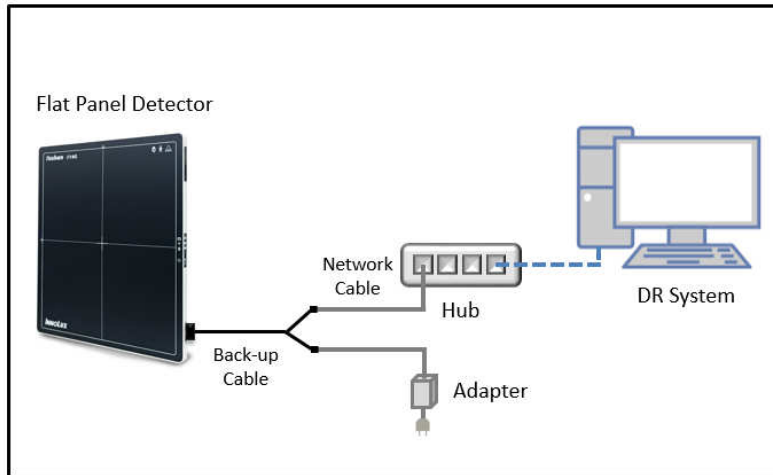

## 3.2 Unit Names and the Functions

### 3.2.1 Flat Panel Detector

Unit names and the functions of the Yushan FPD are described below.

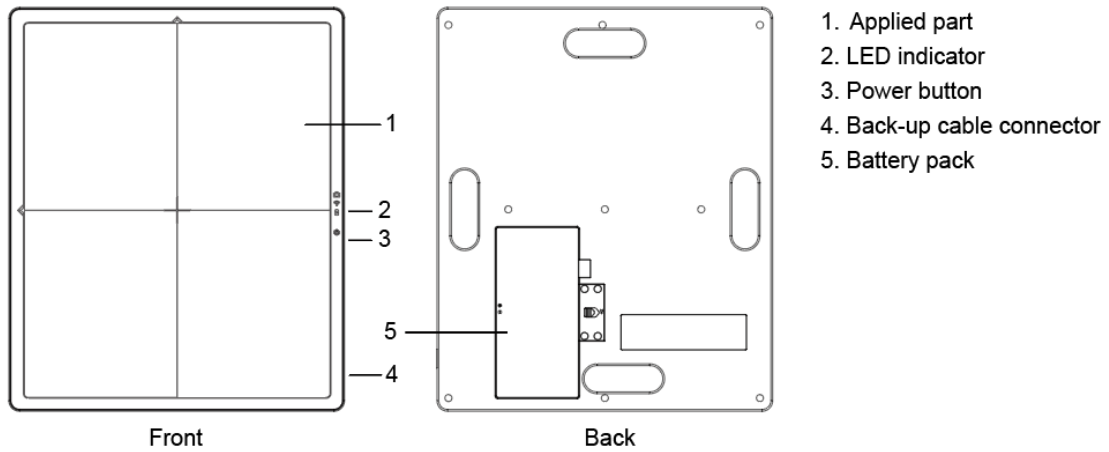


1. Applied part
2. LED indicator
3. Power button
4. Back-up cable connector
5. Battery pack

**Flat Panel Detector**


### 3.2.2 Accessories

#### Back-up cable

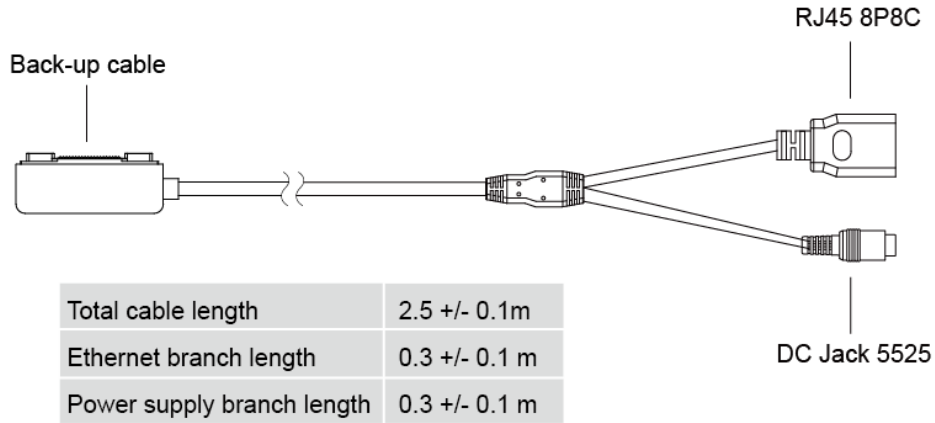

**Description:** A cable which one side connects the flat panel detector and the other side split into two cable, one is RJ45 connector to connect with ethernet port, and the other side is DC Jack 5.5x2.5 to connect with the output of AC/DC adapter. Cable length: 2.5m


#### Adapter

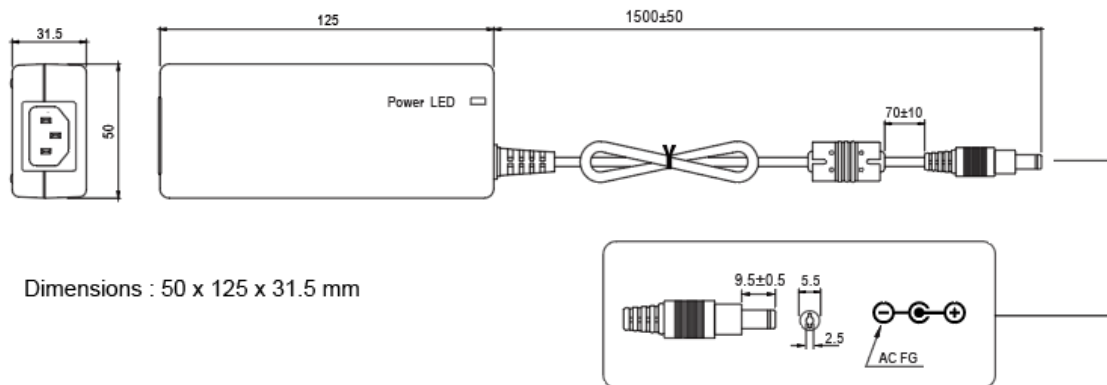

**Description:** Medical grade AC/DC adapter which is used to provide power either to detector or use to charge the battery installed on FPD.

- Input : 100-250V ~ 1.5-0.75A 50-60Hz
- Output : 24V = 2.5A
- Length : 1.5M

## Battery (Optional)

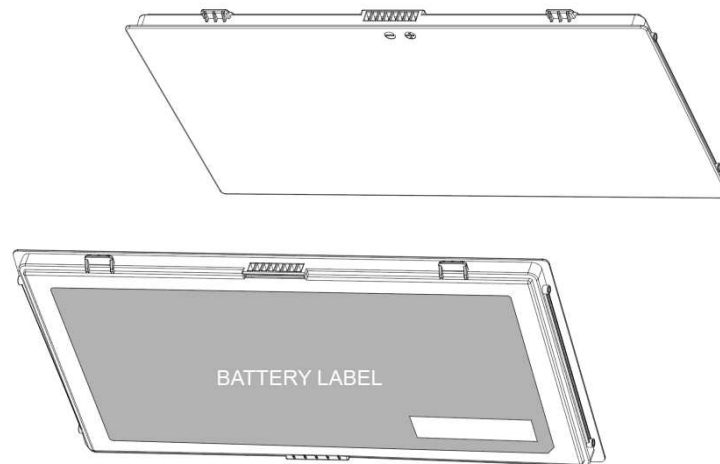

**Description:** Rechargeable Li-Ion battery designed for Yushan FPD.


- Capacity: 4212 mAh
- Rating voltage: 11.4V, TYP:4212mAh/48Wh
- Weight: ~260g
- UL certified Safety

## Power Cord

**Description:** Connection between AC inlet and adapter input.


### EU

- PLUG : 6A/250V, CONNECTOR: 10A/250V
- Length : 3M

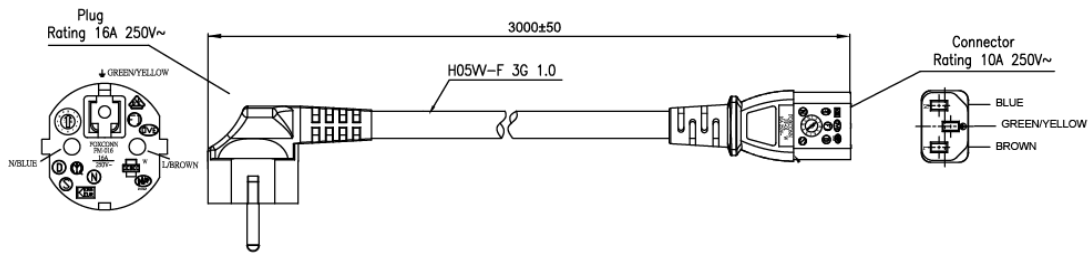

### US

- PLUG : 10A/125V, CONNECTOR: 10A/125V
- Length : 2.5M

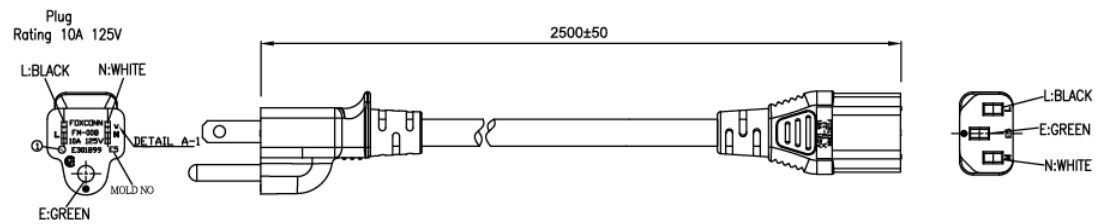

### Note:


Every accessory can't be sale or used as an independent product, which is not compatible with other devices. Therefore, every provided accessory is identified to have not received prior 510(k) clearance.

Do not use parts or accessories which are not certified by INNOLUX Corporation or not approved by CE or FDA.





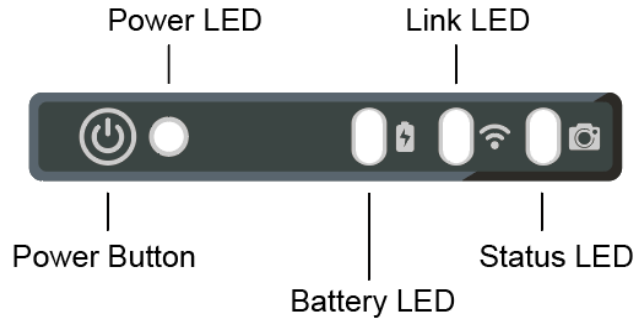

### 3.3 LED indicators

4 LED indicator are on the side of Yushan series FPD.



The behavior and the relations to the FPD status are described in the table below.

Item	State	Description
	Off	Power off
	Green	Power on
	Yellow	Internal system error
	Green flash	Emergency mode
	Yellow flash	Count down to shutdown
	Off	No charging
	Green	Charging
	Yellow	Low battery
	Green flash	No primary battery
	Off	No connection
	Green	FPD connected
	Green flash	FPD has IPv4 or IPv6 address configured ( Not include AP mode)
	Off	Wait and standby
	Green	Detector selected
	Yellow	Internal storage error
	Green flash	Busy

# Chapter 4 Basic Operation

## 4.1 Preparing the Flat Panel Detector

This section describes how to prepare the flat panel detector.

### 4.1.1 Type of Flat Panel Detector


Yushan V14C, Yushan V14G

### 4.1.2 Number of the Connectable Flat Panel Detector


Up to 5 panels can connect to system at the same time.

### 4.1.3 Connecting/Disconnecting the Flat Panel


#### Detector

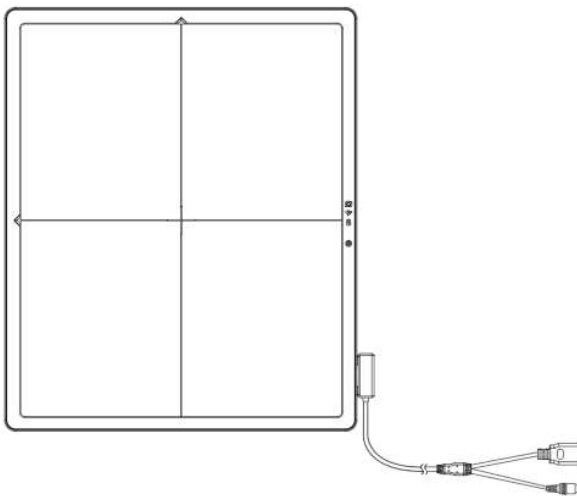

▼ Connecting the FPD

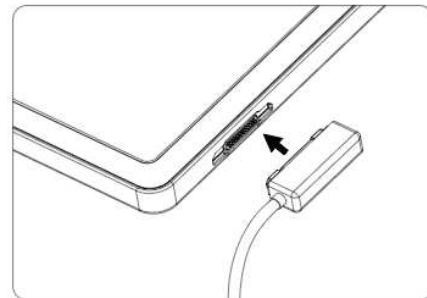


▼ Disconnecting the FPD

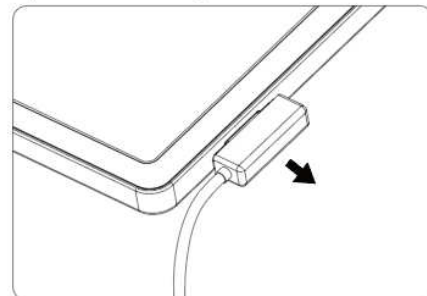

- ⚠ Do not place the connector on floor.
- ⚠ Make sure that the latches on both sides are properly engaged when connecting the connector. If the connector is inserted incompletely, the power may turn off.

---

 **Caution**

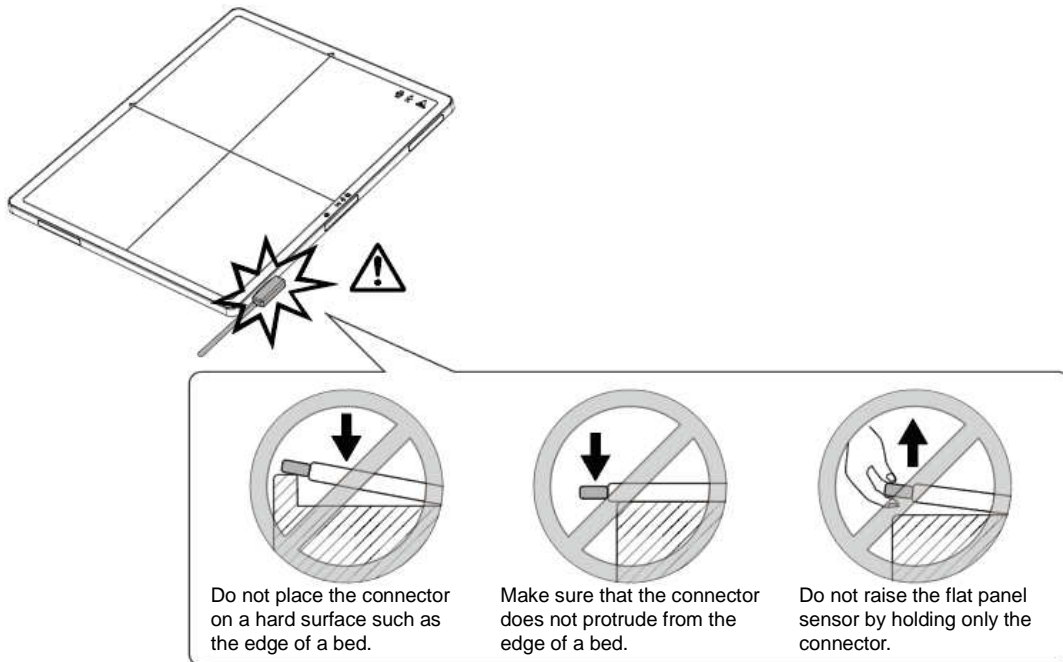

---

#### 4.1.4 Charging the Battery Pack for the Flat Panel

##### Detector

Use the battery charger recommended by INNOLUX Corporation. For details on operations, refer to the instruction manual for the battery charger.


#### 4.1.5 Installing/Removing the Battery Pack for the Flat

##### Panel Detector

Follow the procedure below to install/remove the battery pack for the flat panel detector. When installing/removing the battery pack, place the flat panel detector on a flat place.

Do not remove the battery pack until a processed image appears in the window of the image processing unit after the exposure.


### Remove the battery pack

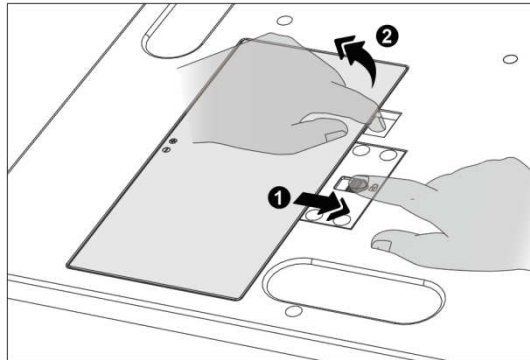

Step ① Slide the arrow switch toward right side and hold it.

Step ② Pick up the edge of battery by finger, then lift it.

### Install the battery pack

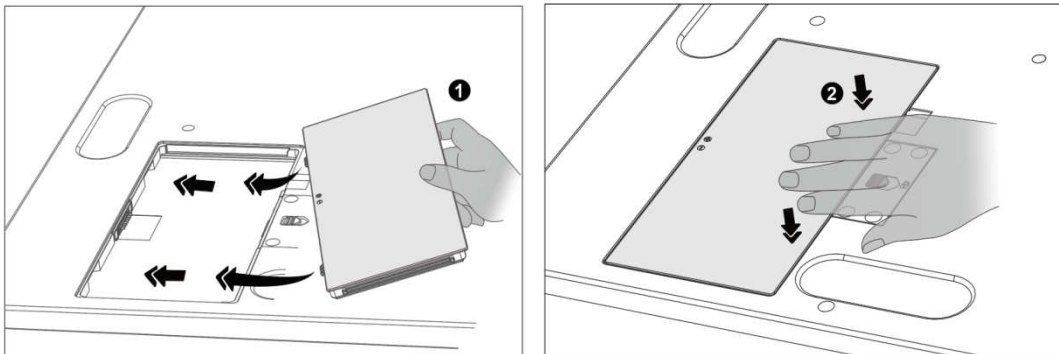


Step ① Following the picture and install the battery.

Step ② Push the battery pack and make sure the battery is tightly installed.

### 4.1.6 LED Indicator to show battery capacity

By short press LED at any time, even when power is off. LED will show current remaining capacity. If the main battery is not exist, it will show the second battery capacity by flashing light.

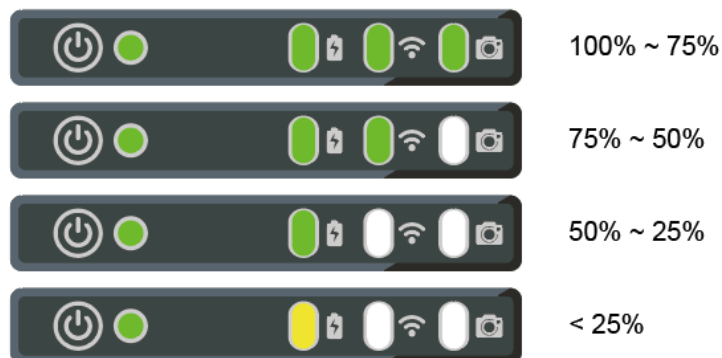


## 4.2 Detector Connection

Yushan detector can be connected to system either by wireless or by wire cable. Yushan FPD use Dynamic Host Configuration Protocol (DHCP) to assign IP address automatically.

### 4.2.1 Wire connection

The default address of Yushan is 192.168.11.100. User need to set the computer to the same domain as Yushan. First, open the IPV4 of computer's local network management interface, and change the IP address to 192.168.11.1 and subnet mask to 255.255.255.0.

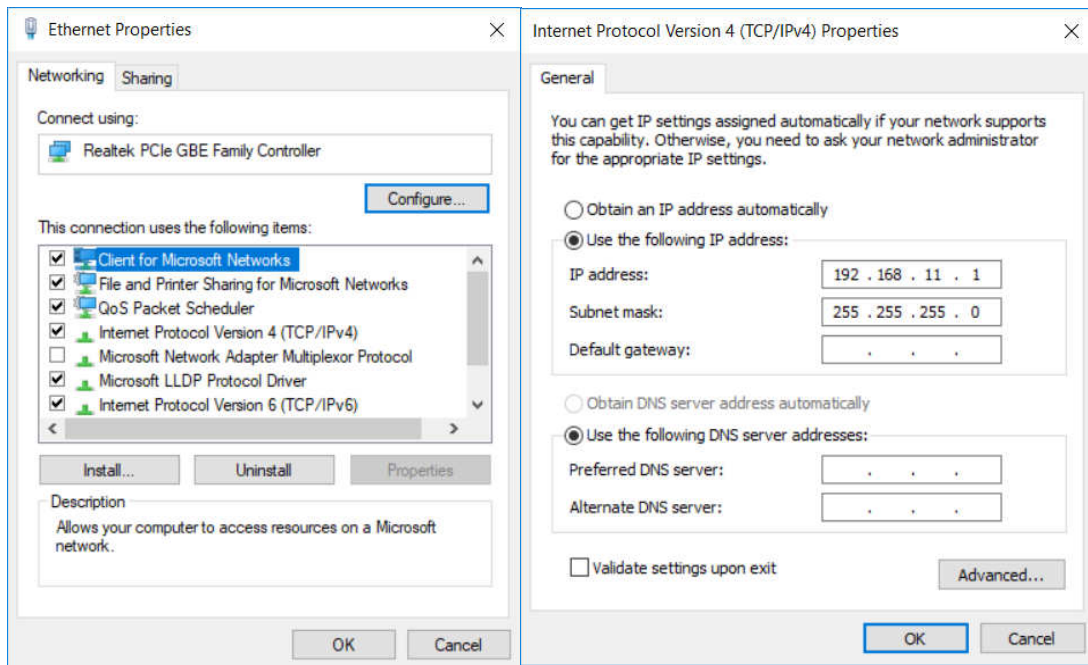

After connection established, open Cmd.exe and enter 'ping 192.168.11.100' then press enter to check the network. If success, there is a reply from FPD as below.


```
Command Prompt - ping 192.168.11.100 -t
Microsoft Windows [Version 10.0.17134.48]
(c) 2018 Microsoft Corporation. All rights reserved.
C:\Users\z7611>ping 192.168.11.100 -t

Pinging 192.168.11.100 with 32 bytes of data:
Reply from 192.168.11.100: bytes=32 time=2ms TTL=64
Reply from 192.168.11.100: bytes=32 time=2ms TTL=64
```


## 4.2.2 Wireless connection

SDK command is used to setup SSID and password of wireless access point. This setup to FPD need to be done first before wireless connection.


Users can set client mode parameters through the software interface. Choose wireless in configuration interface.

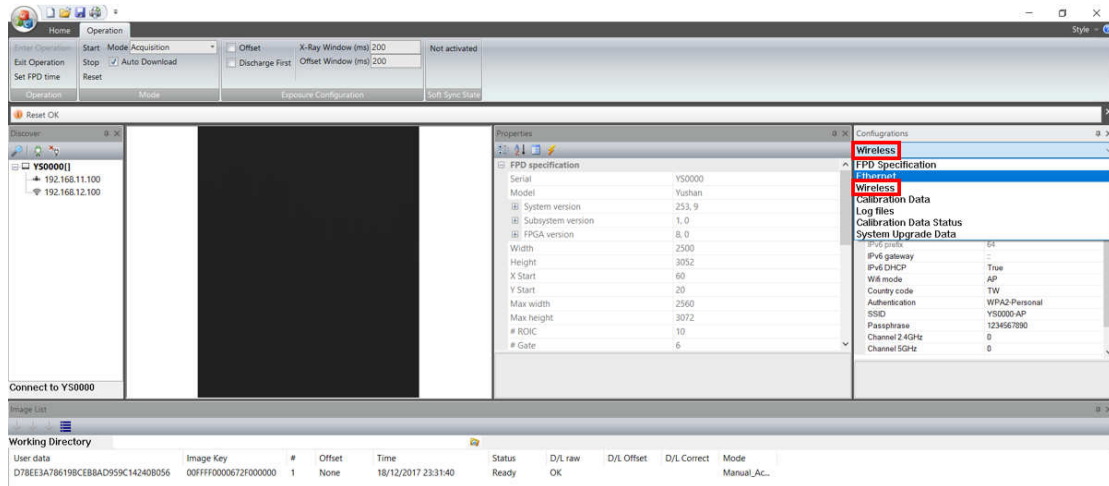

Choose the "Client".

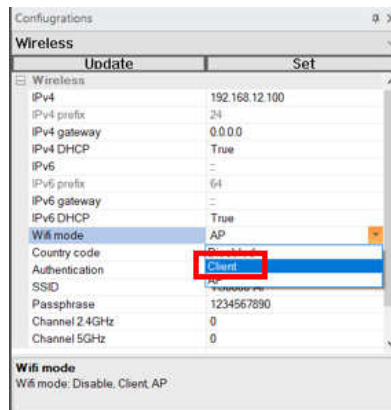


Set the SSID and pass phrase of AP to be connected to the product.

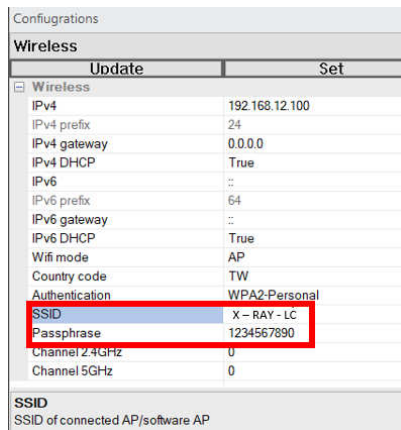


### 4.2.3 AP mode connection


Users can set AP mode parameters through the software interface.  
Choose wireless in configuration interface.

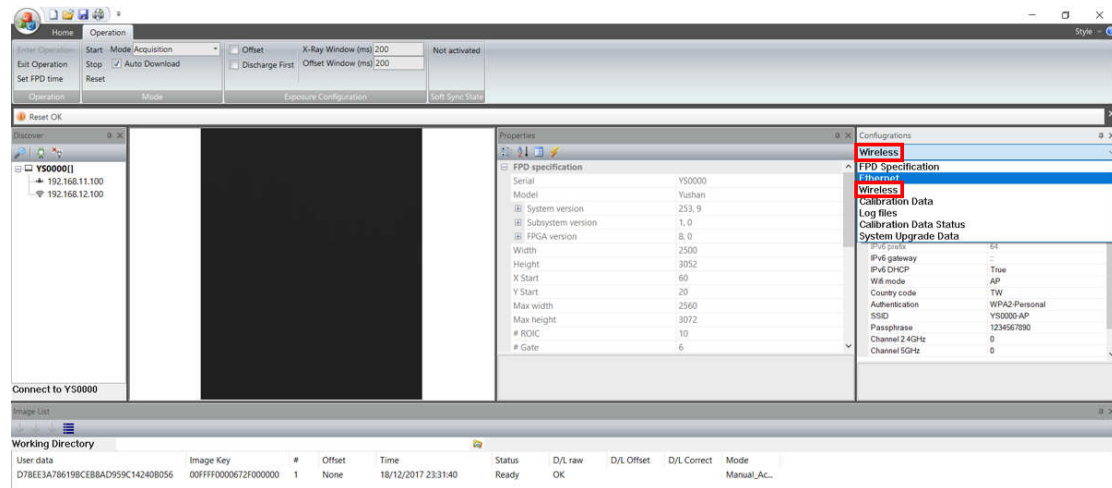


Choose the “AP”.

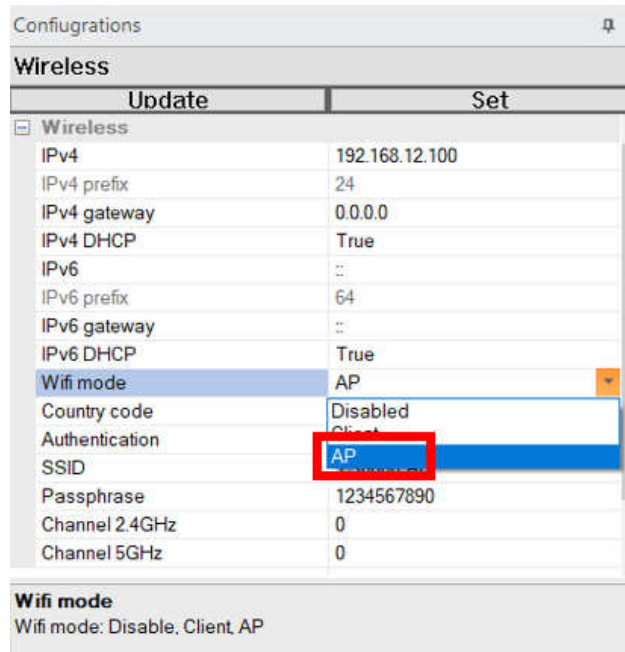

Set the SSID and pass phrase of product.

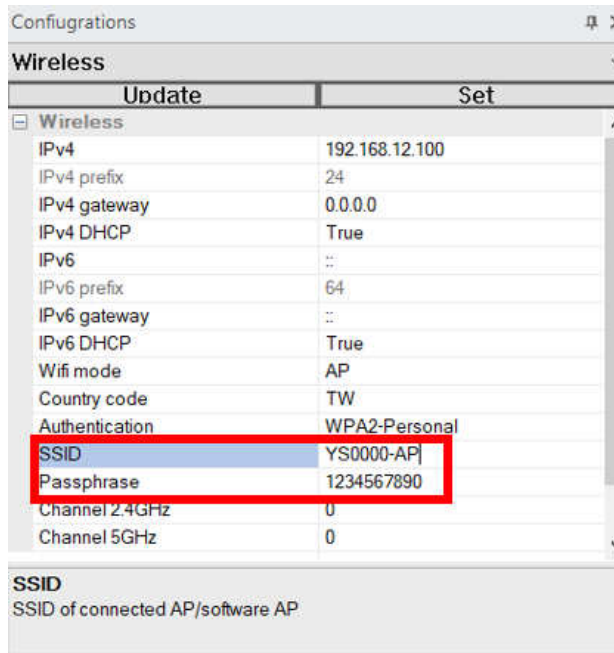

User needs to set the computer to the same domain as Yushan (Wifi). First, open the IPV4 of computer's local network management interface, and change the IP address to 192.168.11.1 and subnet mask to 255.255.255.0.

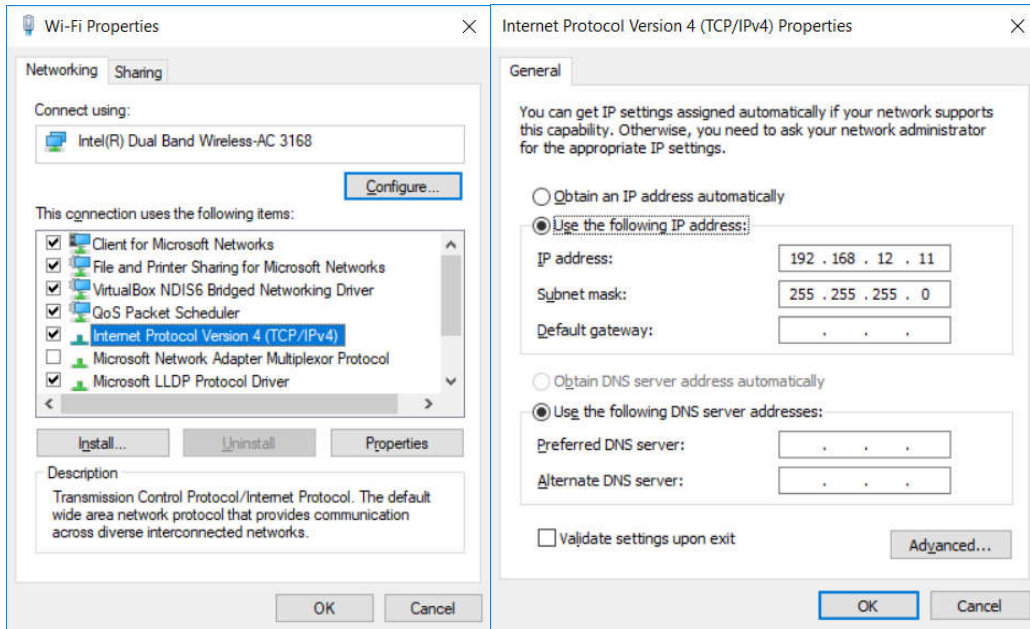




## 4.3 Starting Up and Shutting Down the Yushan FPD

This section explains how to start up and shut down the Yushan.

### 4.3.1 Starting Up the Yushan

Press the power button for more than 2 second to start the FPD.


1. Make sure that the power LED light in green.
2. Wait Status LED stop flashing
3. Wait Link LED light in green
4. Wait Status LED light in green (ready for exposure)

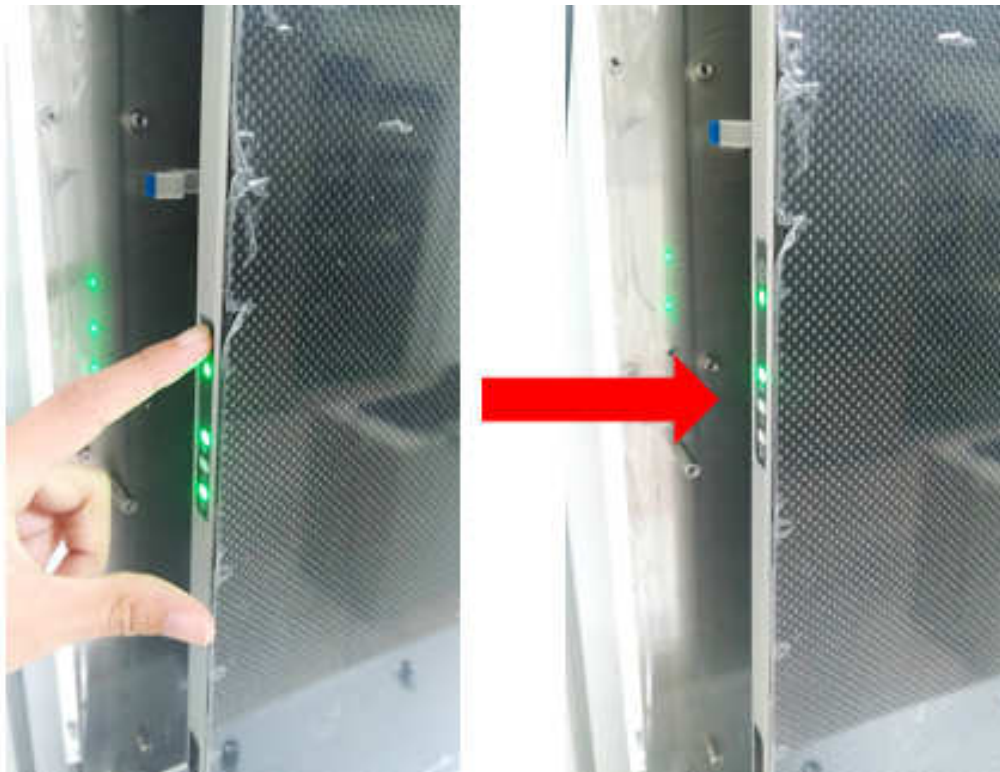

### 4.3.2 Overview of User Interface

- Discover: The area is used to list all FPD those are discovered and reachable.
- Operations: The tool bar contains different exposure mode.

- Message: The area is used to display important message to users.
- Diagnosis: The area lists several diagnostic messages of FPD that is currently connected.
- Image List: The area lists the images created by exposure operations.

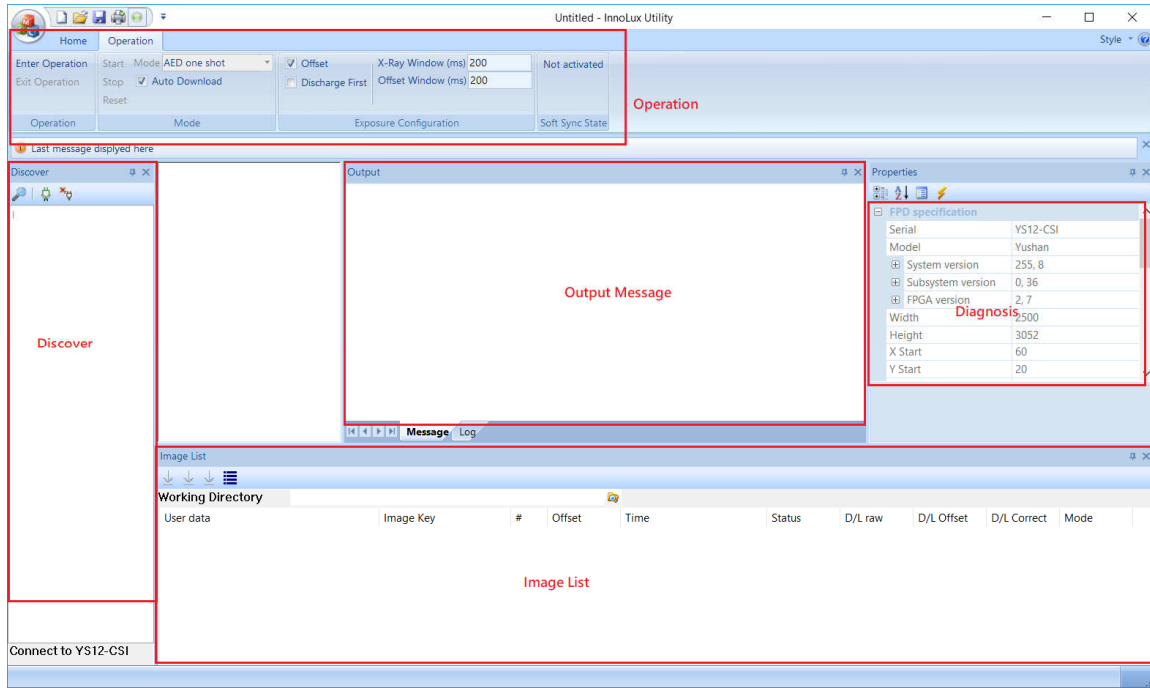


### 4.3.3 Search and connect FPD

Click 1 to discover all reachable FPDs




Expand the selected entry of the list, and select connection by Ethernet or Wifi. Then click 2 to connect the specified FPD. Note: Please make sure the IP of Host PC and the selected connection is in the same local area network.

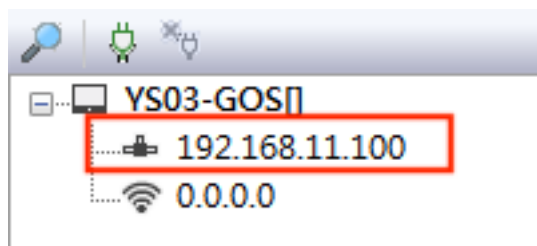


### 4.3.3.1 Wireless connection

Expand the selected entry of the list, and select connection by Ethernet. Then click 2 to connect the specified FPD. Note: Please make sure the host PC is connected to the specified AP.


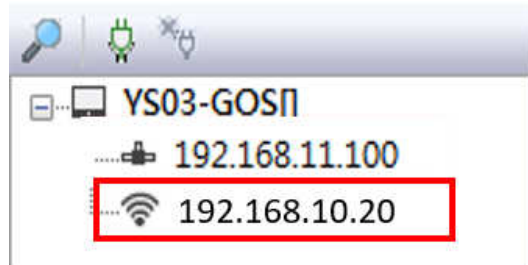


After establishing the connection, you can check the diagnosis field on the right to confirm the information of Yushan products. The message appears to indicate a successful connection.

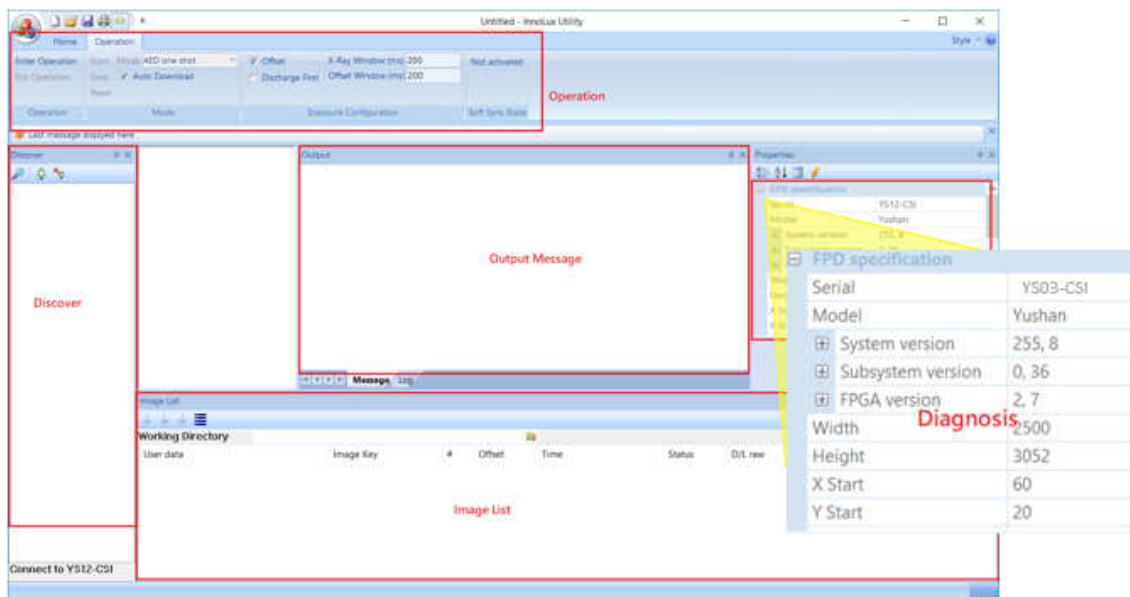

After connection is established, you can use 3 to disconnect the FPD. Note: Connecting to other FPD disconnect the currently connected FPD automatically

### 4.3.4 Enter the operation and select the mode

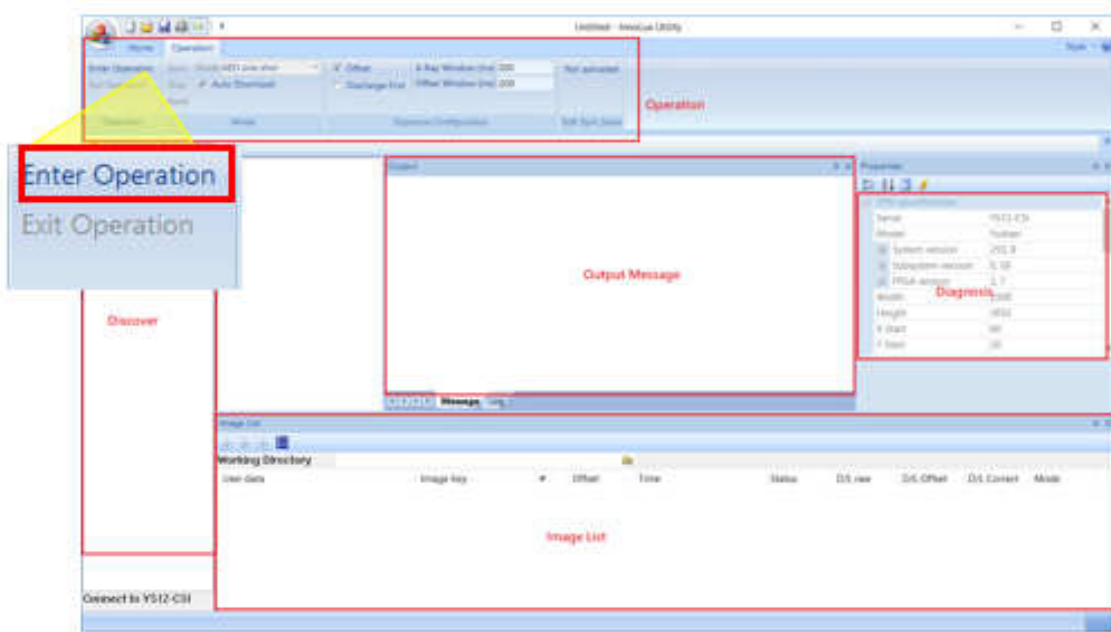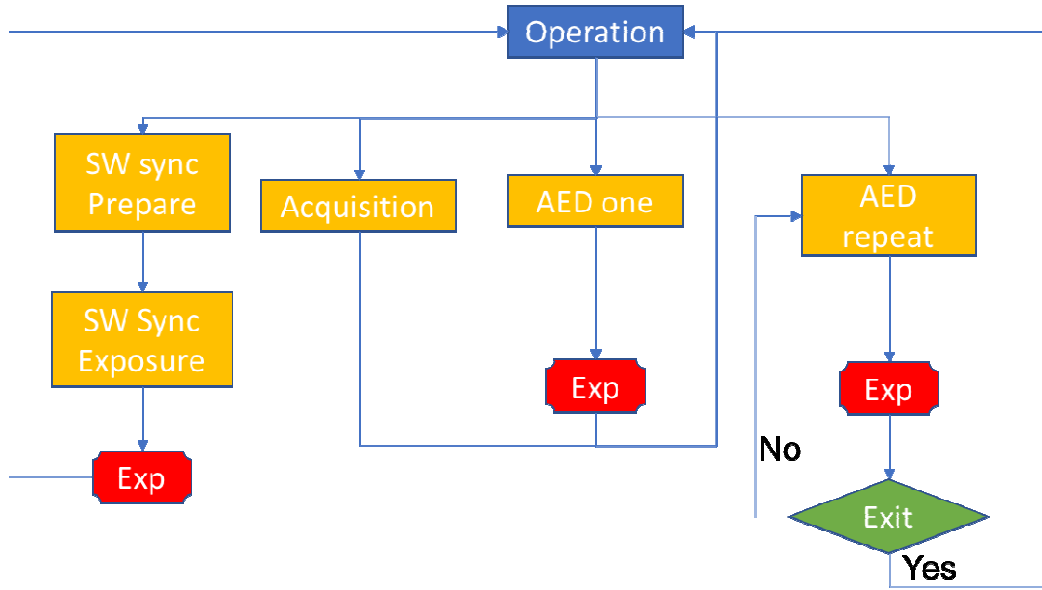

#### Enter operation mode / Exit operation mode:


The FPD must enter operation mode before take exposure or dark image. After FPD enters operations mode, user can perform several exposure mode.


### 4.3.5 Performing an Exposure and Image Acquisition

When performing an exposure, make sure that the STATUS LED light in green and not flashing. Otherwise image may loss.

#### 4.3.5.1 AED one shot mode

- After selecting the **AED one shot** mode, press Start.
- When FPD is in **AED one shot** mode, it returns to operation mode after each X-Ray exposure is detected. User can use Stop to exit **AED one shot** mode before X-Ray exposure is detected.
- If the X-Ray exposure is successfully detected by FPD, the newly created image is listed in the Image List area. This utility application will try to download the image to local storage, and the file name is the image key.
- If the working directory is not specified, the images will be saved in the folder as the utility.

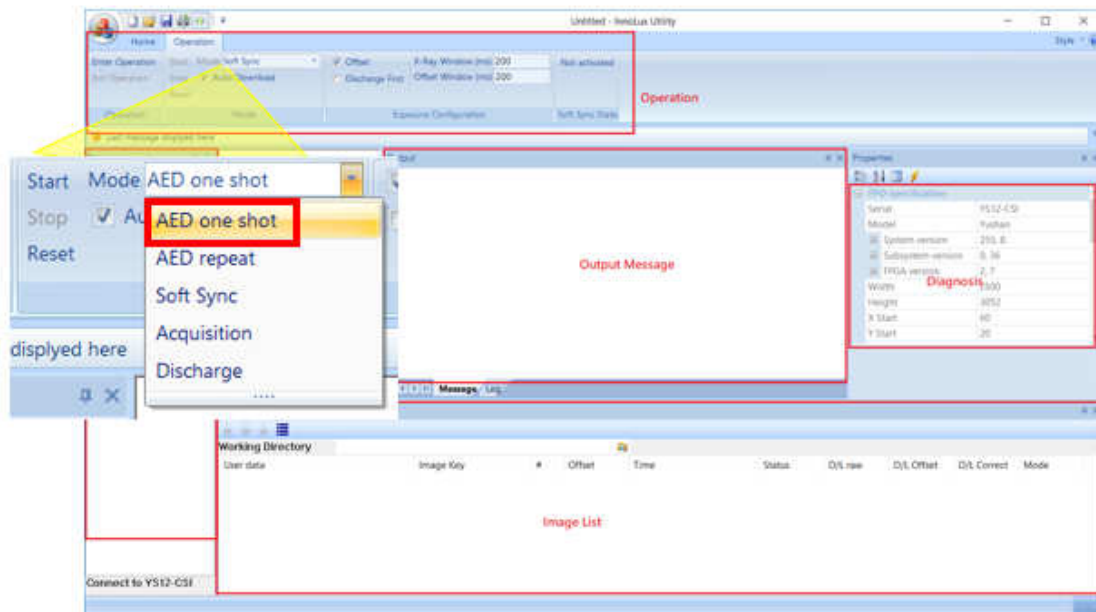

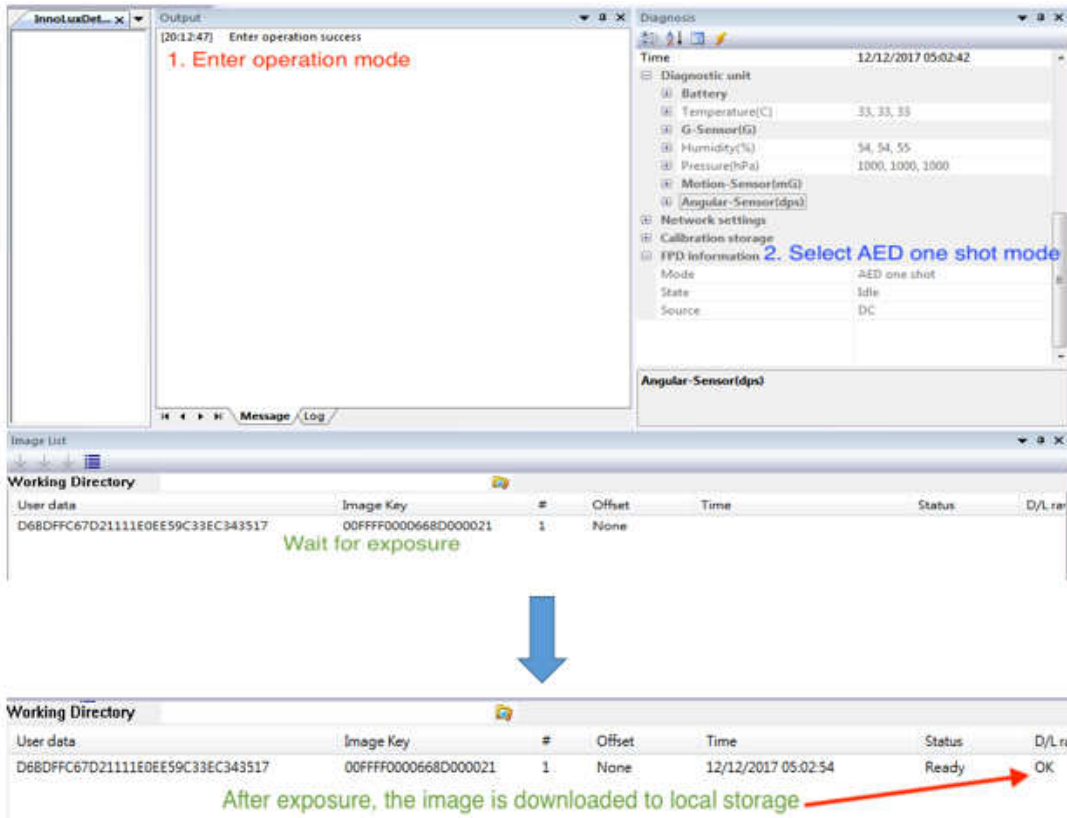

#### 4.3.5.2 Software sync mode (Emulation of two-stage exposure button)

- This group only serves as emulation of two-stage exposure button, and verifies the API functionality.
- After selecting the **Software sync** mode, press Start.
- When performing an exposure, make sure that the STATUS LED light in green and not flashing.

#### 4.3.5.3 AED repeat mode

- After selecting the **AED repeat** mode, press Start.
- When FPD is in **AED repeat** mode, it stays in **AED repeat** mode after each X-Ray exposure is detected. User can use Stop to exit **AED repeat** mode.
- The utility doesn't not download the exposure image after each exposure, it only updates the information of the newly created exposure image in the Image List area.

Output

```
[20:17:57] Start AED "repeat mode";
[20:16:33] Start AED "repeat mode";
[20:12:47] Enter operation success
```

Diagnosis

Time 12/12/2017 05:08:04

Diagnostic unit

- Battery
  - Temperature(C) 33, 33, 33
- G-Sensor(G)
- Humidity(%) 54, 54, 54
- Pressure(hPa) 1000, 1000, 1000
- Motion-Sensor(mG)
- Angular-Sensor(dps)

Network settings

Calibration storage

FPD information

Mode	AED repeat
State	Busy
Source	DC

Angular-Sensor(dps)

Image Key	#	Offset	Time	Status	D/L ra
36E428CCC62935A	00FFF0000668D00002A	0	None	12/12/2017 05:07:47	Ready
36E428CCC62935A	00FFF0000668D000028	0	None	12/12/2017 05:07:47	Ready
3C4CA187A1EC54B	00FFF0000668D000026	0	None	12/12/2017 05:06:28	Ready
3C4CA187A1EC54B	00FFF0000668D000024	0	None	12/12/2017 05:06:28	Ready
779CAF1F452483	00FFF0000668D000022	2	OAX	12/12/2017 05:04:10	Ready OK
EE59C33EC343517	00FFF0000668D000021	1	None	12/12/2017 05:02:54	Ready OK

The image of each exposure is listed but not downloaded

The steps of downloading exposure images are

- Exit the **AED repeat** mode
- Use the Get Image List to update the Image List (Optional)
- Select the image and click Download Raw button

2. Download Raw

1. Get Image List

3. Image downloaded

FPD information

Mode	Operation
State	Idle
Source	DC


Angular-Sensor(dps)

Image List

Working Directory

User data	Image Key	#	Offset	Time	Status	D/L
843E3511A421FAAF441B25A24AA58F11	00FFF0000668A00001E	1	None	03/10/2001 17:08:03	Ready	
8C1A8BDA3D92C779A3D224798EBED7C5	00FFF0000668A00001F	1	None	03/10/2001 17:07:27	Ready	
395CFCB5AD8727700FA020064930300	00FFF0000668A000020	1	None	03/10/2001 17:06:51	Ready	
6CCCB706585477FA1B6E428CC62935A	00FFF0000668D00002A	2	OAX	03/10/2001 17:03:15	Ready	OK
6CCCB706585477FA1B6E428CC62935A	00FFF0000668D000028	2	OAX	03/10/2001 17:03:51	Ready	
5AF9083FE0DE317D3DC4CA187A1EC54B	00FFF0000668D000026	2	OAX	03/10/2001 17:04:27	Ready	
5AF9083FE0DE317D3DC4CA187A1EC54B	00FFF0000668D000024	2	OAX	03/10/2001 17:05:03	Ready	
FC7A5E89B49E49FFE9779CAF1F452483	00FFF0000668D000022	2	OAX	03/10/2001 17:05:39	Ready	OK
D6BDFFC67D21111E0EE59C33EC343517	00FFF0000668D000021	1	None	03/10/2001 17:06:15	Ready	OK

### 4.3.5.4 Other Operations

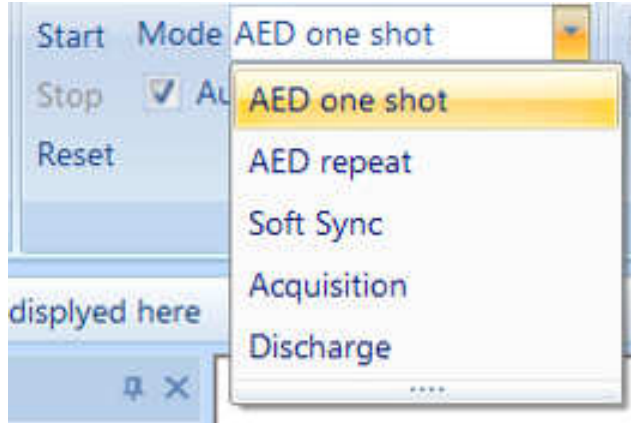


- Discharge: Discharge panel.
- Acquisition: Discharge panel and acquire image immediately.
- Reset operation mode: If errors occur in the exposure workflow, user must use this button to reset.


### 4.3.6 Download image from FPD

- **Auto download**

The software has an automatic download function, as long as the automatic download is checked, the image can be automatically downloaded in modes other than AED repeat mode.

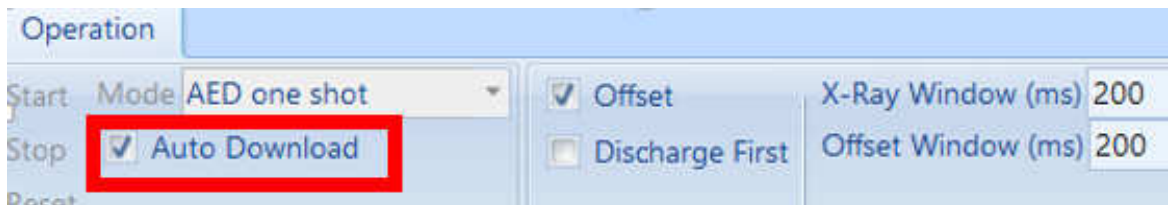

- **Image ready**

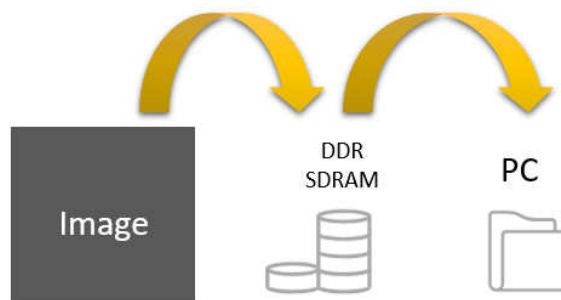

Status → Ready → Download OK



User data	Image Key	#	Offset	Time	Status	D/L rate
D68DFFC67D21111E0EE59C33EC343517	00FFFF0000668D000021	1	None	12/12/2017 05:02:54	Ready	

The image data is first stored in the DDR SDRAM and then transferred to the PC. When the status of image has been show “ready”, it means the images is already stored in the DDR SDRAM.


- **Image download OK**

User data	Image Key	#	Offset	Time	Status	D/L rate
D68DFFC67D21111E0EE59C33EC343517	00FFFF0000668D000021	1	None	12/12/2017 05:02:54	Ready	OK

When the status of image has been show “OK”, it means the images is already stored in the PC.

- **Archive after continuous image taking**

If the automatic download function is not checked, the image will be stored in the DDR SDRAM. First, press (3) to list all the information of the image, and then press the arrow of (1) to output the original image to the PC. In addition, if there is an image with offset set at the beginning of the exposure, press the (2) arrow to output the offset image to the PC.

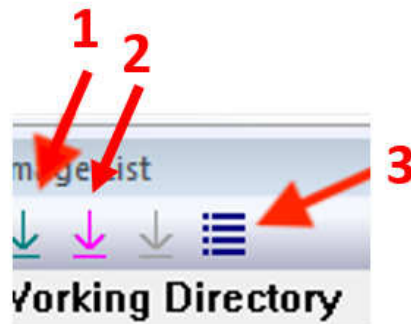

### 4.3.7 Shutting Down the Yushan

Press the power button for more than 2 second to shut down the FPD.

## 4.4 Correction procedure

### 4.4.1 Collect images

When the program is executed, the corrected images are collected in sequence. First, 16 dark images were collected, then the X-ray energy was set to 20 uGy and the X ray window and offset window were both set to 200 ms and collected 16 times. The LSB average of the light image is obtained. It needs to fall within the range of 15,000 to 20,000. According to the above image, the offset map (D) and the gain map (G) are calculated. And use the following algorithm to correct for raw images.


$$C = \frac{(R - D) * m}{(F - D)} = (R - D) * G$$

- C = corrected image
- R = raw image
- F = flat field image
- D = dark field or dark frame
- m = image-averaged value of (F-D)
- G = Gain =  $\frac{m}{(F - D)}$


## 4.4.2 Correction process

First get 16 dark images and 16 light images, sort and store the images in the same folder, and create another folder named "calib". And store the image to be corrected in the same folder and fill in the name of the folder.

Click "Noise": The "mode" represents the level of noise. The "()" represents the number of pixels.

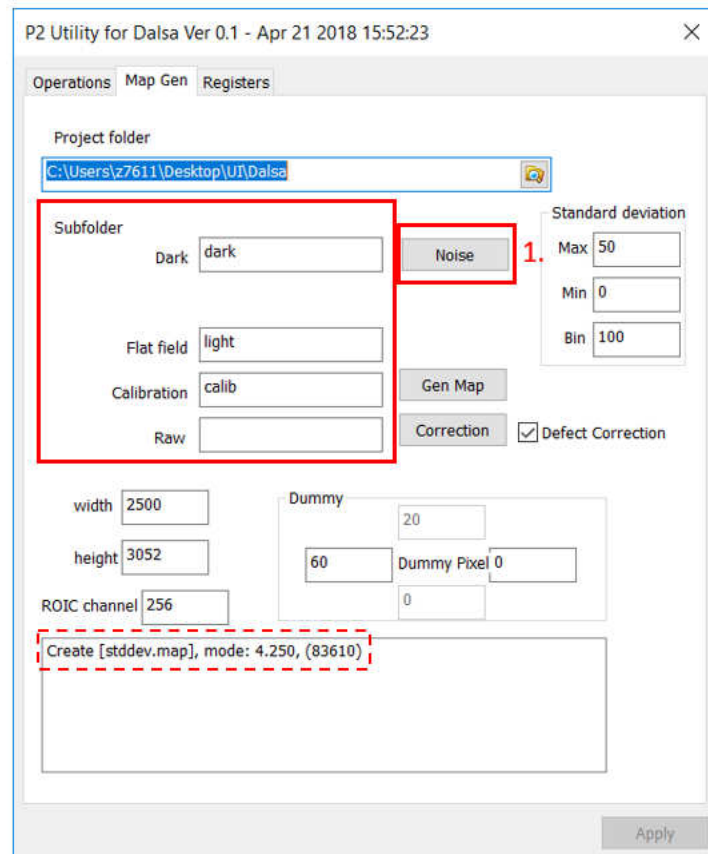


Click “Gen Map”: Will output the map to be calculated

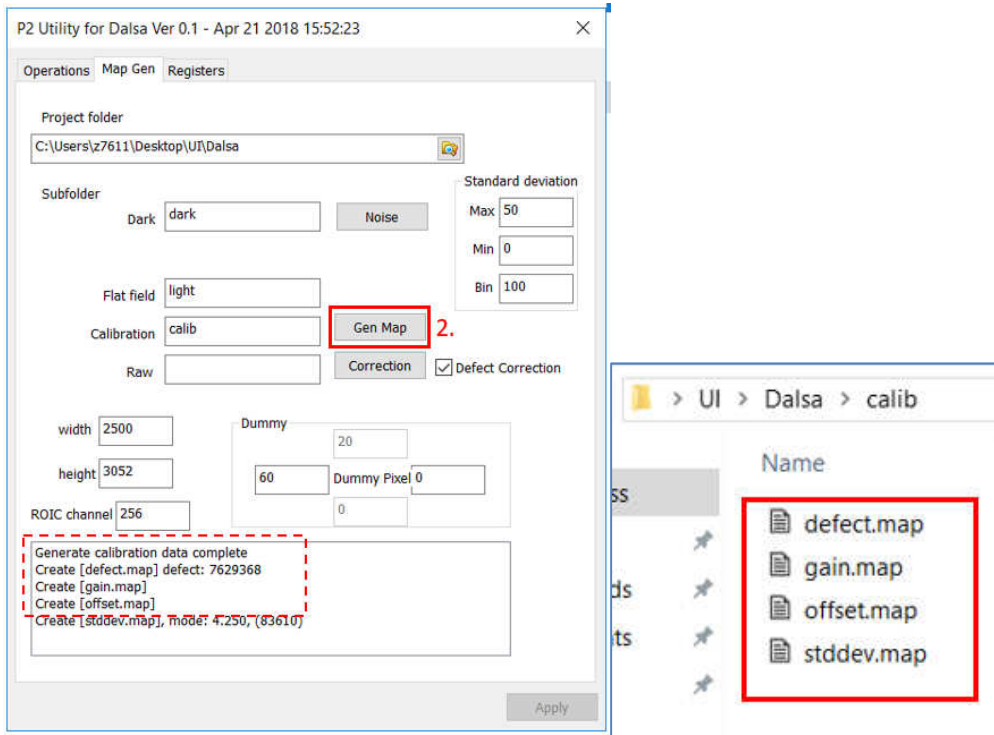

Click “Correction”


The Detector has been successfully corrected if a “correction of all raw images is complete” appears in the windows as seen in the figure.

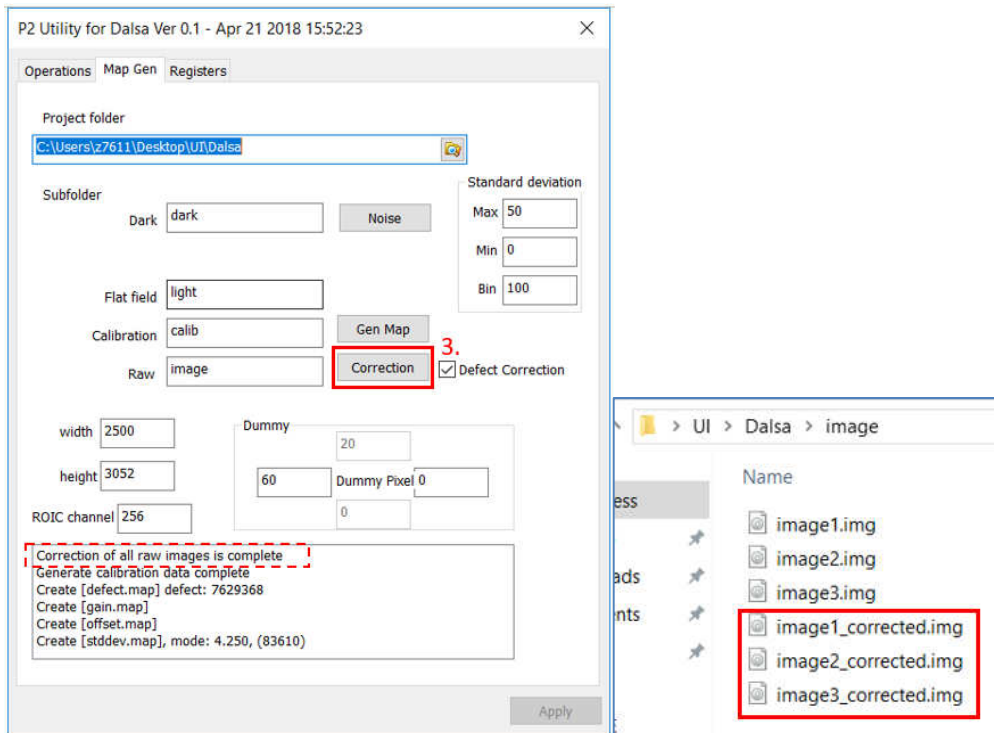


### 4.4.3 Storage of calibration data

10 sets of calibration data can be stored inside Yushan. Users can update or get calibration data for the product.

After selecting the class of the map, click “File to send”. Click on the file to be updated to the inside of the product.

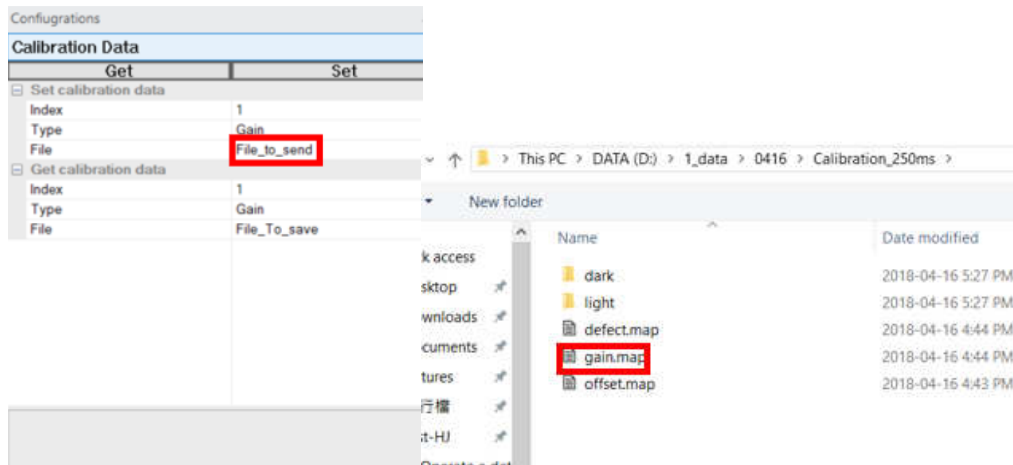

Click “Set” and complete the update.

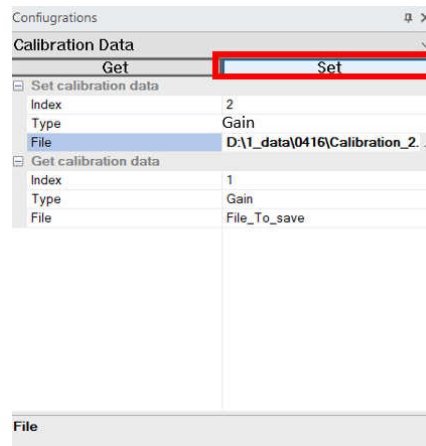

These actions are carried out by Authorized Personnels not End Users.

# Chapter 5 Daily Inspection and Maintenance

## 5.1 Daily User Inspection and Maintenance

During maintenance and inspection, strictly observe precautions contained in “Chapter 2 Safety and Regulatory” in this manual for you to use the device under best conditions.

### 5.1.1 Periodical Inspection

<b>End User</b>	
Remove any dirt or dust accumulated in each part of the equipment using a vacuum cleaner or air duster, clean each part with a slightly moistened soft cloth and then wipe off any moisture with a dry cloth.  See “3.2 Unit Names and the Functions”.	Every Three Months
<b>Authorized Personnel</b>	
There are no parts/components that needed to be replaced by Innolux or any authorized personnel.	



**Caution**

Be sure to turn off the power before cleaning each part of the device.

Ensure sufficient space when cleaning the equipment on a table, etc.

## Chapter 6 Troubleshooting

- In case of smoke, fire, abnormal high temperature, remove battery and power supply and turn off detector immediately.
- In case of yellow error LED on, it may trigger the impact. Please read the manual to release the status and restart the power; if the yellow light cannot be removed, contact INNOLUX service engineer or local representative
- In case of abnormal image, calibrate detector by standard method. Contact INNOLUX service engineer or local representative if error is not removed by calibration.
- Other technical issue, contact INNOLUX service engineer or local representative.
- When Yushan cannot be turned on, please check if the battery has power or if the DC power supply is connected.
- When the software shows that the network can't connect, please confirm the signal of Yushan first. If it is wireless network, the third light needs to be solid green to indicate that it has been connected. If the green light flashes, it means that Yushan is not connected. At this time, please check the network settings of the environment and the computer, and have successfully connected.

# Appendix A Specifications of FPD

## A.1 Specifications

Specifications of the Yushan are shown below.

### A.1.1 Reduced Equivalent


Peak reduced equivalent on the front panel of the flat panel detector: 0.57 mAl

### A.1.2 Power Supply Conditions when Using Back-up

#### Cable

The DC power supply to Yushan FPD is delivered with its AC/DC adapter.

Item	Min	Typ.	Max	unit
Input Voltage Range	80		264	VAC
Nominal Input Frequency	50		60	hz
Input Frequency Range	47		63	hz
Output voltage		24		VDC
Output current			2.5	A
Input connector (AC inlet)		Standard IEC 62320-C14		
Output connector type		DC JACK 5525		

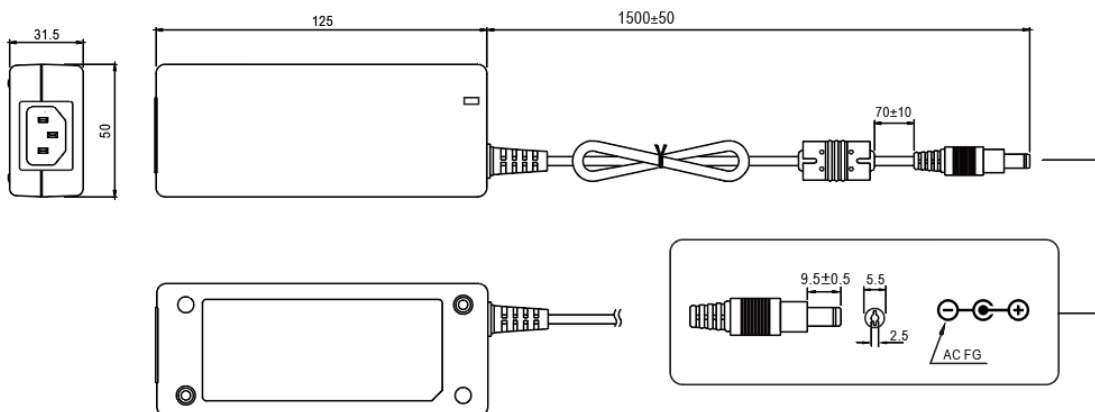



### A.1.3 Power Supply to Battery Charger

A 2-slot battery charger is provided along with the Yushan V14C/V14G.

The battery charger is delivered with its AC/DC adapter which is the same as A.1.2.

### A.1.4 Power Consumption and Battery Specification

Item	Specification (typical)	Unit
Power (Operating)	16	W
Power (Stand-by)	10	W
Battery voltage	11.4	V
Battery capacity	4212	mAh
Battery operating time in stand-by mode	6	hours
Battery operating time in Operating mode	4	hours
Charging time (by cable)	4	hours

### A.1.5 Environmental Conditions

#### A.1.5.1 Storage and Transport

Item	Min	Max
Temperature	-10℃	50℃
Humidity <sup>1</sup>	10%	90%
Pressure <sup>2</sup>	700mBar	1060mBar
Battery storage temperature <sup>3</sup>	-10℃	50℃

- note1: Non-condensing
- note2: 700mBar is equivalent to 3000m in altitude
- note3: Less than 1 month at 50℃

#### A.1.5.2 Functional Operating

Within the range of functional operating conditions, the Yushan V14C, Yushan V14G are safe and functional.

Functional means that the detector is able to deliver an image, whereas the image quality and electro-optical performances are not guaranteed. It also

means that, except if mentioned explicitly, any perturbation occurring within those conditions are reversible and not existing anymore when the detector is turned back within the performance operating conditions.

Item	Min	Typ.	Max
Temperature	5°C	25°C	35°C
Humidity	15%		80%
Pressure	700mBar	1013mBar	1060mBar
Weight (uniform load)			300 kg
Weight (local load on 4 cm diameter)			120 kg
Vibrations (10 -200 Hz / 3 axis )			2 g
Drop height (the reversibility to full performance is not guaranteed for this item)			70 cm

### A.1.5.3 Performance Operating

Performance is guaranteed in the operating range below.

Item	Min.	Typ.	Max.
Temperature	15°C	25°C	30°C
Humidity	15%		80%
Pressure	700mBar	1013mBar	1060mBar
Distance to calibration temperature	-12°C	0°C	+12°C
Weight (uniform load)			300 kg
Weight (local load on 4 cm diameter disc)			120 kg
Vibrations (10 -200 Hz / 3 axis )			2 g
Drop (The Yushan-V14C/G remains at full performance after a drop test of following height)			70 cm



When the flat panel detector is used in high temperature condition for long period of time, it may cause image artifacts and/or failure of the device.

## A.1.6 Image Performance

Each flat panel detector complies with IEC 62220-1 (MEDICAL ELECTRICAL EQUIPMENT - CHARACTERISTICS OF DIGITAL X-RAY IMAGING DEVICES) as a general X-ray radiography equipment.

To ensure optimal image quality, it is recommended that you do not use the flat panel detector near devices (motor, transformer, switching supply, etc.) that generate electromagnetic noise.

### V14C (Csl Version) Performance at RQA5

Item	Min	Typ.	Max	unit
MTF @ 1 lp / mm	0.62	0.69	0.76	
MTF @ 2 lp / mm	0.35	0.39	0.43	
MTF @ 3 lp / mm	0.18	0.22	0.26	
DQE @ 0 lp / mm	0.45	0.51	0.56	
DQE @ 1 lp / mm	0.43	0.48	0.53	
DQE @ 2 lp / mm	0.35	0.39	0.44	
DQE @ 3 lp / mm	0.23	0.29	0.35	
Sensitivity @2uGy	488	574	660	LSB/uGy
Dark noise		4.5	6	LSB
Maximun linear dose	75	88	101	uGy
Maximun clipping dose	100	120	140	uGy

### V14G (GOS Version) Performance at RQA5

Item	Min	Typ.	Max	unit
MTF @ 1 lp / mm	0.47	0.52	0.57	
MTF @ 2 lp / mm	0.20	0.23	0.26	
MTF @ 3 lp / mm	0.06	0.11	0.16	
DQE @ 0 lp / mm	0.27	0.31	0.35	
DQE @ 1 lp / mm	0.23	0.27	0.31	
DQE @ 2 lp / mm	0.11	0.16	0.21	

DQE @ 3 lp / mm	0.04	0.09	0.14	
Sensitivity @2uGy	503	592	681	LSB/uGy
Dark noise		7.9	10	LSB
Maximun linear dose	75	88	101	uGy
Maximun clipping dose	100	120	140	uGy

### A.1.7 Radio Waves

Wireless specifications for the flat panel detector and the access point are as follows.

Item	Specification
Wireless specification	IEEE802.11 ac / a/g/n
Transmit frequency	5.2, 5.3, 5.6, 2.4 GHz
Antenna	2
Date rate	866Mbps (802.11ac)
Authentication	WPA/WPA2-PSK
AP mode	Work only in 2.4Ghz


- Note: Use of 2.4GHz: ch11 for N.America, ch13 for Europe (ETSI)
- Transmit frequencies available vary, depending on the country.
- Radio waves available outdoors vary, depending on the country where the system is used.
- When the Yushan and any other wireless equipment are operating on the same frequency channel in a hospital, it may take time to show an image on the image processing unit monitor.
- This equipment uses wireless LAN (WLAN) radios for transferring images. The WLAN power levels and antenna configurations have been tested and certified compliant through specific absorption rate (SAR) limit set by FCC testing with separations as small as 0 cm between the panel antennas and human tissue.

### A.1.8 Reliability and Lifetime

Item	Specification	unit
X-ray hardness	100	Gy
Expect service life	10	years

## A.2 External View and Weight

Item	Width	Length (mm)	Height (mm)	Weight (Kg)
Yushan V14C	460	383	15	2.7
Yushan V14G	460	383	15	2.7

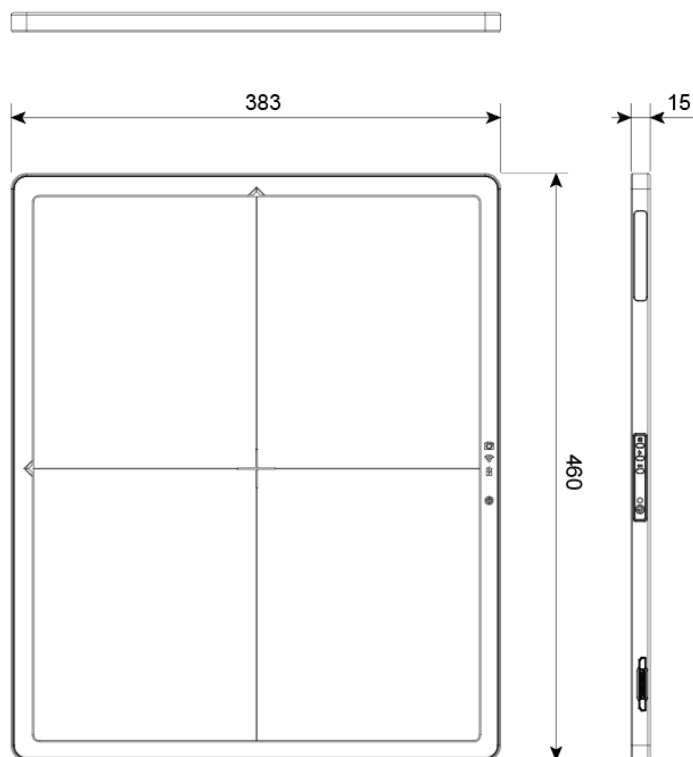

Unit : mm

# Appendix B Electromagnetic Compatibility (EMC)

## B.1 Yushan FPD

This equipment has been tested and found to comply with the limits for medical devices to the IEC 60601-1-2:2014/EN 60601-1-2:2015.

These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity.

However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to other devices, which can be determined by tuning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) are connected.

If the problem cannot be solved with the above measures, stop using this equipment and consult the manufacturer, our official dealer or INNOLUX Representative for help.



Do not place devices generating electromagnetic wave near this equipment.

If a device(s) other than those specified is connected, predetermined EMC performance can not be guaranteed.

## B.2 Further Information for IEC 60601-1-2 (EN 60601-1-2)

1. Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents.

2. Portable and mobile RF communications equipment can affect medical electrical equipment.

3. Information regarding the cable affecting EMC is as follows.

Name	Connection	Maximum length	General specification
Network cable	Extend Back-up cable to PC or network switch.	20m	Cat 5e or more UTP type and straight cable
Power cord	From adaptor to AC outlet	Use a hospital grade power cord	

4. The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by INNOLUX Corporation as replacement parts for internal components, may result in increased emissions or decreased immunity of the Yushan.

5. The Yushan should not be used adjacent to or stacked with other equipment.

If adjacent or stacked use is necessary, the Yushan FPD should be observed to verify normal operation in the configuration in which it will be used.

6. Basic performance of the equipment and the system

After image data are acquired from the flat panel detector, offset data correction is performed by flat pane detector and then the image is send to DR system via wireless connection or wired connection.

7. Test items (Tables 1 to 4)

## Waiting for EMC report.

# Appendix C Radio Frequency (RF) Compliance Information

## **Federal Communications Commission (FCC) Statement**

You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## **This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:**

- 1) this device may not cause harmful interference and
- 2) this device must accept any interference received, including interference that may cause undesired operation of the device.

## **FCC RF Radiation Exposure Statement:**

1. This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.
2. This equipment complies with RF radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated



with a minimum distance of 0 cm between the radiator and your body.

**INNOLUX**

---

 Manufacturer  
**INNOLUX CORPORATION**  
No.3, Sec. 1, Huansi Rd., Tainan City,  
Southern Taiwan Science Park, 74147, Taiwan (R.O.C.)

**CE** **E2460**

**EC REP** European Authorized Representative:  
Innolux Europe B.V.  
Stationstraat 39-G, 6411 NK Heerlen, The Netherlands