

Magic Motion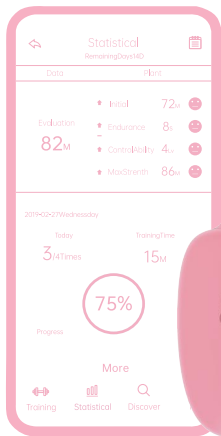


Keegel Coach

User Guide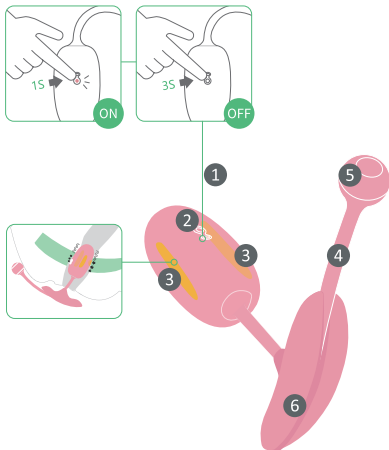

- ❶ Power on/off/Touche Marche/Touche bascule/Einschalten / Ausschalten/Кнопка питания/Ligar /desligar/Encendido / apagado/Accensione / spegnimento /電源オン/オフ/전원 켜기 / 끄기/開/關機鍵/开/关机鍵
- ❷ LED Light/Luminosité LED/LED-Licht/Светодиодный индикатор/Luz LED/LEDライト/Luce a led/LED 라이트/LED指示灯/LED指示灯
- ❸ Touch sensor areas/Zones de capteur tactile/Sensorflächen berühren/Сенсорные датчики/Áreas de sensor de toque/Áreas sensoras táctiles/タッチセンサーエリア/Área del sensore tattile //터치 센서 영역/傳感器區域/传感器区域
- ❹ Antenna/Antenne/Антенна/Antena/안테나/アンテナ/天線/天线
- ❺ Charging port/Port de chargement/Porta di ricarica /Aufladestation/Разъем для подключения зарядного устройства/Porto de carregamento/Puerto de carga/充電ポート/충전 포트/充電口 /充电口
- ❻ Anchor/ Ancre/Анкер/アンカー/ Ancora/Anchor/고정기/ Ancla/ Âncora/限位器

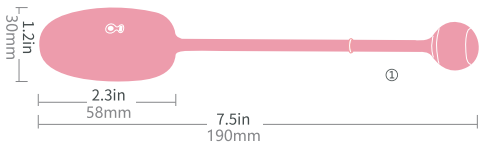②

③

④

⑤


- ① Kegel Coach/凯格尔初恋
- ② Anchor/ Ancre/Анкер/アンカー/ Ancora/Anchor/고경기/ Ancla/ Âncora/限位器
- ③ Magnetic USB Cable/ Magnétique Cable USB/Магнитный USB-кабель/충전 라인/磁気充電器/
Magnetischer Ladeanschluss/Cavo magnetico USB/ Puerto de Carga Magnético /磁吸充电线
/Porta Magnética de Carregamento
- ④ User Guide/ Guide de l'utilisateur/Руководство пользователя/Benutzerhandbuch/Guida utente/
Stauraumtasche/사용 설명서/ Guía del Usuario/ Guia de usuário/取扱説明書/ 使用指南
- ⑤ Storage Bag/ Sac de Stockage/Чехол/Bolsa de Almacenaje/ Saco de almacenamiento/
持ち運び用ポーチ/Borsa di stoccaggio /보관 가방/ 收納袋/收納袋


Preparation for use/Vorbereitung zur Verwendung/Подготовка к использованию/Préparation à l'utilisation/Preparación para el uso/Preparação para uso/Preparazione per l'uso/사용 준비/使用準備/使用准备

Touch the sensor/Berühren Sie den Sensor/Сенсорный датчик/Touchez le capteur/Toca el sensor/Toque no sensor/Тосса il sensore/센서를 터치/センサーに触れる/觸摸傳感器/触摸传感器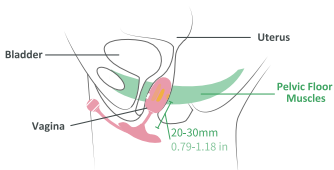





Find pelvic floor muscle/Finden Sie Beckenbodenmuskel/Найти мышцы тазового дна/Trouver le muscle du plancher pelvien/Encontre o músculo do assoalho pélvico/Encuentra el músculo del piso pélvico/骨盤底筋を見つける/골반 근육 찾기/Trova i muscoli del pavimento pelvico/尋找盆底肌/寻找盆底肌

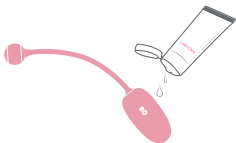A water-based lubricant/Un lubrifiant à base d'eau/Ein Gleitmittel auf Wasserbasis/Смазка на водной основе/Um lubrificante à base de água/Un lubrificante a base de agua/Un lubrificante a base d'acqua/水性潤滑劑/수성 윤활제/水溶性潤滑劑/水溶性潤滑劑


Using Kegel Coach with anchor/Использование Kegel Coach/Utilisation de Kegel Coach/Usando Kegel Coach/Kegel Coachを使用する /Kegel Coach 사용/配合限位器使用Kegel Coach

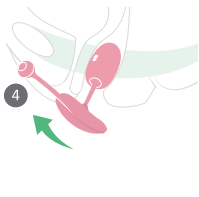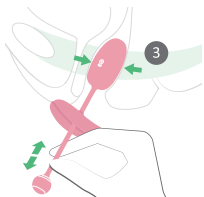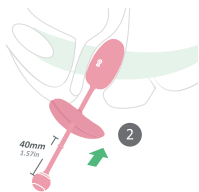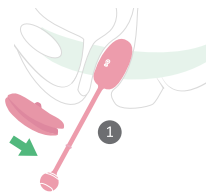Force-Feedback Training Mode/Force-Feedback-Trainingsmodus/Mode d'entraînement à retour de force/Режим обучения с обратной связью/Modo de treinamento Force-Feedback/Modo de entrenamiento de retroalimentación de fuerza/Modalità di allenamento Force-Feedback/強制フィードバックトレーニングモード/강제 피드백 훈련 모드/力反饋訓練模式/力反馈训练模式

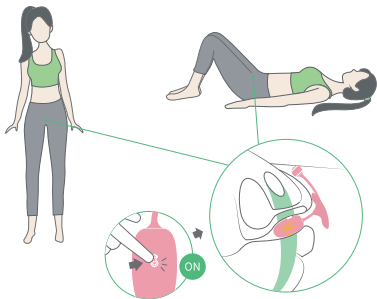Smartphone Workout Mode/Smartphone-Trainingsmodus/Режим тренировки смартфона/Mode d'entraînement sur smartphone/Modo de treino para smartphone/Modo de entrenamiento de teléfono inteligente/Modalità allenamento smartphone/スマートフォンワークアウトモード/스마트 폰 운동 모드/APP交互訓練模式/APP交互训练模式

- 1 Download APP/Herunterladen APP/Скачать APP/Télécharger APP/Descargar APP/Baixar APP/Scarica APP/ダウンロード APP/다운로드 APP/下載APP/下載APP

Magic Kegel

Available on the
App Store

Google play

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EN: Precautions

Do not use device

- If You have given birth within the last six weeks
- You experience persistent pain when inserting or using device, even with water-based lubricant
- You have a medical condition affecting your pelvic health (for example, abnormal vaginal discharge or severe prolapse)

You can use Rebelle during pregnancy as long as you are not suffering from any complications. device is an exercise tracker for core muscle strengthening. It must not be used to treat or prevent any medical condition. If you are unsure whether or not you should use device, please consult a qualified medical professional. For hygiene reasons, do not share your device. Do not leave your device in for more than one hour at a time.

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Magic Motion

www.magicmotion.fun

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning: changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment