FCC and Industry Canada Information

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions: 1) This device may not cause harmful interference, and 2) this device must accept any interference received, including interference that may be caused by undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modification not expressly approved by the party responsible for compliance could void your authority to operate the equipment.

Serene Group, Inc.
14731 Carmenita Road, Norwalk, CA 90650
www.sereneinnovations.com

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Serene Group, Inc.
14731 Carmenita Road, Norwalk, CA 90650
www.sereneinnovations.com Rev 10-20

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HearAll[™]
Cell Phone Amplifier

Pairing the SA-40 with Your Cell Phone:

1. To perform the initial pairing, activate the *Bluetooth*[®] feature of your cell phone.
2. Turn on the SA-40. It will blink as it pairs to your cell phone. The SA-40 will remain paired until you delete it from your cell phone.

Select an Operating Mode:

1. **HS** – Use as a handset.
2. **T** – Couple with your T-Coil hearing aid.
3. **SPKR** – Use as a speakerphone.

Answer a Call on the SA-40:

1. When your phone rings, press  to answer.
2. Adjust volume by pressing the **VOL (+)** or **VOL (-)** buttons.
3. To enhance clarity, adjust tone by moving **TONE**, until you hear the best.
4. Press  to hang up.

Making a Call on the SA-40:

1. Press  to start.
2. If your cell phone is equipped with voice-dial, speak the desired name or number into the SA-40.
3. If your cell phone does not have voice-dial, enter the number using key pad of your cell phone.
4. Press  to hang up.

Please Note:

The SA-40 amplifies your caller's voice up to 100 times louder than your cell phone. Use care when setting phone volume. The SA-40 should always be set to (HS) Handset Mode when handset is pressed to the ear.

Need Help? Call Toll-free Help Line:
866-376-9271 (9AM - 5PM PST, M-F)

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*Bluetooth is a trademark of Bluetooth SIG, Inc.

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