

正面

bebird

Solve the problem of ear cleaning for the entire family

Open bebird Note3

IOS
Search for "bebird" in App Store

Android
Search for "bebird" in Google Play

1 Scan QR code to download APP

2 Enter APP
Select bebird Note3

3 Open the cover
Power on automatically

4 Open the cover
Power on automatically

Blue lighting flashing

Connect Product WIFI

1 Connect product
WIFI "bebird-Note3-xxxx"

2 Back to bebird APP
Click on "Start" to use it

3 Click on the tweezers sign to
switch between tweezers
opening and tweezers clamping

4 Miniature robotic arm
Open the visual tweezers
Clamp earwax

Accessory replacement

Cover the tweezers with
the ear spoon
Fasten the ear spoon

Cover the tweezers with
the ear spoon
Fasten the ear spoon

The ear spoon
must be assembled in
place before using
to prevent falling off.

Ear canal protection ring
Move up and down to
limit insertion depth

Please put the ear canal
protection ring on the rod
when you use it lying on
your back

Storage base

Spin the cover of the base to open it

Swap accessories

Spin
Open the storage base

Accessory

1 Classic O-elastic Ear Spoon x 4
Smooth use with no harm to the ear
A set separated for each family member

2 Microfleece Cover x 1
Cover the magnifying target with the
plug the ear cleaning rod in the lens cover,
it turns into an electronic magnifier

3 Special Cleaning Ear
Spoon x 4
Dig deep into the inner ear for hidden corners

4 Ear Canal Protection Plug x 1
Avoid accidental injury to the
vulnerable ear canal

5 Curved-necked Annular
Ear Spoon x 4
Easy for scraping
Professional in removing stubborn earwax

6 Alcohol Cotton Swab x 5
Softening Cotton Swab x 5
Clean the ear canal
Soften stubborn earwax

7 Mini Brush x 1
Clean lens, tweezers and ear scoops

8 Type-C cable x 1
Fast and convenient charging

背面

Product Size

Outside diameter of lens tube 5.0mm

Troubleshooting

Trouble	Solution
Start up and no response	Please try several more times according to the correct boot-up method. Please use the attached charging base for charging. The charging time is about 1h.
Camera does not turn on	Shut down and restart the ear cleaning rod.
APP quits quickly	Delete the existing APP and re-install the APP code for installation.
The power indicator of the ear cleaning rod shows blue light flashing softly	Charge the ear cleaning rod with the charging base from the original packing.
WIFI connection is unstable, sometimes it is disconnected in seconds	Restart ear cleaning rod, reconnect bebird APP.
The product failed to display the real-time image after successful connecting the bebird APP	Disconnect the phone from WIFI, restart the product and reconnect the WIFI.

If the above operation still fails to solve the problem, please contact Customer Service for further assistance.

Product parameter

bebird smart visual ear cleaning rod
Model: bebird Note3

CE 210-153770
FCC ID:2AUB3-BEBIRDNOTE3

Brand bebird

Product Name bebird smart visual ear cleaning rod

Model bebird Note3

Rod weight 128g

Network standard IEEE 802.11 b/g/n

Antenna Wire antenna

Working frequency 2.4GHz

Image transmission rate 20fps

Image sensor CMOS

Working temperature -10 - 50°C

Battery 300mAh lithium battery

Battery life About 70 minutes

Charging time 1h

Input parameters 5V\times2.0A

Working voltage 4.2V

Rated power 1.5W

Lens diameter 3.7mm

Optimum focal length 1.5 - 2cm

Pixel 1000W

Gyroscope 6 axis

CE Certification

Warning card

User information

User name: Telephone number:

Purchase date:

Seller: Telephone number:

Hazardous substance

Part name	Hazardous material				
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB) (PBDE)
Printed Circuit Components	X	○	○	○	○
Charging Base	X	○	○	○	○
Case	○	○	○	○	○
Plastic Rubber Accessory	○	○	○	○	○

This form is formulated in accordance with SUT 11364
○ : represent the hazardous material is below GB17520-2011
X : represent the hazardous material is above GB17520-2011