

Material: 105G + dumb oil

Size :55x179.5mm

Front

bebird®


A visual ear cleaning experience that will amaze the whole family

Download the APP

IOS
Search for "bebird" in App Store


Android
Search for "bebird" in Google Play

QR scan the QR code to download the app




2

Open the cover
Power on
automatically

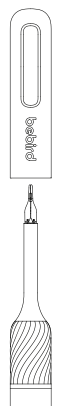


3

Open the APP
Connect device WIFI

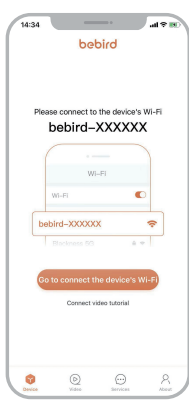


APP Connection

- Connect product WIFI "bebird-xxxxxx"
- Back to bebird APP
Click on "GO" to use it
- Click on the tweezers sign to switch between tweezers opening and tweezers clamping
- Press the body button
Control the opening and closing of the tweezers

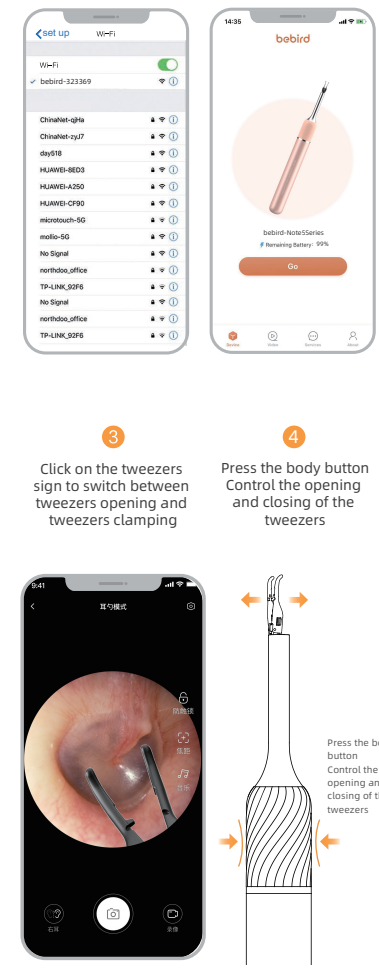

Product Images

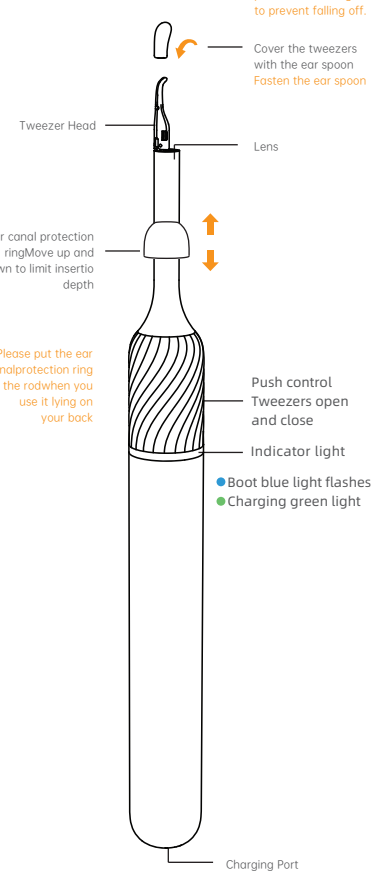

The ear spoon must be assembled in place before using to prevent falling off.

Cover the tweezers with the ear spoon
Fasten the ear spoon

Ear canal protection ring: Move up and down to limit insertion depth

Push control: Tweezers open and close


Indicator light
● Boot blue light flashes
● Charging green light

Charging Port

Please replace the ear tip with a new one every 3 months to avoid aging and breakage.


Accessories

- Classic O-elastic Ear Spoon x 2
- Curved-necked Annular Ear Spoon x 2
- Ear tip container
- Ear Canal Protection Plug x 1
- Magnetic charging cable*50cm

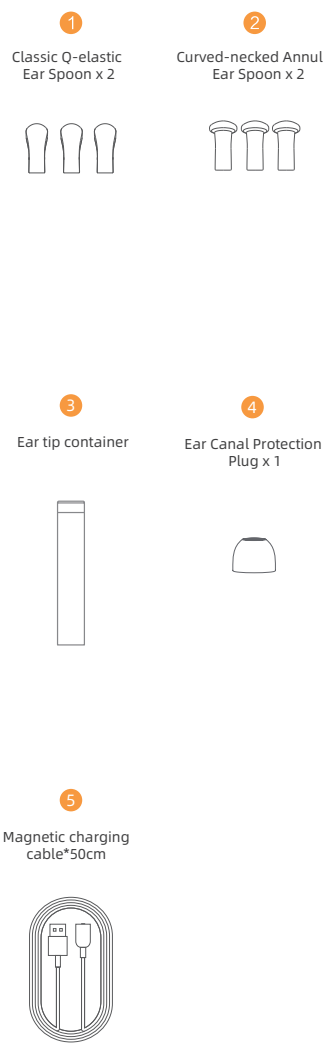

Back

Product Size

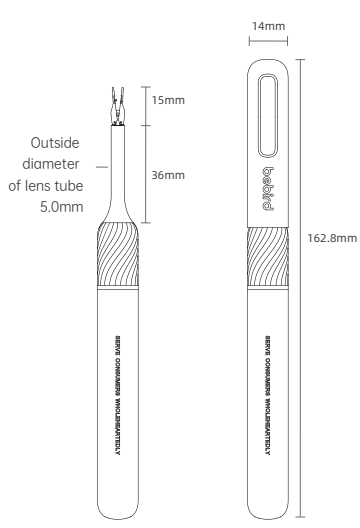

Outside diameter of lens tube 5.0mm

15mm, 36mm, 14mm, 162.8mm


Troubleshooting

Failure Phenomenon	Solution
No response to power on / No sign of power	Please use the matching charging cable for charging
Camera does not light up	Turn off and restart the earwax removal
App keeps crashing	Uninstall the existing APP, re-scan the QR code for installation
Ear cleaner indicator shows red light	Please use the matching charging cable for charging, the full charge time is about 60 minutes
The product successfully connected to the APP does not display the real-time image normally	Disconnect the phone from the earwax removal WIFI, restart the earwax removal and reconnect the WIFI
WiFi connection is unstable, sometimes disconnected in a few seconds	Restart the earwax removal and reconnect to the APP
The lens is blurred	When cleaning the lens, please wipe it carefully with matching double-headed cleaning brush or alcohol swab
Wrong direction when using	Please switch to the left/right ear mode at the bottom of the APP interface
Tweezer head cannot be controlled	Please switch to "Tweezer mode" at the bottom of the APP interface and click the APP tweezers button to control the opening and closing

If the above operation still does not solve the problem, please contact with bebird customer service. We will further provide you with practical solutions

Product parameter

Brand	bebird
Model	Note5
Pixel	1000W
Lens diameter	3.5mm
Optimum focal length	12-15mm
Depth of field	10-50mm
Gyroscope	3 axis
Battery	230mAh
Battery life	About 60 minutes
Charging time	1h
Image transmission rate	20fps
Working temperature	-10-45°C
Input parameters	DC 5V 0.5A
Rod weight	30g
Working frequency	2.4Ghz
Network standard	IEEE 802.11b/g/n
Image sensor	CMOS



FCC Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications to this device not explicitly approved by manufacturer could void your authority to operate this equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

RF Exposure Information
The device has been evaluated to meet general RF exposure requirement.
The device can be used in portable exposure condition, compliance with exposure requirements.

bebird®

3-in-1 Multifunctional Earwax Removal

Hazardous substance

Part name	Hazardous material					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Printed Circuit Components	X	○	○	○	○	○
Case	○	○	○	○	○	○
Plastic rubber accessory	○	○	○	○	○	○

This form is formulated in accordance with 5.81 1564
○ represent the hazardous material ratio is below GB/T26572-2011
X represent the hazardous material ratio is over GB/T26572-2011

Attention

- When cleaning the lens, please wipe it carefully with the equipped alcohol cotton swabs.
- The Ear tip must be screwed on properly and make sure the silicone tip is not loose before use, to prevent it from falling off.
- Please observe the surrounding environment before use.
- Do not use it in an environment that is susceptible to impact.
- Prohibit children from using this product alone to avoid inadvertent injury.
- Please do not put the device inside any form of liquid.
- The battery must be charged once a month for effective use.
- Avoid direct sunlight exposure on the accessories to prevent them from softening.
- When using, the product temperature will rise slightly (from bebird laboratory data, the maximum temperature of the whole machine does not exceed about 45 °C). Please feel free to use.

Warranty card

User information

User name: _____ Telephone number: _____

Seller: _____ Shipping address: _____