

WARNING - SILENT MODE

The audible alarm can go silent on the monitor when using the silencer port. If using a pager with a silencer and pager or component is out of range of wireless monitor, caregiver will not receive an alert from pager.

PLEASE TEST RANGE REGULARLY TO ENSURE PROPER FUNCTION

Customer Service (800) 650-3637

433-CWU-40
R3
Made in China



CONFORMS TO
UL 60601-1

00000000000000000000000000000000

IMPORTANT INFORMATION: This wireless device and compatible components rely on wireless technology which is subject to physical and environmental considerations. These products do not have an out of range function and as such should be tested periodically in the setting in which they are to be used to understand their area of effective operation. Please be aware that the transmitter will not be 100% accurate if it is out of range at anytime. Wireless systems and monitors are intended as an adjunct to good care giving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use.

WARNING: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not substitute for direct visual monitoring by a caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. Read the instructions and legal disclaimer. The Smart Caregiver Corporation wireless device is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to verify that Smart CordLess products are properly installed and tested successfully, before each use, in the setting in which they are to be used. Further, the system is not designed to replace good care giving practices including, but not limited to:

- A.) Direct patient supervision
- B.) Adequate training for staff personnel in fall management and elopement.
- C.) Testing of the system before each use.