
# **User Manual**

HUIYANXIAOBAO(Machine name: EYE2CH)

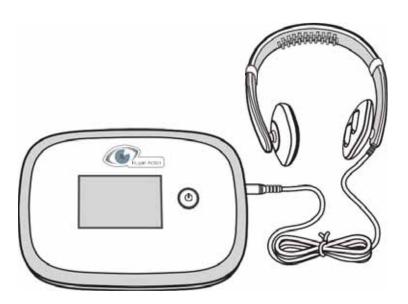



#### 

To make sure safe usage P.1, 2, 3, 4
The name of each part (including accessory)
The preparation before usage P67
Operational method····· P8, 9, 10
The precautions of management-safekeeping P11
Before applying for the after-sale P12
Specification P12
The introduction of clients consultation room Back co

- \* Thank you for purchasing this product.
- \* Thank you for purchasing this product.
- \* Please always keep this booklet at hand.
- Before using it, please make use the safety of accessory without damage.
- Please strictly obey the ways, and precaution of usage and how to use this product correctly.

Shenzhen kangjian and Health Management Service Co., Ltd.

### Please make sure obey it for the safety usage

Please obey the rule of contents as below in order to avoid users and other staffs being damaged or belongs losses.

We will differ the degree for explanation as below(graphs and language) for disobeying the dangers and damages occurred by content of instruction and errors of usage methods.

### Danger

The content of "Death or heavy dangers possibly caused" will be explained aiming at the wrong using methods

### ∕!\Warning

The content of "Death or heavy dangers possibly caused" will be explained aiming at the wrong using methods

The content of "Death or heavy dangers possibly caused" will be explained Attention Ine content of December 1 Attention aiming at the wrong using methods

The content which must be obeyed will be differed by following marks to be explained (below is the example of icons).



"prohibiting" contents which cannot be allowed. expresses the



expresses the "mandatory" content which must be carried out.

### Danger to make sure the safe usage



It might happen wrong operation if it is used with following medical electric appliances together, so please avoid being used.

Implanted medical appliances of cardiac-pacemaker and ICD which are relatively easy to be influenced by electromagnetic interference.

Please contact with doctors before following groups of people use this product.

It might happen wrong operation if it is used with following medical electric appliances together, so please avoid being used.

- The people who are at the unstable first stage of pregnancy or just finished the baby-born
- The people who must keep the quiet environment



- The people who has the initiative barriers with high peripheral circulation disorders due to diabetes mellitus
- The people who have abnormal situations of applicable parts of skins (infectious diseases and trauma).

Please stop using it and contact with doctor if it is no effects after using it or has abnormal feeling for the body.

Please stop using it and contact with doctor if it has rashes, redness, itching due to using this product.

### ⚠ Attention to make sure the safe usage

#### ご使用前の注意

Please use this product under the accompanies by nursing staffs for those disabled or long-term bedridden.

#### It will produce the accident.

Please do not touch the power supplies with your wet hands if it has pins, garbage or water on it.

Please connect with it tightly and correctly to make sure that all the insulated wire are not easy to break away.



Please make sure whether it can be working normally and safely before using the electrical appliance which has not been using for a long period.

#### · It will produce accident.

Make sure using its special guiding part, sponge and AC adaptor of this product.

 If the guiding parts, and insulated wire of other machine are connected into this products, it might produce barrier or electric shock. Especially for AC converter, which is the special converter for this products. Please make sure using this special product.

Please do not use it with other therapeutic apparatus.

#### · It will produce the accident.

Please do not let children use it. ( however, they can use it under the supervision of quardian, doctor or expert.)

In addition, do not tread on the machine and its guiding parts or play. Please do not use this product when drinking, or after eating the sleeping pills.

#### It will produce accident.

Note the last

Please do not use it in the places which are easy to be wet or have serious wetness, like bathroom.

 It will shorten the using service life of machine or may be produce electric shock or fault.

When the insulated wire of AC converter or plug is damaged, the socket is loose to be touched or just not good, please do not use this product.

 Please contact with sales store for spot inspection and maintenance.

Please absolutely do not use this product out of household power supply of AC 100 $\sim$ 240V.

It will produce mechanical fault or fire disaster.

### Attention to make sure the safe usage

#### ご使用中の注意

Please stop using this product immediately if there is malfunction for it and cut off the electric power.

Please stop using this product immediately if there is any discomforts, nausea or abnormal situation for your body during the period of usage. Please stop electrifying and unplug the power supply connector from socket if there is thunder.



Please stop using it immediately and replace it with new products if found there was insulated wire of AC converter and map crack on other insulated electric wires.

It will produce serious stimulation

Please do not let necklace touch guiding parts

It will produce serious stimulation

Please use stimulation(short time) at the first time of using and adjust its strength after adapting it little by little.

Please cut off the electric power for the time being if you need to change the position of guiding part halfway and move the position after stopping export.

It will produce the serious stimulation.

Please do not spread the water or tea on the machine or AC converter.

It will produce electric shock or fire disaster.

Please do not pull the insulated wire, or thorn the pin and needle.



It will produce machine fault or fire disaster.

Please do not bend extremely or pull insulated wire and its body or jointing parts of quide.

It will produce the machine fault.

Please do not sit on the body, or fall down, push down the body to provide strong shock.

It will produce the machine fault.

Please do not place it near the stove.

It will produce fire disaster or machine fault.
 Please do not cover the cotton quilt or carpet on the machine.

It will produce fire or machine fault.



### Attention to make sure the safe usage

#### ご使用後の注意

Please cut off the electrical power after pressing the electric power button. Please do not take the insulated wire when removing insulated electric wire but take the plug and then unplug.



It will produce electric shock or fire disaster.

Please unplug the energy plug from the socket if no need to use for a long time.

 It will shorten the service life of machine, and possibly cause the electric shock or fault.


Please do not repair it causally if malfunction happens, but just contact with sales store.



Please do not absolutely decompose, repair or transform this product,


 It might cause fire or electric shock and injury due to abnormal action.



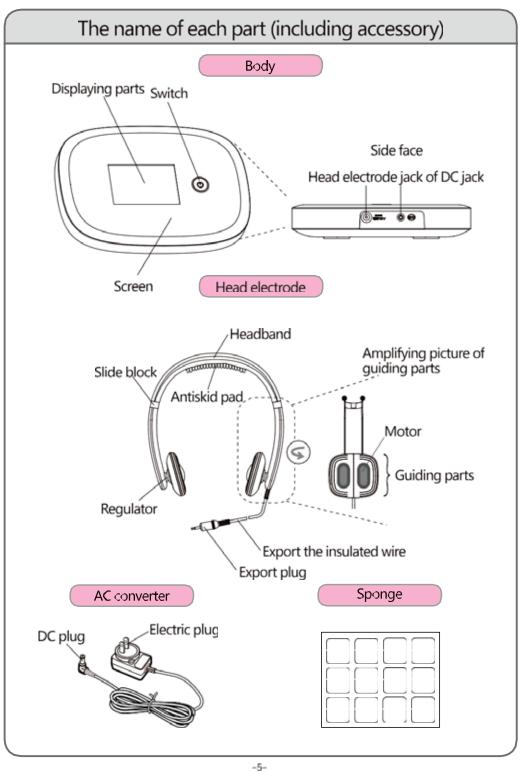

### The preparation before using


 Install the electrode parts of guide at the head motor with slightly having some water shown as below.

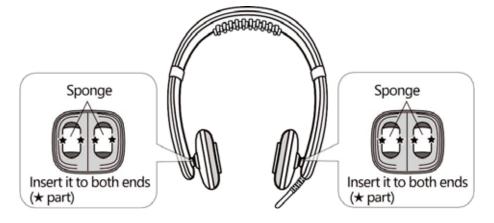

- 2 Insert DC plug of converter into the body itself.
  - Insert the plug to the end tightly.




- 3 Insert electric plug of converter into the socket.
  - Insert the plug into the socket of AC100~240V.

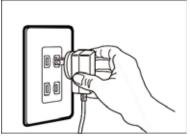

### The preparation before using


- Insert the export plug into the head electrode jack of the product body.
  - · Insert the plug into the end tightly.

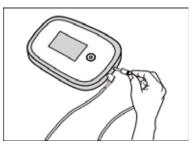

5 Put on the head electrode shown as right figure.

Point

(1) Make sure the headband at the back side of the head when wearing it.

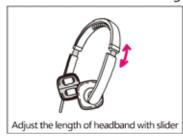




Point

(2) Adjust the position of guide part, which can make the electrode of guide paralleled to the zygoma.




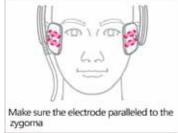

Please pay attention to the part of temple which does not clip the hairs. In addition, please use it before remove the make up of touching part if the users make up.


### Operational method

- This product is not medical appliance. Not suitable for diagnosis or other symptom and in addition, it is also not suitable for healing, relief, treatment and prevention of disease.
- 1 Long-press the button.
  - 'The displaying part of the body will appear the picture as below.

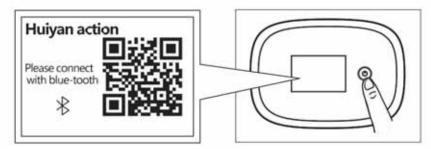


- 2 Open the wet Chat in the smart mobile phone.

  Select the "Huiyan action" in the dialogue list. Select "Scan code for using", scan the body itself to display the part of two-dimension.
  - · It will display the setting picture on the smart mobile phone.

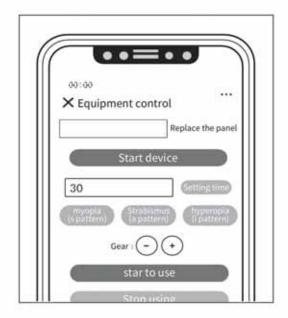


### Operational method


3 Select "Opening device" on the picture of smart mobile phone, and start to set.

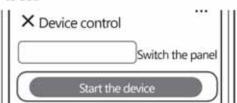

4. Next, set time and mode to start using.

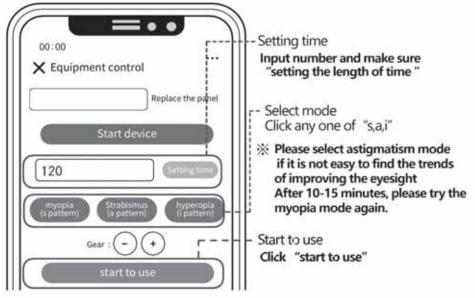


 When setting, the presentation parts of the product itself will be shown as below.

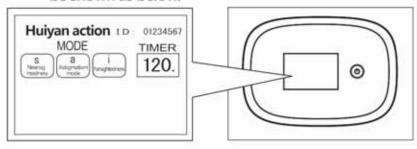


### Operational method


**5** Switch strength and mode in accordance with interests

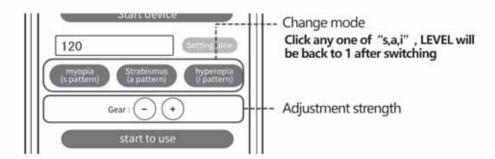

 When setting, the the presentation part of product itself will be shown as figure below.

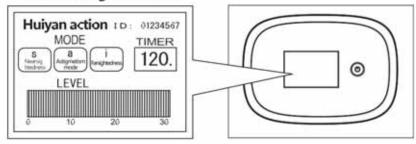

- **6** The product will automatically stop by timer.
  - \*Please press "stop using" if you want to stop it midway.

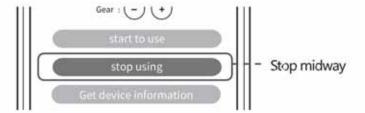

### The precautions about management-safekeeping

#### Sponge

The sponge will be moldy or breeding hybrids.
 Please clean it with water fully after using and keep it after dry it.

 Please replace with newly products if the sponge is broken and deteriorated (in taters).

### **Attention**

Please take care of it at the place where the infants and kids cannot touch.

It will produce the accidents.



Please clean this product and its accessories in place to make sure there is no barrier when using next time, and then sort out and keep it at the place where the moisture is relatively less. When keeping, the high temperature should be avoid to select the place where the moisture and dust are less.

 It will shorten the service life of machine, which also might cause the electronic shock or fault.



Please do not use diluents, gasoline or alcohol.

 It will produce damage or discoloration. The plastic parts of this product are needed neutral detergents for cleaning and then use the soft cloth for dry.

#### Purchase about consumable

Please consult the stores which purchase the commodity if the consumable is needed (sponge).

#### The precautions when discarded

Please obey the rule that the district regulates for dealing with in order to reduce the influence to the environment if this product is needed to discard.

#### **FCC Warning**

- Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.
  This device complies with part 15 of the FCC Rules. Operation is subject to the following
  - (1) This device may not cause harmful interference, and

two conditions:

(2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- -Reorient or relocate the receiving antenna.
- -Increase the separation between the equipment and receiver.
- -Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- -Consult the dealer or an experienced radio/TV technician for help.

This equipment complies with FCC radiation exposure limitsset forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 20cm between the radiator & your body.

## Before application of after sales service

X Encourage the point inspection for daily usage. If find any abnormal situation, please stop using it immediately, unplug the power plug and go to the sales store for spot inspection.

When the situation happens	Please check these places	The way to deal with correspondingly	
	Is the power plug correct to insert into the socket?	Please insert the power plug into the socket.	
No lights and no actions at the displaying place	Is the DC plug correct to insert the body case?	Please insert the DC plug into the body case.	
	Does it put through the electric power?	Please press the button of electric power	
	Is there any condensation for this body case?	The machine cannot be run normally due to the condensation possibly. Therefore, please let it stand still for about 1 hour and then insert into the electric power again.	
Relatively weak stimulation	Is the sponge fully inserted by water?	Please let the sponge absorb water fully.	
The body case becomes warm	_	Maybe due to the long-time usage, the machine will be caused heating so as to make the body case warm, which is not abnormal.	
AC converter detaches the power plug	_	Please install it to the original place practically.	

- If there is any action which is not in line with the given action except for above mentioned, please unplug the
  electric plug for the time being, wait for a while (about 1 minute), and then put the electric power through.
- If the corresponding method is adopted, the abnormal problem is still not solved, please contact with seller.
   In addition, when this condition is happened, the commodity, purchasing time and fault need to be reported to the seller in details.

Specification				
Commodity name	Huiyanxiaobac			
Machine name	EYE2CH	Regulated voltage	AC100-240V	
Regulated frequency	50/60HZ	Using voltage (special AC converter)	DC12V/1ACCC certification	
Size	Body case 235*168*32mm Head electrode 185*160*63mm	Weight	Body case:800g Product:1100g Package:1500g	

MEMO		
		ı

MEMO		
		ı

#### The introduction of clients' consulting room

\*Please contact with the selling store that you purchase the commodity firstly of you want to consult with some matters concerned with repairs or purchasing components.

The consulting about operating methods and repairs

O755-27850919

Shenzhen Kangjian and Health Management Service Co., Ltd.
The time 900-1800 of working time
(except for weekends, holiday and rest-day of the company)

#### The processing method of client's personal information about consulting window

Shenzhen Kangjian and Health Management Service Co., Ltd. (hereinafter referred to as "our company") carried out in accordance with the subordinate regulations aiming at the client's name, address, and other personal information provided by clients (hereinafter referred to as "personal information").

- \*This company will use personal information of clients to the related matters of corresponding, repairs and its confirmation for consulting with our company. The related records of consulting contents will be left for these purposes.
- In addition, when we entrust the external coordination of our company to repair and confirm the business, or in accordance with related decrees, we must fulfill the obligation or rights unless there are other legal reasons, otherwise you cannot exhibit or provide personal information to the third party except for our company.
- · Our company will manage the personal information of managing the clients appropriately and properly.
- Please contact with consulting window if you need to consult with the related matters about the personal information of clients.