DEUTSCH

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US ENGLISH

USER INSTRUCTIONS Widex SoundConnect

⚠ Read these instructions carefully before you start using the device.

Intended use
The device is intended to transmit audio signals from a Personal Computer directly to supported hearing aids.

NOTE:
The device uses a USB A connector. If your computer is without an USB A configuration, you may use a converter.

HOW TO USE

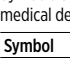
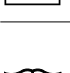
Connect the device to your computer. (Ilus. 1 and 2)

Restart your hearing aids. The hearing aids will then be in pairing mode for 3 minutes. (Ilus. 3)

Press and hold the button on the device until the light indicator is flashing blue, then

SYMBOLS

Symbols commonly used by Widex A/S in medical device labelling (labels/IFU/etc.).

Symbol	Title/Description
	Manufacturer: The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.
	Catalogue number: The product's catalogue (item) number.
	Consult instructions for use The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.
	Warning: Text marked with a warning symbol must be read before using the product.

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LIGHT INDICATIONS

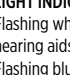
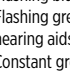
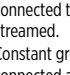
Flashing white: Device has not been paired to hearing aids.
Make sure that the SoundConnect device and your hearing aids are within 1 m range of each other.
The pairing sequence will start automatically.
The light indicator will glow constant green, when the pairing is successfully completed and if sound is played from the computer the audio will be streamed to your hearing aids. (Ilus. 5)

The SoundConnect can only be paired to one set of hearing aids. To pair a new set of hearing aids, you must perform the same steps and the new pairing will overwrite the existing one.

NOTE
After pairing and you unplug the SoundConnect and reconnect the device, it will keep the pairing.

RANGE
In line of site up to 8 meters
Outside in open air up to 5 meters

SYMBOLS
Symbols commonly used by Widex A/S in medical device labelling (labels/IFU/etc.).

Symbol	Title/Description
	WEEE mark: “Not for general waste”. When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances.
	CE mark The product is in conformity with the requirements set out in European CE marking directives.
	RCM mark The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.
	Medical device Indication that the device is a medical device.

REGULATOR INFORMATION FCC AND ISED STATEMENTS
FCC ID: 2XMDT-WSC
IC: 2624B-WSC
Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and
(2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE:
This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee

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