SKIIN

Operations and
Safety Manual

Before using Skiin Underwear, please read this manual.

Myant Inc.
100 Ronson Drive, Toronto, ON,
M9W 1B6 Canada
+1 (416) 423-7906
info@myant.ca

LB00044
Rev. 1.1

Warranty information

The Skiin Underwear biomonitoring device and its accessories, when supplied and delivered new, in the original device packaging to the original purchaser, are covered under a one-year warranty that covers damage due to defective materials and or failure of the equipment to operate in accordance with the information in this Operation and Safety Manual for Skiin Underwear.

Version information

System: Skiin Underwear

System Software: Skiin Connected Life App Version 1.0.0

Operation and Safety Manual for Skiin Underwear System Revision 1.1

Original Instructions PP002

Table of Contents

1. Introduction.....	4	7. Travelling with Skiin Garments	12
2. Safety Information.....	4	8. Limited Warranty.....	13
3. Contents.....	5	9. Device specifications.....	13
4. Concept of Operation.....	6	10. Symbols Glossary.....	14
5. Getting Started.....	6	11. Warnings.....	16
Step 1: Try the Underwear.....	6	12. Cautions.....	17
Step 2: Charge the Skiin Pod.....	7	13. General Safety Precautions.....	17
Step 3: Download the Skiin Connected Life App.....	7	14. Other.....	18
6. Skiin Underwear System Components.....	8		
6.1 Skiin Pod.....	9		
6.2 Skiin Charging Kit.....	9		
6.3 Skiin Underwear.....	10		
6.4 Skiin Connected Life App.....	11		
6.4.1 Sections in the App.....	11		
6.4.2 Getting Help.....	12		
6.4.3 System State Messages.....	12		

1. Introduction

This *Operation and Safety Manual* for the Skiin Underwear System provides instructions for the safe and efficient use of the Skiin Underwear System for biometric monitoring application.

Skiin Underwear uses textile electrodes embedded in the band with electronics in the Skiin Pod to measure and analyze your biometric data.

The underwear is machine washable and can be laundered with your other clothes (using the included wash bag).

The Skiin Underwear System is used with a user-supplied compatible smart device.

Contraindications

The Skiin Underwear is not for use by:

- Persons under 18 years of age
- Pacemaker and/or (ICD) implantable cardioverter defibrillator users
- Pregnant women

This device is not intended for use by children. Do not leave the device near babies, infants or children. This device is not intended for use by individuals with limited physical capacities or limited experience and, or knowledge unless they are supervised by an individual responsible for their safety or receive instructions from the latter as to how the device should be used.

2. Safety Information

Before using the Skiin Underwear System, please become familiar with the information in this *Operation and Safety Manual*.

WARNING EN-W1		Unauthorized modification of this equipment is not permitted and may compromise the safe operation of the equipment.
NOTE EN-N1		Follow the instructions in this booklet to learn how to use and take care of your Skiin Underwear System.

Risks for children and individuals requiring assistance

WARNING EN-W2		Failure to follow safety instructions and/or using the equipment for purposes other than those described in Skiin Underwear <i>Labeling</i> constitutes improper use.
-------------------------	---	---

3. Contents

The standard configuration for the Skiin Connected Clothing System includes the following components:

- (A) 1 Skiin Pod
- (B) 1 USB Charging Kit:
 - 1 USB Charging Dock
 - 1 Micro-USB Cable
 - 1 Wall Plug
- (C) Skiin Garments: Each kit has one "Try Me On" garment that you must try first to check for fit before opening the other garment boxes.
- (D) 1 Wash Bag

WARNING
EN-W3

Use of accessories, and cables other than those specified or provided by the manufacturer of this device could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

NOTE
N-N2

Some features described in this guide may not be available with your configuration of Skiin Garments.

4. Concept of Operation

The Skiin System collects your bio-signals such as heart rate and steps. It is recommended to keep your smart device close to you whenever possible. When you put on the Skiin Underwear, it can start sensing your heart rate and physical activity, which will be captured and stored as data by Skiin Pod and transmitted to the Skiin Connected Life App. As you use the Skiin underwear you become part of the Skiin Connected Clothing System and your health information is stored securely so you can view it from your App at any time.

5. Getting Started

STEP 1: Try The Underwear

Open the 'Try Me On' box and put on the underwear to check the fit. It is important to ensure your underwear is the correct size, as good contact of the sensor zones in the band with the skin surface is required to get a good signal for biometric recording. Proper fit will ensure accurate heart rate readings. Your underwear should be snug, that means it will not ride up, twist, or slip down. When trying on the underwear, the band should not constrict or dig into the body. The band should sit comfortably on your body and not be too tight or too loose.

Located in the band are 5 sensor zones, 3 on the left hip and 2 on the right hip. Make sure the five sensor zones located in the band of the underwear are in contact with your skin. If your skin is too dry, it may affect the performance of the electrodes.

Sensor Zones

Apply moisturizer to improve the performance. Wash your underwear after each wear as sweat, dead skin and dirt build up can prevent the electrodes from functioning properly. Ensure all components are dry prior to use, do not operate while wet or moist.

If the underwear does not fit, visit: www.skiin.com/returns for instructions on how to return or exchange for the correct size. You may return the unopened boxes of underwear to exchange for the correct size. Please visit www.skiin.com/faq/returns for more info. The following [sizing charts](#) show waist measurements per size. The waist measurement should be taken around the hips, keeping a relaxed posture without holding in the abdomen.

NOTE
EN-N3

The underwear without the 'Try Me On' label on the box, cannot be returned if opened.

SIZE	RANGE (INCHES)
Women's	
XS	29.5-32
S	32-34.5
M	34.5-37
L	37-40.5
XL	40.5-45

SIZE	RANGE (INCHES)
Men's	
XS	30-32.5
S	32.5-35
M	35-37.5
L	37.5-41.5
XL	41.5-45

STEP 2: Find & Charge the Skiin Pod

Use the supplied Charger to charge the Pod. Insert the Pod into the Charging Dock. You should feel the magnets pulling the Pod followed by an audible “Snap”. Connect the Charger and the Wall Outlet using the Micro-USB Cable and plug it into a power source. When the light on Pod shows magenta color, the Pod is activated. Take out the Pod from the Charger, you should see a blinking yellow light that indicates it is ready to be paired.

NOTE
EN-N4

It takes Approximately 1.5 hours for the Pod to be fully charged from a discharged state.

STEP 3: Download the Skiin Connected Life App

On your smartphone, download the Skiin Connected Life App from the Apple App Store or the Google Play Store. Open the App and follow the onboarding steps to setup and pair the Skiin Pod, start logging your health data and connect to the Skiin System. Please refer to the Skiin website (<https://www.skiin.com/#compatibility>) for the latest list of compatible devices and operating system versions.

1. Open the App on your phone.
2. Create an account, or login with your Skiin username and password.
3. Set up your account.
4. Activate your Skiin Pod by charging it. Your Skiin Pod should be partially charged when received. Charge the Skiin Pod using the provided charger (see Section 6.2 Charging Instructions).
5. Ensure that Bluetooth has been enabled on your phone. In the Skiin Connected Life App, proceed to the Garments Page and select 'Add a new device' on the screen. The App will indicate when the Skiin Pod is connected, and the light on the Skiin Pod will change from a blinking yellow light to blinking blue light.
6. Slide the Skiin Pod into the Garment Dock i.e. a small rigid pocket in the band of the underwear, located on the band of the underwear. The magnets will help automatically snap the Pod

into the Garment Dock. Verify that the Pod is flat in the pocket and not in a skewed position.

- The App should begin sensing your biometric signals. The Skiin App will show you a demo of this.

6. Skiin Underwear System Components

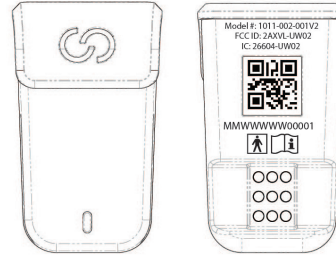
<p>WARNING EN-W4</p> 	<p>Only the components listed in this section may be connected to the Skiin Underwear System. If components other than those specified in this section are connected to the system, the Skiin Underwear Software may fail and cause injury to the user or operator.</p>
---	---

6.1 Skiin Pod

Your Skiin Pod records and securely sends your health information to the Skiin Connected Life App. Pair it with your phone, then slide it into the Garment Dock. The Pod will easily 'snap' with the magnets when inserted in the Garment Dock. The Skiin Pod has an LED light that changes colours to indicate different states of charge and connection.

Low Battery

The Pod's LED blinks RED when the battery is low and the Pod should be charged. Use the Skiin Connected Life App to see the detailed Pod battery level.

Picture of Pod with Model Number, FCC ID, IC, Serial Number and markings

Pod Connected State

Once the Pod is connected to your phone via Bluetooth the Pod's LED turns into a BLUE blink, once every second, indicating that it is connected.

Pod Disconnected State

If the Pod has no Bluetooth connection the Pod's LED blinks YELLOW indicating that the Pod is not connected and is searching for a connection. Please check your phone settings to ensure that Bluetooth is activated or launch the Skiin Connected Life App to check the connection status.

Turning the Device On/Off

The Skiin Pod is always "ON" as long as it is charged and has power. You cannot turn the Skiin Pod OFF, but you can unpair it from your phone in order to stop transmitting data. Refer above to Pod Disconnected State. To unpair your Pod, on your Skiin App, in the Devices page please open the Device Settings for the chosen Pod and select unpair.

Data Storage and Syncing

Your Skiin Pod will automatically sync data to your smart device when it is within range and the app is opened or in the background.

When out of range or disconnected from your phone, the Skiin Pod will store biometric data for up to 45 minutes and heart rate for up to 24 hours depending on the amount of data saved. When your phone reconnects, your data will continue to sync again. It is recommended to keep your smart device close to you whenever possible to avoid data loss.

Cleaning the Pod

Do not wash or immerse in water; the Pod, Charging dock, USB Cable or Wall Plug. The surface of the Pod can be wiped clean with a soft, moistened cloth. Do not use any abrasive or solvent based cleaners.

Storing the Pod

Store the Pod in a clean, dry, safe location. Do not store in extremely hot, cold, humid, wet, or bright conditions.

Resetting the Pod

Insert the Pod into the Charging Dock, locate the pinhole under the Charger. Use a pin to press into the hole and hold until the LED flashes to indicate it is reset.

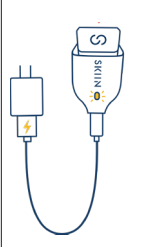
6.2 Skiin Charging Kit

Charging The Pod

When the Pod is inserted into the Charging Dock and connected to a power source, the Pod's LED will appear YELLOW to indicate that the Pod is currently charging. The Pod's LED will remain on and display a YELLOW light until the Pod has fully charged. We recommend that you charge the Pod for 1.5 hours before using.

Fully Charged

If the LED appears GREEN while charging, this indicates the Pod is fully charged and ready to use.

	Magenta	The Pod is activated
	Yellow	The Pod is not fully charged
	Green	The Pod is fully charged
	Blue	The software on the Pod is being updated
	Red	A hardware issue is detected, contact support@skiin.com for assistance

NOTE EN-N5 	While the Pod is charging it will not indicate Bluetooth connection status through the LED light.
--	---

	
Solid Blue	The software on the Pod is being updated
Single Blink Blue	The Pod is connected to the App
Single Blink Yellow	The Pod is NOT connected to the App
Single Blink Red	Low battery level & the Pod is connected to the App
Rapid Blink Red	Low battery level & the Pod is NOT connected to the App

6.3 Skiin Underwear

Materials

Skiin Garments can be washed like regular clothing. The elastic band includes Carbon and Silver yarns. Carbon yarn is used in the textile electrodes of the band and it serves as a conduit to human skin. Conductive silver yarn is used to connect textile electrodes within the band to the textile integration system. Underwear bottoms are made of 95% Bamboo / 5% Spandex - 260 GSM – Single jersey fabric. Bamboo is naturally antibacterial, comfortable, very breathable, moisture-wicking, fast drying and thermal regulating. Bamboo fabric is also anti-static so it doesn't cling to other materials. The band is made of 69% Nylon, 26% Spandex and 5% other fibres.

EN-W5 WARNING 	If you have an allergic or skin reaction please discontinue use and contact customer support at support@skiin.com . If necessary you should seek medical attention.
NOTE EN-N6 	If the Skiin Pod is not inserted into the garment you are wearing, no data will be collected or transmitted.

Laundering Instructions

To care and maintain your Skiin Garments, remove the Pod and place the garments in the wash bag before putting them in the washing machine.

- Before wearing for the first time, please wash
- Wash after wearing the garment for 24-48 hours
- Remove the Pod and put all garments in the washbag, use cold water when washing
- Do not use bleach or fabric softener
- To dry, keep garments in washbag and dry on low
- When you change your garment, take out the Skiin Pod and place it in a clean garment.

EN-W6 WARNING 	Remember to remove the Pod from your garment before washing.
NOTE EN-N7 	Don't air-dry the garment. A machine dryer should be used to ensure elasticity and longevity of the garment.

-  Machine Wash on Cold
-  Tumble Dry on Medium
-  Do not Wring the Garment
-  Do Not Iron
-  No Bleach, No Softener
-  Do Not Dry Clean

Storing the Garments

Store garments in a clean, dry, safe location. Do not store in extremely hot, cold, humid, wet, or bright conditions.

6.4 Skiin Connected Life App

6.4.1 Sections in the App

The Skiin Connected Life App is used to record, display and store your health information. The Skiin Pod can be paired with any android mobile device with Android OS 9.0 or above and any iPhone with iOS 13.0 and above. We recommend you use a mobile device which is compatible with Bluetooth 4.2 for the best App performance.

Home

The Home page for the app provides a summary of your biometric data.

Connections

The Connections page helps you to connect, share and communicate with your friends, family and health care professionals. You can connect with your connections individually and also as a part of a group, sharing all your captured biometric and contextual data. When connecting with your family, friends and/or HCP (healthcare professional), you will be able to specify and control the data which you would like to share with them. You can also reach out to your Contacts using a quick access button to initiate a message, audio call and video call.

Metrics

The Metrics page is where system tracked biometric data (through Skiin devices) is presented to you. The main page is presented in the form of cards for each biometric data parameter that is captured. It gives you a quick view of the most recent system-captured biometric for Heart Health, Activity, Sleep, Stress, Location and more.

Devices

The Devices page lets you add and manage your Skiin devices. The connected devices on this page help capture the biometric data points of heart health, activity, sleep, stress and more vitals. You can see the real-time connectivity and status of your Skiin device here.

NOTE
EN-N8

Your smart device must be connected to the Skiin Pod and the Internet for full functionality.

6.4.2 Getting Help

Navigate to *Menu (User Settings)* > *Help* to access Contact Information for Technical Support and Frequently Asked Questions

Navigate to *Menu (User Settings)* > *About* for information about Software, Terms and Conditions and the Privacy Policy

If you have any questions or feedback, please contact the Skiin Customer support at support@skiin.com 24 hours/day, 7 days/week or drop us a note at <https://skiin.com/pages/contact-us>.

To order more garments, see additional styles or for more information, visit www.skiin.com

6.4.3 System State Messages

The Skiin Garment software provides information to operators when unusual operating conditions occur that affect the *System State*. The type of condition determines the effect on the Skiin Garment System State.

NOTE EN-N9		For Technical Support contact us at: +1 (844) 501-0252 or support@skiin.com
NOTE EN-N10		In the event that an unexpected software error occurs, an error message will be displayed on the App.

Battery Level:  Battery Empty  20%  40%  Battery Full  Charging	Garment Signal Quality:  Good Signal  Noisy Signal  No Signal
Bluetooth Connectivity:  Good  Poor  No Signal	

7. Travelling with Skiin Garments

Flying while wearing a Skiin Garment is straightforward. Simply show the security statement (below) during screening, and if required/or asked to remove the Skiin Pod from the Skiin Garment Dock to disable the Skiin System health data tracking for the time-being of travel. Skiin Pods contain lithium rechargeable battery and should not be included in checked luggages.

Airplane Mode Usage

To prevent the Skiin Pod from transmitting data, remove the Pod from the Garment Dock. Store it safely in your bag or pocket for use after your flight.

Post deboarding the airplane, to reconnect to the Skiin System again simply insert the Skiin Pod in the Garment Dock and make sure you turn on Bluetooth service on your mobile device. Pair the Pod to the Skiin Connected Life App to get connected to the Skiin System again.

Security Screening Statement

This person is wearing a Skiin garment. This garment worn by the user tracks the user's heart rate, body temperature, and other biometric data via the Skiin Pod. The Pod can be removed from the Skiin Garment to stop the transmission of data to their Bluetooth connected mobile device for ease of screening while travelling if required.

8. Limited Warranty

The Skiin Garment has a limited warranty of 30 days (on the date of purchase), and the Skiin Pod and accessories have a limited warranty of 1 year. If during this time the product malfunctions or does not perform as intended the user can request a new product to be shipped to them. Myant will only replace the defective part of the product after it has been returned and found to be defective due to normal use. Myant will not replace any products that have been tampered with or found to have been broken due to user negligence. For information on how to submit a warranty claim please email support@skiin.com.

9. Device Specifications

Performance Characteristics

ECG Channel.....	3 Channels
ECG Dynamic Range.....	+/-400mV
Warranty.....	1year
Frequency Range	0.5 Hz to 40 Hz
CMRR.....	80 - 100 dB
Input Impedance	> 1 GΩ
Sampling Rate	>=320 samples/second
ADC Resolution	24 bits DC
Communication Interface Protocol.....	
Bluetooth Low Energy 4.2	
Operating Frequencies.....	2.4-2.48GHz
Frequency Deviation	2 MHz/channel
Flash Memory.....	8MB

Power Requirements

Battery Type	Rechargeable Lithium Polymer
Battery Capacity.....	200mAh, 2-3 days of use
Battery life	minimum 24 Hours
Operational Time, 1 year warranty	

Physical Characteristics

Skiin Pod	12.0 grams +/- 0.5g 27.9x46.5x10.6mm
Skiin Underwear	Men's and
Women's sizes range from 24" to 47"	

Environmental Specifications

Operational Temperature5 to 40°C
Operational Humidity15% to 90%, non-condensing, water vapour partial pressure <50hPa
Operational Altitude.....700hPa to 1060hPa
Storage Humidity15% to 90%, non-condensing, water vapour partial pressure <50hPa
Storage Temperature.....-10°C to 45°C (for 1 month) or -10°C to 35°C (for 6 months), -10°C to 28°C (over 6 months)

Ingress Protection Marking

Skiin Pod is IP22 rated, which means it is protected from touch by fingers and objects greater than 12 millimeters. It is also protected from water spray less than 15 degrees from vertical. Skiin Pod is compliant with standard IEC60601-1-11:2015.

Expected Service Life

The expected service life for the Skiin Pod is 2 years and 50 washes each pair of Skiin Underwear .

Warm up time

There is no warm up time but for optimal performance it is recommended that the user Applies moisturizer to the skin location where textile electrodes contact prior to use.

User Interface

Five textile electrodes are exposed on the inside of the band of the Skiin Underwear. These electrodes make contact with the user's skin. The Skiin Underwear Electrodes may be cold washed and tumble dried low 50 times without noticeable performance degradation.

10. Symbols Glossary

The tables in the following sections contain brief explanations of the safety symbols and other information used to label the Skiin Underwear System, its accessories and consumables.

Warnings, Cautions, and Notes are used to describe serious and non-serious system conditions. The symbols used to describe these safety conditions are as follows:

WARNING EN-W#		Messages with this heading indicate serious adverse reactions, potential safety hazards, and limitations in use imposed by a condition. The warning will identify steps that should be taken if the incident occurs.
CAUTION EN-C-#		Messages with this heading indicate information regarding any special care to be exercised by the operator and/or user for the safe and effective use of the device and to prevent damage. All caution statements should be followed to ensure data and system integrity.
NOTE EN-N#		Messages with this heading provide additional information that will increase the operator's understanding of the operation of the system.

Symbols on the Skiin Underwear System Device Label (Skiin Pod)		
Symbol	Meaning	Reference
MMWWWWW 00001	Serial Number	
2AXVL-UW02	FCC Number	
26604-UW02	IC ID	
	Refer to operating instructions	ISO 15223-1, Clause 5.4.3 IEC 60601-1 symbol 10 in table D.2
	Type BF applied part	IEC 60601-1, Table D.1, Symbol 20 IEC 60417- 5333

Symbols on the Skiin Underwear System Package		
Symbol	Meaning	Reference
	Legal manufacturer	ISO15223-1 symbol 5.1.1
	Device serial number	ISO 15223-1 symbol 5.1.7
	Manufacturer's catalog number	ISO 15223-1 symbol 5.1.6
	Refer to instruction manual	IEC 60601-1 Table D.2, Symbol 10

	Indicates authorization under the FCC Declaration of Conformity procedure.	FCC title 47, Part 18.209 b (symbol shown in 47 CFR 15.19 b)
	Temperature range limitation	ISO 15223-1 symbol 5.3.7
	Do not use the Skiin Underwear system if its packaging has been damaged.	ISO 15223-1 symbol 5.2.8
	Non Sterile	ISO 15223-1, Clause 5.2.7 ISO 7000-2609
	MR Unsafe. Items should not enter the MRI scanner room. Users with MR Unsafe devices should not be scanned.	ASTFM F2503
	MR Unsafe. Items should not enter the MRI scanner room. Users with MR Unsafe devices should not be scanned.	IEC 60601-1-2:2007, Clause 5.1.1
	When disposing of this equipment it must be sent to the appropriate facilities for recovery and recycling.	ANNEX IX of directive 2012/19/EU for WEEE
	Voltage/frequency/current standard For NA (US and Canada): 110-120 VAC 50/60Hz 600VA.	Alternating current is indicated using symbol 1 table D.1 in IEC 60601-1
	The product has been tested for radio testing, medical electrical safety, Medical EMC, hazardous components. The production facility has undergone an inspection.	60601-1-1, 60601-1-11, 60601-2-47

Symbols on the Wall Plug		
	Indicates authorization under the FCC Declaration of Conformity procedure	FCC title 47, Part 18.209 b (symbol shown in 47 CFR 15.19 b)
	UL Classification Marks for Canada and the United States	60950-1
	External Power Supply energy efficiency standard.	U.S. Department of Energy, International Efficiency Marking Protocol
	When disposing of this equipment it must be sent to the appropriate facilities for recovery and recycling.	ANNEX IX of directive 2012/19/ EU for WEEE
	For indoor use only	IEC 60335-2-29
	Class II equipment	IEC 60051-1

Full name of Standard
IEC 60601-1:2012-Ed.3.1 Medical Electrical Equipment Part 1: General requirement for basic safety and essential performance
IEC 60601-1-2:2014-Ed.4.0 Medical Electrical Equipment Part 1: General requirement for basic safety and essential performance, Collateral standard Electromagnetic Compatibility (EMC) Requirement and test
IEC 60601-1-11:2015 Medical Electrical Equipment – General requirements for basic safety and essential performance – Collateral standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
IEC 60601-2-47:2012 © 2016 Medical Electrical Equipment Part 2-47: Particular requirements for the basic safety and essential performance of ambulatory electrocardiographic systems
IEC60601-1-6:2013 Medical Electrical Equipment - Part 1-6: General requirements for basic safety and essential performance - Collateral standard: usability
ISO 10993-1 :2018 Biological evaluation of medical devices part 1: Evaluation and testing with risk management process
ISO 10993-10 :2010 Biological evaluation of medical devices part 10: Evaluation and testing for irritation and skin sensitization

11. Warning

<p>WARNING EN-W7</p> 	<p>Risks for children and individuals requiring assistance. This device is not intended for use by children. This device is not intended for use by individuals with limited physical capacities or limited experience and/or knowledge unless they are supervised by an individual responsible for their safety or receive instructions from the latter as to how the device should be used.</p>
<p>WARNING EN-W8</p> 	<p><u>Contraindications</u> The Skiin Underwear is not for use by:</p> <ul style="list-style-type: none"> • Persons under 18 years of age. • Pacemaker and/or (ICD) implantable cardioverter defibrillator users. • Pregnant women.
<p>WARNING EN-W9</p> 	<p><u>DO NOT</u> expose the device to a magnetic resonance (MR) environment.</p>
<p>WARNING EN-W10</p> 	<p>Do not ingest the Skiin pod, Skiin Charging Kit or Skiin Garment.</p>
<p>WARNING EN-W11</p> 	<p>Warning: If you have an allergic or skin reaction, please discontinue use and contact customer support at support@skiin.com. If necessary, you should seek medical attention.</p>
<p>WARNING EN-W12</p> 	<p>Do not operate the device if the device feels hot to touch.</p>

12. Cautions

<p>EN-C1</p> 	<p>Do not drop the device <i>It is important that the user takes care when handling the Pod. Repeated drops from 1m or over may damage and/or affect the performance of the Pod.</i></p>
<p>EN-C2</p> 	<p>DO NOT use in the presence of flammable anesthetics, drugs or pressurized oxygen.</p>
<p>EN-C3</p> 	<p>The Skiin Pod contains magnets, use with caution around magnetic strips and metallic objects.</p>

13. General Safety Precautions

Explosion Hazard

<p>WARNING EN-W13</p> 	<p>The equipment is not designed to be used in potentially explosive environments. Do not operate the Skiin Underwear System in the presence of flammable liquids or gases.</p>
--	---

Electrical Safety

CAUTION EN-C4		Do not spray or drip any liquid onto the Skiin Underwear System's Pod, as this could affect reliable operation.
CAUTION EN-C5		After a year of use, dispose of the Skiin Pod and Skiin Garments according to policies set out by your local governing authority.
WARNING EN-W14		Do not use the Skiin Pod, or Skiin Charging Kit or Skiin Underwear if they are damaged in any way.
WARNING EN-W15		Do not put your finger into the Skiin Charger.
WARNING EN-W16		Do not operate the device while wet or moist.
WARNING EN-W17		Do not hold the Skiin Pod, or Skiin Charging Kit directly to skin for extended periods of time.

14. Other

FCC Regulatory Statements

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference.
(2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the Federal Communication Commission (FCC) rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment causes harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by doing one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE: THE GRANTEE IS NOT RESPONSIBLE FOR ANY CHANGES OR MODIFICATIONS NOT EXPRESSLY APPROVED BY THE PARTY RESPONSIBLE FOR COMPLIANCE. SUCH MODIFICATIONS COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.

RF Exposure Warning

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

ISED Regulatory Statements

This device complies with ISED Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must

accept any interference, including interference that may cause undesired operation of the device.

CAN ICES-3 (B)/NMB-3(B)

RF Exposure Information

This equipment complies with ISED RSS-102 radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Table 1 – Emission Class and Group Compliance

The Skiin Underwear is intended for use in the electromagnetic environment specified below. The customer or the user of the Skiin Underwear should assure that it is used in such an environment.		
Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	The Skiin Underwear uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	The Skiin Underwear is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Complies	

Table 2 – Immunity Test Level Compliance

The ME EQUIPMENT is intended for use in the electromagnetic environment specified below. The customer or the user of the ME EQUIPMENT should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV Contact ±15 kV Air	±8 kV Contact ±15 kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient/Burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5 kV, ±1 kV line to line	±0.5 kV, ±1 kV line to line	Mains power quality should be that of a typical commercial or hospital environment.

<p>Voltage Dips, short interruptions and voltage variations on power supply input lines</p> <p>IEC 61000-4-11</p>	<p>0% U_T (100% dip in U_T) for 0,5 cycle</p> <p>0% U_T (100% dip in U_T) for 1 cycles</p> <p>70% U_T (30% dip in U_T) for 25 cycles</p> <p>0% U_T (100% dip in U_T) for 5 sec</p>	<p>0% U_T (100% dip in U_T) for 0,5 cycle</p> <p>0% UT (100% dip in U_T) for 1 cycles</p> <p>70% UT (30% dip in U_T) for 25 cycles</p> <p>0% UT (100% dip in U_T) for 5 sec</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the ME EQUIPMENT requires continued operation during power mains interruptions, it is recommended that the ME EQUIPMENT be powered from an uninterruptible power supply or a battery.</p>
<p>Power Frequency Magnetic Field (50/60 Hz)</p> <p>IEC 61000-4-8</p>	<p>30 A/m</p>	<p>30 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>
<p>NOTE UT is the a.c. mains voltage prior to application of the test level.</p>			

Table 3 – Immunity Test Level Compliance

The ME EQUIPMENT is intended for use in the electromagnetic environment specified below. The customer or the user of the ME EQUIPMENT should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the ME EQUIPMENT including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d=1.2\sqrt{P}$ $d=1.2\sqrt{P}$ 80 MHz to 800 MHz $d=2.3\sqrt{P}$ 800 MHz to 2.7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a should be less than the compliance level in each frequency range ^b
Radiated RF IEC 61000-4-3	6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz	6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz	
	3 V/m 80 MHz to 2,7 GHz	3 V/m 80 MHz to 2,7 GHz	
	RF communication equipment inside 80 MHz to 6 GHz	RF communication equipment inside 80 MHz to 6 GHz	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ME EQUIPMENT is used exceeds the applicable RF compliance level above, the ME EQUIPMENT should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the ME EQUIPMENT b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Table 4 – Recommended Separation Distances

Recommended separation distances between portable and mobile RF communications equipment and the ME EQUIPMENT			
The ME EQUIPMENT is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the ME EQUIPMENT can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the ME EQUIPMENT as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter w	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 2.7 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.24
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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