1. Safety Precautions Storage Use 2. Introduction to BULO **Product Contents** List of Parts Details on the LED Signal Light 3. Instructions to Features Turning on the Device Connecting the Bluetooth Charging 4. Information for Optimum Use 5. Troubleshooting 6. Product Specifications 7. Product Storage and Maintenance --- 13 Transport Cleanliness Mouthpiece Usage Period 2 3. Instructions to Features Turning on the Device Press the button once to turn on the device. Connecting the Bluetooth 1. Make sure that the device is turned on. 2. Install the correlated smartphone app. 3. Follow the smartphone app instructions and press the button once to connect. 4. When connected, the LED signal light will turn blue. (The details of the LED signal light are included in the section "Introduction

to BULO")

to charge it.

to BULO")

1. Connect enclosed USB charging cable

2. The LED signal light will turn on when

(The details of the charging status are

3. When done charging, separate the USB

included in the explanation about the

LED signal light under the "Introduction

charging cable and device and store each

8

connection is successful.

in the optimum fashion.

to the C-Type charging port on the device

Charging

Table of Contents

4. Information for Optimal Use

1. Safety Precautions

objects.

of the product.

1. Avoid direct sunlight or areas close to

flammable spray or other flammable

2. Do not place heavy objects on top

3. When charging the product, please

4. When charging the product, install

can be operated easily.

in areas with radiation.

direct sunlight.

the product in an area where the plug

· There is a danger of an explosion or fire.

5. For optimum performance, avoid storage

6. After each use, place the product in the

enclosed pouch and store away from

· Foreign substances can cause malfunctions.

avoid plugging it into an outlet that

heating devices. Do not store or use near

· There is a danger of an explosion or fire.

· There is a danger of an explosion or fire.

already has multiple electronics plugged in.

· There is a danger of an explosion or fire.

- 1. Use after checking the connection status through the LED signal light. 2. Use after checking that the mouthpiece
- and device are properly assembled.
- 3. Use after checking the battery status. (Battery status can be checked through the device or connected app with the touch of a button. If the device needs to be charged, the LED signal light will turn red. In this case, use the enclosed cable to charge the device.)
- 4. Use after separating the device from the USB charging cable.
- 5. Place the mouthpiece in the mouth during use to avoid wind caused by breathing.
- 6. After each use, place the device in the enclosed pouch and store away.
- 7. After disconnecting the bluetooth, after a given time.
- 8. Do not move the device while charging.

5. Troubleshooting

with dry hands.

1. Safety Precautions

electric shocks.

electric shocks.

or drink.

cable.

shocks.

1. Except for the mouthpiece, keep the

product away from any contact with liquids.

This is to avoid the possibility of fires or

2. In case of smoke or strange smells, stop

use of the product immediately. Quickly

from the charging cable, call the service

This is to avoid the possibility of fires or

3. Separate product from charging cable

when there is lightning and thunder.

4. Do not use shortly after ingesting food

5. Do not attempt to dismantle, repair or

· Fires, electric shock or injuries can be

6. Do not heat, bend, twist, inscribe on or

damage the charging cable in any way. · This is to avoid the possibility of fires or electric shocks caused by a damaged

7. Handle and hold the charging cable only

· This is to avoid the possibility of electric

· May affect sensor performance.

reconstruct the product.

caused by malfunctions.

· This is to avoid causing a fire.

turn off the power and after separating

Before reporting a problem, please check the items listed below.

Issue	Measure
The device isn't turning on.	· Use the USB charging cable to charge the device.
The Bluetooth isn't connecting.	· Make sure that the smartphone can use Bluetooth.
	Restart the smartphone's Bluetooth function and try connecting to the Bluetooth again.
	· Restart the device and try connecting again.
	· Clean up any possible obstructions between the smartphone and the device.
	1

5. Troubleshooting

1. Safety Precautions

electric shocks.

the product.

own discretion.

occur.

8. The charging cable must be removed

· This is due to the possibility of fires or

9. Contact the service center when you have

any complications or inquiries about

compensation if they attempted to

10. Only use the product in accordance

· If the product is used for any purpose

is not responsible for any damage,

besides health maintenance, the company

compensation or legal issues that may

with health maintenance.

disassemble or repair the product at their

· Customers will not receive any

from the product after charging.

please check the items listed below.

Issue	Measure
The device isn't operating well.	· Use the device in a proper area that is not in polar extremities of temperature or humidity.
	·Use the device indoors avoiding wind.
	· Use the device in areas that cannot be reached by wind from fans, air conditioning, open windows or air purifiers.

6. Product Specifications

2. Introduction to BULO

Mouthpiece

Charging USB Cable

1

1 LED Signal Light

3 Removable Handle

4 USB Charging Port

List of Parts

Enclosed Pouch

BULO

Instructions Manual


2 Button

Product Contents

Model	BULO-H01
Input	DC 5V, 2A
Output	3.7V, 40mA
Weight	66g
Dimensions (W x H x D)	140 x 45 x 37(mm)
Materials	PC/ABS, PP
Communications	Bluetooth 5
Battery	Internal Li-PO Battery (300mAh)
Charging Type	USB C-Type
Range	10L/M ~ 840L/M
Recommended Temperature	5°C ~ 45°C

Details on the LED Signal Light

2. Introduction to BULO

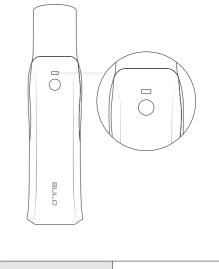

Red
Green
Flickering Blue
Blue
Bright green / Red (Under 20%)
Power off after flickering red twice

This product is a non-medical device FCC ID is on the rear view of the body and it can be viewable

1. When the product is purchased, it will be

Transport

placed in a packaging box and transported to the customer. Cleanliness

7. Product Storage and Maintenance

1. Use a non-abrasive lint-free dry cloth

- to clean the device.
- 2. Avoid using any liquid or chemical products when cleaning this device as doing so can result in deterioration or other complications. Only use dry cloths when cleaning.
- 3. The mouthpiece can be cleaned with water but avoid using any detergent, soap or other chemical products.
- 4. Boiling or hot water can cause deterioration or morphing so only use cold water to clean the mouthpiece.
- 5. Pay attention to avoid damaging the filter when cleaning the mouthpiece.

Mouthpiece Usage Period

- 1. We recommend cleaning the mouthpiece and changing it regularly to maintain personal hygiene as the enclosed mouthpiece is only semi-permanent.
- 2. If the connecting portion of the mouthpiece is separated or the filter is damaged, foreign objects may enter the mouth during calibration. In this case, change to a new mouthpiece immediately.
- we recommend keeping your mouth piece to yourself without sharing with others.

FCC ID · 2AXY8-BUI O-H01 FCC Compliance Statement (for products subject to part 15) This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions (1) This device may not cause harmful interference, and (2) this de must accept any interference received, including interference that may cause undesired operation

(Part 15 Subpart A, 15.21)

of the FCC Rules.

The users manual or instruction manual for an intentional or unintentional radiator shall caution the user that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. (Part 15 Subpart B. 15.105) NOTE: This equipment has been tested and found to comply

with the limits for a Class B Digital Device, pursuant to Part 15

Which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or

These limits are designed to provide reasonable protection

against harmful interference in a residential installation This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instruction, may cause harmful interference to radio communication However, there is no guarantee that interference will not occur If this equipment does cause harmful interference to radio

more of the following measures : Reorient or relocate the receiving antenna. - Increase the separation between the equipment and receiver - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

- Consult the dealer or an experienced radio/TV technician for help.

or television reception

Telephone: (+82) 2-6085-1105 Homepage: https://www.breathings.co.kr


Address: #815 Gieobdosi-ro 200,

Jijeong-myeon, Wonju-si, Gangwon-do, South Korea

and should be used appropriately.

by removing the mouthpiece without any tool.

Instructions Manual

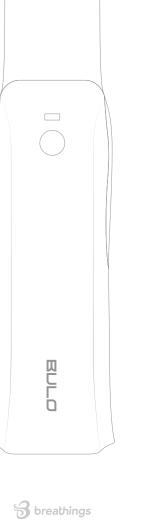

the device will automatically turn off

turning on.	cable to charge the device.
The Bluetooth isn't connecting.	· Make sure that the smartphone can use Bluetooth.
	· Restart the smartphone's Bluetooth function and try connecting to the Bluetooth again.
	· Restart the device and try connecting again.
	·Clean up any possible obstructions between the smartphone and the device.
The device isn't operating normally	· Push the reset button (small hole) located besides the serial number on the bottomof the device with a pin.

Before reporting a problem,

Issue	Measure
The device isn't operating well.	· Use the device in a proper area that is not in polar extremities of temperature or humidity.
	· Use the device indoors avoiding wind.
	· Use the device in areas that cannot be reached by wind from fans, air conditioning, open windows or air purifiers.

to the surroundings and the type of Bluetooth device.

* The Bluetooth may not connect well due

10

11

12

3. To protect users from infections,

13