



Tap-2-Pair™

Posey®

Single Patient Use 30-Day Wireless Chair Sensor Pad for use with Sitter On Cue PRO™

FIRST USE DATE:

____ / ____ / ____


EXPIRATION DATE:

____ / ____ / ____

USE THIS SIDE UP

1. Pair

Locate and align pairing
icons together

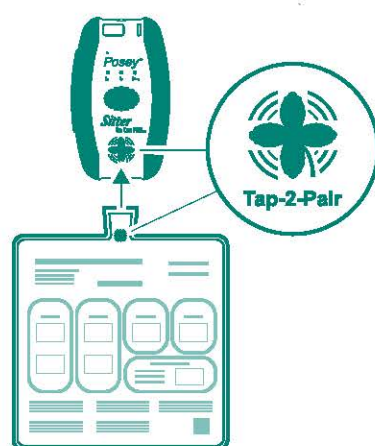


Blue light means paired




2. Place

Place green grip,
then white pad

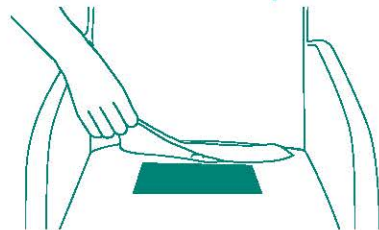


Position pad with neck facing
back of chair or wheelchair

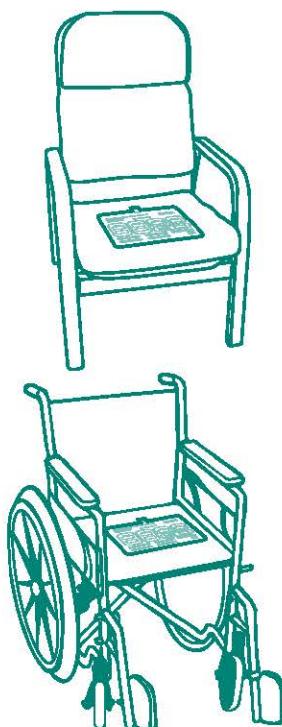


3. Test

Press pad and watch status light
Green means go

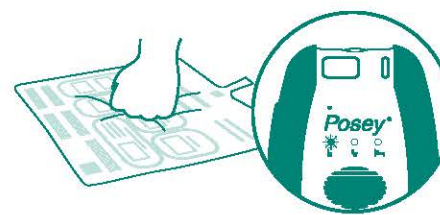

Release pad and listen
for alarm

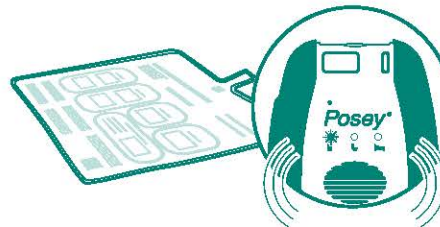

4. Monitor

Place patient on pad
Green means go
Blue light means paired



Optional: Nurse Call Adapter

Locate and align pairing
icons together

Nurse Call Adapter blue light
means paired

Plug into facility's nurse call
system port



Unpair

- To replace sensor pad, press HOLD button to pause monitoring. Repeat steps 1-4.
- To unpair inactive sensor pad, select from these options:
 - Press and hold the MODE button
 - Turn off the fall monitor
 - Move the sensor pad outside the range of the fall monitor

Cleaning

Dampen (but DO NOT soak) a clean cloth with disinfectant. NEVER use any cleaning substance that contains Phenol, immerse in liquid, or sterilize with heat. Use a clean, DRY cloth to dry all parts.

Discard

Utilize electronic recycling programs. Sensor contains foam, electronic components, and a lithium battery. Consult Instructions for Use for additional discarding information.



TIDI Products, LLC • 570 Enterprise Drive • Neenah, WI 54956 USA • tidiproductions.com
Tel 800.447.6739 • Fax 800.767.3933
MADE IN MEXICO

WARNING

- Use only with chairs or wheelchairs
- Use only with Posey Sitter On Cue PRO Fall Monitor
- Ensure all parts of the system are operational before leaving patient unattended.
- This system does not prevent falls or injury from falls and is not a substitute for patient care.
- This product should be used in conjunction with your facility's falls management program.
- Never crease sensor pad. This will damage sensor pad and may cause false alarms or no alarm.

WARRANTY USEFUL LIFE:

Thirty (30) days from date of first use.

FCC ID: 2AZB4-8309WL

This device complies with part 15 of FCC Rules. Operation is subject to the following two conditions:

This device may not cause harmful interference, and this device must accept any interference received, including interference that may cause undesired operation.

L9608M Rev A 06142021



Caution: Consult
Instructions for Use

www.tap-2-pair.com

For more information please visit
WWW.TIDIPRODUCTS.COM
May be patented: see
www.tidiproductions.com/virtual-patent-marking

REF 8309WL - 30-Day Chair Pad, Wireless

TIDI
Support Caregivers.
Protect Patients.