




Instruction

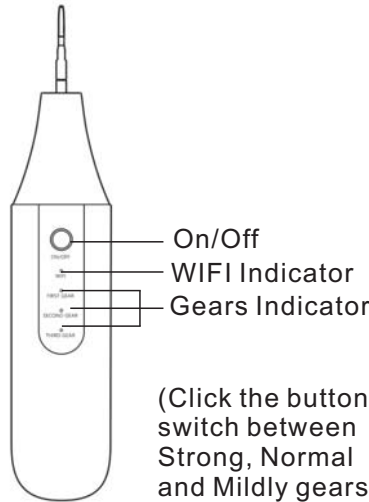

1. Scan the QR code below, download and install the App.



(Scan the QR code above to download and install)

2. Hold down On/Off Button for 1 second to turn on the device and WIFI Indicator flashes.
3. Confirm that WIFI of your smart phone or tablet PC have been turned on. Then return to the main interface and click Start to enter the application.



4. Open the App named WIFI ENDOCAM. Click the green arrow shape on the right to enter the WIFI selection page and select the WIFI named T03-PVx-xxxx to connect; After confirming the WIFI connection is successful, please return to the interface and click "Start" to enter the application;
5. If the green WIFI indicator light stops flashing, it indicates that the WIFI connection is successful. You can start to use it; If the green WIFI indicator light still flashing, it indicates that the WIFI connection is failed. You need to restart the device and reconnect with the APP.

FAQs

1. Is gum bleeding normal during use this product? How to deal with it?
If someone who originally suffer from periodontal or gum disease brushes teeth or bits hard food, it will easily cause gum bleeding. The gums of these patients are likely to bleed as well when they use ultrasonic dental scaler to blast dental tartar or calculus; If the gum is bleeding, just stop using it and try gargling with cold water.
2. What is going on when the device is turned off? It is indicated that the battery power is too low when the device cannot start or the three gear indicator flashes. Please use it after charging.

Cautions

1. Please don't wash this product with water. Please wipe it with a damp cloth if it is dirty.
2. It is normal that the temperature of the product will rise slightly during use or charging.
3. Please keep this product out of the reach of children.
4. Please don't expose this product to sunlight for long time and don't place it in a humid environment.

5. Please don't disassemble this product without authorization to prevent electric shock or battery short circuit and you are responsible for the losses caused by this.
6. If you don't use it for a long time, please take it out and charge it once every two months.
7. Please don't use it in the following conditions.
 - a. Patients with dental diseases such as periodontitis and gingivitis.
 - b. During gum bleeding.
 - c. During dental treatment such as fillings and dental prosthesis.
 - d. Patients with oral diseases.
 - e. Patients with cardiovascular and cerebrovascular diseases.
 - f. Patients with pacemakers and heart-brain bypass surgery.

Package Contents

- 1*Ultrasonic Dental Scaler
- 1*Base Holder
- 1*USB Charging Cable
- 1*User Manual
- 1*Cleaning Supplies

Product Specification

Charging Current: 500mA
 Charging Time: 2 Hours
 Working Frequency: 35~45KHZ
 Product Weight: 370g
 Product Size: 200*51*51mm

Handy Tips

1. For security purposes, there is only a small amount of electricity in the battery when the product leaving our factory. Because the device is not fully charged when you receive it, it needs to be charged before use.
2. This product will automatically shut down after ten minutes of inactivity. If you want to use it again, you need to restart the device and the App.
3. If the Gear Indicators flashes, it means the battery is low. Please charge it in time.
4. Please wipe ultrasonic dental scaler and store it after use. We strongly recommended that the stainless steel scraper should be disinfected with alcohol and then wiped it dry for next use.

FCC Caution:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.