Product Specifications

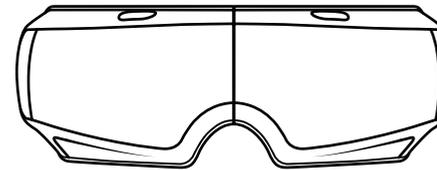
Product Name		Product Model	MLS-RY02
Product Power	5W	Charging Voltage	5V $\overline{=}$
Operating Voltage	3.7V $\overline{=}$		
Lithium Battery Capacity	1200mAh		
Compliance Standards	GB 4706.1-2005/GB 4706.10-2008		

Nantong Manlaishi Electronic Technology Co., Ltd.
Building 3, 2nd Floor, 30 Binshui Road, Development Zone, Nantong City
Website: www.manlaishi.com
Phone: 400-1087-178

EYE MASSAGER
MLS-RY02

User Manual

Please read this manual carefully before using the product and keep it for future reference.

4.Maintenance

Daily maintenance

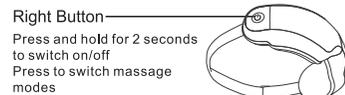
1. In order to avoid affecting the effect of eye moisturizing, add distilled water or purified water, do not add oily, sticky or toxic and harmful liquid in case of product and user harm.
2. Do not wash with water. If there is dirt, wipe it on the surface of the product.
3. Please keep the fog hole clean. Do not use sharp objects to clean the fog hole to prevent damage to the fog hole. Use cotton swabs to clean.
4. Please keep it properly if it is not used for a long time.

Q&A


1. Failed to start up. Please confirm the power or long touch the power button.
2. Can't spray, please add professional eye drops.
3. Can't charge, Check whether the charging cable and charger are properly connected.

12

6. The indicator on the right side is blue when working normally.
7. Warm tip: The product will automatically shut down after running for 15 minutes. If you need to extend it, please turn it on again.
8. Bluetooth: The first time using the Bluetooth, search for Bluetooth device "runyananmoyi" by your phone and connect it; Next time using the Bluetooth, the product will connect your mobile phone automatically if is in the range. Warm tip: If using the Bluetooth, the music that comes with the product will not play.

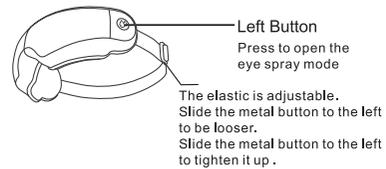
Right Button
Press and hold for 2 seconds to switch on/off
Press to switch massage modes


Left Button
Press to open the eye spray mode
The elastic is adjustable. Slide the metal button to the left to be looser. Slide the metal button to the left to tighten it up.

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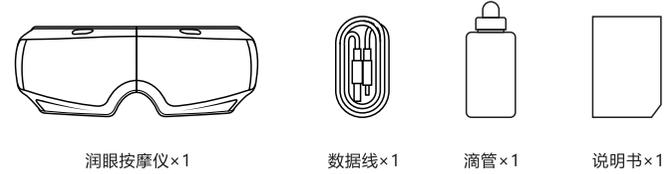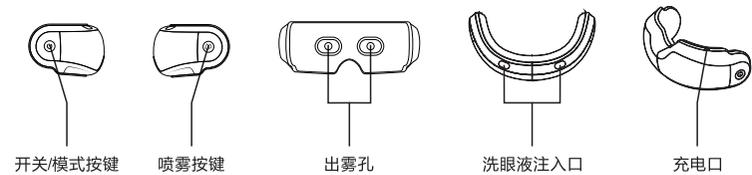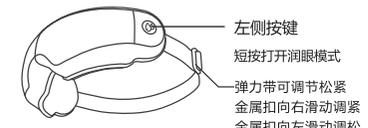
2

3.Warning

1. This product is a personal care eye massage instrument and cannot be used as a substitute for medical instrument or medicine.
2. Do not use the product while driving or walking.
3. Use with caution in patients with glaucoma, cataract, fundus bleeding, etc.
4. The box is washable; Do not inject harmful liquid into the container to avoid eye damage. Use with caution if you have undergone eye or nose surgery or have plastic materials placed on your face.
5. Please stop immediately if you feel uncomfortable during use.
6. Do not disassemble the product without permission.
7. There are lithium batteries in the product, please do not expose to high temperature environment.
8. For diabetics, please use it under the guidance of a doctor.

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4

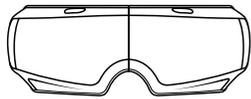
2.Use instruction

When using the product for the first time, or the battery power loss after a long time, please charge to ensure normal use.

1. The product is controlled by touching "⦿" on the right side, and it will start up for 2 seconds after long touching, and shut down for 2 seconds after long touching.
 2. Press the left "⦿" button to enable or disable the spray function. Distilled water or purified water is recommended to add in.
 3. Press the "⦿" button on the right to switch eye health mode, massage mode and relax modes. (Pay attention to the voice prompt after pressing the button)
 4. Press "⦿" and hold for 2 seconds to automatically shut down.
 5. When the product is short of power, the red light on the right side will be on, accompanied by voice prompts. The machine will shut down automatically.
- Please use the mobile phone charger and the cable provided with the product to charge the battery. The charging time is about 3 hours, and the indicator light turns green when fully charged.

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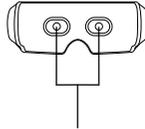
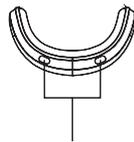
FCC Warning

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.