

EYE MASSAGER

Product specification

Please read the instructions carefully before using the product
This instruction manual contains service guidelines and safe use

S-3

Intelligent eye protection instrument

Product parameter

[Model]	S-3	[Function]	Relieve eye strain
[Rated voltage]	DC5V	[Battery capacity]	1200mAh
[Rated current]	1A	[Product size]	200x 75x 35mm
[Rated power]	5W	[Executive standard]	GB4706.1-2005 GB4706.10-2008.

Product feature

Automatic timing, multi-frequency vibration, intelligent voice broadcasting
Bluetooth music, USB charging, 180° folding portable storage

Prepare and charge before use

Pre-use preparation

Aim the eye protector at your eyes. Put on the machine and adjust the elastic band to the most comfortable size. Adjust the machine up and down left and right to the most comfortable position.

Please remove glasses, contact lenses, hair clips, hair accessories, etc. before wearing. This avoids discomfort when using. Do not use this product with wet hands. Do not fall and tread on this product.

Charging instructions


When the battery is out of power, the voice prompts you to charge it in time.
This means the product needs to be charged (Bluetooth models)

1. Insert the USB plug of the USB data cable into the USB socket of the power adapter
2. Insert the power adapter into the power socket
3. Plug the USB data cable into the Type-C jack of the product

Warm reminder

The product cannot be turned on during the charging process.
(This product is equipped with charging protection program)
The product will automatically stop working when it is charged.
Charging time is about 3 hours. Please disconnect the power adapter after the charging is complete.

Product structure

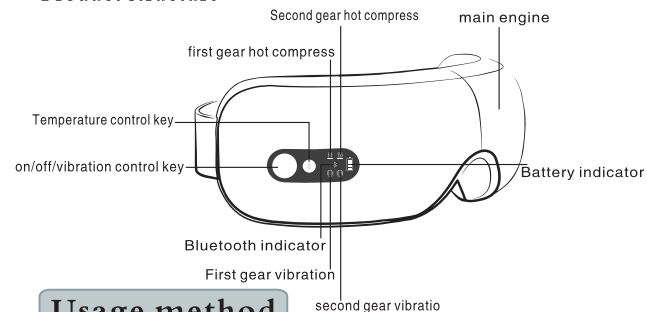

Usage method

In the startup state, double-click the key switch mode. Click the on key to switch the vibration gear.

Open product

Keep pressing the "on key" with your finger for 2 seconds to turn on the machine. After startup, the indicator light is on, the machine starts to work and defaults to automatic mode. The working time is 15 minutes with music.

Close the product

Keep pressing the "on key" with your finger for 2 seconds while the product is working, turn off the machine.

Volume control

Long press the hot Compress button to turn off the volume.
Long press the "hot compress key" again, the volume gradually increases, the order is mute, bass, alto, treble respectively. This pattern is repeated in turn.

Gear control

In mode, the temperature is in second gear by default. Short press the temperature key to cycle successively for third gear hot compress, first gear hot compress and second gear hot compress. Mode mode vibration default is second gear vibration massage. Short press the vibration key in turn cycle for three vibration, first gear vibration, second gear vibratio

Switching mode

Double click “on the key” for less than a second to switch modes. Switch to active mode once. Do it twice and switch to sleep mode. Operate three times and switch back to automatic mode. This patter is repeated in turn

Mode introduction

Automatic mode: Second gear is vibration massage + second gear hot compress + music(temperature and vibration can be adjusted)

Vitality mode: Second gear is vibration massage + music (vibration adjustable)

Sleep mode: Second gear is hot compress + music (temperature adjustable)

Bluetooth connection

After the device is turned on, the Bluetooth of the eye protector is turned off by default. When you need to connect Bluetooth, turn on the Bluetooth of your mobile phone and search for Bluetooth

Name: S-3 (manual connection is OK)

After successful connection, the eye protector can play mobile phone music. When the Bluetooth is disconnected, the device plays built-in meditation music.

Cleaning and storage

Please use a dry soft cloth to clean the product (please turn off the power when cleaning. Do not use benzene or diluent and other corrosive cleaners, otherwise it will cause fading, surface aging, structural damage, etc.). Please store the product in a well-ventilated and dry room. Please cover with protective cover if not used for a long time. If the storage time exceeds 2 months, check and charge the device for 1 hour

Warning This product contains batteries. Do not approach high temperature places or put them into fire.

Matters needing attention

Please read the instructions carefully and understand all of them. Please follow the instructions.

1. Do not use it when there are symptoms of eye disease. It is prohibited for patients with serious eye diseases such as glaucoma, cataract and retinal detachment.
2. Do not use water to clean the product directly. Please wipe with a cotton cloth soaked with alcohol or water.

3. This product has a hot surface. Please be careful when using for people who are not sensitive to heat. Minors, the disabled, the elderly, etc., should be used under adult supervision.
4. Sensitive skin, facial allergic dermatitis, facial plastic materials and such special people. Please use this product under the guidance of your doctor.
5. Please use the national standard power adapter to charge the product and pay attention to safety.
6. Disassembly of the product is prohibited. This product contains batteries, do not place in high temperature environment or put into fire.
7. This product is a personal health care massage product. It is not a substitute for drugs or medical devices.
8. Use of this product will block the line of sight. Do not use it during walking, driving, etc.
9. After using this product, the eyes need to return from a relaxed state to a tense state. Temporary blurry vision is normal. Please close your eyes for 1 to 3 minutes. Your vision will become clear.
10. Due to product hardware and software upgrade, product functions are subject to actual conditions.

Common problems and solutions

Problems encountered	Possible cause	Solution
Automatic shutdown after turning on	Low battery power	Charge
Unable to turn on	Low battery power	Charge
Low hot compress temperature	Low battery power	Charge
The charging light is off	The USB port is improperly connected or the power supply is improperly connected	Plug in the USB charging cable/power cable

If the above processing, still abnormal phenomenon, please turn off the power and contact the company. Do not disassemble or repair the product without authorization. The company shall not be responsible for any product damage, personal injury or other loss caused by this.

Product function and description:

declaration

The company manual mainly introduces the luxury Bluetooth version of the configuration, the same model upgrade version is no voice broadcast. Please know the built-in music has Bluetooth music function.

Upgrade payment:

Automatic mode: second gear is vibration massage + second gear hot compress (temperature and vibration can be adjusted)

Vitality Mode: Second gear is vibration massage (adjustable vibration)

Sleep mode: Second gear is hot compress (adjustable temperature)

Luxury model

Automatic mode: Second gear is vibration massage + second gear hot compress + music (temperature and vibration can be adjusted)

Vitality mode: Second gear is vibration massage + music (vibration adjustable)

Sleep mode: Second gear is hot compress + music (temperature adjustable)

If you have any questions, please contact our customer service or business personnel

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Radiation Exposure statement

The device has been evaluatec to meel general RF exposure requirement. The device can be used in porlable exposure condition without restriction.