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---

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## 01 Introduction to EX-MODE

1. Product Function, Components .....	21
2. Product Name, Operation Method .....	22
3. Product Specifications .....	23

## 02 EX-MODE Usage Method

1. APP Download .....	25
2. APP Usage Method .....	26
3. Exercise Method .....	28

## 03 Precautions and Warranty

1. Precautions for Use .....	31
2. Warranty Regulations .....	32
3. Product Warranty .....	33
4. Manufacturer Information .....	34

# 01 Introduction to EX-MODE


---

1. Product Function, Components .....	21
2. Product Name, Operation Method .....	22
3. Product Specifications .....	23

# Introduction to EX-MODE

## 1 Product Function, Components

The EX-MODE is a personal Kegel exercise device that induces contraction and relaxation of pelvic floor muscles to assist in self-exercise. It measures the contraction/relaxation status of the pelvic floor muscles through the dedicated app and allows the recording of daily, weekly, and monthly exercise amounts, enabling consistent exercise.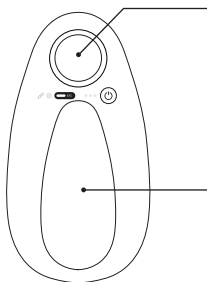


### 1) Massage Mode

Perineum massage mode stimulates blood circulation in the perineal area

### 2) Exercise Mode


Kegel exercise mode that relaxes the glute muscles by contracting/relaxing muscles

## [Components]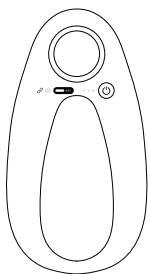

Device 1EA


Color Options  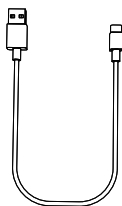


Charging Cable 1EA  
C-TYPE

# Introduction to EX-MODE

## 2 Product Name, Operation Method

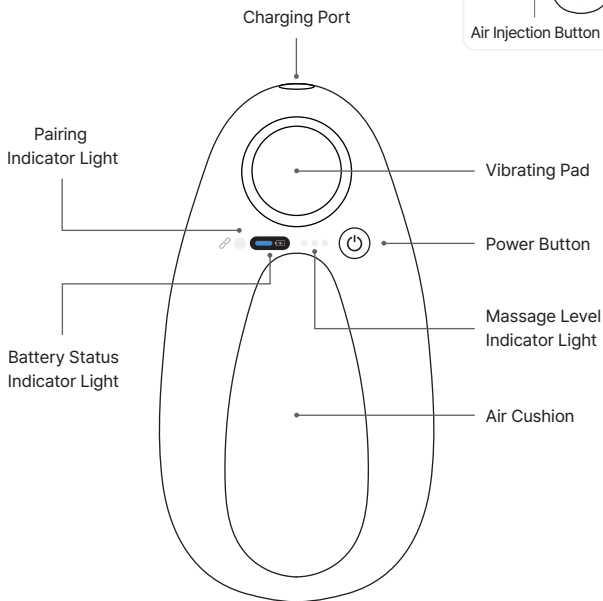### [Button Operation Method]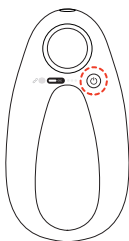

#### Device power ON/OFF

- Device power ON : Short Press
- Device power OFF : Long Press

#### Setting massage level(vibration)

When the power is turned ON, press briefly to change the level.

- 1st level vibration : ●
- 2nd level vibration : ● ●
- 3rd level vibration : ● ● ●

### 3 Product Specifications

This product's specifications may be subject to change without separate notice for performance improvement.

Model Name	EXMODE		
Product Name	Exercise care device		
Size	W170 x H24 x D87 mm		
Weight	160g	Material	ABS
Input	DC 5V2A	Battery	500mAh
Charging Cable	C-TYPE	Charging Time	About 120 minutes
Usage Time	About 180 to 300 minutes (Depends on settings)		
Manufacturer/Seller	JLU Co., Ltd.		
Date of Manufacture	Separate Notation		
Address	410~412 office, Busan Knowledge Industry Center, 111, Hyoyeol-ro, Buk-gu, Busan, Republic of Korea		
Customer Center	+82 51-331-7185		
A/S Center	+82 10-4505-7989		

#### [Battery Status]

FULL

more than 90%

HIGH

between 70% and 90%

MID

between 50% and 70%

LOW

between 30% and 50%

## 02 EX-MODE Usage Method

---

1. APP Download .....	25
2. APP Usage Method .....	26
3. Exercise Method .....	28

# EX-MODE Usage Method


## 1 APP Download

Install the EX-MODE dedicated app to measure the contraction/relaxation status of the pelvic floor muscles and record daily, weekly, and monthly exercise amounts, enabling consistent exercise. Install 'EX-MODE' from the application store/market and start exercising.


\*The UI system may vary depending on the smartphone model (Android, iOS).

\*The APP environment may change without notice to improve usability and performance.

### 1) Install 'EX-MODE' from the Play Store·One Store·App Store.

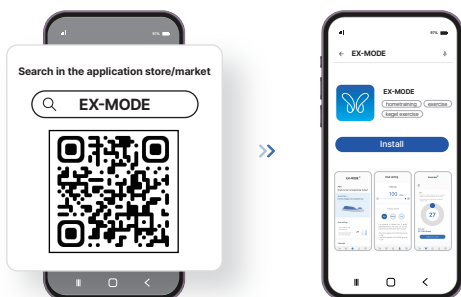
### 2) Run the app after the installation is complete.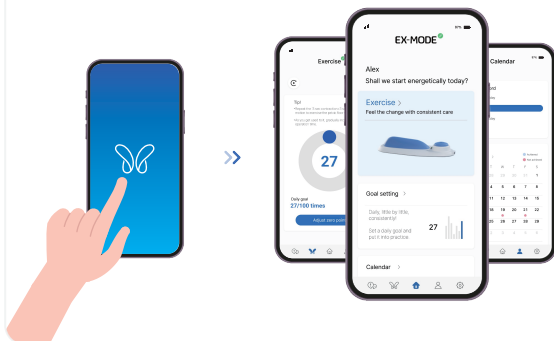


# EX-MODE Usage Method

## 2 APP Usage Method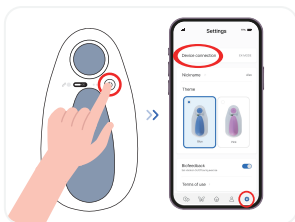

### STEP 01

First, turn on the device, run the EX-MODE app, and then pair it by going to 'Settings > Device connection'.

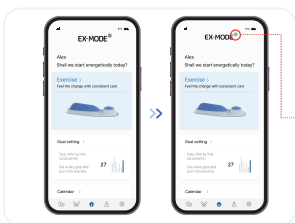
### STEP 02

When the device is connected, the top indicator light will turn green.


● Connected ● Disconnected

\*If there is a connection error, search for the device in 'Settings > Device connection' and pair it.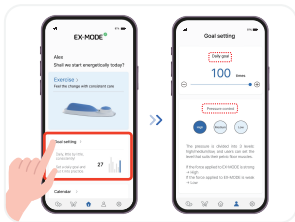

### STEP 03


Before starting exercise, set the daily goal in 'Goal setting.'  
(Can select pressure levels: high/medium/low)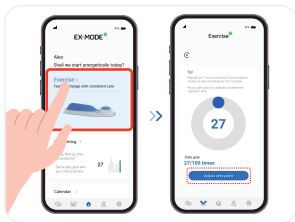

### STEP 04


Adjust the zero point in 'Exercise' before starting the exercise.

## 2 APP Usage Method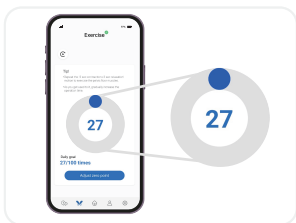


### STEP 05

In the 'Exercise' follow the daily goal and perform contraction/relaxation exercise.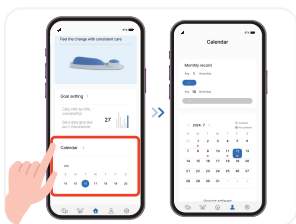


### STEP 06

You can check your monthly records and goal achievement rate in 'Calendar'.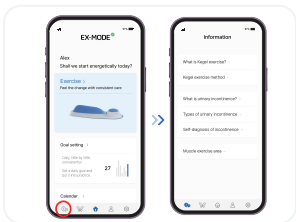

### [Information]

In the 'Info' tab of the bottom bar, you can find information on Kegel exercise methods, targeted muscles, and more.

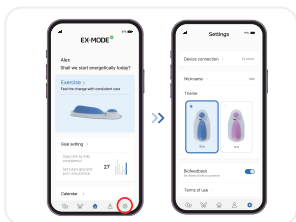
### [Settings]

In the 'Settings' tab of the bottom bar, you can connect equipment, change the theme, and set your nickname.

## EX-MODE Usage Method


### 3 Exercise Method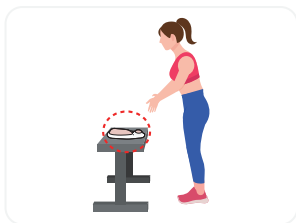

#### STEP 01

Place the device on the hard chair


\*Soft or uneven chairs can prevent accurate exercise of the pelvic floor muscles.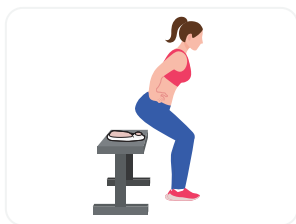

#### STEP 02

Sit on the device with the clothes on and your back straight.


\*Avoid thick materials such as jeans and wear thin clothing.

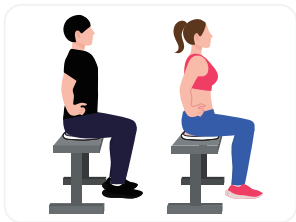#### STEP 03

Place the vibration pad facing the front and the air cushion facing the back.

\*For men, the air cushion is between the scrotum and anus. For women, the air cushion is between the vulva and anus.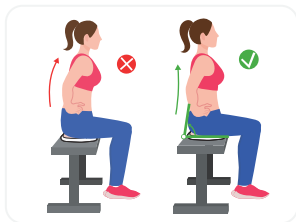

#### STEP 04

In a comfortable sitting position, tilt your pelvis slightly forward to maintain proper posture.

\*Correct posture leads to optimal contraction of the pelvic floor muscles.

### 3 Exercise Method

#### STEP 05

Exercise the pelvic floor muscles by repeating the '3 seconds contraction > 3 seconds relaxation' motion while looking at the 'Exercise' screen in the app.

#### Posture Tip!

\*Exercise with correct posture to achieve optimal contraction of the pelvic floor muscles.

#### [Incorrect Postures]

- Slouching posture
- Excessive movement of arms and legs during pelvic floor muscle training
- Excessive contraction of abdominal muscles, thigh muscles, buttock muscles

Incorrect muscles and posture prevent effective pelvic floor exercises.

#### Exercise Tip!

\*Try using imagination for more effective exercises.

#### [Imagination Therapy]

01

Imagine holding in a fart while in an elevator.

02

Imagine sucking up through a straw from your perineum.

03

Imagine pulling tissue up through your perineum.

## **03** Precautions and Warranty

---

1. Precautions for Use .....	31
2. Warranty Regulations .....	32
3. Product Warranty .....	33
4. Manufacturer Information .....	34

# Precautions and Warranty

## 1 Precautions for Use

For safe and correct use, please be sure to read the user manual before using the product.

### CAUTION

---

1. This product is not a medical equipment.
2. Please keep out of reach of children.
3. Not intended for use by individuals with reduced physical-sensory, or mental capabilities(including children) or those lacking experience and knowledge, unless supervised or given instructions on device use by a responsible person.
4. Adjust the amount of exercise according to your condition, and stop exercising if you feel uncomfortable.
5. Please use it while wearing clothes.  
Avoid thick materials such as jeans and wear thin clothing.
6. Use the product with the air cushion fully inflated.  
If the air cushion feels deflated and loses its elasticity,  
press the air injection button on the back to use it with sufficient air injected.
7. If a detachable power supply is provided, only use the device with the power supply provided with it.
8. Do not modify or disassemble-repair the product. If the user modifies·disassembles·repairs the product, free A/S service cannot be provided within the warranty period, therefore be sure to use the designated A/S center.
9. Do not wash or immerse the product in water.  
Be careful as it may cause electric shock-malfunction.
10. Be careful as the strong impact or drop of the product can cause malfunction.
11. If the product becomes dirty, wipe it with a soft cloth or tissue.  
If necessary, use a small amount of alcohol on a soft cloth or tissue.
12. If a malfunction or accident occurs during use, immediately stop using the product and contact customer service.
13. We are not responsible for any malfunction, damage to the product, or injury caused by user negligence.

## Precautions and Warranty

### 2 Warranty Regulations

**\*Note: The product warranty period is 1 year from the date of manufacture.**

These warranty regulations apply only to the scope of the warranty period and conditions provided by our company. For A/S service, please contact the place of purchase or the A/S center.

#### Within the warranty period

Free maintenance within 1 year from the day of production  
(product registration is required)

During the warranty period, if a problem occurs because of the production process of the equipment, free A/S service can be provided.

#### After the warranty period

Paid A/S service can be provided for products that are not discontinued after the warranty period, and A/S service may not be available for discontinued products.

#### Exceptions

In the following cases, paid A/S service is provided even within the warranty period.

- 1) Lost or stolen
- 2) Damage caused by user negligence
- 3) User arbitrarily modified, disassembled, or repaired by unauthorized personnel
- 4) Improper storage condition
- 5) Damage caused by natural disasters (fire, salt damage, flood damage, etc.)

### 3 Product Warranty

## Product Warranty

[EX-MODE]

Thank you for purchasing JLU products.

This product is a genuine product manufactured by the headquarters and shipped after passing strict quality control and inspection processes.

Free service within the warranty period is available if you complete the product registration through the homepage.

Please access the QR code and proceed with product registration.

## Precautions and Warranty

### 4 Manufacturer Information

#### JLU Co., Ltd.

410~412 office, Busan Knowledge Industry Center, 111, Hyoyeol-ro, Buk-gu, Busan, Republic of Korea

 +82 51-331-7185

 +82 10-3752-8240 (EN line)

 +82 10-4505-7989

 [jlukorea@naver.com](mailto:jlukorea@naver.com)

 [jlu\\_korea\\_official](#)

 [JLU Co.,Ltd](#)

 [jlu\\_korea\\_official](#)

 [blog.naver.com/jlukorea](http://blog.naver.com/jlukorea)

#### FCC Compliance Statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

#### FCC Interference Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

#### FCC Caution

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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410~412 office, Busan Knowledge Industry Center, 111, Hyoyeol-ro,  
Buk-gu, Busan, Republic of Korea