halo

microchip
scanner

User Manual

EN: User Manual	1-5
FR: Guide d'utilisation	6-10
DE: Bedienungsanleitung	11-15
NL: Handleiding	16-20
ES: Manuale de usuario	21-25
IT: Manuale d'uso	26-30

www.haloscanner.com


halo microchip scanner

Product Diagram


What's in the box?

Check that you have all these parts.

- 1) Halo Scanner
- 2) USB Cable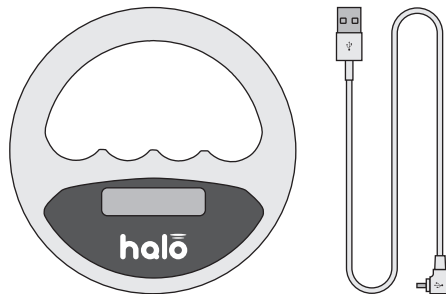

How to take a reading

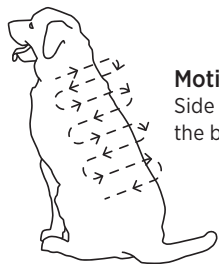Motion Pattern 1
Side to Side across
the body

Motion Pattern 2
Up and down
the body

Halo Microchip Scanner

The **Halo Microchip Scanner** complies with ISO 11784 and ISO 11785 norms. The **Halo** is Scanner Angel® compatible and is supplied with a USB lead for charging the **Halo's** internal rechargeable battery.

Recharging

Before using your **Halo** for the first time, it will need to be charged for 3½ hours.

will appear on the display when the battery is running low with the message "Battery Low". To recharge the battery, plug the USB cable into the back of the **Halo** and into an available USB port on either a computer or wall plugin charger. During the charging cycle, the display will switch off. To see if the battery has finished charging, press the start button and the display screen will inform you if it is "charging" or "charged".

will appear on the display when the battery is fully charged.

Available Software

Depending on your application, there is a range of software available including the Scanner Angel® software for the **Halo** which can be downloaded from www.haloscanner.com. Please note there is no Mac OS compatible software at the moment.

You do not need to install any software to use the **Halo** as a normal traditional microchip scanner.

Scanning

To start your **Halo Scanner**, simply push the on/off  symbol on the back panel and release. The display will show “Halo Scanner Scanning” and the scanner will emit a single beep. If no valid microchip is found within 20 seconds, then the display will show “No Tag Found” and the scanner will again emit a single beep. To start scanning again just press the  symbol again. The **Halo** will automatically switch off after 50 seconds.

The **Halo** will also display the microchip's temperature of compatible temperature sensing microchips. The temperature will only be displayed if the temperature is between 25°C and 43°C.

The outer surface of your **Halo** is the scanning area and when a valid microchip is found the scanner will emit a double beep and the microchip number will be shown on the display.

To review previously scanned numbers, wait until the unit has powered down. Then press and hold the power button on the back for four seconds and then release. To scroll through the list, press the power button once to see the next number. To exit this mode, press and hold the power button for four seconds and release.

Cleaning

Your **Halo** should never be immersed in liquid or placed in any sterilization equipment. It can be wiped with an antibacterial wipe.

Disposal of products and used batteries

This symbol on the product and / or accompanying documents means that used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated recycling points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product. Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling.

Warranty

The **Halo Scanner** comes with a 2 year return to base warranty from the date of purchase. The warranty is limited to any defects in materials components or manufacture including any defects in workmanship under normal use. Faults arising from misuse, neglect, disassembly are not covered by this warranty.

If a fault arises which is covered by the warranty then the manufacturer will either replace or repair the product free of charge at the manufacturer's discretion. In the event of requiring warranty support then the manufacturer should be contacted in the first instance on info@idporte.com. Your statutory rights are not affected.

Disclaimer

The **Halo Scanner** and associated facilities is not intended to replace traditional Veterinary advice. Microchip temperatures are not sufficient for determining the status of an animal's health and should be determined by a Veterinary professional. We expressly disclaim any liability for any loss, damage or fees that arise from using this product.

Technical Specifications

- In compliance with ROHS, CE and ISO 11784 norms.
- Dimensions: 135mm diameter x 33mm
- Weight: 163g
- Boxed Dimensions: 147mm x 144mm x 38mm
- Boxed Weight: 250g
- Scans per Charge: 8 second scans x 1000
- Standby Time: 2 months (typically)
- Battery: Internal lithium polymer rechargeable
- Recharge via USB port
- Charge Time: 3.5 Hours
- Reads: FDX-B, FDX-A (FECAVA), EM4102
- Reads the temperature from Thermo chips
- Operating Temperature: 0°C to 50°C
- Display: LCD 2 lines x 16 characters
- Tones: Audible beep
- Memory: Last 2000 scanned numbers
- Operation: Simple to start scanning and automatic turn off after 50 seconds
- Communications: USB port
- Manufactured in Great Britain
- Colour Options: White, Black, Pink, Blue, Green, Purple

All specifications subject to change without prior notice.

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Address: iD Porte Ltd, Rue Mahaut, St Saviour, Guernsey, GY7 9XD

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

CAUTION: The grantee is not responsible for any changes or modifications not expressly approved by the party responsible for compliance. Such modifications could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment has been tested and meets applicable limits for radio frequency (RF) exposure.

EC DECLARATION OF CONFORMITY

Document No. IDP_Halo_05_03_Iss F
Manufacturer Name: iD Porte Limited
Address: Rue Mahaut, St. Saviour, Guernsey, GY7 9XD
Declare, under its sole responsibility that the product
Product: Halo Microchip Scanner
Markings: 5060352120004, 5060352120011, 5060352120028
5060352120042, 5060352120059, 5060352120035

is in compliance with the essential requirements of the Radio Equipment Directive (2014/53/EU) by application of:

Documents No.	Title
EN300 330 V2.1.1	Short Range Devices (SRD); Radio Equipment in the frequency range 9KHz to 25MHz and inductive loop system in the frequency range 9KHz to 30MHz
EN301 489-1 V2.2.3	ElectroMagnetic Compatibility (EMC) standard for radio equipment and services, Part 1
EN301 489-3 V2.1.1	ElectroMagnetic Compatibility (EMC) standard for radio equipment and services, Part 3
EN62368-1:2014 +A11 : 2017	Audio/video, information and communication technology equipment part 1: safety requirement.
EN 55014-2 : 2015	ElectroMagnetic compatibility. Requirements for household appliances, electric tools and similar apparatus immunity
EN 50364 : 2018	Product standard for human exposure to electromagnetic fields from devices operating in the frequency range 0Hz to 300GHz, used in Radio Frequency Identification (RFID) and similar applications.

and the Directive (2011/65/EU) on the restriction of the use of certain hazardous substances in electrical and electronic equipment (recast) + Delegated Directive (EU) 2015/863 (RoHS)

Signed for and on behalf of: iD Porte Ltd

Name: N.A.J. Smith

Signature:

Position: Technical Director

Date:

24th September 2022

Manufactured at: iD Porte Ltd, Rue Mahaut, St Saviours, Guernsey, GY7 9XD

Manufactured by:

www.idporte.com

IDP_EN_Halo_00_07 Iss F