SAFETY INFORMATION AND IMPORTANT NOTES

GERMAN

PORTUGUESE

FRENCH

SPANISH

ENGLISH

ITALIAN

Deep Care GmbH
Königsallee 43, 71638 Ludwigsburg, Germany

Basic Operation

Power On

Press **Power button** for around 2 seconds and the device will start up and display the lock screen. Slide up to unlock the device, if needs password or pattern, please input them to unlock device.

Power Off / Restart


Press **Power button** for around 2 second and choose **Power off** in the popup, or choose **Restart** to re-boot the device.

Standby

When the device is working, press **Power button** shortly to shut the screen and it goes into standby mode. Press **Power button** again to awake the device, and slide the lock to continue.

Volume + / -

At most interfaces, the volume buttons are used to adapt sound. At some interfaces, the volume buttons can be used to switch between selections, or zoom in / out, or take a picture.

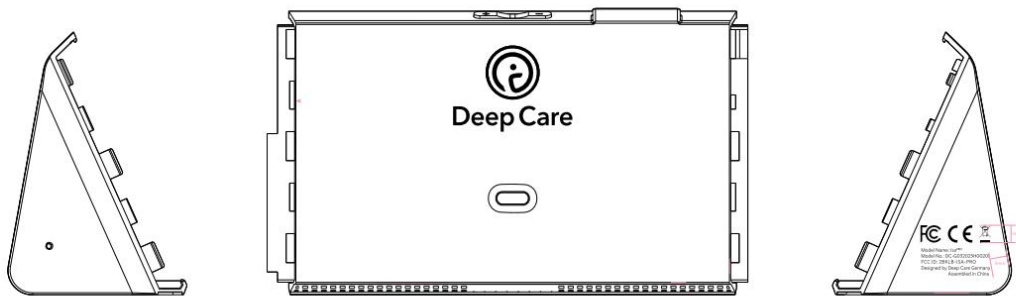
- **Notification Bar:** At the top of screen.

It displays time, battery, Wi-Fi signals, and notification icons or previews, etc.

- **Notification Panel:** Slide down from Notification Bar.

It displays switchers of functions and notifications. Tap the switchers to turn ON/OFF functions. Tap the notifications to open it, or slide it leftwards quickly to clear the notification. You can also slide the notification leftwards or rightwards slowly, and click  to delay the notification, or click  to set notification type.

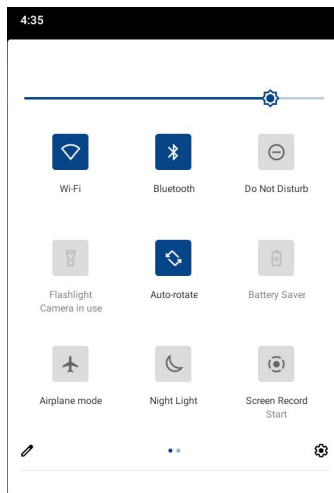- **Navigation Button:** At the bottom of screen.

	Back Button Tap it to go to the previous page.
	Home Tap it to go to Home Screen.
	Recent Task Tap it to view recent tasks. Tap the task frame to open it, or long press it to remove from list.

- **Google Search Widget:** Tap the Widget to input keywords or tap the icon for voice search (need network).

- **Enter App Menu:** At Home Screen, slide up to enter App menu, in which you can view all Apps list and search Apps.
- **Add Home Screen Apps:** You can add App icon to Home Screen from App menu by long pressing App icon and move to Home Screen before loosen your finger. Click App icon to open it. Long press App icon to see most used functions and choose one to open.
- **Remove Home Screen Apps:** Long press App icon and move to top of screen to remove it.
- **Home Screen Settings:** Long press blank area of Home Screen, you can set WALLPAPERS, WIDGETS and SETTINGS from pop-up options. Tap SETTINGS and make general Home Screen settings.
- **Set Wallpaper:** Long press blank area of Home Screen, choose WALLPAPERS, and choose a way to open wallpapers. After choose a picture, tap Set wallpaper, set wallpaper as Home screen, or Lock screen, or Home screen and lock screen.
- **Add Widgets to Home Screen:** Long press blank area of Home Screen, choose WIDGETS, and press a widget and move to Home Screen before loosen your finger.
- **Home Screen Pages:** Long press a widget or App icon on Home Screen, and move to left or right edge of screen, the widget or App icon will be moved to another page (if no existed page, it will create a new page). At Home Screen, slide leftward or rightward to switch between pages.

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ENGLISH

Safety Information and Important Notes

This manual contains important instructions and safety precautions about your ISA.

Do not use ISA in ways other than those described in this manual, as this could result in fire, electric shock or other unexpected accidents.

Safety standard markings and declarations apply only to the supported voltages and frequencies in the respective countries or regions.

Delivery Items:

- 1x ISA
- 1x Power Supply
- 1x Operating manual

© 2021 Deep Care GmbH. All rights reserved.

Deep Care GmbH, Königsallee 43, 71638, Ludwigsburg, Germany.

www.deep-care.de

Electricity Supply:

- Always keep the area around the mains supply clear so that you can easily disconnect the power cable if necessary.
- Never disconnect the plug by pulling on the cable.

General Information:

- Do not place anything on the ISA. Be especially careful with metal parts such as paper clips and staples and with containers containing flammable liquids such as alcohol or thinner.

Laser Product Class 1:

- This product is classified as a Class 1 laser product in accordance with EN/IEC 60825-1, edition 3 (2014).
- The sensor in this product complies with FDA performance standards for laser products except for conformance to IEC 60825-1 Ed. 3 as described in Laser Notice No. 56 dated May 8, 2019.

Do not use ISA in the following cases:

- Metallic foreign objects or liquids have entered your ISA.
- ISA emits smoke or unusual smells or noises occur.
- The Power cable or plug are overheated, broken, bent or damaged.
- The ambient temperature exceeds 35° C.

Failure to observe the following points may result in a fire, electric shock or injuries:

- Do not open or modify ISA.
- Only use the original Power Supply and cable delivered with your ISA..
- Do not connect the device voltages or frequencies other than those specified.
- Plug the Power cable fully into the power socket.
- Do not connect and disconnect the power supply with wet hands.
- Do not damage the power cable by twisting, bundling, tying, pulling or excessive bending.
- Do not place any objects on the cable.
- Do not leave ISA connected to the mains supply during a thunderstorm.

The location for operating your ISA should comply with the following points:

- Keep out of the reach of children.
- Do not place ISA in a location that is unstable or subject to vibration.
- Do not place ISA in a location that becomes very hot (direct sunlight or near a heat source), humid, dusty or outdoors.
- Do not place the ISA on a carpet.
- Keep the visible area in front of the ISA as clear as possible. Blocking the sensor field of view will reduce performance.
- ISA can be placed anywhere on a desk. However, it is recommended that you place ISA at a 45° angle to the left or right in front of you.
- The best performance is achieved when ISA is placed directly on the desk top. Please avoid placing ISA on other objects such as books

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FCC Caution:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

To maintain compliance with FCC's RF Exposure guidelines, This equipment should be installed and operated with minimum distance between 20cm the radiator your body: Use only the supplied antenna.