

to charge the Battery Capsules, as well as the mobile phones.

LED Indicator Lights to Show the

USB Output for Charging the Mobile Pho

3 Input for Charging the Power Pack


5) Internal Mesh Pocket

LED Charging Status Indicator Lights

4 LED on: 100% 3 LED on: 75%


2 LED on: 50% 1 LED on: 25%


Remaining Capacity

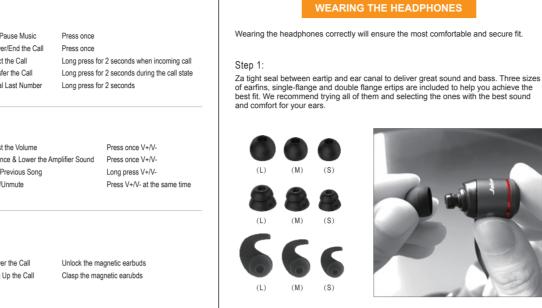





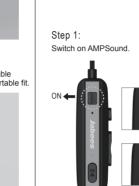

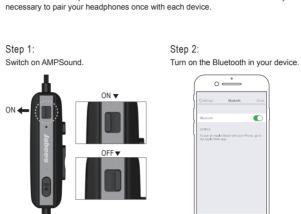


AMPSound must be paired to a Bluetooth device before it can be used. It is only






pairing mode and go to the idle mode automatically.



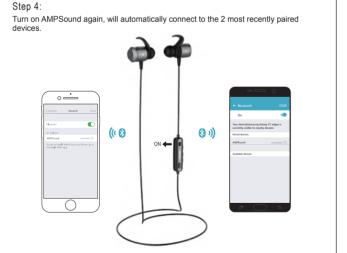

for 3 seconds when the headphones are on but not connected with any device. This will

ensure the better music performance.

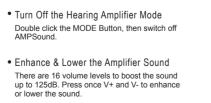

HEARING AMPLIFIER MODE LUETOOTH MUSIC & HANDSFREE MODE

AMPSound by Jabees is the Bluetooth Personal Hearing Amplifiers that amplify sounds and enhance your hearing experience so your conversations become more effective in the car, in social situations, and at a distance. It is designed for people of any age with hearing loss of less than 100dB.

Maximum Saturated Sound Pressure Level (SSPL)	≤125dB ±3dB
High Frequency Average Gain	118±3dB
Average Full-on Gain	58 dB±5dB
Standard Frequency Response Curve	300Hz-5KHz
Equivalent Input Noise (EIN) Level	≤18dB
Harmonic Distortion	≤4%
Battery Life	Up to 7 hrs(Subject to the surrounding noise leve
	.1

 Turn On the Hearing Amplifier Mode Switch on AMPSound, then long press the MODE Button for about 2 seconds.

AMPSound can be used as a conventional Bluetooth Stereo Headphones for Music Streaming and Phone Call after pairing and connecting with your device. (Please refer to 'BLUETOOTH PAIRING & CONNECTION' section.)

 Wireless Music Streaming Wireless Handsfree AMPSound features the aptX technology for the CD-like quality music when you are listening to your favorite tune.

you want to listen to music.

AMPSound can be used to receive and make calls, and voice message on Whatsapp, Wechat, Skype and so on.







Amplifier Mode will be automatically switched Note: Please turn off the Hearing Amplifier Mode if the Bluetooth Handsfree mode. When the call ends, the headphones will be switched back t the Hearing Amplifier Mode.

HARGING THE BATTERY CAPSULE The Power Pack of AMPSound is with built-in 3000mAh battery capacity. It can be used

Turning the Battery Capsule counterclockwise to remove the battery.









Lights turn on when charging.



Press the button on the Power Pack to recharge the Battery Capsules, the LED

The LED Charging Status Indictor Lights turns off when fully charged.

To restore AMPSound to factory setting, you will have to turn on AMPSound first, and make sure they are not connected with any device. Press and hold the MF Button for about 5 seconds. The blue and red LED lights turn on for about 2 seconds and then turn off, the factory reset is successful.

or push other objects into this product, as this may cause short circuit, which led to the Jabees is a registered trademark of FreeTek International Co, Ltd. Other brands and product names are trademarks of their respective owners. Jabees warrants this product against all defects in material and workmanship for a period of twelve (12)months from the date of original purchase of the product. The conditions of this guarantee and the extent of responsibility of Jabees under this guarantee can be read in the Warranty Card.

anent hearing loss may result from long-term exposure to sound at high volumes Use as low volume as possible and avoid prolonged use of the earphone at excessive sound pressure levels. Please read the safety guidelines below prior to using the safety guidelines below prior to using this earphone.

(Prior to using this product follow these steps) Before putting on the earphone, turn the volume control to its lowest level. Put the earphone on, and then slowly adjust the volume control to a comfortable level.

During the use of this product)

Keep the volume at the lowest level as possible.

2. If increased volume is necessary, adjust the volume control slowly. 3. If you experience discomfort or ringing in your ears, immediately discontinue using

the earphone. With continued use at high volume, your ears may become accustomed to the sound level, which may result in permanent damage to your hearing without any noticeable discomfort.

1. If you have a cardiac pacemaker or other electronic medical equipment, you should

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following

fire or get an electric shock. 4. None of the components can be replaced or repaired by users. Only authorized dealer or service center may open the product. 5. If any parts of your product require replacement for any reason, including normal

2. This product contains small parts that may be hazardous to children. Keep out of the

reach of children. The bags themselves or the many small parts they contain may

Be sure to ensure safe driving, avoid distractions and follow local laws.

consult your physician before using this product.

3. Never attempt to disassemble the product yoursel

cause choking if ingested

wear and rear or breakage, please contact your dealer. - Reorient or relocate the receiving antenna. 6. If the product overheats, has been dropped or damaged or has been dropped in a - Increase the separation between the equipment and receiver. liquid, discontinue use.

-Connect the equipment into an outlet on a circuit different from that to which the -Consult the dealer or an experienced radio/TV technician for help

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and

(2) this device must accept any interference received, including interference that may cause undesired operation.

FCC ID: ALE12011

BTHS20160808_V1.0