

The Dash ^{PRO}

tailored by Starkey®

Quick Start Guide

v1.0, 2017-01, V3C QSG English

BT: XXXX-XXXXMHz **NFMi:** XX,XXXMHz
FCC ID: XXXXXXXXXXXXXL **FCC ID:** XXXXXXXXXXXXXR
IC: XXXXX-XXXXXXXXXX



Starkey Hearing Technologies, Minneapolis, MN, USA & **Bragi GmbH**, Munich, Germany

Starkey is a trademark of Starkey Laboratories, Inc., registered in the U.S. and other countries.

All rights reserved. Patents pending. BRAGI, The Dash, "The World's First Hearable", "Listen. Track. Communicate.", and the B logo are trademarks of Bragi GmbH in the U.S. or one or more other countries. The absence of a product or service name or logo from this list does not constitute a waiver of trademark or other intellectual property rights concerning that name or logo.

Thank you.







Charge & Activate.



Charge your Dash
before first use.

Charge.

Before using The Dash, charge it for at least 1 hour. This will also activate your Dash.

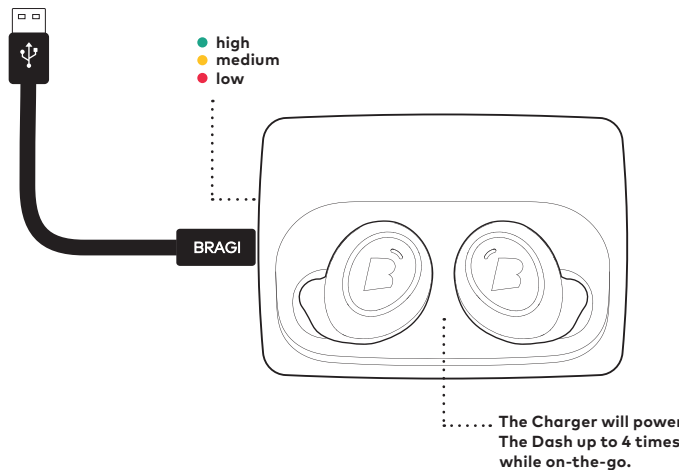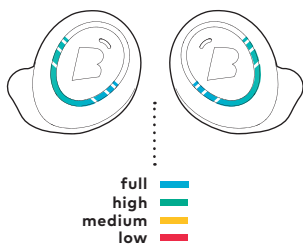



Shake.

See the current battery status by shaking your Dash.

With The Dash in The Charger, connect to your computer or a power supply via USB.

Update.

Your Dash learns new features when updated to the newest **Bragi OS**.
Download from update.bragi.com

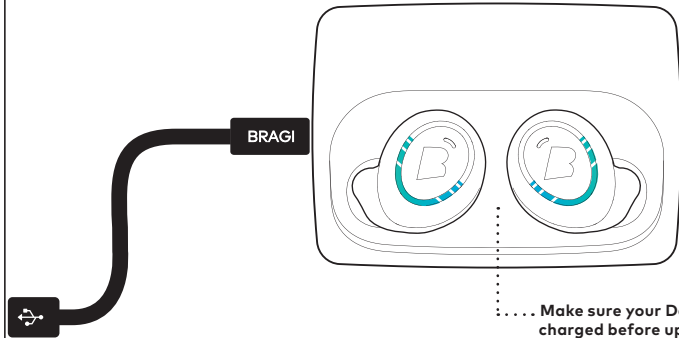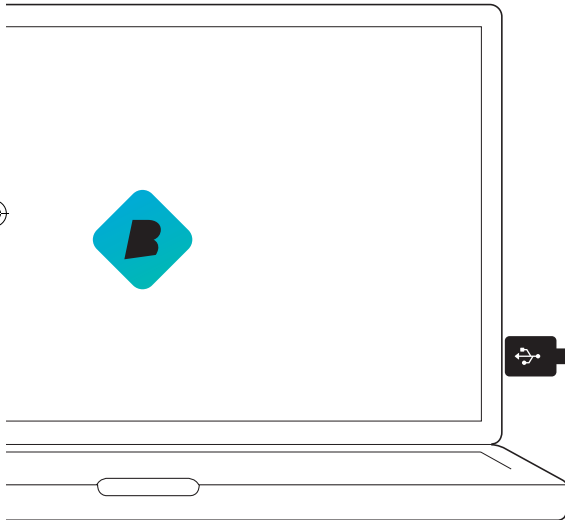



..... Make sure your Dash is fully charged before updating.

▶ Video tutorials on bragi.com/support


update.bragi.com

Insert.

1. Tilt The Dash slightly forward.
2. Insert the tip firmly in your ear canal.
3. Adjust The Dash by rotating it backwards.

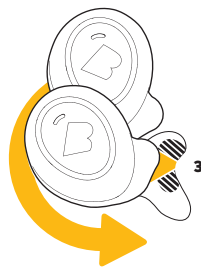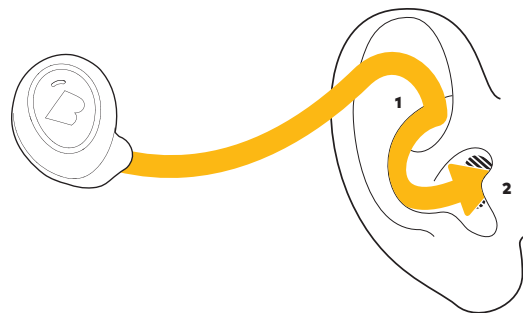


The PerfectFit

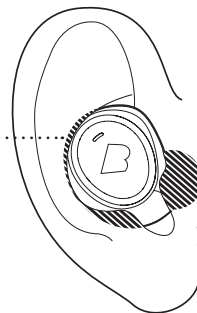


HearClears.

Replace your wax guard...

1. Place the blank side of the applicator in your HearClear® and twist it clockwise.
2. Pull it out.
3. Insert the new HearClear® from the other end of the applicator.

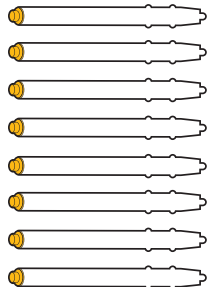


..... HearClear®

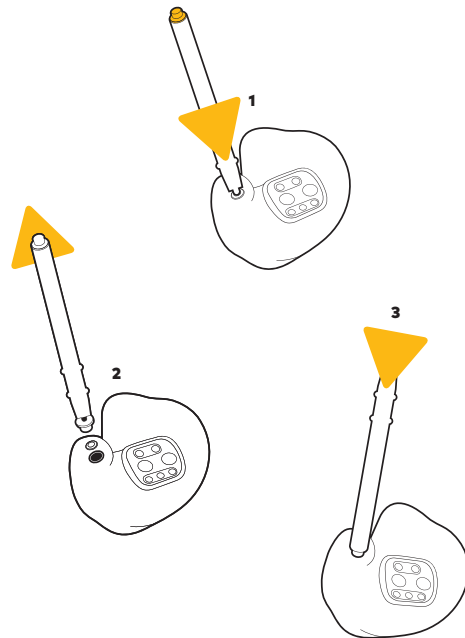

! Replace the HearClears of your Dash, if the sound is muffled.



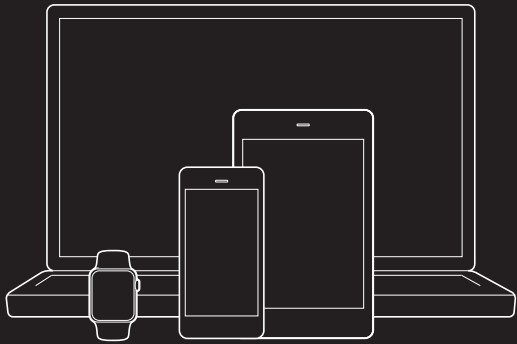

Connect.





Control.

Pairing.

Insert The Dash into your ear and search for **The Dash** in the Bluetooth® menu of your device.

! If your Dash does not show up in the list, do a long hold (5 sec) on the Right Dash.

Connect Audio.

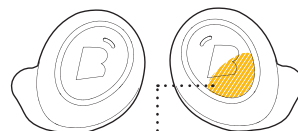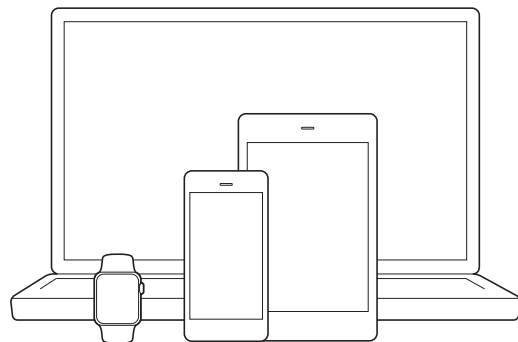


Long Hold on the
Right Dash to initiate
Bluetooth® pairing

Pairing.

To send activity data from the Dash sensors to your device, open the Bragi App and follow the instructions on how to connect.

Connect Sensors.

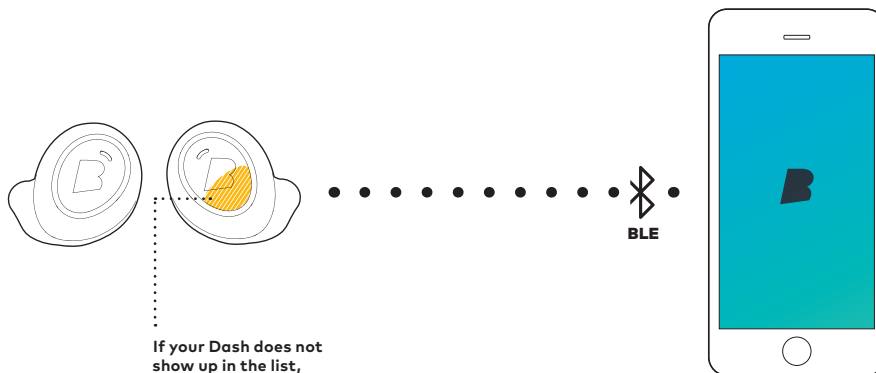


If your Dash does not show up in the list, do a long hold on the Right Dash to initiate Bluetooth® Sensors pairing.

Customize.

Bragi App.

Download the Bragi app to set up and personalize your Dash. You can also use your smartphone to control the features of The Dash.

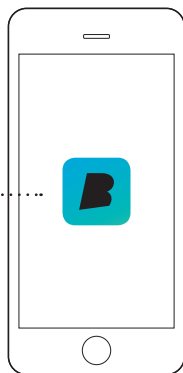


Use the Bragi App to set up:
Activities
MyControls
Sound
Settings
Profile



Free download!


EarTouch.

Control your Dash with tap, hold and swipe gestures using the EarTouch Interface.

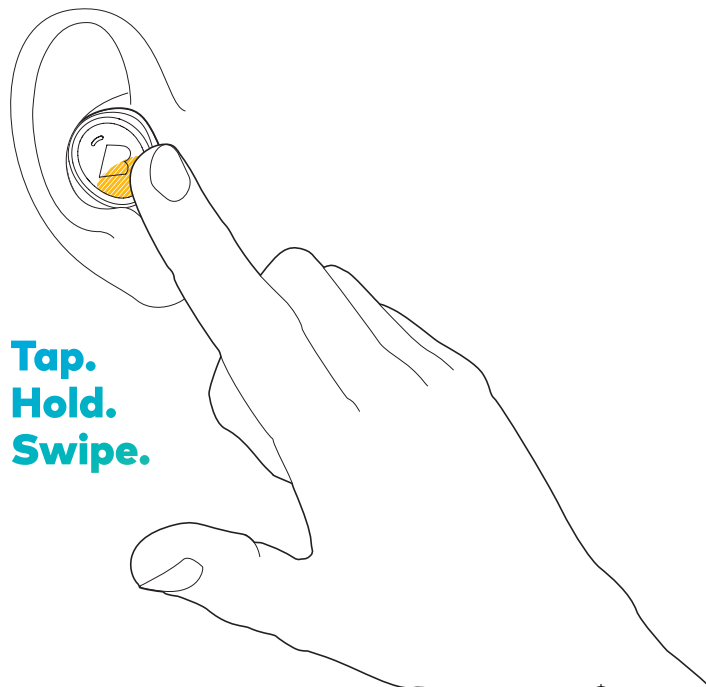

Tap.
Hold.
Swipe.

The EarTouch Interface is located on the lower half of each Dash.

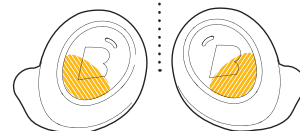

Hold on both Dash simultaneously to turn **TouchLock** on



Basic Controls.



..... Listen & Communicate



Right Dash

Tap	Play/Pause	●
2xTap	Next song	●●
3xTap	Previous song	●●●
Swipe	Volume Up/Down	↔
Hold 1sec.	Main menu	▬
Hold 5sec.	Bluetooth® settings	▬

..... Control Applications




Left Dash

Swipe	Audio Transparency	↔
Hold 1sec.	App menu	▬

Menu navigation


Tap	Select	●
Swipe	Next / Previous menu item	↔
Hold 1sec.	Close menu	▬

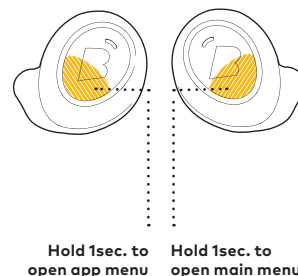


See more features on
bragi.com/support

Languages.

Use the Bragi Updater to change the language of your Dash. **Download from update.bragi.com**

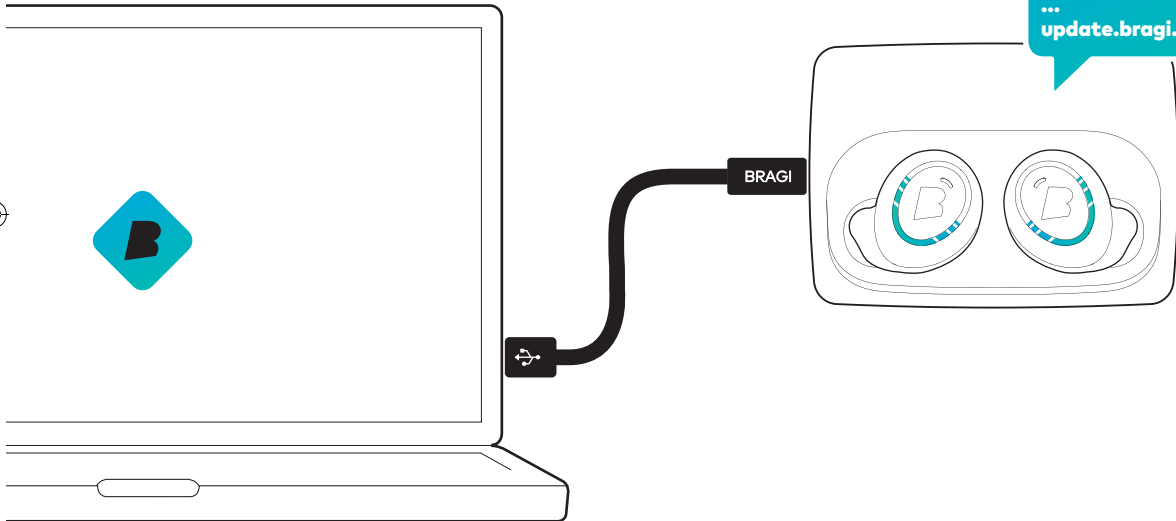

▶ Video tutorials on bragi.com/support

update.bragi.com



Questions?



We are here to help.

Starkey Limited Customization Refit

If you are not satisfied with the fit of The Dash - tailored by Starkey Hearing Technologies, you shall have 30 days from receipt of The Dash - tailored by Starkey Hearing Technologies purchase to notify the practice where purchased of User's desire to seek a replacement custom fit at no cost.

support.bragi.com

