

# SURFLINK<sup>®</sup> WATCH



## Operations Manual

### Table of Contents

<b>Overview</b>	
SurLink Watch Overview	4
<b>Operation</b>	
Modifying Date and Time	6
Toggle Between Seconds View and Date View	6
Illuminate Display	7
Generate Unique Address Code for Pairing with Hearing Devices	7
<b>Battery Replacement</b>	10
<b>Watch Care</b>	13
<b>FCC Information</b>	14

### Overview

The SurLink Watch features:

- Remote control
- A watch
- 12/24-hour time formats
- American/International date formats

### Overview



### Overview

1) Special Feature Functionality	Press <b>STAR</b> (*). This button may have different functionality depending on the hearing device setup performed by your hearing professional.
2) Memory or Program Change	Press <b>PROGRAM</b> (#). Repeat as needed.
3) Volume Increase	Press <b>VOLUME UP</b> (Δ). Repeat as needed.
4) Home or Hearing Device Reset	Press <b>RESET</b> (☀). Hearing devices will return to their power-up settings.
5) Volume Decrease	Press <b>VOLUME DOWN</b> (∇). Repeat as needed.

### Operation

#### Modifying Time and Date

- Press and hold **SET** (\*) for 3 seconds. HOURS digits will flash.
- Press **UP** (Δ) to advance. Go through 12 hours for AM or PM.
- Press **MODE** (#) to select MINUTES, then **UP** (Δ) to advance.
- Repeat to set SECONDS, YEAR, MONTH, DAY, 12/24-hour TIME display, and Month-Day/Day-Month DATE display.
- Press **SET** (\*) when done.

#### Toggle Between Seconds View And Date View

- Press and hold **MODE** (#) until display changes (about 3 seconds).

### Operation

#### Illuminate Display

Display illumination becomes visible in low light situations.

- Press and hold **LIGHT** (☀) for at least 1 second to illuminate watch face.
- Note:** Pressing **LIGHT** (☀) for less than 1 second may cause hearing devices to return to their power-up settings.

#### Generate Unique Address Code for Pairing with Hearing Devices

**Watch Only:** This allows a hearing professional to generate a unique address code, which is used during a programming session, to pair the SurLink Watch with your individual hearing devices.

### Operation

- Press and hold **MODE** (#) until 6-digit address appears (about 10 seconds).
- Write down 6-digit address displayed on your watch and give to your hearing professional.
- Press **SET** (\*) when done.

**Watch Plus Another Watch Control:** If you already use another Watch control, such as a SurLink Watch with your wireless hearing devices, a hearing professional can adjust the watch's address code to match your existing Watch control. Both Watch controls can then make hearing device adjustments.

- Press and hold **MODE** (#) until 6-digit address appears (about 10 seconds).

### Operation

- To match another Watch control pairing address, press **UP** (Δ) to advance.
- Press **MODE** (#) to advance to next address field, then **UP** (Δ) to advance.
- Repeat until 6-digit address matches the other Watch control.
- Press **SET** (\*) when done.

## Battery Replacement



**A.** Open the four screws on the back of watch with small Phillips screwdriver.



**B.** Remove metal back of watch. Be careful not to damage the gasket (dotted line above) between the watch and the metal back.

10

## Battery Replacement



**C.** To remove the battery, place small Phillips screwdriver tip between battery and battery holder clip (small metal tab).



**D.** Push the tab away from battery (as shown above). Remove old battery.


11

## Battery Replacement



**E.** Insert a new CR2032 battery ("+" facing up) into the empty circular recess. Push the battery holder clip until it snaps into the watch. **Note:** If the battery holder clip does not snap into the "catch," it may need to be bent inward slightly.

12

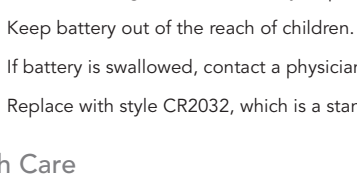

**F.** Set the gasket in its position and replace the metal back with the four screws. Do not overtighten the screws. Following battery replacement, the Watch control functionality with your hearing devices should resume.

13

## Important Battery Notes:

- Follow local regulations for battery disposal.
- Keep battery out of the reach of children.
- If battery is swallowed, contact a physician immediately.
- Replace with style CR2032, which is a standard 3 volt lithium battery.

## Watch Care

### Water Resistance

This watch resists splashes and rain. However, it should not be used while taking a shower, swimming, diving, surfing, snorkeling, or other water sports. Make any adjustments to the watch before exposing it to excessive water. Pressing the buttons in water or a heavy rainstorm may cause water to enter the watch.

14

## FCC Information

SurfLink Watch is designed to comply with the most stringent Standards of International Electromagnetic Compatibility. However, it is still possible that you may experience interference caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges. If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your SurfLink Watch might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance.

### Use on Aircrafts

SurfLink Watch should not be used on aircrafts unless specifically permitted by the flight personnel.

15

## FCC Information

**International Use**  
SurfLink Watch is approved to operate at a radio frequency that is specific to your country or region and might not be approved for use outside your country or region. Be aware that operation during international travel may cause interference with other electronic devices, or other electronic devices may cause interference with your SurfLink Watch.

FCC ID: EOA-WW  
IC: 6903A-WW

### FCC Notice

This device complies with part 15 of the FCC rules and with RSS-210. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any

16

## FCC Information

interference received, including interference that may cause undesired operation of the device.

**Note:** The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Starkey Hearing Technologies  
6700 Washington Ave. South  
Eden Prairie, MN USA

### FCC Notice

This device complies with part 15 of the FCC rules and with RSS-210. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any

17