## TV Clear Earbuds

Models: TVCE1-EB  
TVCE1

## Transmitter

Model: TV Connector

- EN Quick Guide
- DE Kurzanleitung
- FR Guide rapide
- ES Instrucciones resumidas
- PT Guia rápidos
- NL Beknopte handleiding
- IT Guida rapida
- DA Snabbguide
- SV Bruksanvisning

## SENNHEISER

Instruction manuals and product support  
Bedienungsanleitungen und Produktbetreuung  
Manuels d'utilisation et assistance produit  
Instrucciones de uso y soporte del producto  
Manuali d'istruzioni e supporto prodotto  
Bedieningsinstructies en productondersteuning  
Manuais de instruções e assistência ao produto  
Instrukcje obsługi i wsparcie produktowe  
Brugsanvisninger og produkthjælp

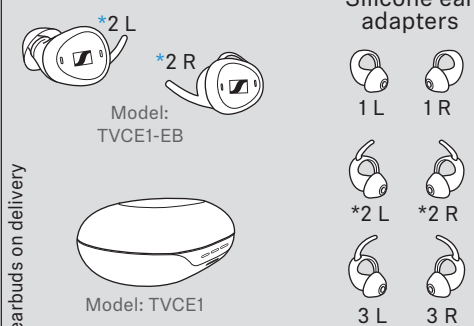
[www.sennheiser-hearing.com/TV-Clear](http://www.sennheiser-hearing.com/TV-Clear)  
[www.sennheiser-hearing.com/download](http://www.sennheiser-hearing.com/download)

Sonova Communications AG  
Herrenschwandweg 4, 3280 Murten, Switzerland  
029-3347-72/V1.00/2021-11


Package includes | Lieferumfang | Contenu de la livraison | Contenido del paquete | O pacote incluí | Inhoud van de verpakking | Contenuto della confezione | Pakken indeholder | Paketet inneholder

### TV Clear Set


#### TV Clear Earbuds


\* mounted on earbuds on delivery



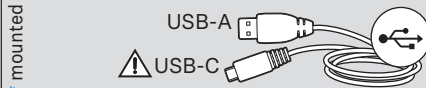
#### Transmitter
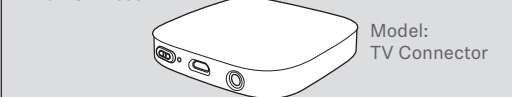
#### Optical cable

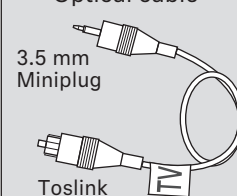
#### Analog 3.5 mm cable

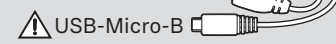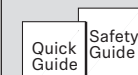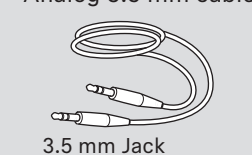
[www.sennheiser-hearing.com/download](http://www.sennheiser-hearing.com/download)  
Detailed instruction manual


### Using the set | Verwenden | Utilisation | Utilizar | Utilizar | Gebruiken | Utilizzo | Anvendelse | Användning

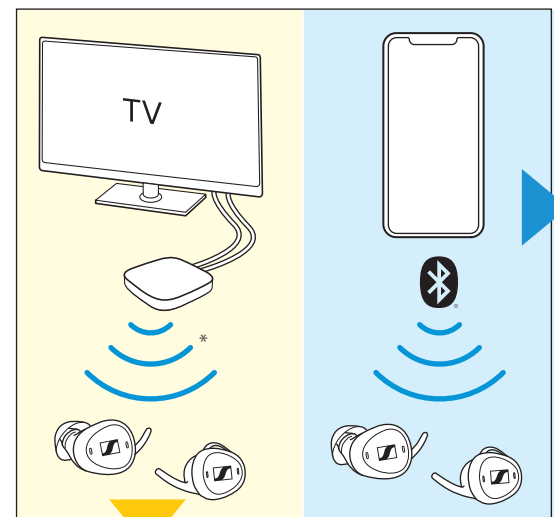
**1** Pairing the earbuds with your smartphone | Ohrhörer mit Smartphone koppeln | Appairer les écouteurs avec votre smartphone | Sincronizar los audífonos con el smartphone | Emparelhar os auriculares com o smartphone | Oortelefoons met smartphone koppelen | Accoppiare le microcuffie con lo smartphone | Parring af øretelefoner med Smartphone | Parkoppla hörlurarna till en smartphone

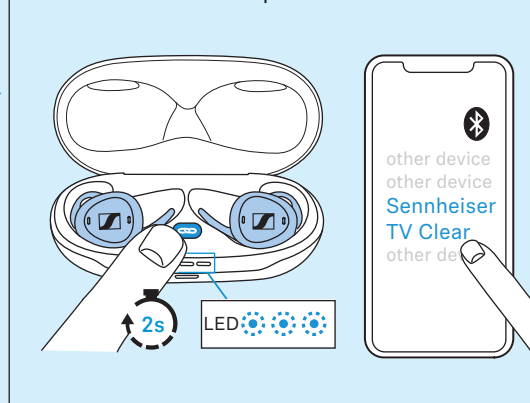
**2** Downloading the app for configuration and updates | App für Konfiguration und Aktualisierungen herunterladen | Télécharger l'appli pour la configuration et les mises à jour | Descargar app para configuración y actualizaciones | Descarregar a aplicação para a configuração e atualizações | App voor configuratie en actualiseringen downloaden | Scaricare l'app per la configurazione e gli aggiornamenti | Download af app for konfiguration og opdateringer | Ladda ner appen för konfiguration och uppdateringar

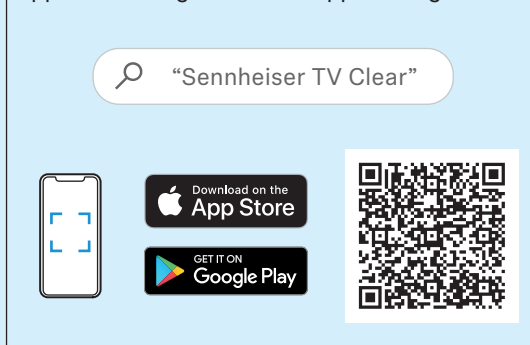
**3** Inserting the earbuds into your ears and selecting suitable ear adapters | Ohrhörer in Ihre Ohren einsetzen und passenden Ohradapter auswählen | Insérer les écouteurs dans vos oreilles et sélectionner des embouts d'oreille appropriés | Colocar los audífonos en las orejas y seleccionar los adaptadores para los oídos adecuados | Colocar os auriculares nos seus ouvidos e escolher o adaptador auricular adequado | Oortelefoons in de oren doen en passende ooradapter selecteren | Inserire le microcuffie nelle orecchie e scegliere i gommini adatti | Isætning af øretelefoner i dine ører og valg af passende øreadapter | Sätta i hörlurarna i öronen och välja passande öronadapter

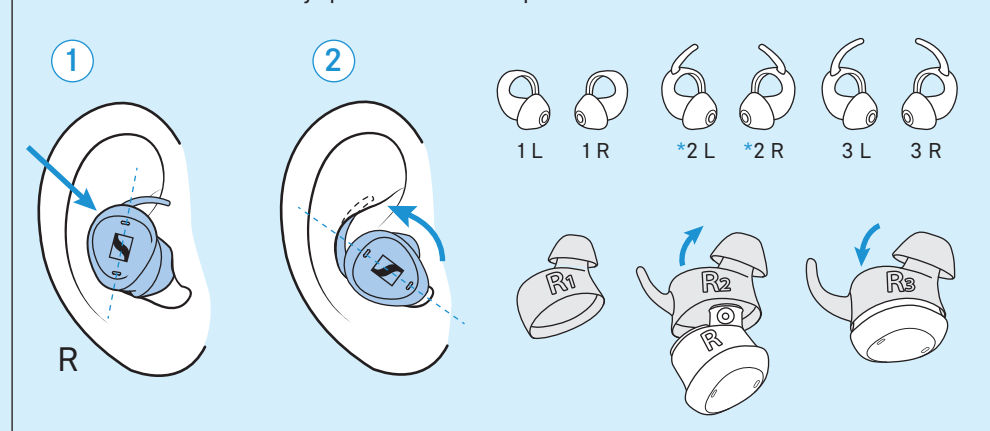Charging the earbuds and the charging case | Ohrhörer und Ladebox laden | Charger les écouteurs et le boîtier de chargement | Cargar los audífonos y la caja de carga | Carregar os auriculares e a caixa de carregamento | Oortelefoons en laadbox laden | Caricare le microcuffie e la stazione di carica | Opladning af øretelefoner og ladeboks | Ladda hörlurarna och laddaren

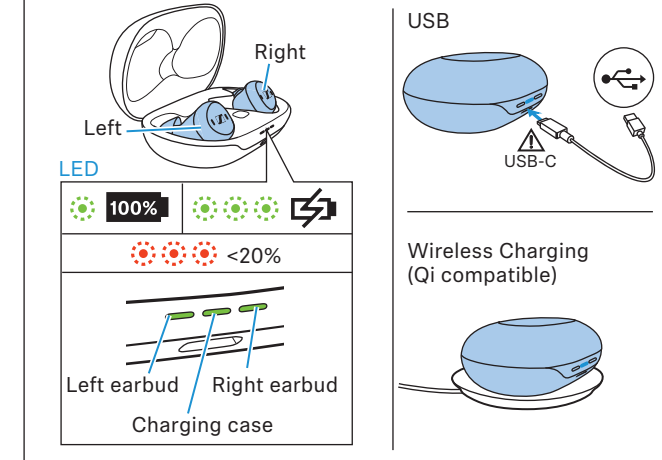
**1** Connecting the transmitter to your TV | Sender mit TV verbinden | Raccorder l'émetteur à votre télé | Conectar el transmisor con el televisor | Conectar o emissor à televisão | Zender met tv verbinden | Collegare il trasmettitore con la TV | Forbindelse af sender med TV | Ansluta sändaren till en TV

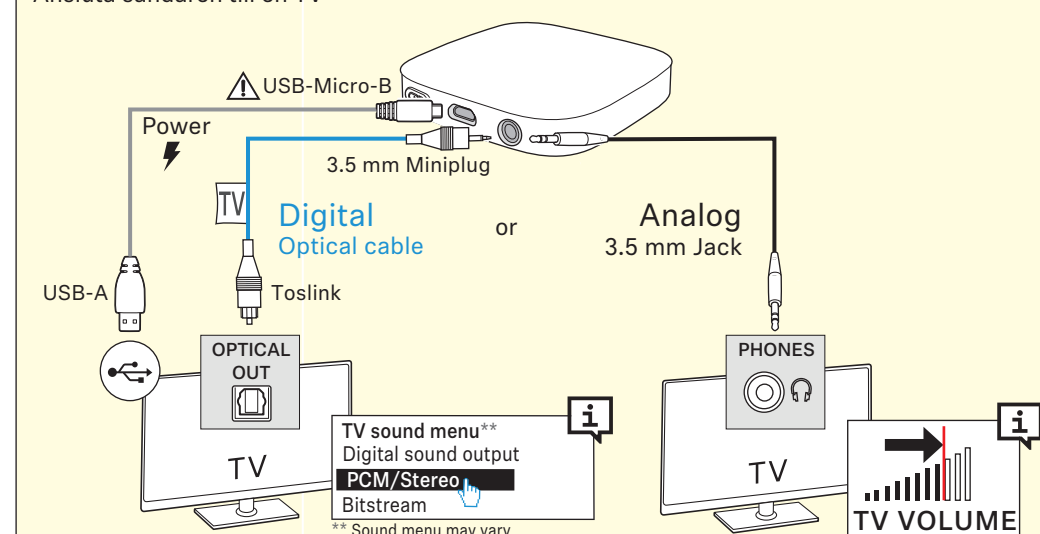
\* AirStream™ transmission technology


**2** Pairing the earbuds with the transmitter | Ohrhörer mit Sender koppeln | Appairer les écouteurs avec l'émetteur | Sincronizar los audífonos con el transmisor | Emparelhar os auriculares com o emissor | Oortelefoons met zender koppelen | Accoppiare le microcuffie con il trasmettitore | Parring af øretelefoner med sender | Parkoppla hörlurarna till sändaren

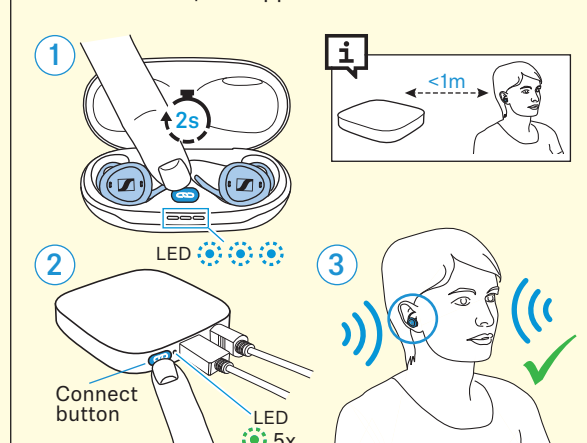**3** Switching on your TV and listening via the earbuds | TV einschalten und mit Ohrhörern hören | Allumer la télé et écouter via les écouteurs | Encender el televisor y escuchar con los audífonos | Ligar a televisão e ouvir com os auriculares | Tv inschakelen en met oortelefoons luisteren | Accendere la TV e ascoltare con le microcuffie | Tilkobling af TV og lytning med øretelefoner | Starta TV:n och lyssna med hörlurarna

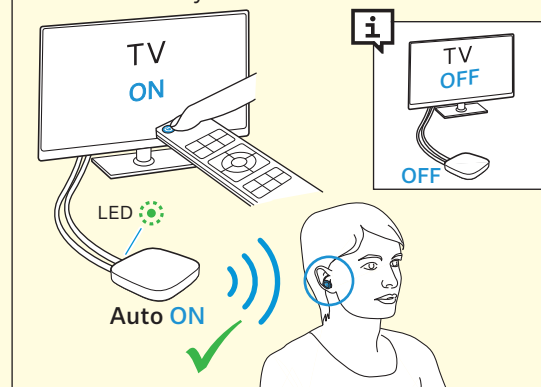Operating the earbuds | Bedienung | Utiliser les écouteurs | Manejo | Operação | Bediening | Utilizzo | Betjening | Användning

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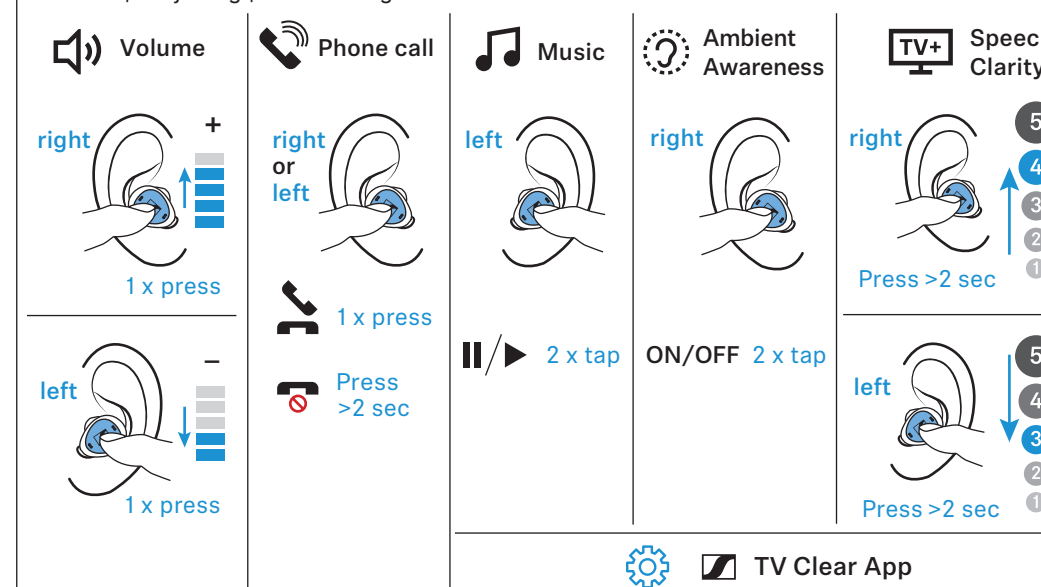
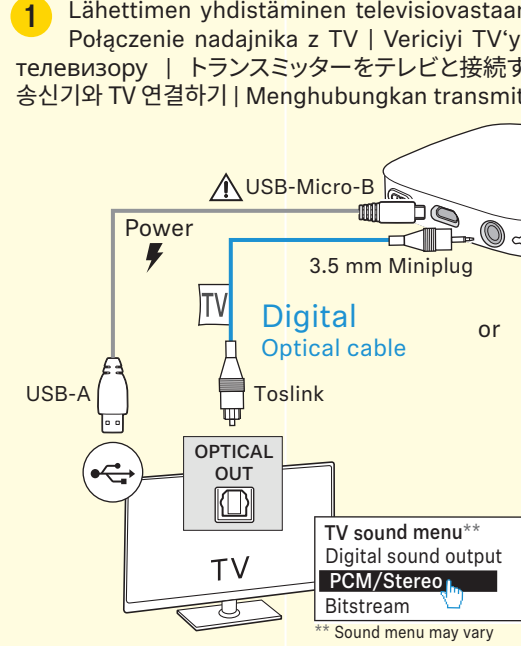
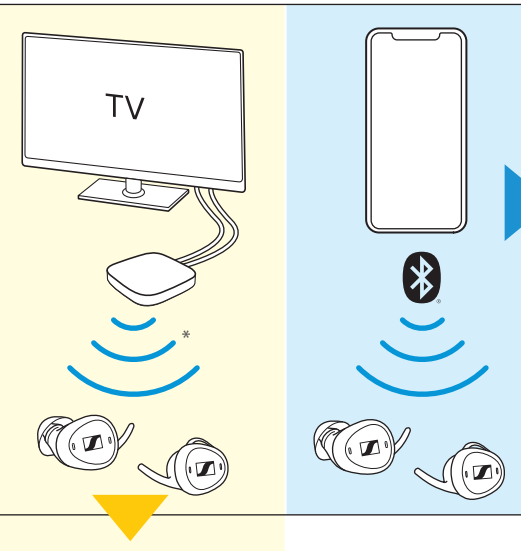
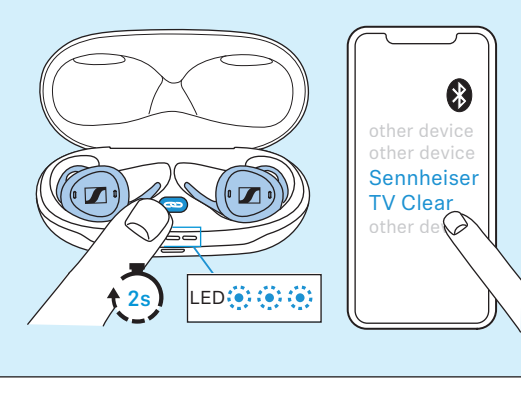
“Sennheiser TV Clear”

Download on the App Store  
GET IT ON Google Play

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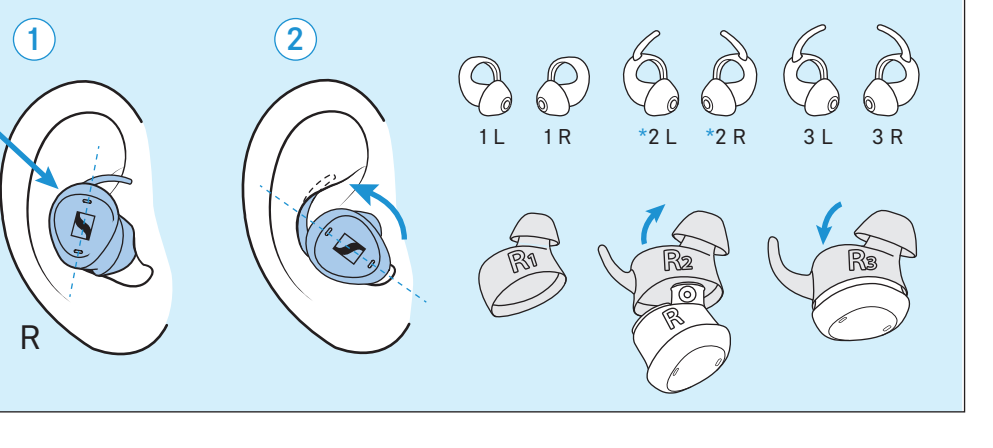LED


100% <20%

Left earbud Right earbud Charging case

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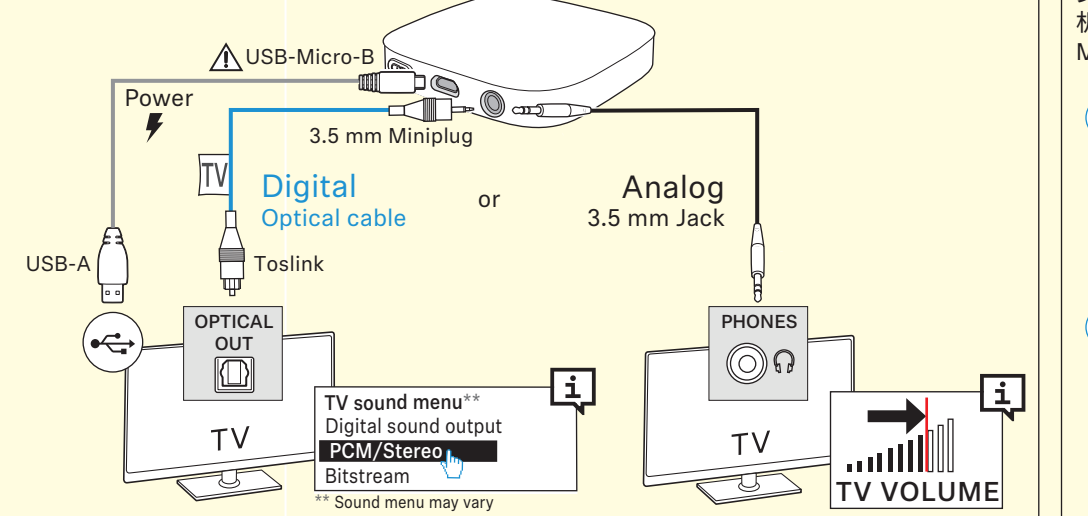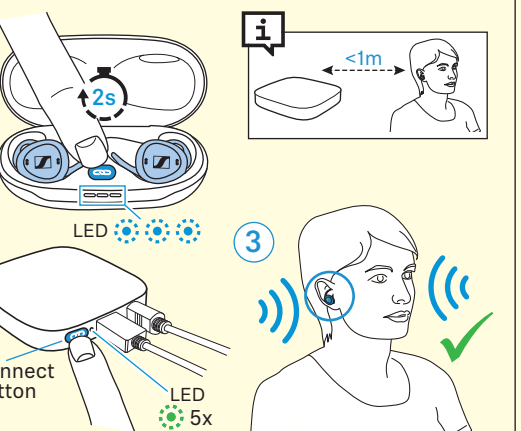
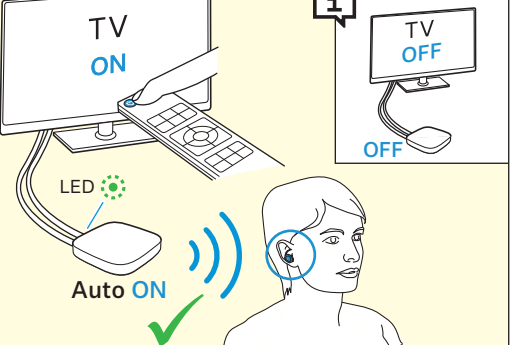
TV Clear App

### Compliance Information Sennheiser TV Clear Earbuds

Frequency band: 2400 – 2483.5 MHz / 111 – 147 kHz

Transmit power: <10 mW

Country/Region	Earbuds Model: TVCE1-EB	Charging case Model: TVCE1
USA	FCC ID: KWC-TWE21EB	FCC ID: KWC-TWE21
Canada	IC: 2262A-TWE21EB	IC: 2262A-TWE21
Europe	  	
United Kingdom	  	
Australia		
New Zealand	<b>R-NZ</b>	

### Compliance Information Sennheiser Transmitter “TV Connector”

Frequency band: 2400 – 2483.5 MHz

Transmit power: <10 mW

Country/Region	Transmitter Model: TV Connector
USA	FCC ID: KWC-TVCONNECTOR
Canada	IC: 2262A-TVCONNECTOR
Europe	  
United Kingdom	  
Australia	
New Zealand	<b>R-NZ</b>

## Statements regarding FCC and ISED

This device complies with Part 15 of the FCC Rules and with Industry Canada's license - exempt RSSs. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

The device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and ICES-003 of Industry Canada. These limits are designed to provide reasonable protection against harmful interference in a residential installation. The device generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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