Blank Page



Don't Forget To Delete Me!

PHONAK

Gebrauchsanweisung
Operating instructions
Mode d'emploi

MICROLINK ML55/ML6

CE

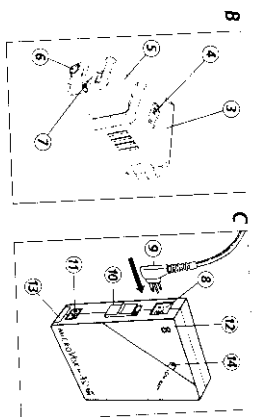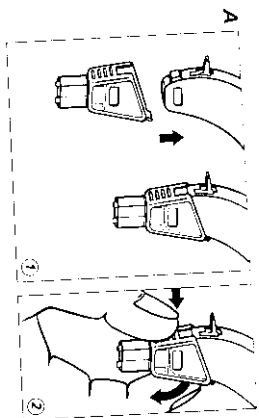D

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GB

Congratulations on your choice of a Phonak FM system. You have selected a high quality Swiss product. When properly handled this product will offer you years of hearing satisfaction.

Yours Phonak

A. To attach and remove the Microlink

1. Place the Microlink FM receiver over the bottom of the hearing instrument until it "clicks".
2. Activate the Microlink FM receiver by setting the user switch of the hearing instrument to the appropriate audio input position, as instructed by your hearing healthcare professional (remember to switch on the transmitter, see point 10).
3. To remove the Microlink FM receiver, twist off as illustrated. The Microlink is now disconnected.

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The Microlink FM receiver is powered by the hearing instrument battery, and has a low battery consumption (less than 1.5 mA). To increase battery life, we recommend that you always remove the Microlink receiver(s) from the hearing instrument(s), when FM is not used.

B. Description of the Microlink

3. Audio shoe part.
4. Number and color code for exact FM channel identification. Must match the channel no. of the transmitter.
5. Microlink receiver with integrated magnetic antenna for normal communication distance.
6. Socket for optional external antenna.

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7. Switch for hearing instrument microphone attenuation (only ML55).

Position «FM+M»: Signal from transmitter and hearing instrument microphone are equally loud.

The FM advantage is enhanced by damping the hearing instrument microphone by 20 dB.

C. Description of the Microvox Transmitter

8. Microphone socket. Used to connect the lapel microphone.
9. Microphone connecting cord. This cord also serves as the antenna and should be kept free of knots at all times.
10. The power On/Off Switch.

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11. Aux. input socket for coupling to external sound sources, such as TV, radio or tape recorder.
12. Number and color code for FM channel identification. Both the Microlink receiver and the transmitter channel numbers must be the same for the units to operate together.
13. Battery compartment.
14. Low battery warning display.

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GB

D. Principle of Operation

The Microlink/Microvox system is the world's smallest FM communication system, specifically designed for use with Phonak BTE hearing instruments, and used to improve hearing in difficult listening environments. The transmitter unit is worn by the person speaking (e.g. the school teacher or the hearing instrument user's conversation partner) or simply placed in front of the sound source (e.g. TV). The transmitter unit collects the desired speech signal and transmits it via radio waves, directly to the Microlink receiving unit attached to the hearing instrument. The incoming speech signal received by the hearing instrument is processed in accordance with the audiological setting of the instrument, providing consistent sound quality. The Microlink FM receiver has a very low battery consump-

tion (less than 1,5 mA) and is powered by the hearing instrument battery.

E. Operating Range

The Microlink/Microvox FM radio system is approved by local government authorities and complies with the rules for Short Range/Low Power Radio Communications Systems. The Microlink FM receiver has a built-in magnetic antenna for improved appearance and convenience, and is the only FM system where it is not necessary to have a small antenna sticking out into the air. New FM technology has made it possible to incorporate the antenna in the audio shoe itself. The Microlink/Microvox communication system covers any normal size classroom, dining- or livingroom, and has an operating range of typically 7 to 15 meters (21 to 45 feet).

An optional external antenna is available and can be connected in for extended operating range.

Reception range is affected by battery condition, local static interference, and radio wave propagation conditions (building construction, metal in walls, doorways, etc.) If FM reception stops, even though the hearing instrument alone still works, replace the battery. (The hearing instrument needs more battery power when the Microlink is connected.)

ML/MV Transmitter

FCC ID: KWC002T

This device may not interfere with TV reception or federal government radar, and must accept any interference received, including interference that may cause undesired operation

CE Made in Switzerland

Microlink Receiver

FCC ID: KWC003R

This device complies with Part 15 of the FCC rules. Operation is subject to the condition that this device does not cause harmful interference.

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Importers in the European countries

Austria: Hagerup & A. 1702, Grosse-Blaugasse, 1140 Wien
Belgium: Phonak Belgium NV, 2000 Antwerpen
Denmark: Phonak Denmark A/S, 2600 Lyngby
France: Phonak France SA, 92000 Boulogne
Germany: Phonak GmbH, 10707 Berlin, Götterstr. 17
Great Britain: P.O. Box 124, London SW12 8LP
Greece: Iliad Hering GmbH, 104 17 Athens
Ireland: Iliad Hering Ltd., 104 17 Dublin
Italy: Phonak Italia S.p.A., 20139 Milano
Japan: Phonak Japan Ltd., 100-0001 Tokyo
Netherlands: Phonak Nederland B.V., 3720 BA Utrecht
Norway: Phonak Norge AS, 0128 Oslo
Poland: Phonak Polska Sp. z o.o., 01-644 Warszawa
Portugal: Phonak Portugal, Lda, 1170-001 Lisboa
Spain: Phonak España S.A., 28002 Madrid
Sweden: Phonak Sverige AB, 171 82 Stockholm
Switzerland: Phonak AG, 8000 Zurich
The Netherlands: Phonak NV, 1118 CA Amsterdam
United Kingdom: Phonak Ltd., 1118 CA Amsterdam

Guarantee

Your local Phonak agent offers you a comprehensive guarantee of one year from the date of purchase. This covers all manufacturing and material defects. Defects arising from improper handling of the instrument and repairs made by an unauthorised party are not covered by the guarantee.

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