

Controls



inspiro


- ① On/Off slider
- ② Color LCD
- ③ Soft Keys
- ④ Keypad lock key
- ⑤ Microphone mute key



- ⑥ 3.5mm audio input
- ⑦ Mini USB for charging



- ⑧ Button to attach the belt clip and the lavalier
- ⑨ Cable wrapper

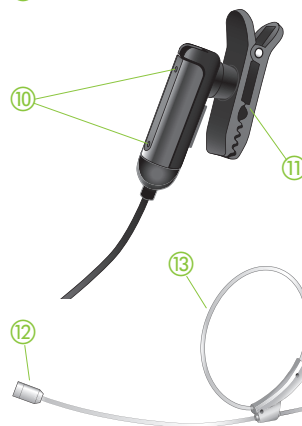

iLapel microphon

- ⑩ Microphone openings (do not cover!)
- ⑪ Wearing clip



iBoom microphone

- ⑫ Adjustable ear loop
- ⑬ Microphone (do not cover!)

Getting Started

Charging inspiro

inspiro contains a re-chargeable built-in state-of-the-art lithium polymer battery with fast-charge capability. A fully-discharged battery takes one hour for an 80% charge and two hours for a 100% charge.

Select an easily accessible socket outlet and plug the charging unit into the socket outlet.



Connect the MiniUSB plug from the charging unit to the inspiro.

⚠ Use only the original Phonak charging unit for inspiro.

The segments in the battery symbol flash during charging. The battery cannot be over-charged.



If the charging unit is removed from the inspiro, the segments in the battery symbol show the actual capacity.



If the battery is fully charged, the battery symbol in the LCD shows four segments.

Getting Started

Low battery alarms


inspiro provides an automatic vibra alarm if the battery is low.

- **Approximately 1 hour before battery is empty:**
A vibra alarm appears. Check the message on the LCD and follow the instructions.
- **Approximately 15 minutes before battery is empty:**
If inspiro has not been charged after the first alarm, the second vibra alarm is automatically activated before the battery is completely exhausted. Check the message on the LCD and follow the instructions. If the battery is not being charged at that time, inspiro switches off automatically after approx. 15 minutes.

If the battery is completely exhausted, there are no segments in the battery symbol anymore and the battery symbol in the LCD changes its color from white to red.

Wearing inspiro

inspiro can either be worn on the belt by using the clip...

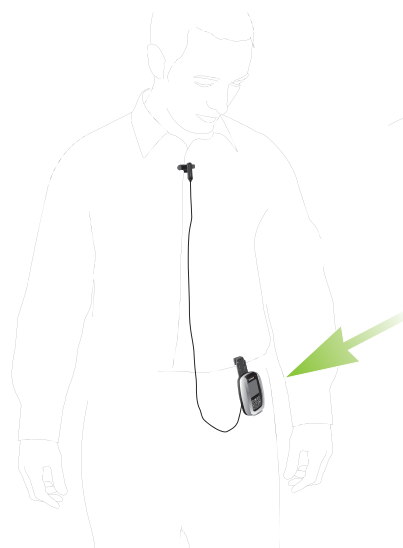

(see page 18)


...or around the neck by using the lavalier




(see page 20)

Getting Started


Attaching the clip to the inspiro:

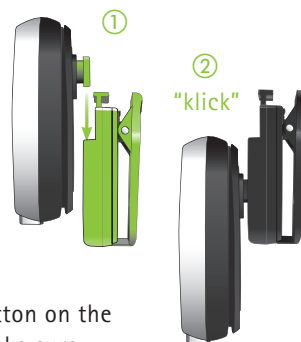

Attach the clip to the button on the backside of the inspiro. Make sure the clip holds securely on the inspiro.

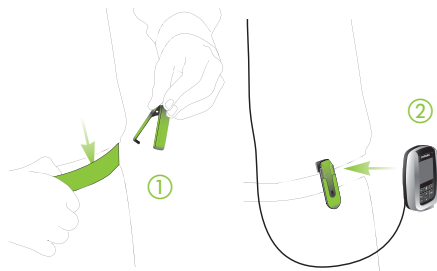


The clip can also be fixed on the belt or trousers before the inspiro is attached. Make sure the clip holds securely on the inspiro.

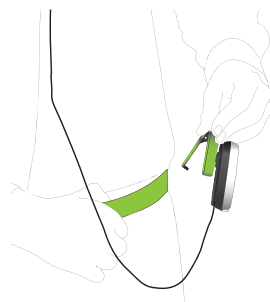

Fixe the clip on the belt or trousers. Make sure the clip holds securely on the inspiro.


To remove the clip from the inspiro, press the button on the clip and carefully remove the clip from the inspiro.

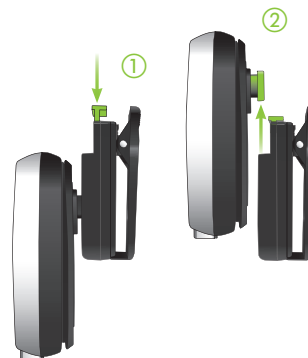

Getting Started

Attaching the lavalier to the inspiro:

Attach the lavalier to the button

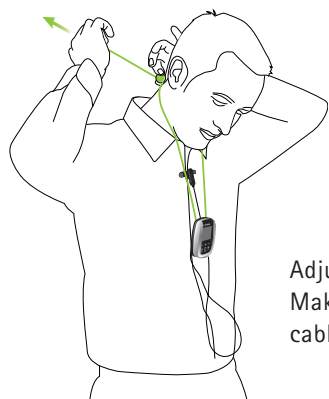

Adjust the lavalier length
Make sure the microphone
cable is unwinded.

17


Wearing iLapel

iLapel is an easy-to-use directional lapel
microphone which provides excellent sound quality.

a) Unwind the iLapel from the inspiro.



b) Fix the iLapel on your
shirt and make sure
the distance between the
microphone and your
mouth is approximately
8 inches (20 cm).

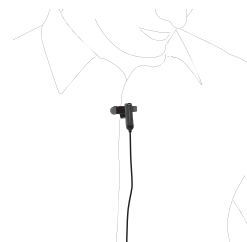

18

Getting Started

iLapel wearing position:




If you are uncertain whether the iLapel is positioned well, activate the Sound check feature in inspiro. Sound check helps to position the iLapel correctly.

Wearing iBoom

iBoom is an ultra-lightweight boom microphone. Extremely comfortable in wear, it can be worn on the left or right ear and adjusts easily to fit all ear and head sizes.


How to fix the iBoom on the ear:

- a) Slightly enlarge the diameter of the ear loop.

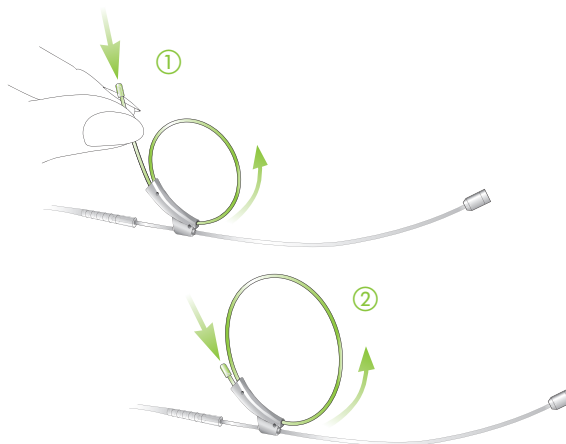


Getting Started


- b) Open the loop carefully and attach the iBoom to the left or right ear.

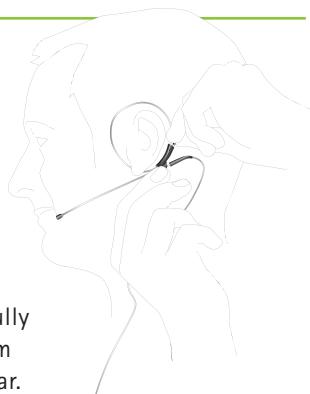

- c) Hold the iBoom with one hand and slightly pull the black nipple with the other hand to reduce the diameter of the ear loop until the iBoom holds securely.

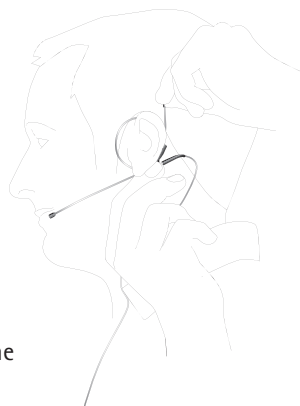


- d) Adjust the boom arm in order to bring the microphone as close to the mouth as possible.

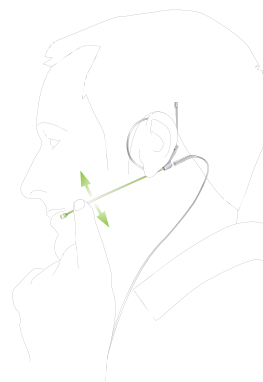

- e) Fix the clip on your clothes as depicted and make sure the cable is not stretched between the clip and the ear.

⚠ Do not bend the boom arm!

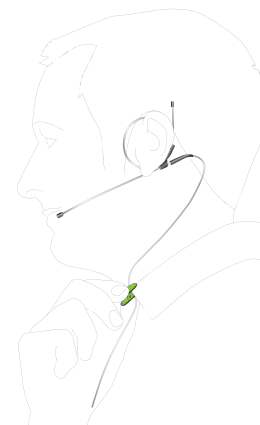

Getting Started

Switch on inspiro

To switch on, push the slider to the left until the green mark appears and the LCD starts up.

To save power, the backlight of the inspiro LCD is switched off after a certain time. Press any button to activate the LCD backlight.

To switch off inspiro, push the slider to the right.



⚠ The inspiro can also be switched off if the keypad is locked!

Activate the key pad lock

It is recommended to activate the key pad lock whenever possible.



To activate the key pad lock, go to the highest menu level as shown above and press the key showing the key symbol, then press OK. A key symbol appears in the status bar.



⚠ If the key pad lock is activated, the six keys are locked. Please note that the on/off slider is never locked and can always be used!

To unlock the keys, proceed the same way.