Angelcare®

Safety 1st

No. 49255

Angelcare™ MD

*Sound & Movement
Nursery Monitor*

Owner's Manual

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Safety 1st

Introduction:

Thank you for purchasing the Angelcare® Sound & Movement Nursery Monitor. This reliable and easy to use product monitors your baby's movements and alerts you if movement stops for more than 20 seconds.

This unique product consists of three main components: Motion Sensor Pad, Baby's Unit, and portable Parent's Unit. The Motion Sensor Pad is inserted underneath the baby's mattress to detect your baby's movements — even the slightest breathing movements. The Baby's Unit transmits your baby's sounds and movements to the Parent's Unit, giving you peace of mind even when your baby is sleeping.

Additional features include sound only monitoring, or sound and motion monitoring. A two-channel frequency selection ensures clear reception. The audible "Tack" option provides parents with further assurance by sounding as baby's breathing movements are detected.

Full user instructions are provided. Please read all the instructions before using the Angelcare® Sound & Movement Nursery Monitor. Keep this booklet handy, preferably in the baby's room for quick reference.

IN CANADA: The carrier frequency on channel A is 49.830 MHz and the carrier frequency on channel B is 49.890 MHz. The field strength is 80 dB microvolts per meter at 3 meters.

OPERATION IS SUBJECT TO THE FOLLOWING TWO CONDITIONS: (1) THIS DEVICE MAY NOT CAUSE INTERFERENCE, AND (2) THIS DEVICE MUST ACCEPT ANY INTERFERENCE, INCLUDING INTERFERENCE THAT MAY CAUSE UNDESIRABLE OPERATION OF THE DEVICE.

Safety Warnings:

- THE ANGELCARE™ SOUND & MOVEMENT MONITOR IS NOT A MEDICAL DEVICE AND IS NOT A DEVICE TO PREVENT CASES OF SUDDEN INFANT DEATH SYNDROME (OR SIDS). MONITORING OF PREMATURE BABIES, OR THOSE CONSIDERED TO BE AT RISK, SHOULD ONLY BE CARRIED OUT UNDER THE SUPERVISION OF YOUR DOCTOR OR HEALTH PROFESSIONAL.
- THIS MONITOR SHOULD NOT BE USED AS A SUBSTITUTE FOR DIRECT SUPERVISION OF YOUR CHILD BY A RESPONSIBLE ADULT. YOU SHOULD CHECK YOUR CHILD'S ACTIVITY AT REGULAR INTERVALS.
- THE MOTION SENSOR PAD MAY DETECT MOTION FROM MANY SOURCES OF VIBRATION FROM BOTH INSIDE AND OUTSIDE YOUR BABY'S ROOM. SOME EXAMPLES OF VIBRATION SOURCES ARE FANS, WASHING MACHINES, TRAINS, AIRPLANES, LOUD MUSIC, ETC. BE ESPECIALLY CAREFUL TO INSURE THAT ALL VIBRATION SOURCES ARE ELIMINATED BEFORE USING YOUR MONITOR.

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CAUTION
RISK OF ELECTRIC SHOCK
DO NOT OPEN!

- TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE COVER OR BACK OF EITHER UNIT, EXCEPT FOR BATTERY REPLACEMENT.
- NEVER TOUCH THE END OF THE ADAPTER CORD WHEN THE ADAPTER IS PLUGGED INTO AN ELECTRICAL OUTLET.
- DO NOT EXPOSE TO WATER OR MOISTURE.
- NO SERVICEABLE PARTS INSIDE. REFER ALL SERVICE QUESTIONS TO SAFETY 1st CONSUMER RELATIONS.
IN CANADA, PLEASE CALL (800) 721-3389
IN EUROPE CALL (44) 181 236 0707

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Safety 1st

Instructions:

READ INSTRUCTIONS: All the instructions should be read before the monitor is operated.

RETAIN INSTRUCTIONS: The instructions should be retained for future reference.

HEED WARNINGS: All warnings on the monitor and in the Owner's Manual should be adhered to.

FOLLOW INSTRUCTIONS: All operating instructions should be followed.

WATER AND MOISTURE: The monitor should not be used near water, for example, near a bath tub, wash bowl, kitchen sink, laundry tub, in a wet basement, near a swimming pool, etc.

VENTILATION: The Parent's Unit and Baby's Unit should be placed so that their location or position does not interfere with proper ventilation. For example, they should never be situated on a bed, sofa, rug or similar surface that may block ventilation.

HEAT: The monitor should be kept away from heat sources such as radiators, heat registers, stoves or other appliances that produce heat.

AC POWER SOURCE: The monitor should be connected only to the AC Adapters supplied with the unit.

POWER CORD PROTECTION: The power supply cord should be routed so that it is not likely to be walked on or pinched by items placed upon or against it. Pay particular attention to the cord at the plug, receptacle, and at the point it exits from the AC adapters.

CLEANING: Use a dry cotton cloth to keep the monitor free from dust. Disconnect the units before cleaning them. Do not immerse in water. Do not spray with cleaning products or solvents; use a dry, lint-free cloth only. The Motion Sensor Pad can be cleaned by wiping with a cloth dampened very slightly with an antiseptic or mild detergent solution.

NON-USE PERIODS: The power cords of the monitor should be unplugged from the outlet when unused for long periods of time. The battery should also be removed during these periods.

OBJECT AND LIQUID ENTRY: Care should be taken so that objects do not fall onto the product, and liquids are not spilled into either unit or sensor pad through the openings.

DAMAGE REQUIRING SERVICE: The monitor should be serviced by qualified personnel when:

- A: The power supply cord or plug has been damaged;
- B: Objects have fallen or liquid has been spilled onto the product;
- C: The monitor does not appear to operate normally or exhibits a marked change in its performance;
- D: The monitor has been dropped or either unit or the sensor pad has been damaged.

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Antenna - Circular antenna for clear reception and easy handling.

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Power On Indicator - Green light appears indicating power is on.

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Channel Selector (A/B Switch) - Two channels available for clear reception.

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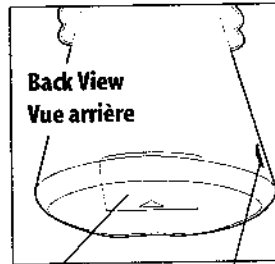
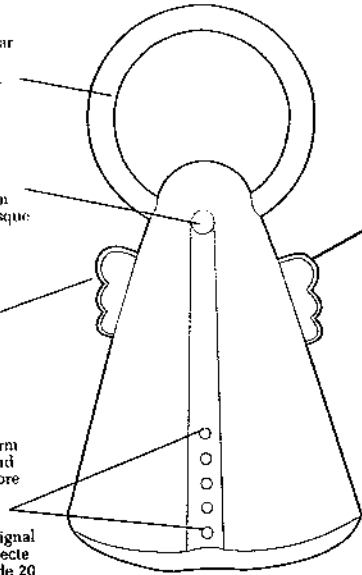
Speaker - Delivers sounds or Alarm from Baby's Unit. Alarm will sound when motion is undetected for more than 20 seconds.

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Power On/Off Switch With Volume Control - Power control allows for three volume settings with a range of up to 200 feet (60 m) from Nursery Unit.

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
Back View
Vue arrière

Battery Compartment - Compartment allows for one 9-volt battery.

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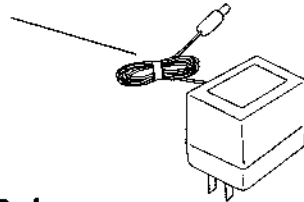
Detects baby's slightest movements - even breathing movements.

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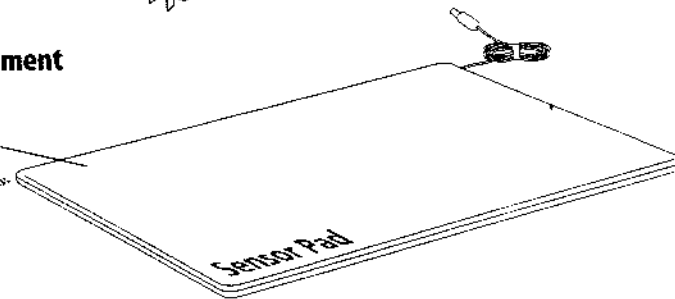
Safety 1st

Setting Up Your Monitor

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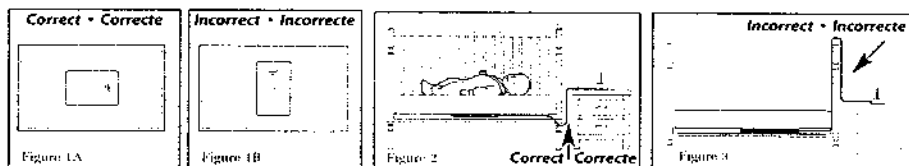Step 1 To position the Motion Sensor Pad, lift up the mattress and place pad, logo side up and lengthwise, directly under where your baby sleeps.

Important Notes:

- To ensure the best monitoring, it is critical that the Motion Sensor Pad is positioned lengthwise in the center of the crib (Figure 1A). Do not position the Motion Sensor Pad widthwise (Figure 1B).
- For the monitor to operate properly, the Motion Sensor Pad must be in close contact with the mattress and rest on a firm surface.
- Make sure there are no bedclothes tucked under the mattress and in between the mattress and the sensor pad. Also, if the mattress support is too soft – i.e. if there is a boxspring or underpadding – it will be necessary to place a board under the Motion Sensor pad to provide a firm surface for it. Ideally, the board should be sized to fit the crib. At a minimum, the board should be larger than the sensor pad.

Step 2 Place the Baby's Unit on a sturdy surface no more than 10 feet (3 meters) from the crib and position with the microphone toward the crib.

Step 3 Plug the Motion Sensor Pad cord into the Motion Sensor Pad Jack located in the back of the Baby's Unit.

⚠ WARNING: Make sure cord is totally out of baby's reach as shown above in Figure 2. See Figure 3 for incorrect positioning.

Step 4 Plug AC Adapter into AC Adapter Jack located in the back of the Baby's Unit.

Step 5 Plug AC Adapter into wall outlet.

Step 6 On Baby's Unit, select channel setting A.

Step 7 Set the Power Selector and Tick Function to the Off position.

Step 8 Plug AC Adapter into AC Adapter Jack on side of Parent's Unit for use with household power.

Step 9 Plug AC Adapter into wall outlet.

Step 10 For battery operation, remove the battery cover underneath Parent's Unit. Install a 9-volt alkaline battery.

Step 11 Select Channel A and set the power switch to Off.

Important Note:

There is no low battery indicator on the Parent's Unit. The battery should be replaced if you note a decrease in the intensity of the Power On Indicator or the quality of the reception. If you do not intend to use the monitor for an extended period of time, remove the battery to prevent it from leaking.

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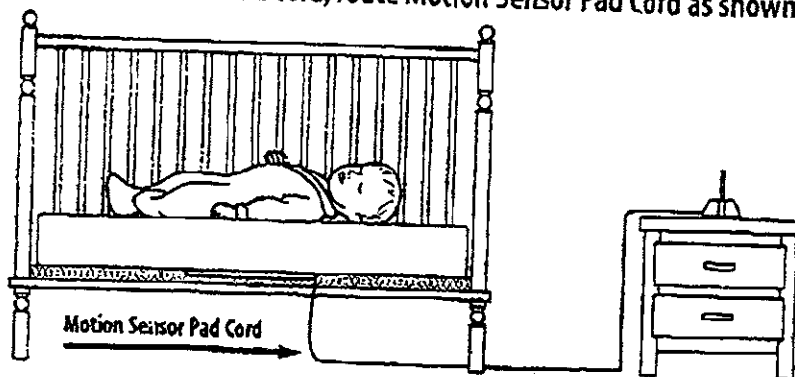
Safety 1st Angelcare[™]

No. 49255 US

Addition to page 8. If head board or foot board of crib has openings which could allow the child to reach the cord, route Motion Sensor Pad Cord as shown below.

! WARNING: Possible entanglement or strangulation injury. Keep all cords out of baby's reach.

! WARNING: Motion Sensor Pad performance could be influenced by environmental factors such as air currents from fans and air conditioners or mechanical vibrations from washing machines, airplanes, trains, etc. Vibration could be sensed by the Motion Sensor Pad and may prevent or delay alarm. For this reason vibration sources must ALWAYS be eliminated when using this monitor.

! WARNING: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is needed.
- Consult the dealer or an experienced radio/TV technician for help

Testing Your Monitor

Step 1 Make sure both units are on Channel A and turned on.

Step 2 Without baby in the crib, switch Baby's Unit power switch to Sound & Motion and the Tick function ON. Switch the Parent's Unit to ON position as well. After about 15 seconds, a loud tick will be heard from the Baby's Unit and on the Parent's Unit. The loud tick will be followed by the alarm and a flashing red light indicator five seconds later on the Baby's Unit. If the alarms on Parent's Unit and Baby's Unit do not sound, you may need to raise the sensitivity. See Sensitivity Adjustment Instructions.

Step 3 To stop the alarm, place your hand lightly on the mattress or place your baby in the crib. The red light will continue to flash for approximately another minute and then stop. You can also stop the alarm by setting the main switch to Off. The movement test has now been completed.

Important Note:

If the alarm does not sound, make sure all wires are properly connected (see Setting Up Your Monitor section).

Important Note:

Continuous vibrations from the floor can delay or prevent the alarm from sounding. You may then have to place the crib in the corner of the room or near a solid supporting wall. A strong draft (from a fan for example) may also delay or prevent the alarm from sounding, so make sure that the crib is not in a strong draft. You may also have to place the Motion Sensor Pad on a rigid board underneath the mattress.

Step 4 To test the sound, switch the Baby's Unit power switch to Sound. Set the volume of the Parent's Unit to the desired level. Ask someone to speak near the Baby's Unit or place a sound source nearby (such as a radio). Test the Parent's Unit both inside and outside your home to ensure proper sound is being received. The sound test has now been completed.

Step 5 Test Channel B by switching to Channel B on both units and repeating Step 4.

Important Note:

Check that the monitor is operating properly before first-time use, and then on a periodic basis. It is important to do so to ensure that normal room vibrations do not prevent the alarm from sounding.

Using The Monitor

Sound Only:

Step 1 Set the Power Selector on the Baby's Unit to Sound. The Power On Indicator light will come on.

Step 2 Set the volume of the Parent's Unit to the desired level.

Step 3 Check which channel, A or B, gives the best results. Make sure both the Baby's Unit and the Parent's Unit are on the same channel. If there is interference on both channels, refer to the Troubleshooting section.

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Sound & Motion:

- Step 1** Once you have made sure that the monitor is operating properly (in Sound only mode), place your baby in the crib directly over the Motion Sensor Pad.
- Step 2** Set the Power Selector on the Baby's Unit to Sound & Motion. The Power On Indicator light will come on. The Alarm Indicator and Motion Indicator on the Baby's Unit will come on for about a second each. The Motion indicator then begins to flash as it detects your baby's movements.
- Step 3** Adjust the volume of the Parent's Unit by placing the Power On/Off Switch with Volume Control to the desired volume level.
- Step 4** Set the Tick Function to On or Off, as desired.

Alarm: If movement or breathing movement is not being captured properly, the red light will flash in time with the green. The red and green lights flashing together may occur normally if your baby is in a very deep sleep, and/or baby has moved away from the sensor pad. If the red and green lights flash constantly when baby has just gone to sleep, it indicates that the monitor is having difficulty capturing movements and you should consult the sensitivity adjustment section.

Important Note:
To avoid needlessly sounding the alarm, get into the habit of always resetting the switch to Sound or Off before removing your baby from the crib. A prewarning tick will sound 15 seconds after baby has been removed from the crib to inform you to turn off the Baby's Unit. 5 Seconds following the loud tick, the alarm will sound.

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Sensitivity Adjustment Instructions

This adjustment, located on the bottom of Baby's Unit, has been pre-set at the factory to accommodate most motion monitoring conditions. However, in certain circumstances, the motion monitoring situation is not ideal. For example, if you are trying to monitor the movement of a newborn baby through a very thick, heavy mattress, it may be necessary to increase the sensitivity to ensure better detection of baby's movements. Do not increase the sensitivity unless you are experiencing false alarms and have already followed the advice in the False Alarms Section.

Before attempting to set the sensitivity, wait until baby has settled and is asleep. Using a small screwdriver or your fingers, increase the sensitivity by turning the screw or adjuster knob slowly clockwise until the green light is flashing on its own. When baby's movements increase as baby gets older, it may be desirable to reduce the sensitivity level.

Care should be taken not to have the sensitivity adjustment set any higher than necessary. In the event that the alarm does not sound when there is no movement (see Testing Your Monitor section), the sensitivity level needs to be decreased. After lowering the sensitivity slightly, perform another movement test as described in the Testing Your Monitor section. If the alarm still is not sounding at the lowest sensitivity setting, consult the Troubleshooting section or call Safety 1st.

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False Alarm

The alarm may sound list of possible causes

Problem

False Alarm

Troubleshoot

Problem

No Transmission

Very weak reception

Static distortion interference

Feedback

Red and green lights flashing together

False Alarms

The alarm may sound even if your baby is moving normally. The following is a list of possible causes and ways to correct them:

| Problem | Possible Cause | Solution |
|-------------|--|---|
| False Alarm | <ul style="list-style-type: none"> You removed your baby from the crib but left the Power Selector Sound & Motion on. The Motion Sensor Pad Cord is improperly connected to the Sensor Pad Cord Jack. Your baby has moved and is not lying directly above the Motion Sensor Pad. The Motion Sensor Pad is not in full contact with the mattress. | <ul style="list-style-type: none"> Set the Power Selector to the Sound setting or to the Off position. Reconnect the cord by pushing it in fully. Make sure that the Motion Sensor Pad is positioned lengthwise in the crib and directly under the baby. Make sure that there are no blankets or bedding lifting the mattress and separating it from the Motion Sensor Pad. If your mattress rests on a boxspring, then it is necessary to place a rigid board under the Motion Sensor Pad. |

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Troubleshooting

| Problem | Possible Cause | Solution |
|--|--|--|
| No Transmission | <ul style="list-style-type: none"> Battery is dead or improperly installed. AC adapters are improperly connected. Parent's Unit is out of range. One of the units is not on. | <ul style="list-style-type: none"> Replace battery. Check connections. Rotate the Parent's Unit or move it closer to the Baby's Unit. Check switches. |
| Very weak reception | <ul style="list-style-type: none"> Parent's Unit is out of range. Parent's Unit or Baby's Unit placed near a large mass of metal. Parent's Unit or Baby's Unit has tipped over. Low Battery. | <ul style="list-style-type: none"> Rotate the Parent's Unit or move it closer to the Baby's Unit. Move the Parent's Unit or Baby's Unit. Ensure that units are upright. Replace the battery. |
| Static, distortion, interference | <ul style="list-style-type: none"> Interference with other monitors, wireless phones, walkie-talkies, etc. Parent's Unit is located near motorized devices, fluorescent lights, television, etc. Parent's Unit is out of range. | <ul style="list-style-type: none"> Make sure both units are on the same channel. If interference is still occurring, switch the channel on both units. Eliminate the cause or move the Parent's Unit away from the source of interference. Rotate the Parent's Unit or move it closer to the Baby's Unit. |
| Feedback | <ul style="list-style-type: none"> Parent's Unit is located too close to Baby's Unit. Volume is too high. | <ul style="list-style-type: none"> Move the Parent's Unit farther away. Decrease volume. |
| Red and green lights flashing together | <ul style="list-style-type: none"> Baby is in a very deep sleep. Baby has moved away from sensor. Monitor is having difficulty capturing movements. | <ul style="list-style-type: none"> Consult sensitivity adjustment section. Reposition baby directly over sensor pad. Consult sensitivity adjustment section. |

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ONE (1) YEAR LIMITED WARRANTY

Safety 1st Inc. warrants to the original owner of this product that the Angelcare™ Sound & Movement Nursery Monitor is free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If this monitor fails to function properly when used as directed under normal conditions, within the one (1) year period, Safety 1st Inc. will either repair or replace the product at our option, free of charge, when it is returned complete and in its original packaging, prepaid postage, and insured to Safety 1st Canada Inc., 804 rue Deslauniers, Ville St. Laurent, Québec H4N 1X1, Safety 1st Europe Ltd., Isopad House, Shenley Road, Borehamwood, Hertfordshire, WD6 1TE Great Britain. The product must be accompanied by a proof of purchase, either a bill of sale, receipted invoice, or other proof that the Monitor is within the warranty period. Safety 1st Inc. will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a Monitor that has been damaged as a result of improper maintenance, accidents, improper voltage supply or other misuse. The warranty is also void if owner repairs or modifies the product in any way. Safety 1st Inc. is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranties are given.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

IMPORTANT NOTE: ONLY USE THE ADAPTERS SUPPLIED FOR THIS NURSERY MONITOR. THE USE OF OTHER ADAPTERS WILL VOID THIS WARRANTY.

IMPORTANT!

Please do not return this product to the store!
Your questions and comments are important to us!

Thank you for purchasing this quality product from Safety 1st. We are confident you will be pleased with its many features and performance. If you are having any problems assembling, installing, or using this product, please call us. We will be very happy to answer any questions that you may have. Many times there is a simple solution to any problem that may occur. Talking to us directly will be convenient and save you time.

Call our Consumer Relations
In Canada, please call (800) 721-3389. In Europe, call (44) 181 236 0707
We will gladly answer any and all of your questions about this product.

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