

# Contents

## 1

<b>The Basics</b> .....	<b>1-1</b>
<b>About This Manual</b> .....	<b>1-3</b>
Manual Purpose .....	1-3
Intended Audience .....	1-3
Revision History .....	1-3
<b>Manual Conventions</b> .....	<b>1-4</b>
Product References .....	1-4
Definitions .....	1-4
Illustrations and Names .....	1-5
<b>Common Operations</b> .....	<b>1-6</b>
“Clicking” the Mouse .....	1-6
Radio Buttons .....	1-6
Check Boxes .....	1-7
Scroll Bars .....	1-7
Popup Lists .....	1-8
<b>Unit Defaults Worksheet</b> .....	<b>1-9</b>
Telemetry Alarm Control Defaults .....	1-9
Telemetry Unit Defaults .....	1-10

## 2

<b>Safety</b> .....	<b>2-1</b>
<b>For Your Safety</b> .....	<b>2-3</b>
Intended Use .....	2-3
Definitions .....	2-3
System Safety .....	2-4
Reference Literature .....	2-12
Classification .....	2-13
Underwriters Laboratories, Inc. ....	2-13
FCC Compliance Information Statement .....	2-14
Equipment Symbols .....	2-15

# 3

## Equipment Overview . . . . . 3-1

### Introduction . . . . . 3-3

### ApexPro Telemetry System . . . . . 3-4

Compatibility with Bedside Monitors . . . . . 3-5

ApexPro Transmitters . . . . . 3-6

ApexPro CH Transmitter . . . . . 3-10

ApexPro FH Transceiver . . . . . 3-11

Apex Oximeter SpO2 Module . . . . . 3-12

Xpod™ Oximeter . . . . . 3-13

Accutacker DX Noninvasive Blood Pressure (NBP) Monitor . . . . . 3-14

DINAMAP® PRO Series Monitors . . . . . 3-16

Antenna System . . . . . 3-17

Receiver System . . . . . 3-17

Unity Network . . . . . 3-17

CIC Pro Clinical Information Center . . . . . 3-18

# 4

## Connection . . . . . 4-1

### ApexPro Transmitter Setup . . . . . 4-3

ApexPro Transmitter Battery Installation . . . . . 4-3

Battery Functional Life . . . . . 4-4

ApexPro Transmitter Leadwires . . . . . 4-5

### Apex Oximeter Setup . . . . . 4-7

Apex Oximeter Battery Installation . . . . . 4-7

Battery Functional Life . . . . . 4-7

Apex Oximeter Connections . . . . . 4-8

Xpod Oximeter Connection . . . . . 4-9

### Accutacker DX Setup . . . . . 4-10

Tips for Monitoring . . . . . 4-10

Battery Installation . . . . . 4-10

Accutacker DX Functional Life . . . . . 4-11

Accutacker DX Connections . . . . . 4-12

### DINALink™ Serial Cable . . . . . 4-16

### Interconnection Cables . . . . . 4-17

# 5

<b>Maintenance</b> .....	<b>5-1</b>
<b>Biocompatibility</b> .....	<b>5-3</b>
<b>Inspection</b> .....	<b>5-4</b>
<b>Cleaning</b> .....	<b>5-5</b>
General Cleaning .....	5-5
Cleaning the Transmitters .....	5-6
More Intensive Disinfecting or Sterilization .....	5-8
Cleaning the Power Unit .....	5-8
<b>Transmitter and Leadwire Storage</b> .....	<b>5-9</b>
Storage Guidelines .....	5-9
<b>Technical Maintenance</b> .....	<b>5-10</b>
Technical Specifications .....	5-10

# 6

<b>Telemetry Setup</b> .....	<b>6-1</b>
<b>Introduction</b> .....	<b>6-3</b>
<b>Telemetry Factory Default Settings</b> .....	<b>6-4</b>
<b>Service Password</b> .....	<b>6-5</b>
<b>Telemetry Unit Defaults</b> .....	<b>6-7</b>
Viewing Telemetry Unit Defaults .....	6-7
<b>CIC Defaults</b> .....	<b>6-14</b>
Adjusting Alarm Volume .....	6-16
<b>Current Telemetry Listings</b> .....	<b>6-17</b>
<b>Full Disclosure Defaults</b> .....	<b>6-19</b>

# 7

<b>Admit/View a Patient</b> .....	<b>7-1</b>
<b>About Admitting</b> .....	<b>7-3</b>
Switching Transmitters .....	7-3
Terminology .....	7-3
<b>Admit Instructions</b> .....	<b>7-5</b>
Admit Procedure .....	7-5
<b>Discharge Instructions</b> .....	<b>7-8</b>
New Patient Button .....	7-9
Clearing a Patient Window .....	7-10
<b>Viewing a Patient</b> .....	<b>7-11</b>
Sample .....	7-11
Relearn .....	7-12
<b>Viewing Another Patient</b> .....	<b>7-13</b>
Viewing in the Single Patient Viewer .....	7-13
Viewing in the Multiple Patient Viewer .....	7-14
<b>Viewing Patients Through Alarm Condition Indicators</b> .....	<b>7-16</b>

# 8

<b>Alarm Control</b> .....	<b>8-1</b>
<b>Alarm Structure</b> .....	<b>8-3</b>
Patient Status Alarms .....	8-3
System Status Alarms .....	8-4
<b>Alarm Control Tab</b> .....	<b>8-5</b>
Accessing the Alarm Control Tab .....	8-5
Parameter Limits .....	8-6
Parameter Alarm Levels .....	8-7
Arrhythmia Alarm Levels .....	8-8
Alarms On/Off .....	8-9
Recalling Unit Defaults .....	8-12
Alarm Help .....	8-12
Printing Alarm Settings .....	8-13
Silencing Alarms .....	8-13
<b>Alarm Pause Breakthrough</b> .....	<b>8-14</b>
<b>Unit Default Settings for Alarms</b> .....	<b>8-16</b>
Telemetry Alarm Control Defaults .....	8-16

# 9

<b>Printing</b> .....	<b>9-1</b>
<b>Initiating a Graph</b> .....	<b>9-3</b>
Automatic Alarm Graphs .....	9-3
Transmitter Initiated Graphs (Manual Graphs) .....	9-4
Graph Messages .....	9-5
<b>Graph All Patients</b> .....	<b>9-6</b>
Initiating a Graph All Patients Request .....	9-7
<b>Graph Location Defaults</b> .....	<b>9-8</b>
<b>Stopping a Graph</b> .....	<b>9-9</b>
<b>Graph Paper Out Indicator</b> .....	<b>9-10</b>
<b>Graph Setup Tab Sheet</b> .....	<b>9-11</b>
Graph Waveforms Controls .....	9-12
Graph Location Controls .....	9-13
Graph Speed Controls .....	9-15
<b>Laser Printer</b> .....	<b>9-16</b>

# 10

<b>Patient Data</b> .....	<b>10-1</b>
<b>Introduction</b> .....	<b>10-3</b>
<b>Alarm Histories</b> .....	<b>10-4</b>
Event Directory .....	10-5
Event Window .....	10-5
Alarm Histories Buttons .....	10-6
<b>Graphic Trends</b> .....	<b>10-8</b>
Trend Directory Window .....	10-9
Cursor .....	10-10
Time Resolution Menu .....	10-11
Graphic Trends Buttons .....	10-11
Printing Graphic Trends .....	10-13

<b>Vital Signs</b> .....	<b>10-14</b>
Sort Mode .....	10-15
Increment .....	10-15
Scroll Bars .....	10-16
Printing Vital Signs .....	10-16
<b>Full Disclosure</b> .....	<b>10-17</b>
Full Disclosure Options .....	10-18
<b>Time Focus</b> .....	<b>10-20</b>

# 11

<b>ECG Monitoring</b> .....	<b>11-1</b>
<b>Introduction</b> .....	<b>11-3</b>
Data Synchronization .....	11-3
<b>Patient Preparation</b> .....	<b>11-4</b>
Skin Preparation .....	11-4
Electrode Placement .....	11-4
<b>Special Considerations for 6-Lead Monitoring</b> .....	<b>11-10</b>
<i>V FAIL</i> Message .....	11-10
Relearn .....	11-10
<b>ECG Monitoring</b> .....	<b>11-11</b>
ECG in the Multiple Patient Viewer .....	11-11
ECG in the Single Patient Viewer .....	11-12
ECG Limits .....	11-13
ECG Artifact .....	11-14
<b>ECG Tab Sheet</b> .....	<b>11-15</b>
Accessing the ECG Tab Sheet .....	11-15
Display Lead .....	11-16
Relearn .....	11-16
Size .....	11-16
Detect Pace .....	11-17
Lead Analysis .....	11-22
Arrhythmia .....	11-24
Full Arrhythmia Conditions .....	11-26
Lethal Arrhythmia Conditions .....	11-29
PVC Limit .....	11-29
ST .....	11-29
Va Lead and Vb Lead .....	11-29
AFIB Identification .....	11-30
<b>ST Analysis</b> .....	<b>11-31</b>
ST Deviation Alarm .....	11-32

<b>Arrhythmia Troubleshooting</b> .....	<b>11-33</b>
<b>Pacemaker Troubleshooting</b> .....	<b>11-35</b>
Interface Connector Ports .....	11-35
<b>ST Troubleshooting</b> .....	<b>11-37</b>

# 12

<b>SpO2 Monitoring</b> .....	<b>12-1</b>
<b>Introduction</b> .....	<b>12-3</b>
SpO2 Probe Safety .....	12-4
<b>Infants and Pulse Oximetry</b> .....	<b>12-5</b>
<b>Signal and Data Validity</b> .....	<b>12-6</b>
Signal Strength Indicator .....	12-6
Error Messages .....	12-7
<b>SPO2 Monitoring</b> .....	<b>12-8</b>
SPO2 in the Multiple Patient Viewer .....	12-8
SPO2 in the Single Patient Viewer .....	12-9
SPO2 Limits .....	12-9
<b>SPO2 Tab Sheet</b> .....	<b>12-10</b>
Accessing the SPO2 Tab Sheet .....	12-10
Rate .....	12-11
Size .....	12-11
<b>Troubleshooting</b> .....	<b>12-12</b>
SpO2 Messages .....	12-12

# 13

<b>NBP Monitoring</b> .....	<b>13-1</b>
<b>Introduction</b> .....	<b>13-3</b>
<b>Safety Considerations</b> .....	<b>13-4</b>
<b>Programming the Blood Pressure Monitor</b> .....	<b>13-5</b>
Setting the Measurement Interval .....	13-5
Setting Test Parameters .....	13-6
Setting Limits .....	13-8
Software and Hardware Versions .....	13-8
<b>Patient Preparation</b> .....	<b>13-9</b>
Microphone Placement .....	13-11

<b>NBP Monitoring</b> .....	<b>13-13</b>
NBP in the Multiple Patient Viewer .....	13-13
NBP in the Single Patient Viewer .....	13-13
NBP Limits .....	13-14
<b>Pressures Tab Sheet</b> .....	<b>13-15</b>
Accessing the Pressures Tab Sheet .....	13-15
Auto .....	13-16
Cuff Size .....	13-16
Clear Message .....	13-16
<b>Troubleshooting</b> .....	<b>13-17</b>
NBP Status Messages .....	13-17
<b>Message Glossary</b> .....	<b>A-1</b>
<b>Message Glossary</b> .....	<b>A-3</b>
Alarm Messages .....	A-3
Graph Messages .....	A-5
Transmitter-Related Messages .....	A-6
System Status Messages .....	A-6
Patient Status Messages .....	A-7
<b>Supplies</b> .....	<b>B-1</b>
<b>Contact Information</b> .....	<b>B-3</b>
<b>Abbreviations and Symbols</b> .....	<b>C-1</b>
<b>Abbreviations and Symbols</b> .....	<b>C-3</b>
Abbreviations .....	C-3
Symbols .....	C-7
<b>ApexPro FH Transceiver</b> .....	<b>D-1</b>
<b>The Basics</b> .....	<b>D-3</b>
About the ApexPro FH Transceiver .....	D-3
Programming Transceivers .....	D-4
<b>Safety</b> .....	<b>D-5</b>
Warnings, Cautions, and Notes .....	D-5

<b>Equipment Overview</b> .....	<b>D-7</b>
ApexPro FH Transceiver Operating Instructions .....	D-7
Push Button Function and Use .....	D-7
Attendant Present/Procedure Alarm Silence (PAS) Unlock Button .....	D-8
Procedure Alarm Silence (PAS) Button .....	D-9
LED Indicators Function .....	D-11
<b>Connections</b> .....	<b>D-13</b>
Connecting the ApexPro FH Transceiver to Accessory Devices .....	D-13
<b>Maintenance</b> .....	<b>D-16</b>
Cleaning .....	D-16
Storage and Usage .....	D-20
<b>Index</b> .....	<b>Index-1</b>

**For your notes**