

**DEXCOM™ STS™ CONTINUOUS GLUCOSE MONITORING SYSTEM USER GUIDE
FOR TRANSMITTER MODEL NUMBER 9300 & RECEIVER MODEL NUMBER 8204**




Starting Continuous Glucose Monitoring

STEP 1- Inserting a New STS™ Sensor

Choose a site on your abdomen (belly) to put the Sensor that is a flat area and where you don't have any scarring or rough patches of skin and clean the site with an alcohol swab. Remove the adhesive backing from the STS™ Sensor Pod and place the STS™ Sensor on the selected site.


1. Pull the safety lock out from the Applicator.
2. Place 2 fingers under the collar, with your thumb on the plunger, and push the plunger down completely.
3. At the same time that you are pushing the plunger down, pull the collar back completely until you hear a click.
4. Press the tabs on the sides of the STS™ Sensor Pod and pull out the STS™ Applicator.

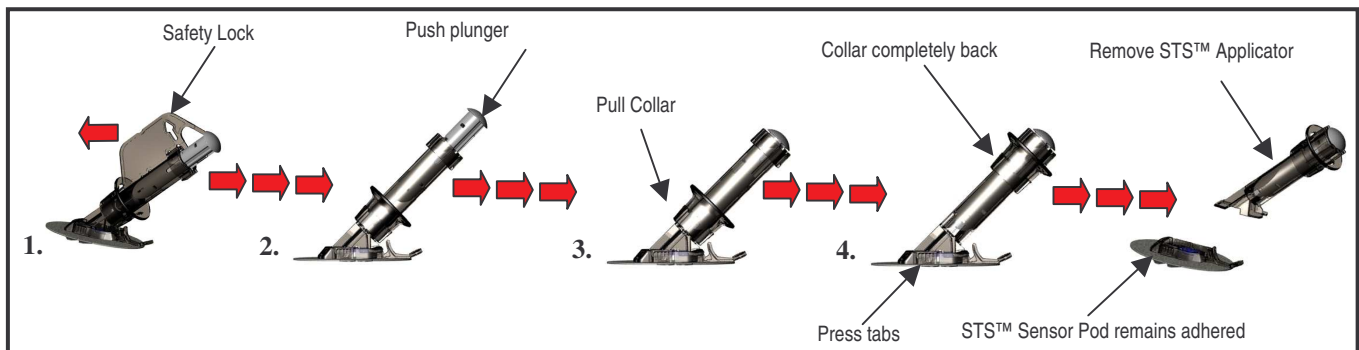

Caution: Investigational device. Limited by Federal (United States) law to investigational use.

STEP 2- Installing Your STS™ Transmitter

- Place the STS™ Transmitter into the STS™ Sensor Pod with the flat side facing down.
- Pull the STS™ Transmitter Latch up onto the STS™ Transmitter to snap it into place.
- Remove the STS™ Transmitter Latch by holding the end of the latch and quickly twisting off the latch away from your body.

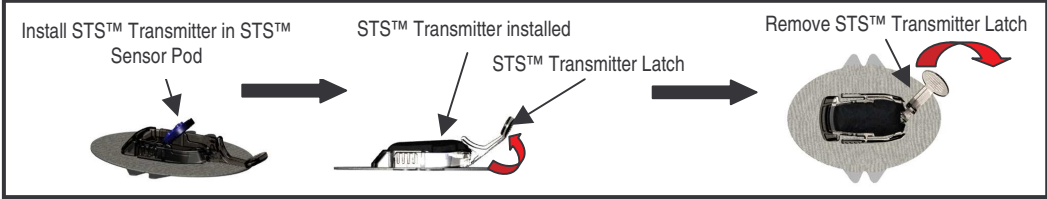


- Press any button on the STS™ Receiver to check for the Antenna icon (Y) before you continue.



STEP 3- Beginning your Continuous Glucose Monitoring Session

- Press the Down ▼ Arrow 3 times until you come to the 9-hour Glucose Trend Screen and then press and hold the Down ▼ Arrow for at least 7 seconds until you see the Sensor Insertion Screen.

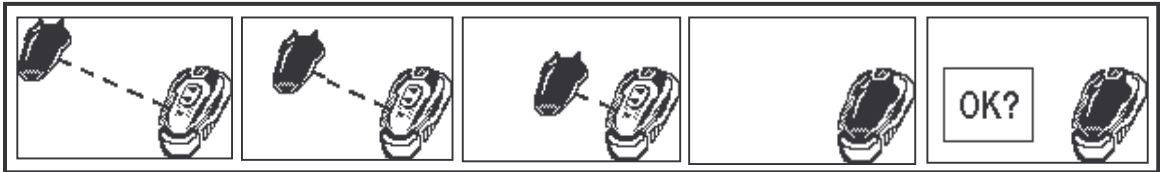


- Press the OK button.

CONTINUOUS MONITORING

Viewing your Glucose Trends

You can view trends and patterns using the 1-hour, 3-hour, and 9-hour glucose screens. (See screens below).

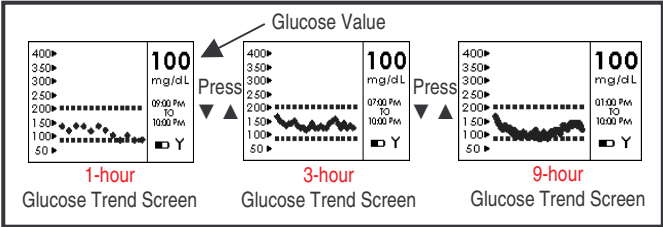

STS™ Receiver Alert Screens

The device may vibrate and/or beep and display one of the following screens during use:


Calibration Update Screen	Out of Calibration Screen	Low Glucose Alert	High Glucose Alert	Low Glucose Alarm	Error Code

Caution: Investigational device. Limited by Federal (United States) law to investigational use.

- If the Calibration Update Screen or the Out of Calibration Screen appears on the STS™ Receiver, clear the prompt, take a BG fingerstick to bring the STS™ System back into calibration.
- When the STS™ Sensor glucose value is above or below the alert settings you have selected (factory settings are 80 and 200), the STS™ Receiver will notify you of a high or low glucose value. Press any button on the STS™ Receiver to acknowledge the alert. Take a fingerstick before you make any treatment decisions. In addition to your personal glucose alert settings, your STS™ Receiver also has an automatic low glucose alarm set at 55mg/dL (3.1 mmol/L).
- The error code indicates the STS™ Receiver is not functioning properly. Note the Error code, use your meter to check your blood glucose readings, and contact Technical Support immediately. Your Continuous Glucose Monitoring Session is finished and you should remove the STS™ Sensor.

ENDING YOUR CONTINUOUS GLUCOSE MONITORING SESSION

STS™ Sensor Expiration Notification

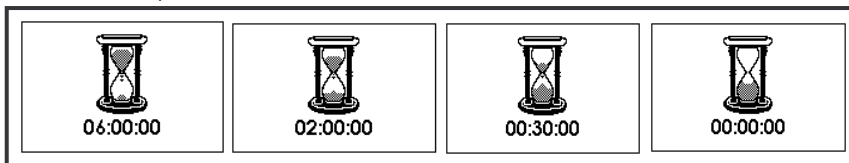

- The STS™ Receiver will display screens that notify you how long you have until the end of your Continuous Glucose Monitoring Session. You will be notified at 6 hours, 2 hours, 30 minutes to the end of your sensing period, and when your session is finished. Press any button to clear the alert.

CONSUMER INFORMATION AND FCC REQUIREMENTS

This transmitter is authorized by rule under the Medical Implant Communication Service (47 C.F.R. Part 95) and must not cause harmful interference to stations operating in the 400.150 – 406.000 MHz band in the Meteorological Satellite, or Earth Exploration Satellite Services and must accept interference that may be caused by such aids, including interference that may cause undesired operation.

This transmitter shall be used only in accordance with the FCC Rules governing the Medical Implant Communications Service. Analog and digital voice communications are prohibited. Although this transmitter has been approved by the Federal Communications Commission, there is no guarantee that it will not receive interference or that any particular transmission from this transmitter will be free from interference.

LBLXXXX, Rev. XX, © 2005, DexCom, Inc.

Caution: Investigational device. Limited by Federal (United States) law to investigational use.