



Smartphone Docking Station with Speaker and Wireless Bed Shaker

For use with the AlarmDock iOS or Android alarm clock app



www.silentcall.com
Silent Call® Communications Corporation
5095 Williams Lake Road
Waterford, Michigan 48329



User's Manual

TABLE OF CONTENTS

Introduction.....	3
Important Safety Instructions.....	3
Danger	3
Warnings	4
Precautions.....	4
FCC Statement.....	4
Specifications USB Power Cube.....	4
Smartphone Application Requirements.....	4
Manual Reviews.....	5
AlarmDock Docking Cradle with Built-in Speaker.....	5
Bed Vibrator, Model # VIB-PJ.....	6
Wireless Bed Vibrator Module, AD200-SM	6
USB Charger/Power Cube	7
Application and Setup.....	7
Shaker Module Setup and Pairing.....	8
Pairing Instructions for the Bed Shaker	8
Volume Control for AlarmDock Wireless Phone Dock.....	9
AlarmDock Application Setup Icons.....	9
Troubleshooting Tips.....	10
Warranty.....	11

INTRODUCTION

Thank you for choosing the AlarmDock™ Smartphone docking station with speaker and wireless bed shaker. This instruction manual contains useful information on appropriate use and maintenance of this product. In order to benefit from its features, read this manual before first use and save it for further reference.

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING: Read all instructions before using this product.

DANGER:

- Do not let water or any other liquid or flammable detergent enter the power jack to avoid electric shock and/ or fire hazard. Do not clean the power cube with water or any other liquid or a (flammable) detergent to avoid electric shock and/or a fire hazard.
- To prevent fire or shock hazard do not expose this product to water, or moisture. Before connecting to the main power make sure that the requirements specified on the power cube unit corresponds to the available power. i.e. AC100-240V~, 50/60 Hz

WARNINGS

- This product is not intended for use by children or persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the unit by a person responsible for their safety.
- Modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

PRECAUTIONS

Children should be supervised to ensure that they DO NOT play with the unit especially the parts.

FCC Statement

This device complies with Part 15 Subpart B of the FCC rules. Operation is subject to the following two conditions: 1) This device may cause harmful interference, and 2) This device must accept any interference received. Including interference that could cause undesired operation.

SPECIFICATIONS USB Power Cube

Switching type AC/DC Power Adapter, Rated Input: 100-240 Vac, 50/60 Hz, 0.8A

SMARTPHONE APPLICATION REQUIREMENTS

Androids: 4.0 or higher

iPhone: iOS 8 or later, compatible with iPhone, iPad, and iPod touch.
App size 2.4 MB.

THIS MANUAL REVIEWS PROPER CONNECTION AND USE OF:

- Bed vibrator, Model # VIB-PJ
- Wireless Shaker Module Model number AD100SM
- USB Charger/ Power Cube
- AlarmDock smartphone application (iOS and Android)


2 BED VIBRATOR, MODEL# VIB-PJ

Best placement for the bed shaker is under the mattress, between the mattress and box spring. Place in a spot that would be closest to your upper back. Once in place, plug the bed shaker into the end of the Shaker Module input jack. Plug the Shaker Module into the USB charger cube, then plug the USB charger cube into the 120 volt wall outlet. Very important "DO NOT USE ANY OTHER MANUFACTURERS BED SHAKER WITH THE Shaker Module, IT COULD DAMAGE THE SHAKER MODULE AND VOID THE WARRANTY".



3 WIRELESS BED VIBRATOR MODULE, AD200-SM


Plug the wireless bed shaker module into the USB charger cube. Plug the USB charger cube into a 120 volt wall outlet. The USB charger cube will keep the Shaker Module powered and ready to activate the bed shaker. When the Shaker Module is in range, approximately 30 feet from your smartphone it will automatically pair itself to the smartphone. A blue light on the Shaker Module will light up and flash on & off when ready to be paired. When paired the blue light will stay steady, no flashing.

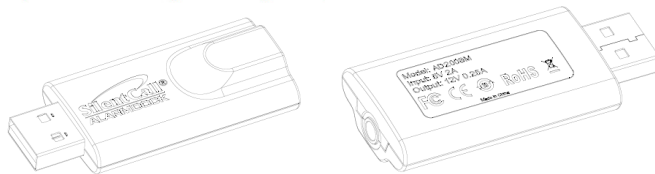

6

4 USB CHARGER/POWER CUBE

The USB charger Cube is designed to power and charge up to four devices at one time. Charge devices like the AlarmDock™ wireless speaker stand, cell phones, & tablets. It also powers the wireless bed shaker module.

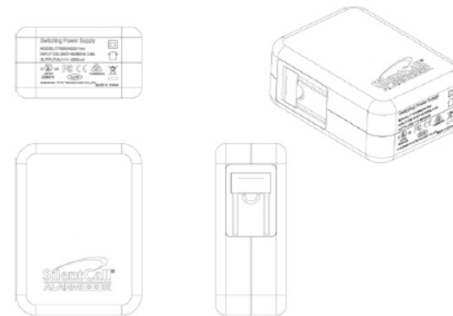

5 APPLICATION AND SETUP


Next you'll need to upload the AlarmDock™ application. For Android's go to the Google play store. For iPhone go to the iTunes App Store. Type AlarmDock™ into the application search field, then look for AlarmDock by Silent Call® Communications. Once you've uploaded the application, you'll need to open it to setup your alarm settings and to pair the wireless bed shaker. To use the AlarmDock™ application one device at a minimum must be paired for it to work. See application requirements under smartphone application requirements above.



7

SHAKER MODULE SETUP AND PAIRING

Plug the bed shaker jack into the jack input at the end of the Shaker Module. Next plug the Shaker Module into the USB Charger Cube and plug the USB Charger Cube into any 120 volt outlet. Make sure the jack on the bed shaker is pushed all the way into the Shaker Module. Place the bed shaker between the mattress and box spring of your bed. For best result, place the bed shaker under the area where your upper back would be positioned when in bed.

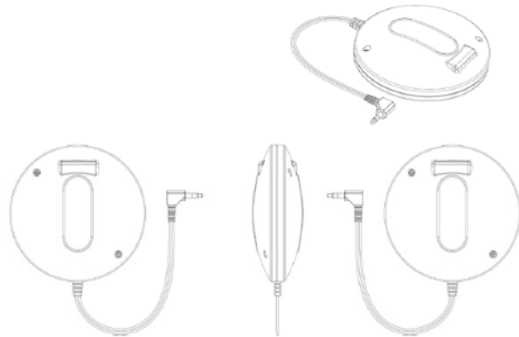

PAIRING INSTRUCTIONS FOR THE BED SHAKER

Bed Shaker Pairing: Once your smartphone is in range of the Bed Shaker approximately 30' or less the Bed Shaker will automatically pair with your smartphone. A blue flashing indicator light will flash on the Shaker Module stick and will change to steady lit no flashing when paired. If the blue LED is still flashing, you'll need to turn on the Bluetooth setting in your phones settings.

ALARMDOCK™ APPLICATION SETUP ICONS (LOCATED AT THE BOTTOM OF THE APP)

There are five icons located at the bottom of the application, below we explain each individually.

Connect icon (pairing the wireless bed shaker/Vibrator Shaker Module)

Alarm icon – Tap on the Alarm icon, then tap the + at the top right corner to setup an alarm. Follow the app instructions. Here you can set multiple alarms, repeat the alarms with days of the week selections. You can name the alarm with labels, select the sound type, and select an alarm fade option from 10 seconds to 5 minutes or none.

Timer icon – Tap the Timer icon to set a timer.

Clock icon with Swipe to Dim Feature – Tap on the Clock icon swipe on the display screen in either direction to select from eight different display colors. Once the color has been selected you can dim the display by swiping left to right or top to bottom.

Settings (located at the bottom right-hand side) – To leave the clock display, tap anywhere on the screen. Tap settings to see all the choices available. Adjust the volume, vibration strength. Select how you want to be alerted, Sound, vibrator or your phones flashing light. Adjust the snooze duration from 5 minutes to 50 minutes, and select the preferred time format 12/24 hour.

TROUBLESHOOTING TIPS

- Q. Wireless Shaker Module won't stop flashing.**
A. Turn on your Bluetooth setting in your phones settings. Also make sure your phone is in range, 30' or less.
- Q. Sound, vibration or flashing light not working.**
A. Go to the app settings icon and turn on the alert.
- Q. Alarm Clock Display too bright.**
A. Swipe with finger from top to bottom or left to right.
- Q. How do I change colors for the clock display?**
A. Swipe with finger from left to right or up and down to see all eight color choices.
- Q. Wireless vibe/bed shaker too weak or not working.**
A. Go to settings and slide the vibration slider to the right. Is the Bed Shaker plugged in all the way to the Shaker Module? Is the Shaker Module plugged in all the way on the power/charger cube?
- Q. Alarm volume too low.**
A. Go to settings and slide the volume slider to the right.
- Q. How do I set 12/24 hour time?**
A. Go to settings, at the bottom select 12 or 24.

WARRANTY

The Silent Call AlarmDock and its components including the power cube and the vibration module are warranted for 90 days from the day of purchase to be free of defects in material and workmanship. The Bed Vibrator, Model # VIB-PJ has a 5 year limited warranty. This warranty is limited to free repair or replacement when the product is shipped prepaid to:

Silent Call® Communications Corporation
5095 Williams Lake Road
Waterford, Michigan 48329

This warranty is void when damage is caused by misuse, neglect, fire, excessive heat, water, or by improper installation or operation. Silent Call will not be responsible for the cost of parts or labor and/or service charges which are performed by others.

To obtain warranty service, call toll free 1-800-572-5227 or email us at technicalsupport@silentcall.com

After this warranty expires, Silent Call will repair or replace any Silent Call product for a fixed cost.

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: 1) this device may not cause harmful interference, and 2) this device must accept any interference received, including interference that may cause undesired operation. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.