SCXMIT10 418Mhz Consolidated Transmitter Series Installation Instructions

Revision *IR*

Date *01/12/2016*

REVISION HISTORY

| Revision # | Initiated By | Revision Date | Approved by | Approval date. | Reason |
|------------|--------------|---------------|-------------|----------------|--------|
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |
| | | | | | |

Retain this record in front of this manual. Upon receipt of this revision, insert and remove pages accordingly. Then enter on this page, the revision number, issue date, insertion date, and your initials.

TABLE OF CONTENTS

| | |
|---|----------|
| SECTION 1: INTRODUCTION..... | 5 |
| 1.1 PURPOSE:..... | 5 |
| SECTION 2: SYSTEM DESCRIPTION..... | 5 |
| 2.1 KEY FEATURES:..... | 5 |
| 2.2 CONSOLIDATED TRANSMITTER (FA3-MC) SPECIFICATIONS: | 5 |
| 2.3 REGULATORY INFORMATION NOTICE: | 6 |
| 2.4 TRANSMITTER FEATURES:..... | 6 |
| 2.5 TRANSMITTER SETTINGS:..... | 6 |
| 2.6 REPLACEMENT BATTERIES: | 6 |
| 2.7 TRANSMITTER MOUNTING AND LOCATION:..... | 6 |
| SECTION 3: CONFIGURED TRANSMITTER INSTRUCTIONS..... | 7 |
| 3.1 BM418-MC Installation: | 7 |
| 3.2 BM418-MC Operation:..... | 7 |
| 3.3 DB2-MC Installation:..... | 7 |
| 3.4 DB2-MC Operation: | 7 |
| 3.5 DB41-MC Installation:..... | 8 |
| 3.6 DB41-MC Operation:..... | 8 |
| 3.7 DW3-MC Installation: | 8 |
| 3.8 DW3-MC Operation: | 8 |
| 3.9 FA3-MC Installation: | 9 |
| 3.10 FA3-MC Operation: | 9 |
| 3.11 FA41-MC INSTALLATION: | 9 |
| 3.12 FA41-MC OPERATION: | 9 |
| 3.13 MAT4-MC Installation | 10 |
| 3.14 MAT4-MC Operation:..... | 10 |
| 3.15 SM5-MC Installation: | 11 |

3.16 SM5-MC Operation:11

3.17 TEL-MC Installation:12

3.18 TEL-MC Operation:.....12

3.19 WT-MC Installation:12

3.20 WT-MC OPERATION:12

SECTION 1: INTRODUCTION

1.1 PURPOSE:

This document contains information relative to the installation and operation characteristics of the Silent Call Medallion Series SCXMIT10 Consolidated Transmitter configurations.

SECTION 2: SYSTEM DESCRIPTION

2.1 KEY FEATURES:

The Silent Call SCXMIT10 Transmitter Medallion Series is a configurable 418Mhz Transmitter to be used in conjunction with the Medallion Series Receivers. The consolidated transmitter can be configured into 10 different transmitter types depending upon the customer requirements. The following is a list of configurable 418Mhz consolidated transmitters.

- BM418-MC—Bed Mat Transmitter
- DB2-MC -- Doorbell Transmitter (with remote button)
- DB41-MC—Doorbell Transmitter
- DW3-MC– Door/Window access Transmitter
- FA3-MC --Fire Alarm Transmitter (contact input)
- FA41-MC --Fire Alarm Transmitter (Battery w/voltage)
- MAT4-MC – TransMatter Transmitter
- SM5-MC– Sound Monitor Transmitter
- TEL-MC– Telephone Transmitter
- WT-MC– Weather Alert Transmitter

2.2 CONSOLIDATED TRANSMITTER (FA3-MC) SPECIFICATIONS:

| | |
|--------------------|--|
| FA3-MC Mass: | .14 kilograms w/batteries |
| FA3-MC Dimensions: | Length = 115 mm Width = 62 mm Height = 28 mm |
| Operating Voltage: | 3volts DC 2 AA Panasonic LR6WXA |
| Current Draw: | .04 milliamps Idle CONT. 8.3 milliamps during Alert |
| FCC ID: | PPJ-SCXMIT10 |
| IC: | 4498A-SCXMIT10 |

2.3 REGULATORY INFORMATION NOTICE:

THIS DEVICE COMPLIES WITH PART 15 OF THE FCC RULES.

This Device Complies With Industry Canada License-Exempt Rss Standard(S).

Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- ***Reorient or relocate the receiving antenna.***
- ***Increase the separation between the equipment and the receiver***
- ***Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.***
- ***Consult the dealer or an experienced radio/television technician for help***

Unauthorized changes or modifications could void the user's authority to operate the equipment.

2.4 TRANSMITTER FEATURES:

SCXMIT10 transmitters are battery operated and will flash a YELLOW LED when the batteries are in need of replacement. Transmitter has a test button for programming to receivers and required follow on operational testing The RED LED will illuminate when the transmitter is actively transmitting signal to the intended receiver.

2.5 TRANSMITTER SETTINGS:

The Silent Call Medallion Series address system is digitally encoded. All Silent Call Medallion Series receivers are capable of receiving one million different address codes. Each Medallion Series Transmitters has a unique code preprogrammed during manufacturing. Please refer to your receiver manual for instruction on programming the transmitter code to the receiver.

2.6 REPLACEMENT BATTERIES:

Transmitter uses two AA 1.5V Alkaline Batteries. AA Batteries may be purchased from local retail stores. Transmitter is shipped with 2 Rayovac Alkaline Batteries Type LR6. FA3-MC and FA41-MC configured transmitter are certified with AA Panasonic LR6WXA batteries only. It is recommended transmitter batteries be changed once a year and checked weekly.

2.7 TRANSMITTER MOUNTING AND LOCATION:

The Consolidated Transmitter Medallion Series Transmitter is designed for indoor use only. The transmitter case has provisions for mounting to a nail or screw on the wall or use provided Velcro for mounting on desktop or wall. Avoid mounting transmitter near large metal objects, other transmitting devices in close proximity or high current appliances that will degrade the transmitter / receiver operation. Test transmitter mounting location with receiver before mounting to ensure proper system operation. Adjust antenna for best results Transmission range with a Silent Call Medallion Series transmitter is about 500 feet.

SECTION 3: CONFIGURED TRANSMITTER INSTRUCTIONS

3.1 BM418-MC INSTALLATION:

1. Place the bed mat under the sheet on the bed in the area of where the users back will lay.
2. Allow the wire and transmitter to hang over the head of the bed so that the user is not lying on either part or does not trip on the wire when getting out of bed.
3. The transmitter is connected to the Bed Mat and must not be mounted directly to a metallic surface. Doing so will severely reduce the range of the transmitter.

3.2 BM418-MC OPERATION:

1. When someone is lying on the bed mat it will not transmit. The user can roll around and it will not transmit.
2. The transmitter will activate once the person gets up out of bed and pressure is removed from the mat.
3. Medallion Series Receiver strobe will activate and illuminate the door LED Icon. The Receiver will indicate alert for 16 seconds. Repeat steps 1 and 2 for system to re-alert.
4. Quick reference Document # 109101.

CAUTION: When handling or shipping the Bed Mat Transmitter, do not bend the mat sharply or fold it. It must be kept flat or rolled loosely or it will be damaged.

3.3 DB2-MC INSTALLATION:


1. Decide where to mount the transmitter on an inside wall near the location for the button.
2. Drill a hole in the wall behind where the transmitter will mount.
3. Pass the wires from the transmitter through the hole and connect them to the terminals in the button.
4. Install the button on the outside wall covering the hole.

3.4 DB2-MC OPERATION:

1. When the remote button is pressed, the Red LED on the face of the transmitter will light up. The transmitter will then send a signal to any Silent Call Medallion Series Receiver activating the receiver.
2. Release the Doorbell button and the transmissions will cease.
3. Quick reference Document # 109105

3.5 DB41-MC INSTALLATION:

1. Locate your doorbell chime unit and remove the cover.
2. Inside the chime unit, locate the terminal block with three terminals marked FRONT, REAR and TRANS.
3. Loosen the two screws but do not disconnect the attached wires.
4. Attach the two wires from the DB41-MC as follows: Red to FRONT, yellow to REAR.
5. Tighten the two screws and tuck and excess wires out of sight. Make sure all wires are clear of the chime mechanism.
6. Replace the chime box cover.
7. Other applications are possible. The DB41-MC will activate when it is connected to a source of voltage between 12V and 24V AC or DC.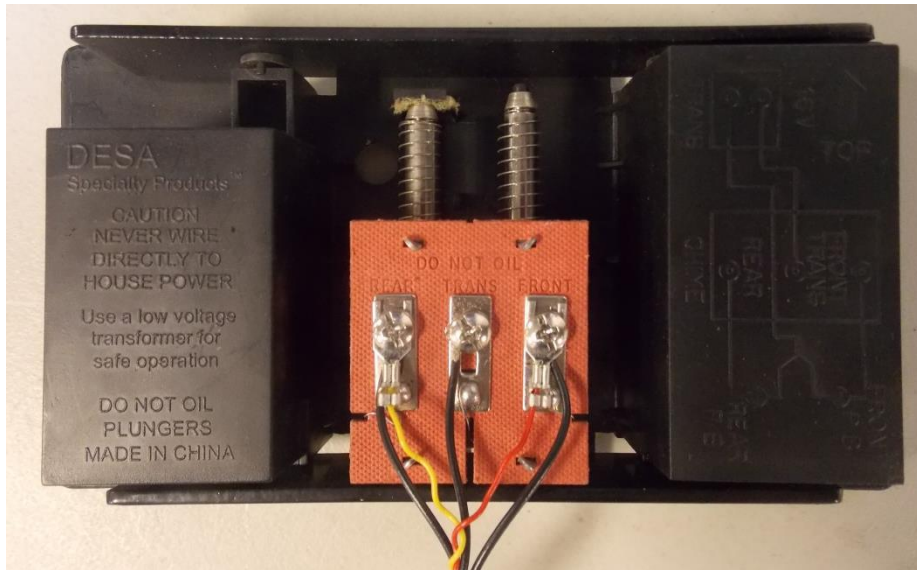

3.6 DB41-MC OPERATION:

1. Whenever the front (or rear, if used) doorbell button is pressed, the chimes will sound as usual, the Red LED on the face of the doorbell transmitter will light up and the transmitter will send a signal to any Silent Call Medallion Series Receiver activating the receiver.
2. Both the front and rear doorbell buttons will activate the doorbell transmitter.
3. Quick reference Document # 109109

3.7 DW3-MC INSTALLATION:

1. Mount the magnet on the door or window so that it clears the door jamb or window frame and does not interfere with normal door or window operation.
2. Mount the magnetic switch so that it is as close to the magnet as possible when the door or window is closed. Spacing should be less than approximately 1/4".

3.8 DW3-MC OPERATION:

1. When the door or window moves enough that the magnet is separated from the switch, the red LED on the transmitter will light up. The transmitter will then send a signal to any Silent Call Medallion Series Receiver activating the receiver. The transmitter will transmit for approximately five seconds and then stop until the door or window is closed. The door or window must be closed for approximately five seconds before the transmitter will reset and be ready to activate again.
2. Quick reference Document # 109111.

3.9 FA3-MC INSTALLATION:

Please contact a licensed fire alarm installer for installation and working with this product. Improper installation can damage your fire alarm and/or prevent proper operation of the fire alarm. Technical support is available at (800) 572-5227.

The FA3-MC in conjunction with a Silent Call Medallion Series receiver is considered an *Supplementary Visible Notification Appliance*.

The FA3-MC is designed to interface with a dry contact output. It will transmit as long as the contact is closed and will stop transmitting when the contact opens.

1. Do not mount inside of the alarm panel box! The metal of the box will detune the transmitter.
2. Connect Red and Black wire Spade terminals to Open/Unused Form A /Form C relay contacts. Contact closure activates transmitter. Damage to Transmitter will occur if connected to any voltage source
3. Reference Silent Call FA3-MC Installation Drawing 20-300-021-01FA3.

3.10 FA3-MC OPERATION:

1. The FA3-MC will transmit as long as the Red and Black wire spade terminals are connected. When the activating contacts are closed, the Red LED on the face of the transmitter will light up. The transmitter will then send a signal to any Silent Call Medallion Series Receiver activating the receiver.
2. FA3-MC will stop transmitting when Alarm is reset and alarm contacts are opened.
3. Quick reference Document # 1098113.

3.11 FA41-MC INSTALLATION:

Please contact a licensed fire alarm installer for installation and working with this product. Improper installation can damage your fire alarm and/or prevent proper operation of the fire alarm. Technical support is available at (800) 572-5227.

The FA41-MC in conjunction with a Silent Call Medallion Series receiver is considered an *Supplementary Visible Notification Appliance*.

The FA41-MC is designed to interface with a voltage input. It will transmit as long as the voltage is positive and will stop transmitting when the voltage reverses polarity or ceases.

1. Do not attach inside of the alarm panel box! The metal of the box will detune the transmitter.
2. Connect Red and Black wire Spade terminals to Alarm Panel NAC Circuit or circuit with a positive voltage for alert.
3. Alert Voltage 12VDC to 24VDC.
4. Reference Silent Call FA41-MC Installation Drawing 20-300-023-01FA41.

3.12 FA41-MC OPERATION:

1. The FA41-MC will transmit as long as positive polarity voltage is applied to the Red and Black wire spade terminals. When positive voltage is present, the Red LED on the face of the transmitter will light up. The transmitter will then send a signal to any Silent Call Medallion Series Receiver activating the receiver.
2. FA41-MC will stop transmitting when Alarm is reset and positive polarity is reversed or voltage is removed.
3. Quick reference Document # 109155.


3.13 MAT4-MC INSTALLATION:

1. Place the mat where it will be stepped on anywhere except the very edge. The water resistant material used to manufacture the TransMATTER is required to have a protective cover over it; this would usually consist of a door rug or similar covering.
2. The transmitter that is connected to the TransMATTER must not be mounted directly to a metallic surface. Doing so will severely reduce the range of the transmitter.
3. The TransMATTER is intended for indoor use only.

3.14 MAT4-MC OPERATION:

1. Whenever the mat is stepped on, the Red LED on the face of the TransMATTER Medallion Series transmitter will light up. The transmitter then sends a signal to any Medallion Series Receiver, activating the Receiver as a Doorbell Alert.
2. If pressure is maintained, the transmitter will continue to transmit. Transmitter will stop transmitting when pressure is removed from the mat.
3. Quick reference Document # 109117.

CAUTION: When handling or shipping the TransMATTER, do not bend the mat sharply or fold it. It must be kept flat or rolled loosely or it will be damaged.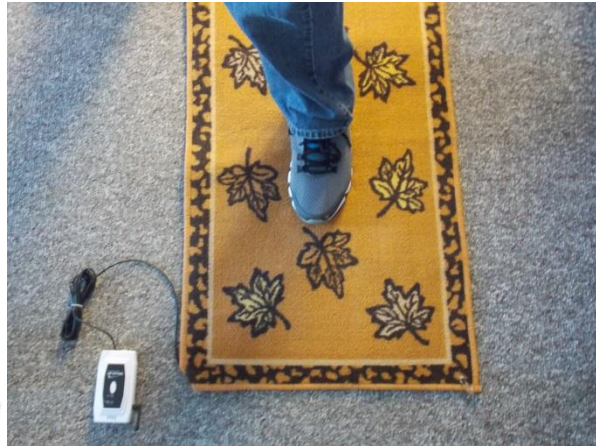


3.15 SM5-MC INSTALLATION:

Please read all of the instructions thoroughly before using your Sound Monitor.

1. Remove your Sound Monitor from the packaging.
2. Place the Sound Monitor as close to the sound source as possible that you want to monitor (but not within a baby's reach). For example, place it on a dresser next to a baby's bed or next to a telephone, alarm clock or apartment doorbell speaker.
3. The Sound Monitor can be mounted to any flat surface with the supplied Velcro strips or you can hang it on a wall using the access hole on the back of the case. This step is not necessary if you will be moving the Sound Monitor from location to location.

3.16 SM5-MC OPERATION:

1. Slide the on/off switch on the side of the Sound Monitor to the ON position.
2. Any continuous loud sound will cause the Sound Monitor to activate and the Red LED on the face of the transmitter will come on. The transmitter then sends a wireless signal, to any Silent Call Medallion Series Receiver activating the receiver.
3. The Sound Monitor is designed so that brief sounds such as door slams, regardless of how loud they are, will not activate the Sound Monitor.
4. The Sound Monitor leaves the factory set to 50% sensitivity, which should be adequate for almost all situations. If for some reason the sensitivity needs to be changed, there is an adjustment dial located behind the battery door. Rotating the adjustment dial clockwise increases the sensitivity.
5. A new feature of the Sound Monitor is the ability to set the Sound Monitor to detect a specific sound. Remove the battery cover and at the bottom left hand corner there is a 3 position switch. Switch # 1 ON is for Sound Alert, Switch #2 ON is for Telephone Alert and Switch # 3 ON is for Doorbell Alert.
6. Quick reference Document # 109119.

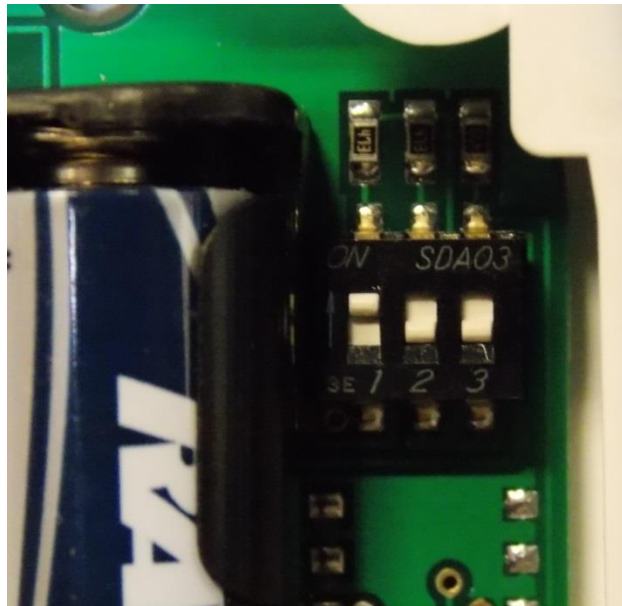Shown set to Sound "Switch 1 ON"

3.17 TEL-MC INSTALLATION:

1. The telephone transmitter will work with standard telephone lines and with most video phone RJ-45 outputs.
2. **To install the TEL-MC telephone transmitter on a conventional telephone line**, plug the telephone transmitter into any telephone outlet. Use the supplied dual adapter if you need to connect a telephone AND the transmitter to the same outlet.
3. **To install the TEL-MC telephone transmitter on a videophone**, plug the telephone transmitter into the telephone outlet on the videophone. Use the supplied dual adapter if you need to connect a telephone AND the transmitter to the videophone.
4. If plugged into a two line telephone outlet, the telephone transmitter will respond to line one only.
5. The TEL-MC transmitter may not work with digital phone systems. Please contact your telephone service provider as filters may be necessary to filter out the modem tones associated with digital (DSL) or VOIP type services.

3.18 TEL-MC OPERATION:

1. Whenever the telephone (or videophone) rings, the Red light on the telephone transmitter will light up and the transmitter will send a signal to your Silent Call Medallion Series Receiver causing the receiver to activate Phone Alert.
2. Quick reference Document # 109121.

Connect TEL-MC to Dual Adapter

3.19 WT-MC INSTALLATION:

1. The WT-MC Weather Alert Transmitter is designed to interface with a variety Weather Alert radios that include a normally open (or open collector) output.
2. Transmitter connector mates with standard female 3.5mm WX Radio output jack.
3. Plug the cord on the weather transmitter into the alarm output jack located on the rear panel of the weather radio.

3.20 WT-MC OPERATION:

1. The WT-MC Weather Alert Transmitter will transmit as long as the weather receiver maintains the alert output and will stop transmitting when the alert output goes back to **normal**.
2. Quick reference Document # 109123.

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

CHANGES OR MODIFICATIONS NOT EXPRESSLY APPROVED BY THE MANUFACTURER COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.

Limited Warranty

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.

Return for Serving To:

Silent Call Communications

5095 Williams Lake Road, Waterford Michigan 48329

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