Fall detector FD 100

Instructions for use

Intended Use

The Fall detector FD100 senses the body's orientation and monitors falls by recognising changes in orientation combined with an impact. The Fall detector FD100 is designed to be worn by a person at risk of a fall. The unit is easy to use and will generate minimal false alarms. The device originates alarms locally through a carephone or remotely to an existing emergency response center.

Preparation

- Remove the battery cover.
- Insert the two batteries as advised in the battery compartment.
- Reprogramme the carephone to detect the fall alarm (see the carephone user manual)
- Replace the battery cover in the correct position.
- Test the range of the Fall detector FD 100

Operation

The Fall detector FD100 must be worn on the torso using the belt clip.

Wearing the fall detector on your arm or leg may cause the device to false alarm upon a real fall.

The fall detector is water resistant. It can be worn in the shower.

Attach the fall detector to a belt by squeezing the clip or by rotating the detector on the belt to ensure it is in normally vertical position.

Wearing the fall detector so that it does not have the correct orientation may cause the device to fail.

The Fall Detector FD100 may not be used outside the registered country.

Care

Cleaning

Wipe the surface with a damp cloth.

Replacement of batteries

Warning : Danger of explosion if battery is incorrectly replaced.

Replace only with the same or equivalent type recommended by the manufacturer.

Replacement of batteries is only allowed for service technicians.

- Remove the battery cover
- Remove the old batteries
- Insert two new batteries (CR 2477)
- Replace the battery cover

Technical Data

	Temp. °C	Rel.Hum. %
Storage	-20 .. 70	5 ... 95
Usage	0 ... 50	10 ... 95
Water resistance	IP55	
Flame rating	UL V0	
Power source	2 CR2477 Lithium batteries	
Battery life time	up to 360 days. Dependant of activity of user	
Frequency range	173.255 MHz (UK version) 167.500 MHz (US version)	
Channel spacing	20 kHz	
Eff. radiated power	0,025 mW	
ITU designation	1K20FD1	

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CE0560 Ⓢ

5702071

©Dräger HomeCare
1st edition - June 2001
Subject to alteration

Dräger HomeCare

Dräger

Dräger HomeCare, P.O. Box 10.100, 5680 GA Best

datum
May 28, 2001
referentie
FDJ/sv/QM01_137

Subject: **Declaration of Conformity**

We Dräger HomeCare
A division of Dräger Nederland B.V.
Kanaaldijk 29
5683 CR BEST
The Netherlands

Declare that the **Fall Detector FD100**:

5730510	Fall Detector FD100, DE	Germany
5730514	Fall Detector FD100, GB	United Kingdom
5730515	Fall Detector FD100, FI	Finland
5730516	Fall Detector FD100, ES	Spain
5730517	Fall Detector FD100, AT	Austria
5730518	Fall Detector FD100, SE	Sweden
5730519	Fall Detector FD100, DK	Denmark
5730520	Fall Detector FD100, CH	Switzerland

is in conformity with the essential requirements as described in Directive 1999/5/EC and satisfies all the technical regulations applicable to the product within this directive

ETSI EN 300 220-1
ETSI ETS 300 683
EN 60950

Equipment Class Identifier ⓘ

This product is a class 2 device under the scope of the R&TTE

Dräger HomeCare
Best

.....
Dr. M.Th.A. Teeeling
Managing Director

.....
G. van Dijk
Manager Research & Development

Dräger HomeCare
a division of Dräger Nederland B.V.
Kanaaldijk 29
5683 CR Best
The Netherlands
Telephone: +31 (0) 499 331 331
Fax: +31 (0) 499 331 222

ABN AMRO Bank N.V. - Zoetermeer
Account nr. 42.98.23.908
Trade Register The Hague 27012085