


FCC submission

Product Manual

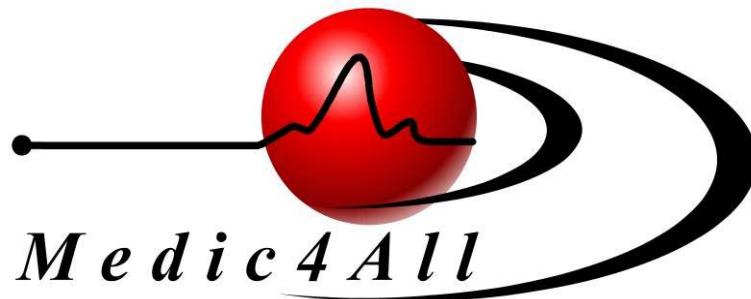

October 21, 2004


**Medic4All Ltd.
POB 4222 Petach Tikva,
Israel 49000**

Operation Manual

Wireless Weight Scale – WS100



The WS-100 is a wireless Weight Scale device that measures, stores, and transfers the weight data to a monitoring center using the MedicGate™ or to a PC using the MiniGate™.

 Please read this booklet carefully in order to achieve optimal performance of the Wireless Weight Scale.

Warnings & Precautions

Consult your local provider concerning any operational difficulties. Do not attempt to repair the Wireless Weight Scale on your own. Any modifications made to the equipment void the device warranty and service contract obligations.

Keep the Wireless Weight Scale away from electro-magnetic emitting devices.

Do not store the Wireless Weight Scale in direct sunlight, at a high temperature, or in a location with excessive humidity.

Use of this device is not intended as a substitute for consulting with your physician, since additional measurements to those supplied by the Wireless Weight Scale may be necessary.


Operating the Wireless Weight Scale

To initiate a measurement session:

Press the MEASUREMENT switch gently.



All display screen characters light up for several seconds. The scale then displays 0.00 Kg (lb) and the OK mark (O).

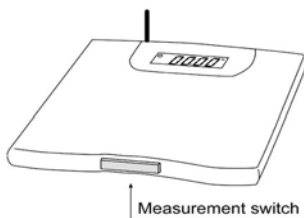


Step on the scale gently.




Your weight is displayed. Remain still until O appears. The communication with the MedicGate™ or MiniGate™ is activated automatically.



The device is automatically turned off after stepping off of the scale.


 Do not allow anyone else to use your Wireless Weight Scale. Remember: Every time you use the Wireless Weight Scale, the results are sent to your personal file. The use of the device by another person will cause an accumulation of erroneous data in your personal file.


Battery Replacement

Replace all four batteries with new ones, when  is displayed on the display screen.

Remove the battery compartment cover. Push the cover up and lift at the arrow indication.

Insert each battery with its positive (+) and negative (-) terminals properly aligned with the corresponding marks on the Wireless Weight Scale.

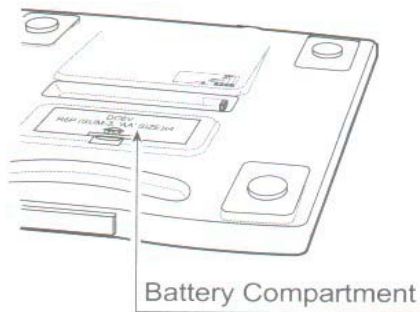

 Do not mix used and new batteries as this may cause damage to the battery or Wireless Weight Scale.

Specifications

Scale measurement

Maximum capacity	150 Kg (350 lb.)
Minimum display	0.05 Kg (0.1 lb.)

Radio Frequency communication

Frequency	433.92 MHz
Max transmission power	+10 dBm (EIRP)
Bandwidth	60 kHz

Power Supply / battery

Power	4 x 1.5v AA batteries
-------	-----------------------

Complies with the CE standard under the Euro Directive.


FCC Part 15 Information

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the Federal Communication Commission (FCC) Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses, and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the inference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

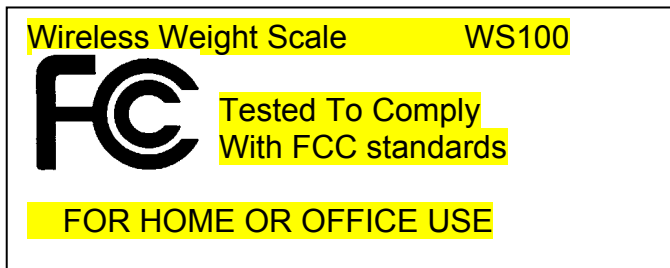

Specifications are subject to change without prior notice.

Manufacturer: Medic4All, Israel

Email: info@medic4all.com

www.medic4all.com

Local provider Details:



Contents

1. Operation Manual
2. Wireless Weight Scale

Warranty

This product is warranted against defects in material and workmanship beginning from the date of shipment for a period of one year (the “Term”).

Under the Term, the obligation of Telcomed Advanced Industries Ltd (the “Company”) will be limited to repair or replacement of the product (not including batteries, cables and spendable parts), according to the Company’s discretion. Such repair or replacement will be made only after the Company’s examination determines that said product is defective in material and/or workmanship.

This product warranty is void if the product has been repaired or altered in any way as to affect its use and operation as determined by the Company. This warranty is also void if the product has been subjected to abuse, misuse, negligence or accident.

No warranties, whether expressed or implied, shall exist beyond the Term.

Under no circumstances shall the Company be responsible for any loss of data or income or any special, incidental, consequential or indirect damages, arising from the product or its operation, howsoever caused.

Under no circumstances the Company will be liable for any claim arising from the product or its operation in excess of the total price paid for this product.

The Company reserves the right to make changes and improvements to the product without prior notice.