English

Magyar

Română

Français

Deutsch

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# miniTek

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# Content

miniTek	4
Intended use	8
General notes on safety	9
Basic functions	12
Bluetooth	16
Using miniTek with cell phones and landline phones	17
Using the transmitter	19
Using the line-in audio input	23
Using miniTek with a third party Bluetooth transmitter	24
Using miniTek with a public loop system	26
Accessories	27
Maintenance and care	28
Disposal information	29
Technical information	30

EN

HU

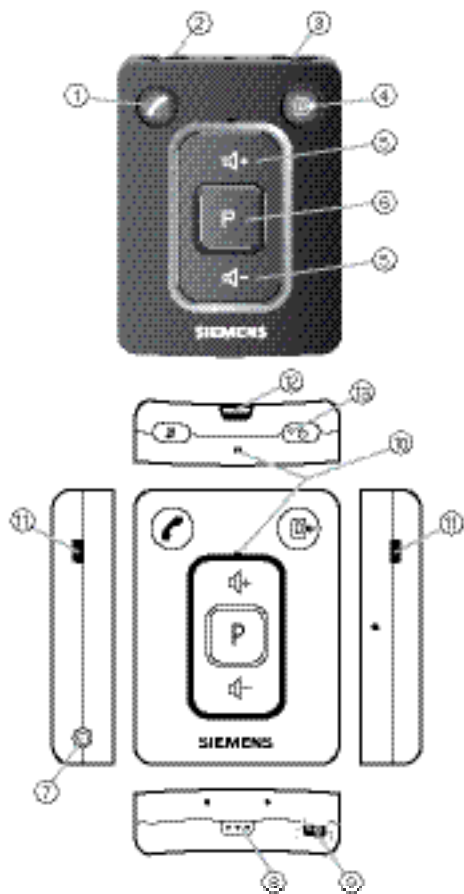
RO


FR

DE

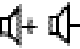AR


## miniTek

- ①  Telephone button
- ②  Stand-by for hearing instruments; mute
- ③  miniTek on/off
- ④  Audio source button
- ⑤  Volume up / volume down
- ⑥  Program change button
- ⑦  2.5 mm stereo socket
- ⑧  DAI port for FM receiver
- ⑨  Socket for power plug
- ⑩ Microphone openings
- ⑪ Slots for fixing the clip
- ⑫ Lanyard attachment
- ⑬ Power LED


### Included in delivery

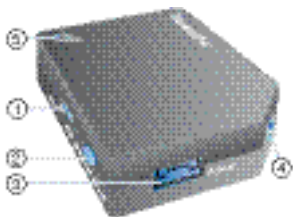- miniTek with lanyard and clip
- Transmitter

- ① On/off switch
- ② Sound level
- ③ Socket for power plug
- ④ 2.5 mm stereo socket
- ⑤ LED

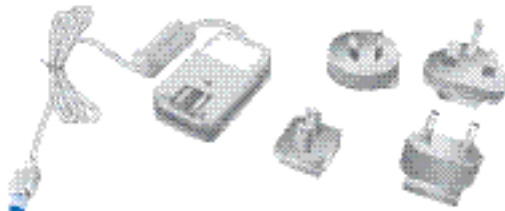- 2 Power plugs with country specific adapters (one for remote control, one for Transmitter)

**■ Cables:**

- 1.8 m audio cable with 3.5 mm stereo plug on one side and a 2.5 mm stereo plug on the other side
- 0.3 m audio cable with 3.5 mm stereo plug on one side and a 2.5 mm stereo plug on the other side

**■ Adapter set:**

- 2.5 mm stereo plug to 3.5 mm stereo socket
- 6.3 mm stereo plug to 3.5 mm stereo socket
- RCA (cinch) to 3.5 mm stereo socket
- RCA (cinch) to SCART

## Intended use

This remote control is intended to operate hearing instruments.

Read and follow the instructions of this user guide to avoid injuries or damage to remote control.

Symbols:	
Be aware of information marked with the warning symbol <b>WARNING, CAUTION</b> or <b>NOTICE!</b>	
	<b>WARNING</b> points out a situation that could lead to serious injuries, <b>CAUTION</b> indicates a situation that could lead to minor and moderate injuries.
<b>NOTICE</b>	<b>NOTICE</b> indicates possible property damage.
	Advice and tips on how to handle your device better.
	Instruction. Indicates that something has to be done.

## General notes on safety

### WARNING

Wireless systems may interfere with measuring devices and electronic equipment.

- ▶ Turn the remote control off when in hospitals or airplanes.
- ▶ Prior to use in conjunction with an electronic implant or life supporting system, have the electromagnetic compatibility verified.

### NOTICE

The remote control is sensitive to extreme heat, high humidity, strong magnetic fields ( $>0.1T$ ), X-rays and mechanical stress.

- ▶ Do not expose the remote control to extreme temperature or high humidity.
- ▶ Do not leave it in direct sunlight.
- ▶ Do not immerse it in water.
- ▶ Do not place your remote control in a microwave oven.

### NOTICE

Operating temperature: charge min.  $0^{\circ}C$  to max.  $45^{\circ}C$ ; discharge min.  $-20^{\circ}C$  to max.  $60^{\circ}C$

Storage conditions: min.  $-20^{\circ}C$  to max.  $60^{\circ}C$ ;  
low humidity

In some countries restrictions for the usage of wireless equipment exist.

- ▶ Refer to local authorities for further information.

The performance of the e2e wireless function may be affected by electromagnetic interference – e.g. from a computer monitor or fluorescent lamp system.

- ▶ Move away from the source of interference if you experience difficulty.

▶ In the event your hearing instruments can be controlled by another person's remote control, return hearing instruments and remote control to your Hearing Care Professional.

### **WARNING**

Risk of interference with cardiac pacemakers.

- ▶ Do not use the remote control if you are fitted with a cardiac pacemaker.

### **WARNING**

Hazard of explosion in gas-filled spaces!

- ▶ Do not use your hearing instruments and remote control in areas where there is danger of explosions (e.g. mining).

EN

HU

RO

FR

DE

AR

**WARNING**

Strangulation hazard when lanyard gets entangled in e.g. moving mechanical parts of machines.

- ▶ Use only the delivered breakaway lanyard. It will automatically detach when entangled.

**WARNING**

Choking hazard posed by small parts.

- ▶ Keep batteries, small parts and remote control out of children's reach.
- ▶ If swallowed consult a physician or hospital immediately.

**NOTICE**

Use only original Siemens power adapter (model no. KSAA0500080W1UV-1; input: 100-240V ~, 50-60 Hz, 180 mA, output: 5 V, 800 mA, protection class II, only for indoor use).

**NOTICE**

Please make sure that the power plug is easily accessible to remove it from power supply if necessary.

## Basic functions

### Before first use

- ▶ Charge miniTek before first use.

### Charging

- ▶ Connect miniTek with the power plug to a power supply.

Charging is complete when the power LED shines green.

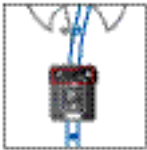
### Power LED state

	Battery full	Battery nearly empty
miniTek off	off	off
miniTek on	flashes green	flashes red
miniTek off/ charging	green constantly	red constantly
miniTek on/ charging	pulses green	pulses red
Turning off miniTek	green / turns off slowly	red / turns off slowly
Turning on miniTek	green for 3 seconds, then flashes green	red for 3 seconds, then flashes red

If you turn on miniTek and the battery is empty, the LED rapidly flashes red for 3 seconds, then turns off.

## Wearing miniTek

- ▶ Wear miniTek no further than 60 cm (24 inches) away from the hearing instruments.
- ▶ Use lanyard or clip to wear miniTek

## Turning the miniTek on/off

- ▶ Press  for about 3 seconds to turn miniTek on. All LEDs shine for 3 seconds, then only the power LED shines.
- ▶ Press  for about 3 seconds to turn miniTek off. All LEDs shine for 3 seconds, then turn off slowly.

## Changing the hearing program

- ▶ Press  to change the hearing program. Each time you press the program button, your hearing instruments switch to the next program, confirming each change with an acoustic signal.

## Changing the volume

- ▶ Press  to increase the volume.
- ▶ Press  to decrease the volume.

### Turning the hearing instruments on/off

- ▶ Press  for about 3 seconds to turn your hearing instruments off (stand-by).
- ▶ Press  for about 3 seconds to turn the hearing instruments back on.

### Changing the external audio source

- ▶ Press  to change the external source.

Each time you press , your miniTek switches to the next available external source according to the following overview:

Sequence of external source (if connected)	
	FM transmitter (DAI)
	Line-in (2.5 mm stereo socket)
	First transmitter
	Second transmitter
	Third party Bluetooth transmitter
	T-coil

 Your Hearing Care Professional has to configure your miniTek for external devices.

 Press the program button  to stop listening to the external audio source and switch back to the hearing programs.

 While using an external audio source you can press  to turn the microphones of your hearing instruments off and on again.

The hearing instruments will switch automatically to the telephone program when your phone rings.

- ▶ Press  to answer the phone.
- ▶ Press  to terminate the call.

Your hearing instruments will change back to the audio program after terminating the call.

OR

- ▶ Press  for about 3 seconds to reject the call.

## Bluetooth

Bluetooth is a wireless standard that provides a way to connect wireless devices like cell phones, TV, headsets and others.

Using a Bluetooth connection, the devices can exchange information such as data, sound or even control commands. The Bluetooth connection has a range of about 10 m (32 ft). Walls or objects between the devices can shorten this distance.

The Bluetooth symbol  denotes if a device is Bluetooth compatible.

The miniTek can be paired via Bluetooth with up to two phones, two transmitters and a third party Bluetooth transmitter (e.g. mp3 player or laptop) at the same time.

The following chapters give instructions about how to connect your phone, TV or mp3 player with your hearing instruments.

## Using miniTek with cell phones and landline phones

EN

When you are making a call with a Bluetooth compatible phone or cell phone, the miniTek allows you to use your hearing instruments as a wireless headset.

HU

Instead of using the telephone microphone, you speak into miniTek and listen through your hearing instruments.

RO

### Pairing

Before you can use miniTek with your phone, you need to pair them:

FR

- ▶ Press  for about 3 seconds to turn miniTek on.
- ▶ Activate the Bluetooth function in your phone.
- ▶ Press  for about 5 seconds to start pairing miniTek with your phone.

DE

When pairing mode has been activated, all 3 buttons (on/off button, telephone button and audio source button) shine for 3 seconds, then the telephone button flashes.

AR

- ▶ Search in the menu of your phone for other Bluetooth devices.
- ▶ Choose "RCU Connect" to pair miniTek and the phone.
- ▶ Enter the PIN "0000" if requested.

- ▶ Refer to the user guide of your connected device for more detailed information on how to link devices via Bluetooth.

### Accepting an incoming call

- ▶ Press  to accept the incoming call.
- ▶ Speak into miniTek's microphone to talk.
- ▶ Press  to terminate the call.

Your hearing instruments switch back to the hearing program you used before taking the call.

If paired with two phones, miniTek uses different ring tones to distinguish incoming calls.

### Rejecting an incoming call

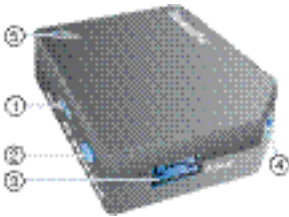
- ▶ Press  for about 3 seconds to reject the incoming call if supported by the phone.

### Making a call

- ▶ Dial the desired number on your phone.
- ▶ Press  twice to transfer the connection from your phone to the miniTek.
- ▶ Press  twice to transfer the connection from the miniTek back to the phone.

If your phone transfers the connection automatically you do not need to press .

## Using the transmitter

- ① On/off switch
- ② Sound level
- ③ Socket for power plug
- ④ 2.5 mm stereo socket
- ⑤ LED

### Before first use

- ▶ Charge transmitter before first use.

### Charging the transmitter

- ▶ Connect the transmitter with the power plug to a power supply.  
While charging the LED shines red.  
Charging time is about 5 hours depending on the status of the rechargeable battery.

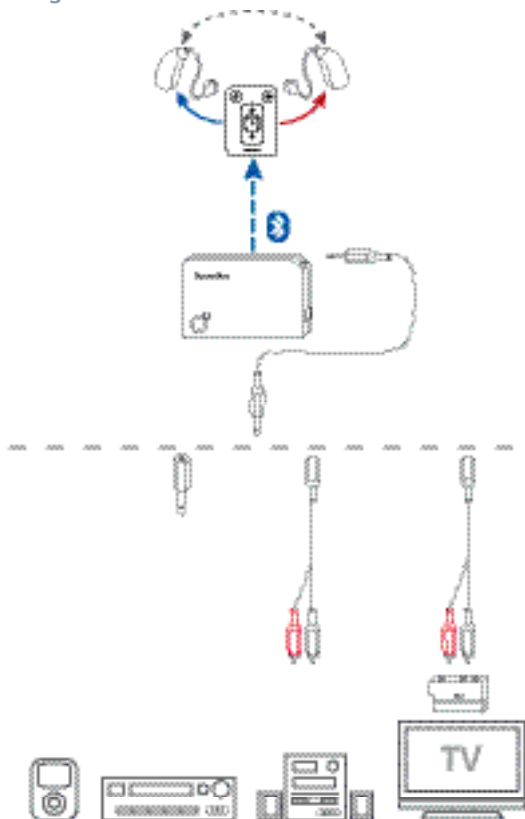
### Turning the transmitter on/off

- ▶ Use the on/off switch to turn the transmitter on.  
The LED flashes blue while searching for a connection to the miniTek, then the LED shines blue.
- ▶ Use the on/off switch to turn the transmitter off.  
Turn the transmitter off when it is not in use.

### Sound level

- ▶ Press the sound level button to change the volume of the audio signal.  
The LED flashes blue. The number of times the LED flashes (up to 5), indicates the volume level.

## Receiving the audio stream from an external device

- ▶ Press  for about 3 seconds to turn the miniTek on.
- ▶ Use the on/off switch to turn the transmitter on.
- ▶ Use one of the supplied cables to connect the transmitter to an external device (refer to graphic on previous page).
- ▶ Connect the transmitter with the power plug to a power supply or charge the rechargeable battery regularly.
- ▶ Press  on miniTek to change to the audio program (see chapter “basic functions” for information about the sequence of external sources).  
 flashes while searching a connection to the transmitter and shines constantly after it is established. The LED on the transmitter confirms that the Bluetooth audio link is established.
- ▶ Start the audio transmission on your device.  
You can now hear the sound from your external device through your hearing instruments.

You can use a second transmitter, which is available as an accessory. This way the first transmitter can always remain connected to your preferred device while the second transmitter is connected with another device (e.g. a second TV in your bedroom).

## Using the line-in audio input

### WARNING

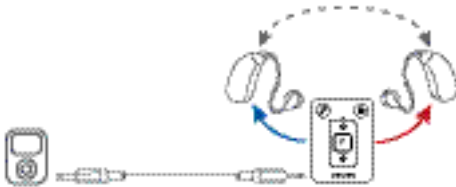
Risk of injuries caused by electric shock!

- ▶ Connect audio input only to a device conforming to IEC 60065 (IEC-Norm for audio, video and other electronic devices).

- ▶ While charging do not use the line-in audio input with devices that are connected to the power supply.

- ▶ Press  for about 3 seconds to turn miniTek on.
- ▶ Use the supplied cables and adapters to connect the external device and miniTek.
- ▶ Press  to change to the audio program (see chapter “basic functions” for information about the sequence of external sources).

If you press  without an audio device connected to it, miniTek will search for a connection to other audio sources.

## Using miniTek with a third party Bluetooth transmitter

miniTek can be connected via Bluetooth with a third party Bluetooth transmitter (e.g. iPhone, iPod, BlackBerry, mp3 player, laptop).

### Pairing

- ▶ Press  for about 3 seconds to turn miniTek on.
- ▶ Activate the Bluetooth function in your device.
- ▶ Press  on your miniTek for more than 5 seconds to prepare your miniTek for pairing.

When pairing mode has been activated, all 3 buttons (on/off button, telephone button and audio source button) shine for 3 seconds, then the telephone button flashes.

- ▶ In the menu of your device choose "Bluetooth setup".
- ▶ Choose "RCU Connect" to pair miniTek and the third party Bluetooth transmitter.
- ▶ Enter the PIN "0000" if requested.

EN

HU

RO

FR

DE

AR

## Using

- ▶ Start the audio transmission on your device.  
You can now hear sound from your device through your hearing instruments via Bluetooth.
- ▶ If the sound does not start automatically press  to change to the audio program (see chapter “Basic functions” for information about the sequence of external sources).

- ▶ Refer to the user guide of your connected device for more detailed information on how to link devices via Bluetooth.

## Using miniTek with a public loop system

Your miniTek is equipped with a telecoil. This is intended for using induction loops, e. g. in churches, theaters or lecture rooms.

- ▶ Press  for about 3 seconds to turn the miniTek on.
- ▶ Press  to change to the public loop receiver (see chapter "Basic functions").

You can now hear the sound from the public loop system through your hearing instruments.

The hearing instruments will switch automatically to the telephone program when your phone rings and will change back to the public loop system after telephoning (see chapter "Basic functions").

# Accessories

EN

HU

RO

FR

DE

AR

## FM transmitter (DAI)

Your miniTek can be connected to any standard FM receiver. This enables you to use FM systems even with hearing instruments which are not equipped with an audio shoe. FM systems help to improve speech understanding in difficult listening situations and are frequently used in schools.

- ▶ Press  for about 3 seconds to turn the miniTek on.
- ▶ Turn the FM transmitter on.
- ▶ Connect the FM receiver with the DAI port of the miniTek.
- ▶ Press  on miniTek to change to the FM transmitter (see chapter "Basic functions").

Now you can hear sound from the FM transmitter through your hearing instruments.

The hearing instruments switch automatically to the telephone program when your phone rings and change back to the FM transmitter after terminating the call (see chapter "Basic functions").

## Maintenance and care

### NOTICE

- ▶ Do not put your remote control in water!
- ▶ Do NOT clean your remote control with alcohol or benzine.

- ▶ Clean the remote control as necessary with a soft tissue.
- ▶ Avoid abrasive cleaning liquids with additives such as citric acids, ammonia, etc.

EN

HU

RO

FR

DE

AR

## Disposal information

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment."

Amended by "Directive 2003/108/EC" (WEEE).

- ▶ Recycle the device and accessories according to national regulations.

With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices and the European Directive 99/5/EC (R&TTE) concerning radio and telecommunications terminal equipment.

## Technical information

miniTek remote control

FCC: SGI-WL500

IC: 267AB-WL500

Operating frequencies:  $F_c=3.28$  MHz

Max. field strength @ 10m: -24.1 dB $\mu$ A/m

Max. field strength @ 30m: -6.2 dB $\mu$ V/m

CE, FCC and IC compliance information can be found on the back of the instrument.

N14203

This Class B digital apparatus complies with Canadian ICES-003.

Changes or modifications not expressly approved by SAT GmbH for compliance could void the users authority to operate the equipment.

### FCC compliance

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada.

Operation is subject to the following conditions:

- this device may not cause harmful interference, and
- this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15

of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### Siemens Transmitter

**IMPORTANT NOTE:** IC Radiation Exposure Statement This equipment complies with IC radiation exposure limits set forth for an uncontrolled environment.

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© Siemens AG, 11.2010 · ANR
Siemens Audiologische Technik GmbH