

CenTrak

Hand Washing Monitor

Hardware Installation


Guide

Warning:

This guide is in draft form and can be changed without notice at anytime by CenTrak. Please contact CenTrak prior to commercial use to receive all the necessary and latest information about the system.

Copyright © 2010 CenTrak, Inc. All rights reserved.

Caution: Any changes or modifications not expressly approved by the party responsible for product compliance could void the user's authority to operate the equipment.

	Document Name:	HWM Installation Guide
	Document Version Number:	201010CT203022MAN
	Effective Date:	10/01/10
	Confidential & Proprietary. DO NOT Copy or Distribute. Page 1 of 6	

How to Install the CenTrak Hand Washing Monitor Hardware



Items Needed..... 3

Small Round Soap Dispenser Installation..... 4

Large Round Soap Dispenser Installation 5

Flat Front Sanitizer Dispenser Installation..... 6



Small Round Dispenser



Large Round Dispenser



Flat Front Dispenser

How to Install the CenTrak Hand Washing Monitor Hardware



Items needed for Installation

- **CenTrak Hand Washing Monitor**



- **Rounded back, double stick adhesive mount (for use on round dispensers)**



- **Flat double stick adhesive mount (for use on flat front dispensers)**


CENTRAK HWM HARDWARE INSTALLATION GUIDE

Small Round Soap Dispenser

A-1. Peel off flat side of adhesive backing and firmly place flat side down onto Hand Washing Monitor as shown in Figure A-1.



Fig. A-1

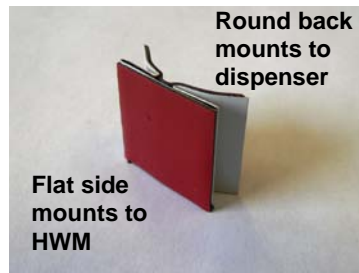

Press on firmly for 15 seconds.



Fig. A-2

A-2. Peel off curved side of adhesive covering and place Hand Washing Monitor (HWM) onto dispenser, making sure to mount more than 3 inches above push mechanism (where soap comes out).

Press firmly and hold for 15 seconds.

CENTRAK HWM HARDWARE INSTALLATION GUIDE



Fig. B-1

Large Round Soap Dispenser

B-1. Peel off flat side of adhesive backing and firmly place flat side down onto Hand Washing Monitor as shown in Figure B-1.

Press on firmly for 15 seconds.

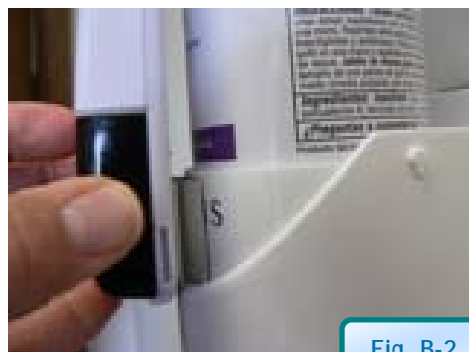
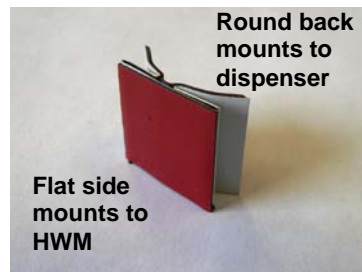


Fig. B-2

B-2. Peel off curved side of adhesive covering and place Hand Washing Monitor onto dispenser, making sure to mount above push mechanism (3" above where soap comes out).

Press firmly and hold for 15 seconds.

CENTRAK HWM HARDWARE INSTALLATION GUIDE

Flat Front Sanitizer Dispenser



Fig. C-1

C-1. Using the flat double stick mounting adhesive, Peel off adhesive backing and firmly place onto back of Hand Washing Monitor. Then remove backing from other side.



Fig. C-2

C-2. Place HWM onto dispenser, 4-5 inches above where soap comes out.

Press firmly and hold for 15 seconds.