

TD-4283
BLOOD GLUCOSE
MONITORING SYSTEM

OWNER'S Manual

Version 1.0
2012-02

Dear TD-4283 System Owner:

Thank you for purchasing the **TD-4283** Blood Glucose Monitoring System. This manual provides important information to help you to use the system properly. Before using this product, please read the following contents thoroughly and carefully.

Regular monitoring of your blood glucose levels can help you and your doctor gain better control of your diabetes. Due to its compact size and easy operation, you can use the **TD-4283** Blood Glucose Monitoring System to easily monitor your blood glucose levels by yourself anywhere, any time.

If you have other questions regarding this product, please contact the place of purchase or call the Customer Care Line.

IMPORTANT SAFETY PRECAUTIONS

READ BEFORE USE

1. Use this device **ONLY** for the intended use described in this manual.
2. Do **NOT** use accessories which are not specified by the manufacturer.
3. Do **NOT** use the device if it is not working properly or if it is damaged.
4. Do **NOT** use the equipment in places where aerosol sprays are being used or where oxygen is being administered.
5. Do **NOT** under any circumstances use the device on newborns or infants.
6. This device does **NOT** serve as a cure for any symptoms or diseases. The data measured is for reference only. Always consult your doctor to have the results interpreted.
7. Before using this device to test blood glucose, read all instructions thoroughly and practice the test. Carry out all the quality control checks as directed.
8. Keep the device and testing equipment away from young children. Small items such as the battery cover, batteries, test strips, lancets and vial caps are choking hazards.
9. Use of this instrument in a dry environment, especially if synthetic materials are present (synthetic clothing, carpets etc.), may cause damaging static discharges that may cause erroneous results.
10. Do **NOT** use this instrument in close proximity to sources of strong electromagnetic radiation, as these may interfere with the accurate operation.

KEEP THESE INSTRUCTIONS IN A SAFE PLACE

TABLE OF CONTENTS

BEFORE YOU BEGIN

Important Information

Intended Use

Test Principle

Meter Overview

Display Screen

Test Strip

Setting the Meter

THE FOUR MEASURING MODES

Control Solution Testing

Performing a Control Solution Test

TESTING WITH BLOOD SAMPLE

Preparing the Puncture Site

Alternative Site Testing

Performing a Blood Glucose Test

METER MEMORY

Reviewing Test Results

Reviewing Blood Glucose Day Average Results

Downloading Results onto a Computer

MAINTENANCE

Battery

Caring for Your Meter

Caring for Your Test Strips

Important Control Solution Information

SYSTEM TROUBLESHOOTING

Result Readings

Error Messages

Troubleshooting

DETAILED INFORMATION

SYMBOL INFORMATION

SPECIFICATIONS

BEFORE YOU BEGIN

Important Information

- Severe dehydration and excessive water loss may cause readings which are lower than actual values. If you believe you are suffering from severe dehydration, consult a healthcare professional immediately.
- If your blood glucose results are lower or higher than usual, and you do not have any symptoms of illness, first repeat the test. If you have symptoms or continue to get results which are higher or lower than usual, follow the treatment advice of your healthcare professional.
- Use only fresh whole blood samples to test your blood glucose. Using other substances will lead to incorrect results.
- If you are experiencing symptoms that are inconsistent with your blood glucose test results and you have followed all the instructions given in this owner's manual, contact your healthcare professional.
- We do not recommend using this product on severely hypotensive individuals or patients in shock. Readings which are lower than actual values may occur for individuals experiencing a hyperglycaemic-hyperosmolar state, with or without ketosis. Please consult your healthcare professional before use.
- The measurement unit used for indicating the concentration of blood or plasma glucose can either have a weight dimension (mg/dL) or a molarity (mmol/L). The approximate calculation rule for conversion of mg/dL in mmol/L is:

mg/dL	Divided by 18	= mmol/L
mmol/L	Times 18	= mg/dL

For example;

1) $120 \text{ mg/dL} \div 18 = 6.6 \text{ mmol/L}$

2) $7.2 \text{ mmol/L} \times 18 = 129 \text{ mg/dL}$ approximately.

Intended Use


This system is intended for use outside the body (*in vitro* diagnostic use) by people with diabetes at home and by healthcare professionals in clinical settings as an aid to monitoring the effectiveness of diabetes control. It is intended to be used for the quantitative measurement of glucose (sugar) in fresh whole blood samples (from the finger, palm, forearm, upper arm, calf and thigh). It should not be used for the diagnosis of diabetes, or testing on newborns.

Professionals may test with capillary and venous blood sample; home use is limited to capillary whole blood testing.

Test Principle

Your system measures the amount of sugar (glucose) in whole blood. The glucose testing is based on the measurement of electrical current generated by the reaction of glucose with the reagent of the strip. The meter measures the current, calculates the blood glucose level, and displays the result. The strength of the current produced by the reaction depends on the amount of glucose in the blood sample.

Meter Overview

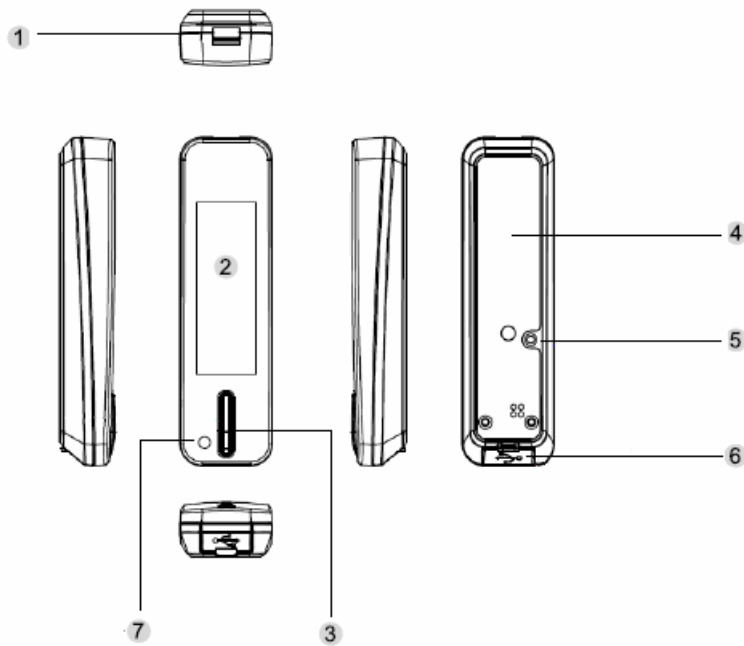

1. Test Slot

Insert test strip here to turn the meter on for testing.

2. Display Screen

3. Main Button

Enter the meter memory and silence a reminder alarm.

4. Battery Compartment

5. Set Button


Enter and confirm the meter settings.

6. USB Port (only for charge)



For battery recharge.

7. Bluetooth/Charging Indication Light

Display Screen

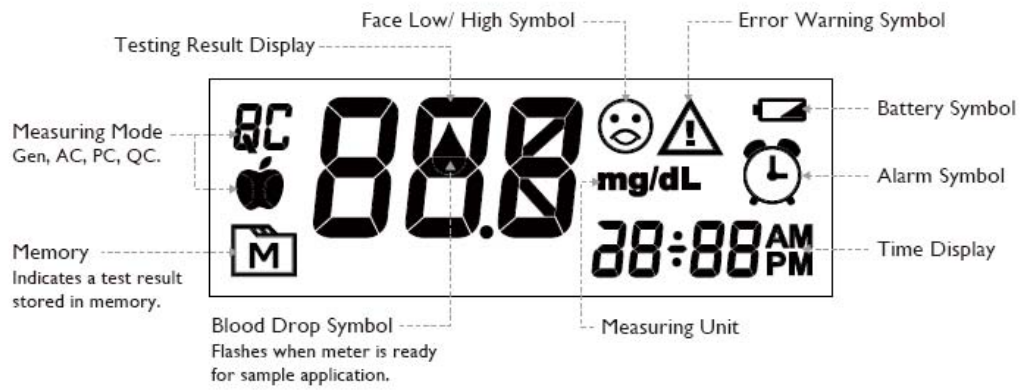

Test Strip

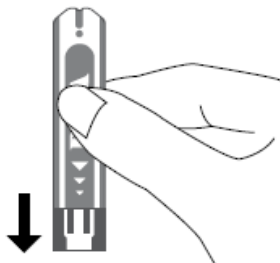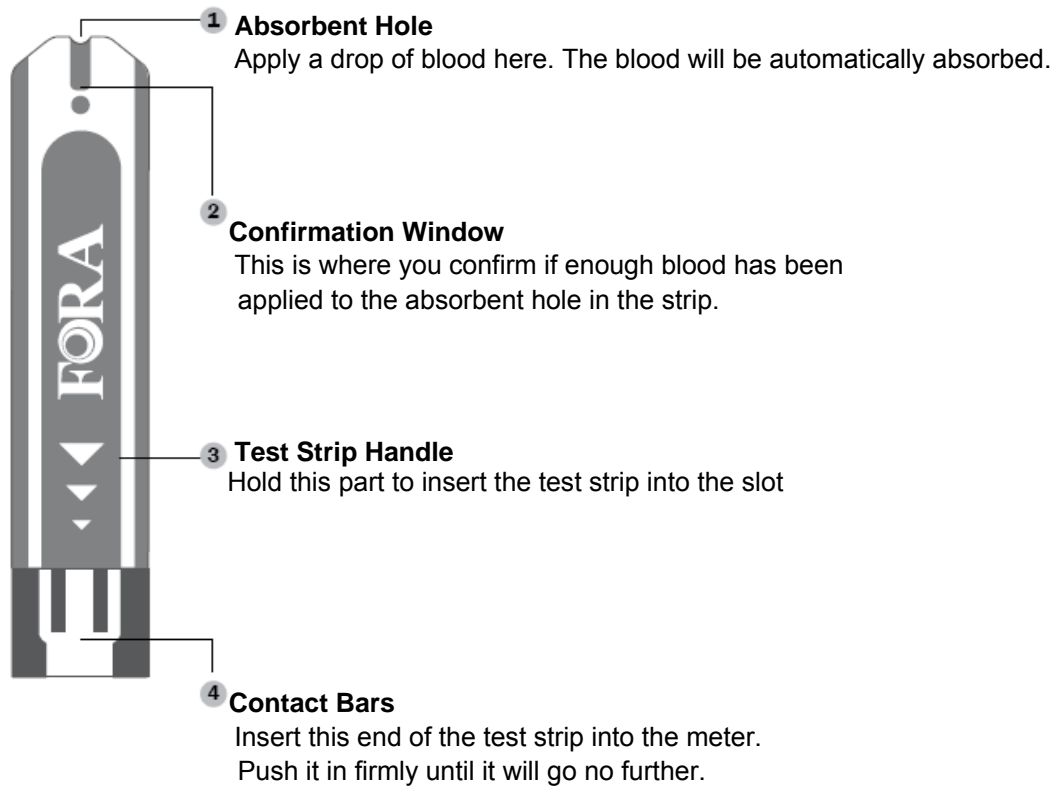

ATTENTION:

The front side of test of test strip should face up when inserting test strip.

Test results might be wrong if the contact bar is not fully inserted into the test slot.

NOTE:

The **TD-4283** meter should only be used with **TD-4283** Test Strips. Using other test strips with this meter can produce inaccurate results.

Setting the Meter

Before using your meter for the first time or if you change the meter battery, you should check and update these settings. Make sure you complete the steps below and have your desired settings saved.

Entering the Setting Mode

Start with the meter off (no test strip inserted). Press SET located on the back to turn on the meter.

1. Setting the Date

With the year flashing, press **Main Button** until the correct year appears. Press SET.



With the month flashing, press **Main Button** until the correct month appears. Press SET.



With the day flashing, press **Main Button** until the correct day appears. Press SET.



2. Setting the time format

Press and release **Main button** to select the desired time format --- 12h or 24h. Press SET.



3. Setting the time

With the hour flashing, press **Main Button** until the correct hour appears. Press SET.



With the minute flashing, press **Main Button** until the correct minute appears. Press SET.




4. Deleting the memory

With “dEL” and a flashing “no” symbol on the display, press SET to keep the results in memory.


To delete all the results, press **Main Button** with a flashing “yes” symbol on the display, press SET to delete all the results and “---” will be displayed.



5. Setting the reminder alarm

You may set up any or all of the reminder alarms (1-4). The meter displays “On” or “OFF” and , press **Main Button** to turn on or turn off to set the first reminder alarm.

Press Main Button to select “On”, then press SET to set the hour. When the hour is flashing, press **Main Button** to add an hour. Press SET to confirm and go to minutes, press **Main Button** to add one minute. Hold **Main Button** longer to add faster. Press SET to confirm and go to the next alarm setting.

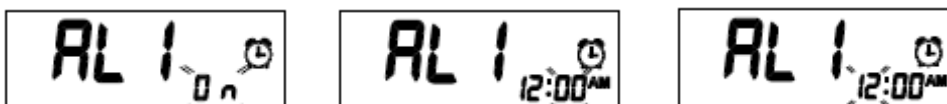

If you do not want to set an alarm, press SET to skip this step.

If you want to turn off an alarm, find the alarm number by pressing SET in the setting mode, press **Main Button** to change from “ON” to “OFF”.


At the time of your alarm, the meter will beep and automatically turn on. You can press **Main Button** to silence the alarm and insert a test strip to begin testing. If you do not press **Main Button**, the meter will beep for 2 minutes then switch off. If you do not want to test at this time, press **Main Button** to switch off the meter.


THE FOUR MEASURING MODES

The meter provides you with four modes for measuring: General, AC, PC and QC.


Modes	Use when
General tests	any time of day without regard to time since last meal
AC	no food intake for at least 8 hours
PC	2 hours after a meal
QC	testing with the control solution

You can switch between each mode by:

1. Start with the meter switched off. Insert a test strip to turn on the meter. The screen will display a flashing “”.

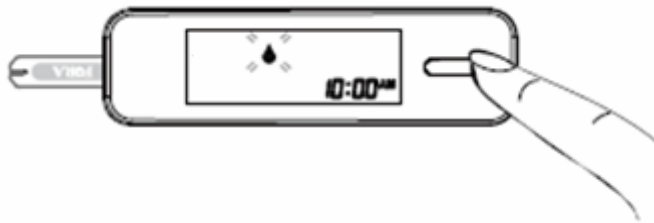

2. Press **Main button** to switch between General, AC, PC and QC mode.

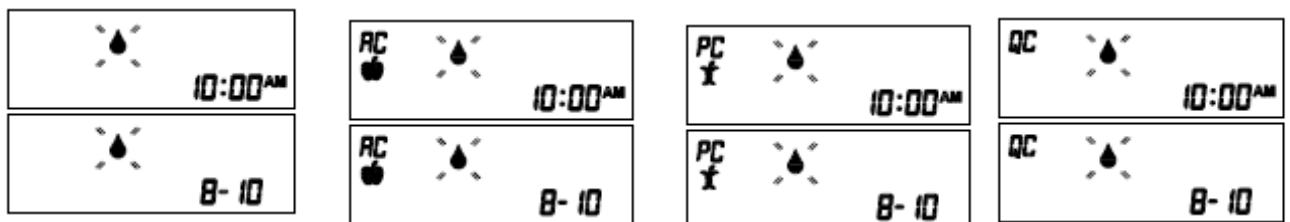

Control Solution Testing

Our Control Solution contains a known amount of glucose that reacts with test strips and is used to ensure your meter and test strips are working together correctly.

Test strips, control solutions, or sterile lancets may not be included in the kit (please check the contents on your product box). They can be purchased separately. Please make sure you have those items needed for a blood glucose test beforehand.

Do a control solution test when:

- you first receive the meter,
- at least once a week to routinely check the meter and test strips,
- you begin using a new vial of test strips,
- you suspect the meter or test strips are not working properly,
- your blood glucose test results are not consistent with how you feel, or if you think the results are not accurate,
- practicing the testing process, or
- you have dropped or think you may have damaged the meter.

Performing a Control Solution Test

You will need:

1. Meter




2. Test Strip



3. Control Solution




1. Insert the test strip to turn on the meter

Insert the test strip into the meter. Wait for the meter to display the “”.



2. Press Main Button to mark this test as a control solution test

With “QC” displayed, the meter will not store your test result in memory. If you press **Main Button** again, the “QC” will disappear and this test is no longer a control solution test.

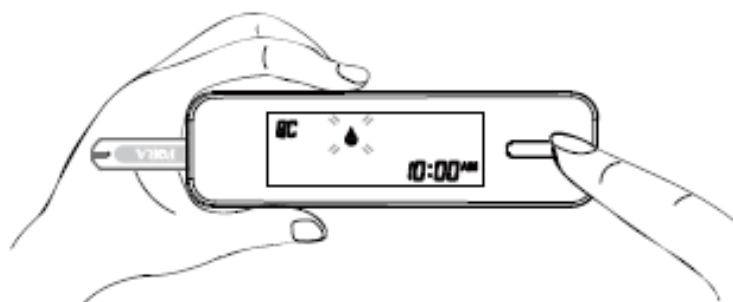

WARNING:

When doing the control solution test, you have to mark it so that the test result will not be stored in the memory. Failure to do so will mix up the blood glucose test results with the control solution test results in memory.

3. Apply Control Solution

Shake the control solution vial thoroughly before use.


Squeeze out a drop and wipe it off, then squeeze out another drop and place it on the tip of the vial cap. Hold the meter to move the absorbent hole of the test strip to touch the drop. Once the confirmation window fills completely, the meter will begin counting down. To avoid contaminating the control solution, do not directly apply control solution onto a strip.



4. Read and Compare the Result

After counting down to 0, the control solution test result will appear on the display. Compare this result with the range printed on the test strip vial and it should fall within this range. If not, please read the instructions again and repeat the control solution test.

(100 mg/dL = 5.6 mmol/L; 109-165 mg/dL = 6.1-9.2 mmol/L)

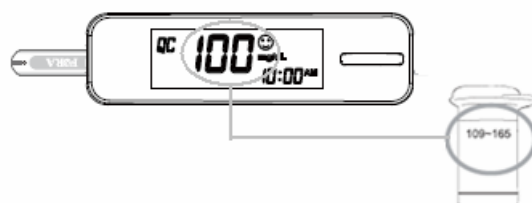

Out-Of-Range Results

If you continue to have test results fall outside the range printed on the test strip vial, the meter and strips may not be working properly. Do NOT test your blood. Contact your local customer service or place of purchase for help.

NOTE:

- The control solution range printed on the test strip vial is for control solution use only. It is not a recommended range for your blood glucose level.
- See the **Meter Maintenance** section for important information about your control solutions.

TESTING WITH BLOOD SAMPLE

Warning:

To reduce the chance of infection:

- Never share a lancet or the lancing device.
- Always use a new, sterile lancet. Lancets are for single use only.
- Avoid getting hand lotion, oils, dirt, or debris in or on the lancets and the lancing device.

Preparing the Lancing Device for Blood Testing

Please follow the instructions in the lancing device insert for collecting a blood sample.

Preparing the Puncture Site

Stimulating blood perfusion by rubbing the puncture site before blood extraction has a significant influence on the glucose value obtained.

Blood from a site that has not been rubbed exhibits a measurably different glucose concentration than blood from the finger. When the puncture site was rubbed prior to blood extraction, the difference was significantly reduced.


Please follow the suggestions below before obtaining a drop of blood:

- **Wash and dry your hands before starting.**
- Select the puncture site either at the fingertip or another part of the body (please see section “Alternative Site Testing” (AST) on how to select the appropriate sites).
- Clean the puncture site using cotton moistened with 70% alcohol and **let it air dry.**
- Rub the puncture site for about 20 seconds before penetration.
- Use a clear cap (included in the kit) while setting up the lancing device.



■ Fingertip testing

Press the lancing device's tip firmly against the lower side of your fingertip. Press the release button to prick your finger, then a click indicates that the puncture is complete.

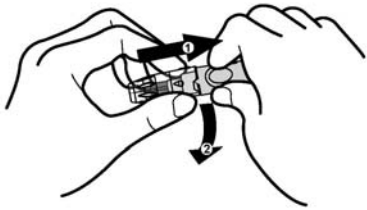

■ Blood from sites other than the fingertip


Replace the lancing device cap with the clear cap for AST. Pull the cocking control back until it clicks. When lancing the forearm, upper arm, hand, thigh, or calf, avoid lancing the areas with obvious veins because of the risk of excessive bleeding.

NOTE:

- Choose a different spot each time you test. Repeated punctures at the same spot may cause soreness and calluses.
- Please consult your health care professional before you begin AST.
- It is recommended that you discard the first drop of blood as it might contain tissue fluid, which may affect the test result.

Alternative Site Testing

You can test on a variety of locations on your body.

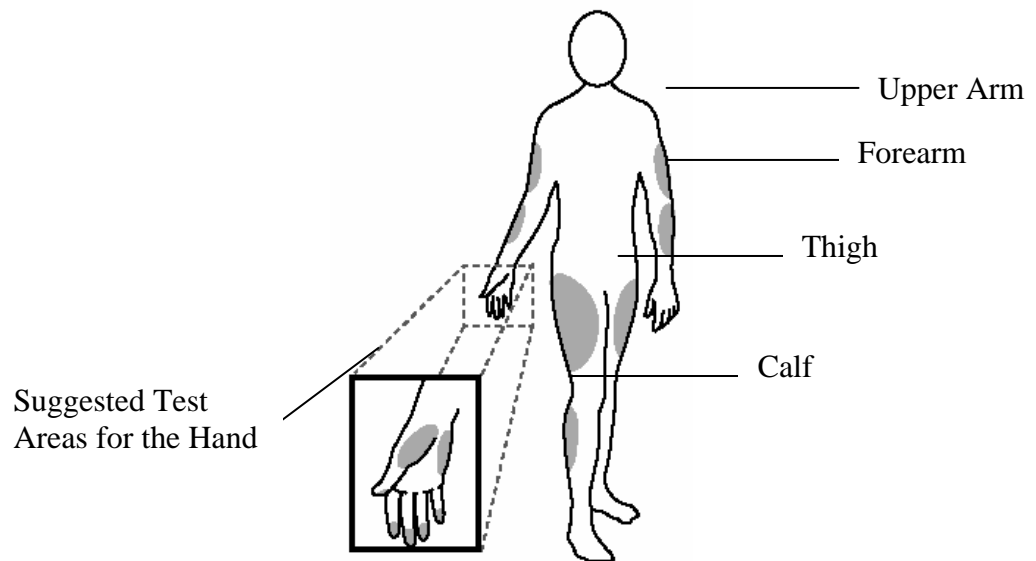

Important:

There are limitations with AST (Alternative Site Testing).

Please consult your health care professional before you perform AST.

When to use AST?

Food, medication, illness, stress and exercise can affect blood glucose levels. Capillary blood at the fingertip reflects these changes faster than capillary blood at other sites. Thus, when testing blood glucose during or immediately after a meal, physical exercise, or any other event, **take a blood sample from your finger only.**

We strongly recommend that you perform AST **ONLY** at the following times:

- In a pre-meal or fasting state (more than 2 hours since the last meal).
- Two hours or more after taking insulin.
- Two hours or more after exercise.

Do **NOT** use AST if:

- You think your blood glucose is low.
- You are unaware of hypoglycemia
- You are testing for hyperglycemia
- Your AST results do not match the way you feel.
- Your routine glucose results often fluctuate.

Performing a blood glucose test


You will need:

1. Meter
2. Test Strip
3. Lancing Device
4. Lancet



1. Insert the test strip to turn on the meter

Wait for the meter to display the “4”.

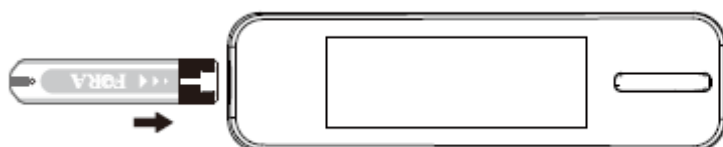

2. Select the appropriate measuring mode by pressing M.

For selecting the measurement mode, please refer to the “FOUR MEASURING MODES” section.

3. Obtaining a blood sample



✓ Correct




✗ Incorrect

Use the pre-set lancing device to puncture the desired site. Wipe off the first appeared drop of blood with a clean cotton swab. The size of the drop should be at least as big as (actual size), which is **0.5** microliter (μL) of volume. After penetration, discard the first drop of blood with a clean cotton swab. Gently squeeze the punctured area to obtain another drop of blood. Be careful **NOT** to smear the blood sample.

4. Apply the Sample

Gently apply the drop of blood to the absorbent hole of the test strip at a titled angle. Confirmation window should be completely filled if enough blood sample has been applied. Do **NOT** remove your finger until you hear a beep sound.

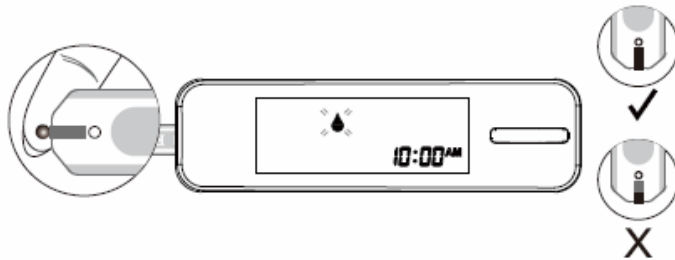


NOTE:

- Do not press the punctured site against the test strip or try to smear the blood.
- If you do not apply a blood sample to the test strip within 3 minutes, the meter will automatically turn off. You must remove and reinsert the test strip to start a new test.
- The confirmation window should be filled with blood before the meter begins to count down. **NEVER** try to add more blood to the test strip after the drop of blood has moved away. **Discard the used test strip and retest with a new one.**
- If you have trouble filling the confirmation window, please contact your healthcare professional or the local customer service for assistance.

5. Read Your Result

The result of your blood glucose test will appear after the meter counts down to 0. Blood glucose test results will automatically be stored in the memory. (100 mg/dL = 5.5mmol/L)

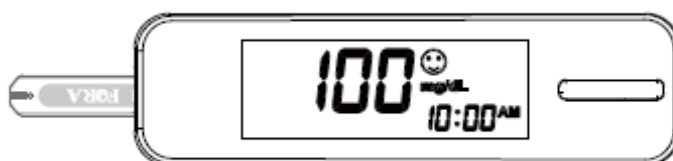

6. Remove the used test strip

Remove the test strip by hand, throw the strip into a disposal container. The meter will turn itself off automatically after the test strip is removed.

Always follow the instructions in the lancing device insert when removing the lancet.

WARNING:

The used lancet and test strip may be biohazards. Please discard them carefully according to your local regulations.

METER MEMORY

The meter stores the 450 most recent blood glucose test results along with respective dates and times in its memory. To enter the meter memory, **start with the meter switched off.**

Reviewing Test Results

1. Press and release Main Button

 will appear on the display. Press **Main Button**

again, and the first reading you see is the last blood glucose result along with date, time and the measurement mode.

(300 mg/dL = 16.7 mmol/L)




2. **Press Main Button** to recall the test results stored in the meter each time you press. After the last test results, press **Main Button** again and the meter will be turned off.

(200 mg/dL = 11.1 mmol/L)



Reviewing Blood Glucose Day Average Results

1. Press and release Main Button.

When  appears on the display, keep pressing **Main Button** for 3 seconds until the flashing “**A 09**” appears. Release **Main Button** and then your 7-day average result measured in general mode will appear on the display.
(58 mg/dL = 3.2 mmol/L)



2. Press **Main Button** to review 14-, 21-, 28-, 60- and 90- day average results stored in each measuring mode in the order of Gen, AC, then PC.

3. Exit the meter memory.

Keep pressing **Main Button** and the meter will turn off after displaying the last test result.

NOTE:

- Any time you wish to exit the memory, keep pressing **Main Button** for 5 seconds or leave it without any action for 3 minutes. The meter will switch off automatically.
- Control solution results are **NOT** included in the day average.
- If using the meter for the first time, “---“ displays when you recall the test results or review the average result. It indicates that there is no test result in the memory.

Downloading Results onto a Computer

Data Transmission Via Bluetooth


You can use your meter and Health Care Software System to view your test results on your personal computer. To learn more about the Health Care Software System, please contact local customer services or the place of purchase for assistance.

1. Obtaining the required cable and installing the software

To download the Health Care Software System, please visit the TaiDoc's website: www.taidoc.com

2. Connecting to a personal computer

With the monitor turned off, the meter will initiate the monitor's data transmission automatically. The Bluetooth light will flash.

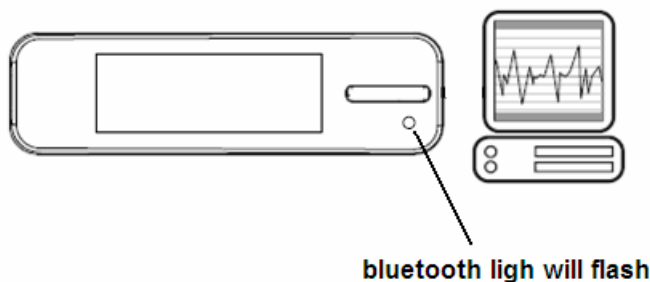

3. Data transmission

To transmit data, follow the instructions provided with the software. Results will be transmitted with date and time.

WARNING:

Make sure your PC has Bluetooth before transmitting the data and the monitor is within the receiving range.


MAINTENANCE



Battery

Your meter comes with a built-in rechargeable Li-polymer battery.

Low Battery Signal

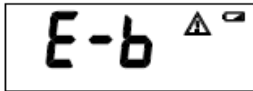The meter will display one of the messages below to alert you when the meter power is getting low.



The  symbol appears with E-b and :
The power is not enough to do a test. Please recharge the battery immediately.

Battery Recharge

1. Connect the USB cable to the USB port of the meter.
2. Connect the other end of cable to a free USB port on your personal computer.
3. The charging indicator will light up in red when battery is recharging. After the battery is fully charged, the red light will turn green.
4. Remove the USB cable and the meter will automatically switch off.

NOTE:

- It should take approximately 2 hours to be fully recharged. Recharging the battery does not affect the test results stored in the memory.
- If you wish to recharge through a wall outlet, please obtain a USB adapter.

Caring for Your Meter

To avoid the meter and test strips attracting dirt, dust or other contaminants, please wash and dry your hands thoroughly before use.

Cleaning

1. To clean the meter exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do NOT rinse with water.
2. Do NOT use organic solvents to clean the meter.

Meter Storage

- Storage conditions: -20°C to 60°C (-4°F to 140°F), below 95% relative humidity.
- Always store or transport the meter in its original storage case.
- Avoid dropping and heavy impact.
- Avoid direct sunlight and high humidity.

Caring for Your Test Strips

- Storage conditions: 2°C to 32°C (35.6°F to 89.6°F), below 85% relative humidity. Do **NOT** freeze.
- Store your test strips in their original vial only. Do not transfer to another container.
- Store test strip packages in a cool dry place. Keep away from direct sunlight and heat.
- After removing a test strip from the vial, immediately close the vial cap tightly.
- Touch the test strip with clean and dry hands.
- Use each test strip immediately after removing it from the vial.
- Write the opening date on the vial label when you first open it. Discard remaining test strips after 3 months.
- Do not use test strips beyond the expiry date. This may cause inaccurate results.
- Do not bend, cut, or alter a test strip in any way.
- Keep the strip vial away from children since the cap and the test strip may be a choking hazard. If swallowed, promptly see a doctor for help.

For further information, please refer to the test strip package insert.





Important Control Solution Information

- Use only our control solutions with your meter.
- Do not use the control solution beyond the expiry date or 3 months after first opening. Write the opening date on the control solution vial and discard the remaining solution after 3 months.
- It is recommended that the control solution test be done at room temperature 20°C to 25°C (68°F to 77°F). Make sure your control solution, meter, and test strips are at this specified temperature range before testing.
- Shake the vial before use, discard the first drop of control solution, and wipe off the dispenser tip to ensure a pure sample and an accurate result.
- Store the control solution tightly closed at temperatures between 2°C and 30°C (36°F and 86°F). Do **NOT** freeze.





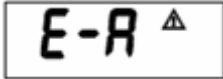
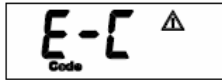
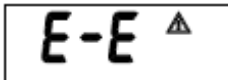
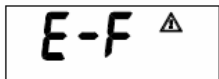
SYSTEM TROUBLESHOOTING

If you follow the recommended procedure but the problem persists, or error messages other than the ones below appear, please call your local customer service. Do not attempt to repair the meter yourself and never try to disassemble it under any circumstances.

Result Readings

MESSAGE	WHAT IT MEANS		
Lo	< 20 mg/dL (1.1 mmol/L).		
	20 - 69 mg/dL (1.1 and 3.8 mmol/L)		
	AC[®]	PC[†]	Gen
	70 - 129 mg/dL (3.9 – 7.2 mmol/L)	70 - 179 mg/dL (3.9 - 9.9 mmol/L).	70 - 119 mg/dL (3.9 - 6.6 mmol/L).
	AC[®]	PC[†]	Gen
	130 - 239 mg/dL (7.3 - 13.3 mmol/L)	180 - 239 mg/dL (10 - 13.3 mmol/L).	120 - 239 mg/dL (6.7 - 13.3 mmol/L).
	≥ 240 mg/dL (13.3 – 33.3 mmol/L)		
Hi	> 600 mg/dL (33.3 mmol/L).		

Error Messages

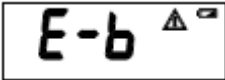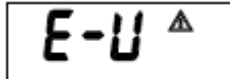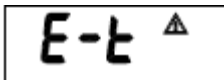MESSAGE	WHAT IT MEANS	WHAT TO DO
	Appears when the battery cannot provide enough power for a test.	Recharge the battery immediately.
	Appears when a used test strip is inserted.	Repeat test with a new test strip. If the error message re-appears, please contact your local customer service agent for help.
	The temperature of the environment, meter, or test strip is outside the system's operating range. You cannot perform a test until the meter and test strip are within the operating range of 10°C to 40°C (50°F to 104°F).	Repeat the test after the meter and test strip are within the operating temperatures.
   	Problem with the meter.	Review the instructions and try again with a new test strip. If the problem persists, please contact your local customer service agent for help.
	You may have removed the strip after applying blood to the absorbent hole.	Keep the test strip in the meter after it has absorbed the blood until the meter displays the test result.

Troubleshooting

1. If the meter does not display a message after inserting a test strip:

POSSIBLE CAUSE	WHAT TO DO
Battery exhausted.	Recharge the battery.
Test strip inserted upside down or incompletely.	Insert the test strip with contact bars end first and facing up.
Defective meter or test strips.	Please contact customer services.

2. If the test does not start after applying the sample:

POSSIBLE CAUSE	WHAT TO DO
Insufficient blood sample.	Repeat the test using a new test strip with a larger volume of blood sample.
Defective test strip.	Repeat the test with a new test strip.
Sample applied after automatic switch-off (2 minutes after last user action).	Repeat the test with a new test strip. Apply sample only when flashing "🔥" appears on the display.
Defective meter.	Please contact customer services.

3. If the control solution testing result is out of range.

POSSIBLE CAUSE	WHAT TO DO
Error in performing the test.	Read instructions thoroughly and repeat the test again.
Control solution vial was poorly shaken.	Shake the control solution vigorously and repeat the test again.
Expired or contaminated control solution.	Check the expiry date of the control solution.
Control solution that is too warm or too cold.	Control solution, meter, and test strips should be at room temperature (20°-25°C/68°-77°F) before testing.
Defective test strip.	Repeat the test with a new test strip.
Meter malfunction.	Please contact customer services.

DETAILED INFORMATION






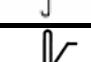









The meter provides you with plasma equivalent results.

Time of day	Normal plasma glucose range for people without diabetes (mg/dL)
Fasting and before meal	< 100 mg/dL (5.6 mmol/L)
2 hours after meals	< 140 mg/dL (7.8 mmol/L)

Source: American Diabetes Association (2010). Clinical Practice Recommendations. Diabetes Care, 33 (Supplement 1): S1–S100.

Please consult your doctor to determine a target range that works best for you.

SYMBOL INFORMATION

Symbol	Referent
	In vitro diagnostic medical device
	Do not reuse
	Consult instructions for use
	Keep away from sunlight
	Keep dry
	Temperature limitation
	Use by/ Expiry date
	Use within three months after first opening
	Batch code
	Manufacturer
SN	Serial number
	Caution, consult accompanying documents
	Authorised representative in the European Community
	Sterilised using irradiation
	Do not use if package is damaged
	CE mark

SPECIFICATIONS

Model No. : TD-4283

Dimension & Weight: 93.03mm (L) x 26.03 mm (W) x 5.57 mm (H),
27.2g (without battery)

Power source: Li-Polymer battery

Display: LCD

Memory: 450 measurement results with date and time

External output: Bluetooth

Automatic detection of electrode insertion

Automatic reaction time count-down

Automatic blood detection

Temperature warning

Operating condition: 10°C to 40°C (50°F to 104°F), below 85% R.H.

Storage/transport conditions: -20°C to 60°C (-4°F to 140°F), below 95%
R.H.

Measurement units: Fixed mg/dL

Measurement range: 20 to 600 mg/dL (1.1 to 33.3 mmol/L)

This device has been tested to meet the electrical and safety requirements of:
IEC/EN 61010-1, IEC/EN 61010-2-101, EN 61326-1, IEC/EN 61326-2-6.
EN 301 489-17, EN 300 328.

FEDERAL COMMUNICATIONS COMMISSION (FCC) STATEMENT

15.21

You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

15.105(b)

Federal Communications Commission (FCC) Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause **harmful** interference and
- 2) This device must accept any interference **received**, including interference that may cause undesired operation of the device.

FCC RF Radiation Exposure Statement:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.

End users must follow the specific operating instructions for satisfying RF exposure compliance.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.